
## HEINE GAMMA G5<sup>®</sup>, G7<sup>®</sup>, GP<sup>®</sup>, XXL LF<sup>®</sup>

CE 0297

HEINE Optotechnik GmbH & Co. KG  
Kientalstr. 7 · 82211 Herrsching · Germany  
Tel. +49 (0) 81 52 / 38 - 0  
Fax +49 (0) 81 52 / 38 - 2 02  
E-Mail: info@heine.com · www.heine.com  
med 0708 1/1.12

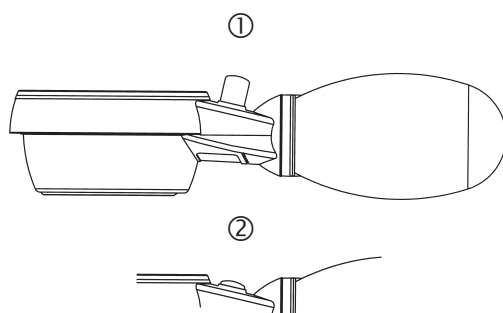## Instructions

ENGLISH

Read these instructions carefully before using the HEINE Sphygmomanometer and keep them in a safe place for future reference.

### Safety Information

**Range of applications:** HEINE GAMMA G<sup>®</sup> aneroid sphygmomanometers are exclusively designed for measuring blood pressure at the upper arm or thigh on healthy skin.

**General safety:** Protect the instrument from impact or shocks ( do not drop) and protect from moisture and dirt. Avoid damaging the cuff with pointed objects such as needles or scissors.

Do not use the sphygmomanometer in the vicinity of strong magnetic fields. Do a regular inspection of the devices and its components regarding to its function.

Do not use the sphygmomanometer if fit is defective in any way.

Do not expose the sphygmomanometer to direct sunlight for long periods of time.

**Important note: All components of this sphygmomanometer are latex-free, the inflation bulb is also phthalate-free.**

### Safety in use:

Do not take your blood pressure for more than 2 minutes!

Wait for at least 2 minutes before taking a repeat measurement!

Accurate performance of the sphygmomanometer can only be guaranteed if original HEINE G-series latex-free cuffs are used. The use of other cuffs is done so entirely at the liability of the user.

A rectangular label is attached to the back of the instrument which carries a year-mark. This shows the year in which the instrument should be re-calibrated. Please refer to the enclosed assembly instructions.

### Fitting the cuff

Fit the cuff so that the lower edge is about 2-3 cm above the elbow (or about 5 cm above the knee). Locate the special mark over the artery. The white index mark should be located in the marked index area.

### Measurements

#### GAMMA G5, G7, XXL LF

Close the air release valve by turning to the right. Pump up the cuff to a value approximately 30 mmHg greater than the expected systolic pressure.

Open the valve slowly and regulate the air release to 2–3 mmHg per second.

After the measurement, open the valve entirely in order to quick-release the air from the cuff.

#### GAMMA GP

Press the button to close the valve completely ①. Pump up the cuff to a value approximately 30 mmHg greater than the expected systolic pressure. Open the valve slowly and regulate the rate of air release to 2–3 mmHg per second. After the measurement, open the valve completely in order to quick-release the air from the cuff ②.

**Please note:** It is not possible to pump up the cuff with the button in position ②.

### Changing the position of the spoon (GAMMA G5, G7, GP)

The spoon is normally set for right-handed use. If you wish to inflate the cuff with the left hand, the spoon can be re-positioned for left-handed use:

- Remove the ball from the instrument with a twisting and pulling motion.
- Loosen the two screws near the air intake but do not screw them out completely.

(A special tool is supplied with GAMMA G7 for this purpose).

- Rotate the base of the spoon through 180° and re-tighten the screws.
- Grip the neck of the ball and push it back onto the air intake with a twisting motion. If necessary moisten the air intake to ease fitting.

### Preparation

The following recommendations must be followed in procedures in hospitals / practises with regard to national hygienic guidelines and / or legal requirements.

#### Manual Cleaning

Clean all accessible surfaces by wiping with a damp cloth (water, pH-neutral, light alkaline detergent). The cuff could be hand washed at 30°C after removing the bladder.

#### Manual Disinfection

Do a cleaning disinfection of all accessible surfaces by wiping with a damp cloth. For disinfection only those products which are approved for use on medical devices made of plastic and aluminium alloy (GAMMA G7) may be use. The manufacturer's recommendations must be followed.

Sphygmomanometer:

Liquid must not penetrate the device. Disinfection by spraying and soaking is not allowed.

Cuff / Bladder:

Cuff and Bladder are released to for a disinfection by spraying and soaking.

#### Machine Cleaning / Ultrasonic Cleaning / Sterilisation

Sphygmomanometer, cuff and bladder are not permitted for these procedures as they will lead to damage of the components.

### Technical data

Range:	0–300 mmHg
Accuracy:	± 3 mmHg
Environmental conditions:	
Normal Use:	0 °C to +40 °C
Storage:	-34 °C to +70 °C
Relative humidity:	≤ 85%

**Note:** A calibration check should be carried out at the latest every two years by the manufacturer or an authorized service technician.

### Applicable standards

DIN EN 1060: "Non-invasive sphygmomanometers",  
Part 1: General requirements, and  
Part 2: Supplementary requirements for  
mechanical sphygmomanometers  
ANSI/AAMI SP9: 1994  
ANSI/AAMI/ISO 81060-1:2007

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