



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

PRESSAN®

Manuale utente - User manual - Notice d'utilisation - Manual de uso

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.


ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

 **GIMA**

ENGLISH

PRESSAN® is a practical hygienic accessory for the sphygmomanometer whilst measuring the blood pressure; it is made of a white non-woven fabric, to ensure the patient both comfort and hygiene

As illustrated in **Fig. 1**, before commencing measurement, PRESSAN® is slipped over the person's hand in the direction of the F arrow and placed on the upper arm. Measurement of pressure can now commence, applying the cuff of the sphygmomanometer over the PRESSAN® (**Fig. 2**). On conclusion of the measurement, PRESSAN® is slipped off the arm and discarded.

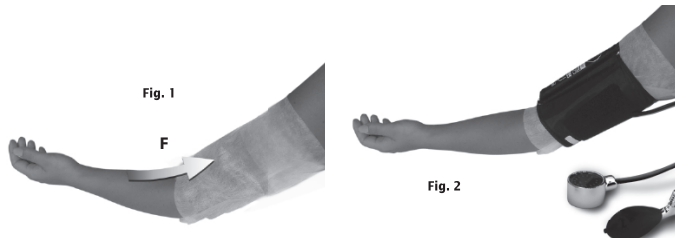

Its use does not in any way alter the blood pressure values of the patient. Its regular substitution at every blood pressure measurement ensures best hygiene both for the sphygmomanometer and for the arm of the patient. It is a light sleeve, foldable and small, which can be used in a doctor's surgery, in hospital or at the chemist's, but could also be carried in a health visitor's bag for measuring blood pressure on home visits.

Fabbricante/Manufacturer/Fabricant/Fabricante
LEVETTA s.a.s. di Maffei Michele & C.
Via dei Cancellieri, 6/8
51100 Pistoia (Italy)
Made in Italy

REF 5208 (GIMA 32858)