



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

## Trolley for DALLAS sphygmo

**REF** 32906



M32906-en-rev.0-09.19




Gima S.p.A. - Via Marconi, 1  
20060 - Gessate (MI) - Italy  
Made in Pakistan


## Assembling insrtuction

The package consists of:


- Fixing rod
- 2 bars with wheel with brake
- 3 bars with wheel without brake
- Central pin for fixing bars
- 5 screws for fixing the bars

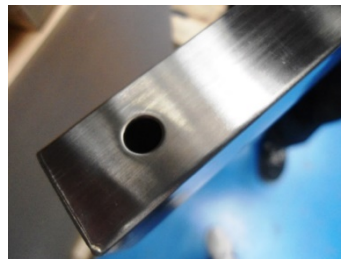

1. Insert the bars into the central fixing bracket while keeping the hole downwards. It is recommended to place all 5 bars before proceeding to the next step. When all the bars are in position, insert the screws, when all the bars are well secured, tighten the screws well..

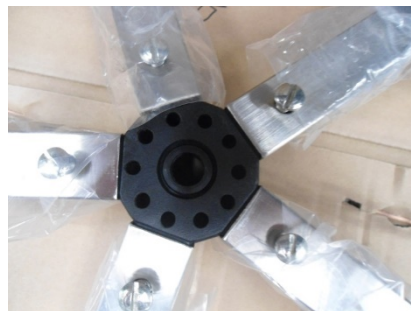

2. To insert the rod, proceed by removing the screw located on the bottom of the rod, insert the rod and secure it to the central pin, making sure that you have tightened the screw fixing

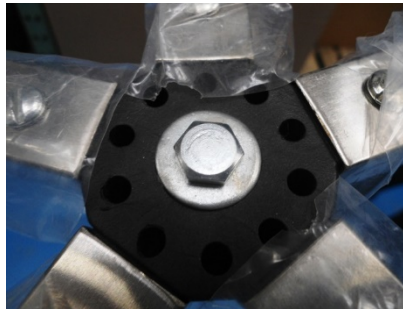









3. Connect the rod to the hole underneath the Dallas sphincter and turn it to the end of the stroke.



## Cleaning

Clean the surface with a damp cloth or soap and water.

## Description of symbols

	Product code
	Caution: read instructions (warnings) carefully
	Keep away from sunlight
	Keep in a cool, dry place
	Medical Device complies with Directive 93/42/EEC
	Manufacturer
	Lot number
	Consult instructions for use