



GIMA

PROFESSIONAL MEDICAL PRODUCTS

INDICATIONS

The Tondaus Surgical Skin Marker is intended to be used to mark on human skin for positioning identifications in Ordinary Surgery, Enterochirurgia, Orthopedics, Neurohormone Surgery, Cardiovascular Surgery and Radiotherapy.

CONTRAINDICATIONS

Do not apply on damaged mucous membranes and skin or when wounds are present.

CAUTION AND WARNING

- 1). Patients should be considered for allergic reactions to Gentian violet/Allura Red AC/Indigo/Titanium dioxide(corresponding to different colors). During use, if the patient is found to be unwell, stop using it immediately and use it properly.
- 2). This product can be used only once in a single patient to avoid cross infection.
- 3). This product should be used on dry skin. If the patient's skin is wet or sweaty, please dry the skin and keep the skin. Dry and fresh, then follow the instructions above.
- 4). Pay attention to the protection of the pen tip when using it. When not in use, cover the pen cover to prevent dry ink.
- 5). The package should be used immediately after opening. Do not use if the package is damaged or stained. After the product is used, it should be disposed as medical waste.
- 6). Do not soak or wipe the product with an organic solvent. Organic solvents may damage the product.
- 7). Operate by trained medical professionals.
- 8). Dispose of as medical waste after use.

DIRECTIONS FOR USE

- 1). Check the integrity of the inner package prior to use. Do not use if the inner package breaks. 2). Disinfect and dry the skin first (note: be sure to use it on dry skin);
- 2). Use a skin marker to mark the skin;
- 3). To remove the mark, use 75% alcohol to erase.

STORAGE AND TRANSPORTATION CONDITIONS

- 1). This product should be stored in a cool, ventilated, dry place, avoiding contact with water, direct sunlight, extreme temperature or humidity, and prevent contact with corrosive gases. Temperature: $\leq 40^{\circ}\text{C}$, humidity: $\leq 90\%$, pay attention to anti-rat, pest control, to prevent damage to the packaging.
- 2). One-time use of medical skin markers allows the use of general transportation vehicles, but must prevent severe shocks, vibrations and rain and snow during transportation.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

	IT - Data di fabbricazione GB - Date of manufacture FR - Date de fabrication ES - Fecha de fabricación PT - Data de fabrico DE - Herstellungsdatum GR - Ημερομηνία παραγωγής DK - Fabrikationsdato FI - Valmistuspäivämäärä NL - Productiedatum PL - Data produkcji CZ - Datum výroby BG - Дата на производство EE - Valmistamise kuupäev HR - Datum proizvodnje HR - Datum proizvodnje HU - Gyártás dátuma LT - Pagaminimo data LV - Izgatavošanas datums NO - Fabrikasjonsdato RO - Data fabricației SK - Datum výroby SL - Datum proizvodnje SV - Tillverkningsdatum AR - تاريخ التصنيع
	IT - Fabbricante GB - Manufacturer FR - Fabricant ES - Fabricante PT - Fabricante DE - Hersteller GR - Παραγωγός DK - Fabrikant FI - Valmistaja NL - Fabrikant PL - Producent CZ - Výrobce BG - Производител EE - Tootja HR - Proizvajalec HU - Gyártó LT - Gamintojas LV - Ražotājs NO - Produsent RO - Producător SK - Výrobca SL - Proizvajalec SV - Tillverkare AR - مصنع
	IT - Data di scadenza GB - Expiration date FR - Date d'échéance ES - Fecha de caducidad PT - Data de validade DE - Ablaufdatum GR - Ημερομηνία λήξεως DK - Udløbsdato FI - Viimeinen voimassaolopäivä NL - Vervaldatum PL - Data ważności CZ - Datum ukončení platnosti BG - Срок на годност EE - Aegumiskuu päev HR - Rok uporabnosti HU - Lejárati dátum LT - Galiojimo laikas LV - Derīguma termiņš NO - Utløpsdato RO - Valabil până la data de SK - Datum expirácie SL - Rok uporabnosti SV - Utgångsdatum AR - تاريخ الصلاحية
REF	IT - Codice prodotto GB - Product code FR - Code produit ES - Código producto PT - Código produto DE - Erzeugniscode GR - Κωδικός προϊόντος DK - Produktkode FI - Tuotekoodi NL - Productcode PL - Numer katalogowy CZ - Kód výrobku BG - Код на продукт EE - Toote kood HR - Koda izdelka HU - Termékkód LT - Prekės kodas LV - Produkta kods NO - Produktkode RO - Cod produs SK - Kód výrobku SL - Koda izdelka SV - Produktkod AR - كود المنتج



0123



MD

STERILE R



GIMA 33183

GIMA 33184



Dogguan Tondaus Meditech Co., Ltd.
3rd Floor, No.27 Rd. Guantai, Chiling
Community, Houjie Town,
Dogguan, Guangdong, China.
Made in China

REF

TR03

TD01

EC REF

MedPath GmbH
Mies-van-der-Rohe-Strasse 8
80807 Munich, Germany



Gima S.p.A
Via Marconi, 1- 20060 Gessate (MI) - Italy
export@gimaitaly.com - gima@gimaitaly.com
www.gimaitaly.com

CE
0123