

case of deterioration of external insulation and connections and/or oxidation of connectors or plugs.

NB: Snap connection models must be immediately destroyed in case electrode doesn't stick perfectly to cable connection.

STORAGE

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

GENERAL NOTES

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

WASTE DISPOSAL

Refuses deriving from health structures must be disposed in according to the regulation in force. While the disposal of refuse deriving from civil residences does not require observance of any particular precautions of the regulations in force regarding urban refuse, it is recommended that the product be treated with disinfectant solutions (chlorine oxides, sodium hypochlorite, etc.) before being disposed of.

RX ONLY-CAUTION: U.S.A. FEDERAL LAW RESTRICTS THIS DEVICES TO SALE BY OR ON THE ORDER OF A PHYSICIAN.

EN

ADAPTERS FOR CONNECTOR CABLES FOR ECG AND ELECTROSTIMULATION

DESCRIPTION AND INDICATIONS

These articles have been designed to enable the adaptation of electrode connector cables which, given the type of connection they are equipped with, would not normally be compatible with the basic equipment cables. The FIAB adapters can be used for the connection of both disposable and multi-use electrodes to equipment for electro-stimulation and for electro-cardiac recording.

The products must be used by qualified healthcare personnel.

INSTRUCTIONS FOR USE

Carefully check packaging, cables and connection plugs. Do not use the product in case of visible damages or defects.

- Having checked the compatibility of the connections, insert the cable connector in the chosen adapter.
- Proceed to apply the electrode to the patient as described in the instructions supplied by the electrode manufacturer.

NB: make the connections with the equipment turned off.

- If it is not required for other use, leave the adapter inserted in the cable-head even when the equipment is not in use.
- To effect the disconnection of the adapter from the cable head, exert opposing tractive force between the adapter cable shaft and the cable-head shaft to which the adapter is connected.

WARNING

- **The product is supplied non-sterile.**
- Do not make connections or disconnection when the equipment is turned on.
- The equipment for electro-stimulation and the electrodes used with this device must all be compliant with the regulations in force.

NB: Only ever use equipment designed for electro-stimulation at low current intensity.

- The device must be connected and started up by qualified personnel.










CLEANING

For cleaning detergent solutions (non-ionic surface-actives) and disinfectant solutions (components with quaternary ammonium of 0.1 - 0.2%) must be used. For the use of these products follow the relative instructions. After cleaning rinse under running water.

NB: Dry well before use.

LIFE OF PRODUCT

The life of the products depends on the number of uses and on maintenance type. The products must be immediately destroyed in

	IT	EN	DE	ES	FR	RU	PT
	Conforme alla vigente normativa Europea sui Dispositivi Medici	Compliant with current European legislation on Medical Devices	Entspricht der aktuellen europäischen Gesetzgebung für Medizinprodukte	Cumple con la legislación europea vigente sobre dispositivos médicos	Conforme à la législation européenne en vigueur sur les dispositifs médicaux	Соответствует действующему европейскому законодательству о медицинских устройствах	Em conformidade com a legislação europeia em vigor sobre Dispositivos Médicos
	Consultare le istruzioni d'uso	Consult instructions for use	Gebrauchsanweisung lesen	Consulte las instrucciones de uso	Consulter les instructions d'utilisation	Обратитесь к инструкции по применению	Consulte as instruções de utilização
REF	Numero di catalogo	Catalogue number	Katalognummer	Número de Catálogo	Code de référence	Каталожный номер	Número de Catálogo
LOT	Numero di lotto	Batch code	Posten-Nummer	Número de Lote	Numéro de lot	Код партии	Número de Lote
	Scadenza	Use by	Verfalldatum	Fecha de Caducidad	Date de péremption	Использовать до	Prazo de Validade
	Fabbricante	Manufacturer	Hersteller	Fabricante	Fabricant	Производитель	Fabricante
	Distributore	Distributor	Verteiler	Distribuidor	Distributeur	Дистрибьютор	Distribuidor
	Data di produzione	Date of manufacture	Herstellungsdatum	Fecha de Producción	Date de production	Дата изготовления	Data de Fabrico
	Limiti di temperatura	Temperature limitation	Temperaturbereich	Límites de Temperatura	Limites de température	Температурные ограничения	Limites de Temperatura
	Proteggere dalla luce solare	Keep away from sunlight	Vor Sonnenlicht schützen	Mantener alejado de la luz solar	Tenir à l'abri de la lumière du jour	Беречь от солнечных лучей	Manter longe da luz solar
	Limiti di umidità	Humidity limitation	Feuchtigkeitsbereich	Límites de Humedad	Limites d'humidité	Ограничения по влажности	Limites de Humidade
MD	Dispositivo medico	Medical Device	Medizinprodukt	Producto sanitario	Dispositif médical	Медицинское устройство	Dispositivo médico
UDI	Identificativo unico del dispositivo	Unique Device Identifier	Einmalige Produktkennung	Identificador único del producto	Identifiant unique des dispositifs	Уникальный идентификатор устройства	Identificação única do dispositivo