



**MEDICAL RECORDING PAPERS  
INSTRUCTIONS**

**APPLICATION:** medical recording papers are projected to recorder traces of physiologic parameters of medical equipments during diagnostic analysis. Use is reserved only at medical and paramedical authorized operators with competence in medical recording equipments.

**PRODUCT IDENTIFICATION:**

The identification data ( device code, LOT production and mark) are in the identification label stuck on the external packaging.

**USE AND INSTALLATION:**

- ✓ Before using this medical device it is always necessary to control that paper size and paper grid (if present) are concerning the use of the instrument.
- ✓ Avoid the use of cutting tools in packaging opening.
- ✓ Before using this medical device it is always necessary to consult the instructions concerning the use of the instrument.
- ✓ Paper (roll or z-fold) has to be placed comfortable in the paper hold of the instruments in order to not obstruct paper dragging. Never force paper introduction in the instrument.
- ✓ The grid, if present and except any different indication, use the mm unit and  $\pm 1\%$  precision.

**STORAGE CONDITIONS:**

- ✓ Before use,

store the paper in a fresh and dry place.  
Sauf si autrement spécifié sur le diagramme, la Long exposures to condensate and temperature dimension de l'éventuel réseau imprimé est exprimée en millimètres et elle a un degré de précision  $\pm 1\%$ .  
en higher or lower than what indicated in label could millimètres et elle a un degré de précision  $\pm 1\%$ .  
cause adverse effects to the paper.  
Avoid the exposures to direct sunlight, bright lights

- ✓ After registration,

to assure long time recording preservation:  
Store the paper in a cool and dry place after use.  
Avoid the exposures to direct sunlight, bright lights (UV).  
Avoid any contact with liquids and solvents such as alcohol, ketone, ligroin or similar.  
Avoid any contact with other kind of chemical and thermal paper.  
Do not insert the paper in any envelopes made of PVC ,but use qualified envelopes.

**DISCHARGING CONDITIONS**

Use, storage and discharging of the paper do not cause any environmental pollution.  
Papers discharging can be made in the appropriate boxes for recycling.  
The inflammability of this product is similar to normal uncoated papers. To extinguish incidental fires of this product use water or normal fire-extinguisher.

**CONTRAINDICATIONS:**

On the basis of previous experiences and several analysis results, a correct use of the product cannot cause any noxious effect.

