


# NOTICE

## Battery Replacement Method

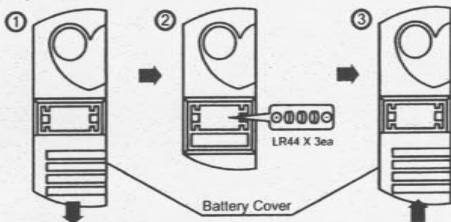

- ① Open the battery compartment cover.
  - ② Insert the LR44 battery as shown in the picture.  
Ensure that the batteries are inserted with correct polarity.
  - ③ Close the battery compartment cover.
- △ CAUTION : Do not use new and used batteries together.

- The battery for the audible alarm lasts for about 30 minutes if the door remains open.  
- Battery Type : LR44 X 3EA
- Turn on the alarm button on the low right.



CU Medical Systems, Inc.

[www.cu911.com](http://www.cu911.com)



### • Head office

Dongwha Medical Instrument Complex  
1647-1 Dongwha-ri, Munmak-eup, Wonju-si,  
Gangwon-do, 220-801 Republic of Korea  
TEL : +82 31 578 5725 / FAX : +82 31 578 5729

### • Europe

Branch Office of CU Medical Systems, Inc. in Germany  
Kuester Strasse 6, 30519 Hannover, Germany  
TEL : +49 511 365 4353 / FAX : +49 511 848 6054

English