




FERRARIS  
CardioRespiratory

# POCKET *Valved Aerosol Holding Chamber* CHAMBER™

for use with Metered Dose Inhalers  
voor gebruik met inhalers met afgepaste dosis  
zur Verwendung in Verbindung mit dem Dosierinhalator per aerosol dosati  
per aerosol dosati  
para utilização com Inaladores de Doses Calibradas  
para utilizar con inhaladores dosificadores  
pour utilisation avec les inhalateurs doseurs  
Όλζύλú Doz Inhaler kullanimi için  
για χρήση με δοσιμετρικές συσκευές εισπνοών

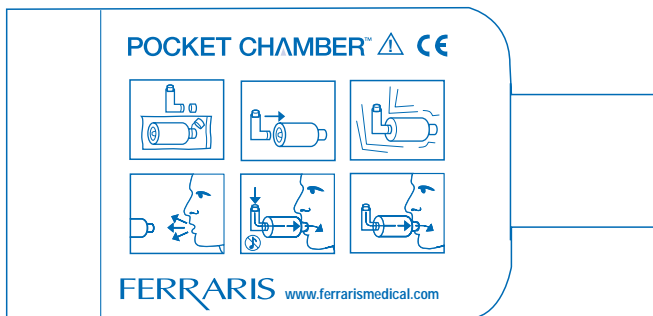

INSTRUCTIONS FOR USE  
GEBRUIKSAANWIJZING  
GEBRAUCHSANLEITUNG  
ISTRUZIONI D'USO  
INSTRUÇÕES DE UTILIZAÇÃO  
INSTRUCCIONES DE USO  
MODE D'EMPLOI  
KULLANIM AÇIKLAMALARI  
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

*treat-asthma.com*


*ferrarismedical.com*

## For Use with Metered Dose Inhalers


Before using the POCKET CHAMBER, please read the instructions carefully.

The purpose of the POCKET CHAMBER is to reduce the aerosol medication dose from the Metered Dose Inhalers (MDI) by dispersing large aerosol particles to a dose that is more effectively inhaled.


### Instructions for Use

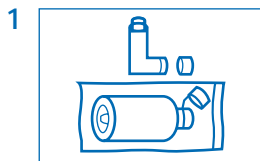


Remove POCKET CHAMBER from bag. Remove caps from both the MDI and the POCKET CHAMBER.

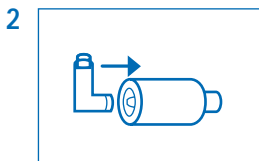


Insert the MDI into the flexible MDI end of the POCKET CHAMBER.

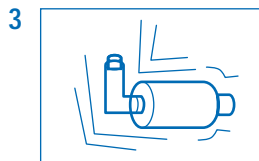


Shake the assembly vigorously.

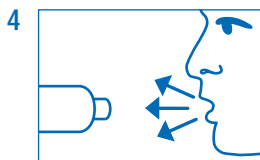

Exhale slowly and deeply then place the POCKET CHAMBER in your mouth.

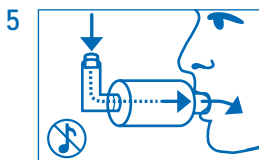

Depress the MDI canister and spray one puff into the POCKET CHAMBER while inhaling slowly, so that you do not activate the whistle.

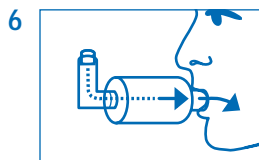

Continue to breathe in slowly and fully.

7 Try to hold your breath for 5 to 10 seconds.

8 Exhale slowly.

9 Repeat Steps 3 - 8 as many times as instructed by your physician.

### Technical Information

The POCKET CHAMBER helps to deliver the MDI medication. The POCKET CHAMBER's recessed baffle plate reduces the non-respirable (non-therapeutic) particle count that would otherwise deposit in your throat or mouth. At the same time, the POCKET CHAMBER delivers an adequate dose of respirable (therapeutic dose) MDI aerosol medication that will be easily inhaled into the lungs. This process helps to reduce unwanted side effects and promotes an effective treatment. If you should have any questions or problems using the POCKET CHAMBER, please contact your physician.

### Warnings and Cautions

The POCKET CHAMBER should be replaced if:

1. The body is cracked
2. The flexible MDI end has split
3. The POCKET CHAMBER's recessed baffle plate is cracked or missing

### Cleaning

With repeated use, medication powder will build up on the POCKET CHAMBER's inner surfaces. This may interfere with the performance of the device. The device along with the MDI plastic elbow should be washed at least once a week using warm water and a liquid dishwashing detergent. Place the whole unit in warm soapy water before trying to remove flexible MDI end. Then wash the unit, rinse with warm clear water and allow to air dry.

**⚠** SEE INSTRUCTIONS FOR USE  
ZIE GEBRUIKSAANWIJZING  
SIEHE GEBRAUCHSANWEISUNG  
SEGUIRE LE ISTRUZIONI PER L'USO  
CONSULTAR AS INSTRUÇÕES DE UTILIZAÇÃO  
VEA LAS INSTRUCCIONES PARA EL USO  
VOIR LES INSTRUCTIONS POUR L'USAGE  
KULLANIM İÇİN AÇIKLAMALARA BAKINIZ  
ΔΕΙΤΕ ΤΙΣ ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ

Contents : 1 Unit for single patient use non sterile class 1 medical device

Inhoud: 1 Eenheid voor eenmalig gebruik door de patiënt niet-steriel klasse 1 medisch hulpmiddel

Inhalt: 1 Einheit zur Einnahme ausschließlich durch einen Patienten. Nichtsteriles medizinisches Produkt der Klasse 1.

Contenuto: presidio medico di classe 1, non sterile, costituito da 1 unità per l'uso da parte di un singolo paziente

Conteúdo: 1 unidade para utilização num único doente Dispositivo médico de classe 1 não estéril

Contenido: 1 unidad para uso por un solo paciente

Contenu: 1 unité pour utilisation sur un seul patient

I indekiler: 1 adet steril edilmemis, tek hasta kullanımlık class 1 medikal r n

Περιεχόμενο: 1 μονάδα για χρήση σε έναν μόνο ασθενή, μη στείρα, ιατρική συσκευή κατηγορίας 1

MANUFACTURED BY:

GEFABRICEERD DOOR:

HERSTELLER:

PRODOTTO DA:

FABRICADO POR:

FABRICADO CERCA:

FABRIQUÉ PAR:

ÜRETİCİ FİRMA:

ΚΑΤΑΣΚΕΥΑΖΕΤΑΙ ΑΠΟ ΤΗΝ:

LOT:



**FERRARIS MEDICAL Ltd**  
4 Harforde Court, John Tate Road,  
Hertford, SG13 7NW, England  
Tel: +44 (0) 1992 526 300  
Fax: +44 (0) 1992 526 320  
email: [info@ferrarismedical.com](mailto:info@ferrarismedical.com)  
website: [www.ferrarismedical.com](http://www.ferrarismedical.com)

**ALSO AVAILABLE TO BUY  
OOK VERKRIJGBAAR OM TE KOPEN  
EBENFALLS KÄUFLICH ZU ERWERBEN  
PRODOTTO DISPONIBILE PER L'ACQUISTO  
TAMBÉM SE ENCONTRA À VENDA  
TAMBIÉN HAY DISPONIBLES PARA ADQUIRIR  
AUSSI EN VENTE  
DİĞER ÜRÜNLERİMİZ  
ΔΙΑΤΙΘΕΤΑΙ ΕΠΙΣΗΣ ΠΡΟΣ ΠΩΛΗΣΗ**



**FERRARIS MEDICAL  
KOKO ELECTRONIC  
PEAK FLOW METER**