

Quick Start Card

Name:ECO Trolley  
Model:TR9000




Check the number of components according to the following list before installation.

Item	Name	Q'ty
1	Chassis Module	1
2	Left Column	1
3	Right Column	1
4	Paper Basket	2
5	Printer Shelf	1

Item	Name	Q'ty
6	Desktop	1
7	Fixture	1
8	Screwdriver	1
9	Screw M3-20	4
10	Screw M3-5	4

Attention: Hand spin screws(Screw3-20 & Screw3-5) are used to install the main unit and the printer. Please keep it properly!

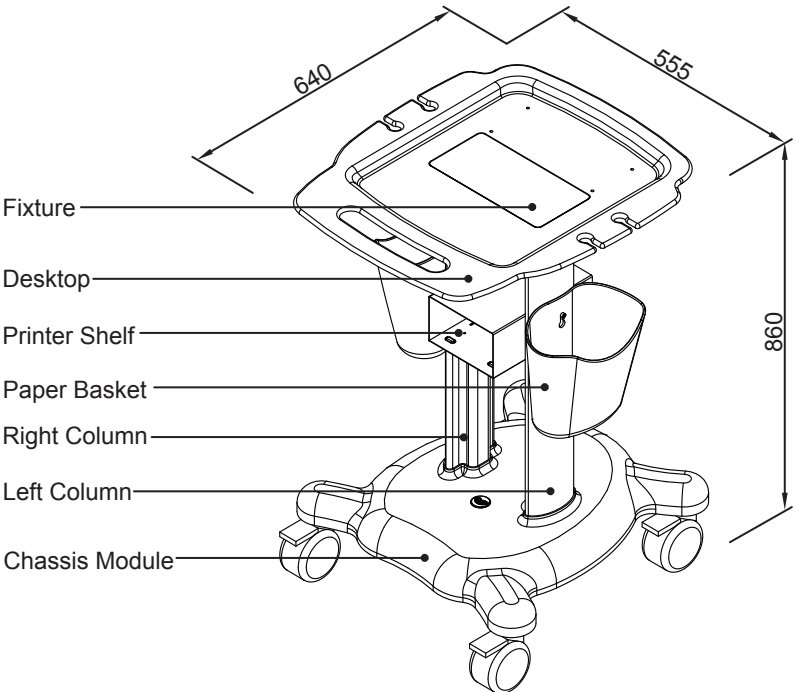


Quick Start Card

Name:ECO Trolley  
Model:TR9000




Installation steps:

- 1 Install right & left column to the chassis module 2 Install desktop to right & left column

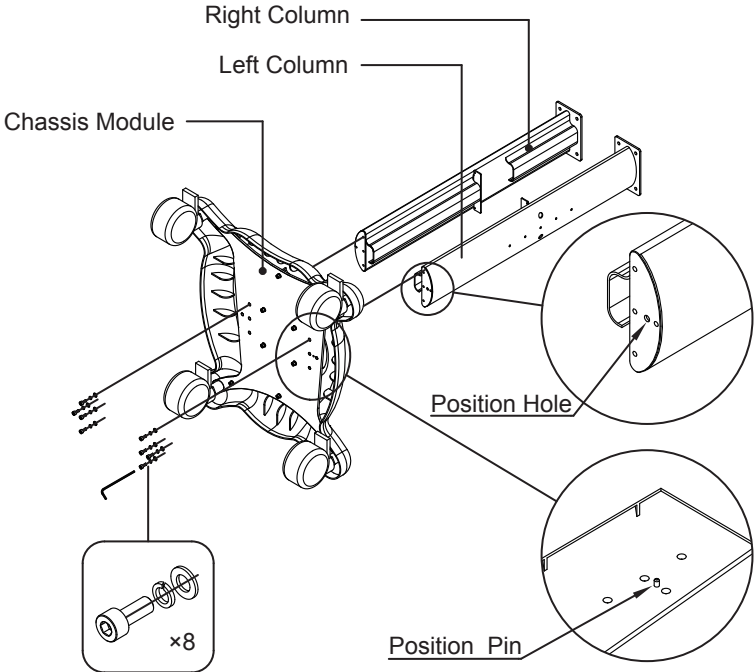


- 1.Insert the left column into the chassis module, position pin should be fixed in the position hole, assemble them together with 4 screws at the bottom of the chassis.  
2.Insert the right column into the chassis module, assemble them together with 4 screws at the bottom of chassis module.

- 3 Install paper basket

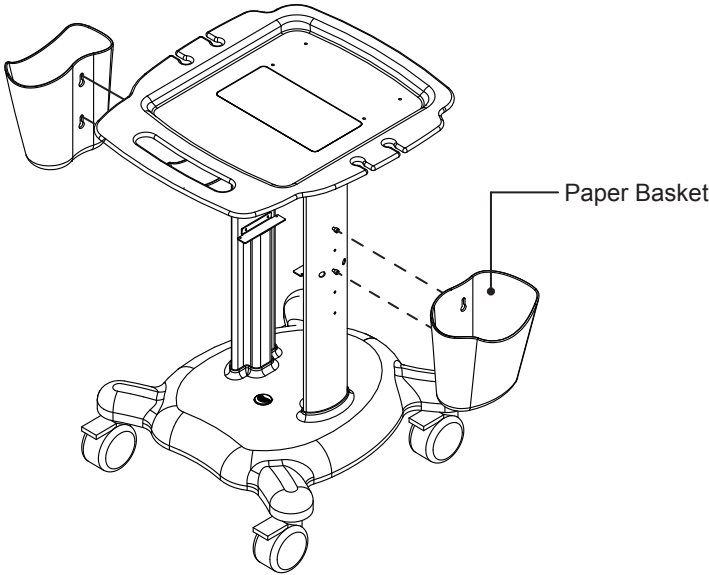


Hang the 2 paper baskets the screws side of the left & right columns. The height of paper basket locket can be adjusted by moving the screws.

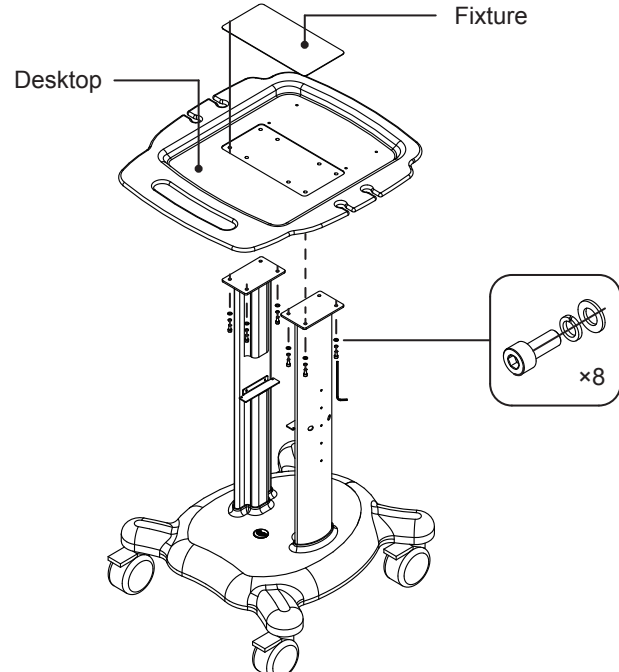

Put the fixture into the desktop, fix them with the left & right column together using 8 screws(M5x12) from bottom.

- 4 Install printer shelf

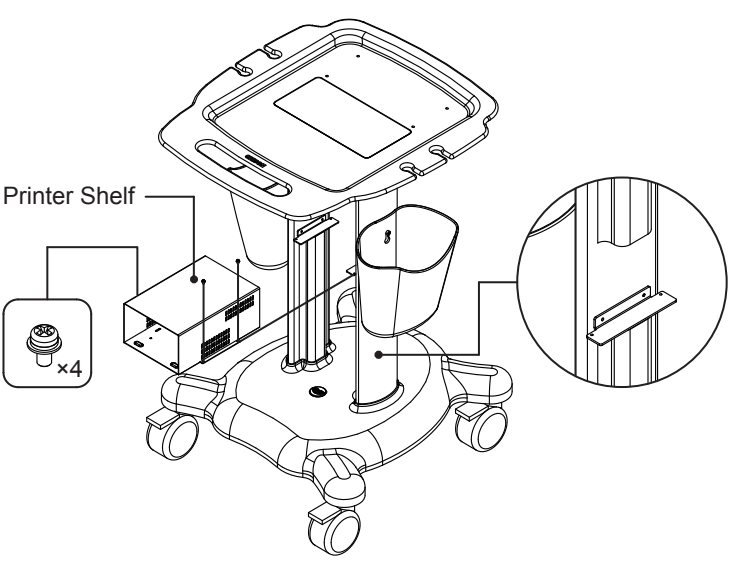

Fix the printer shelf into the corner support between 2 columns using 4 screws(M3x6 ).

**CHISON Medical Technologies Co., Ltd.**  
Addr. :No.228, Changjiang East Road, Block 51 and 53, Phase 5, Shuofang Industrial Park, Xinwu District, Wuxi, Jiangsu, China 214142  
Tel. :0086-510-85310593/85310937/85311707  
Fax :0086-510-85310726  
Website :http://www.chison.com  
E-mail :export@chison.com.cn