

EN - DISPOSABLE VIRAL AND BACTERIAL FILTER INSTRUCTIONS FOR USE

INTENDED USE: Designed to protect patients from infection and contamination from equipment, offers a high degree of protection against bacteria and viruses with very low levels of resistance to ensure the accuracy of the diagnostic result. Filter should be applied only to intact, clean and healthy skin.

PRODUCT IDENTIFICATION: The identification data (code, lot and date of production) are given in the label on the box.

USAGE:

- Pull out the filter and check both the device and the filtering material are intact. Otherwise discard it and go on with another device.
- Insert the filter in the device's sensor without thrust, always following the instructions of the device producer.
- Apply the mouthpiece in the proper side.
- Never reuse the filter and the mouthpiece for other patients to avoid cross-contamination.
- This device is not sterilizable.
- Use only with medical staff assistance.
- This device cannot be used for neonatal purposes.

DISPOSAL AND SAFETY: After use, the product should be disposed in normal non-toxic solid waste containers or in those of general hospital waste.

CONTRAINDICATIONS: Based on the experience and results of various analysis, there are no adverse effects as a result of appropriate use of the product.

NOTES: Packaging in individual heat-sealed envelope (single-wrap) guarantees the absence of bacteria, which is regularly monitored according to the manufacturer's quality system..

Manuale d'uso LUMED Rev. 2.0 01/09/2023



LUMED Srl
Via Staffora 18/9
20073 Opera ITALIA
www.lumed.com
tel +39 (0)2 57606750