



Digital Color Doppler Ultrasound System

Model
i8

OPERATION MANUAL

Direction: CHUMi8-001

V1.0
December 13, 2013

CHISON MEDICAL IMAGING CO., LTD.

We reserve the right to make changes to this manual without prior notice.

Regulatory Requirement



This product conforms to the essential requirements of the Medical Device Directive 93/42/EEC. Accessories without the CE mark are not guaranteed to meet the Essential Requirements of the Medical Device Directive.

This manual is a reference for the i8. Please verify that you are using the latest revision of this document. If you need the latest revision, contact your distributor.

NOTE:

Important

1. No part of this manual may be reduced, modified, copied or reprinted, in whole or in part, without written permission from CHISON.
2. The contents of this manual are subject to change without prior notice and without our legal obligation.
3. Before operating the system, please read and understand this manual. After reading, keep this manual in an easily accessible place. If you have any question or doubt, please contact CHISON's authorized service engineer.

NOTE:

Important information

1. It is the customer's responsibility to maintain and manage the system after delivery.
2. The warranty does not cover the following items, even during the warranty period:
 - a) Damage or loss due to misuse or abuse with system and probes, for example, drop the probe, the liquid or the metal part fall into the system.
 - b) Damage or loss caused by Acts of God such as fires, earthquakes, floods, lightning, etc.
 - c) Damage or loss caused by failure to meet the specified conditions for this system, such as inadequate power supply, improper installation or environmental conditions.
 - d) Damage or loss caused by non approved transportation by CHISON.
 - e) Damage or loss due to use the system outside the region where the system was originally sold.
 - f) Damage or loss involving the system purchased from a source other than CHISON or its authorized agents.
3. Do not make changes or modifications to the software or hardware of this system and probes.
4. During operate the system, if user has any doubt, difficulty or any unclear, please contact CHISON's authorized service engineer immediately. Please describe the situation clearly to solve the question in time. Before solve the question, please don't operate the system.
5. This system shall not be used by persons other than fully qualified and certified medical personnel.
6. It is prohibited to use the device for fetal sex examination, except for necessary medical needs. The device can only be sold to qualified medical institutions or doctors. The users shall fully understand and master the devices before operating. The users shall have got the qualification, and shall comply with the local laws and regulations, the local religion and customs, etc.
7. The System modified or repaired by people other than CHISON's qualified service engineers, CHISON shall not be liable for the system.
8. The purpose of this system is to provide physicians with data for clinical diagnosis. It is the physician's responsibility for diagnostic procedures. CHISON shall not be liable for the results of diagnostic procedures
9. This manual contains warnings regarding foreseeable potential dangers, but user shall always be alert to dangers other than those indicated as well. CHISON shall not be liable for damage or loss that results from negligence or from ignoring the precautions and operating instructions described in this operation manual.
10. Due to negligence not following operation manual, CHISON shall not be liable for the results.
11. Each time before and after ultrasound examination, please check the probe surface, probe cable and sheath whether they are abnormal, such as cracking, peeling and deformation. Also check whether the lens is strongly fixed. Abnormal probes may cause electric shock and injure the patient. Once any abnormal, user must stop using and contact CHISON's authorized service engineer.
12. If the probe is dropped or scratched by hard part, please stop using the probe immediately. And contact CHISON's authorized service engineer to make sure the safety and effectiveness is in good condition before use.
13. If there is any liquid or metal to enter to the system, please power off the system and stop using it immediately. Please first contact CHISON's authorized service engineer to make sure it's safe before restart using it.
14. Please don't use solvents (such as paint thinner, benzene, or alcohol) or abrasive cleansers for cleaning the system (including monitor and probes, etc). It may corrode the system and probes.
15. While the system or probe is over life time, please refer to operation manual section 9.5

16. Important data must be backed up on external memory media. CHISON shall not be liable for loss of data stored in the memory of this system caused by operator error or accidents.
17. Please put this operation manual with the system to ensure operator and manager can reach it at any time.

Caution: It is prohibited to use the device for fetal sex examination, except for necessary medical needs. The device can only be sold to qualified medical institutions or doctors. The users shall fully understand and master the devices before operating. The users shall have got the qualification, and shall comply with the local laws and regulations, the local religion and customs, etc.

Caution: The users should read the operation manual carefully before operating the devices. Turning on the device means the users have read the operation manual and accept the listed cautions, warnings, and notes in the manuals. If the users disagree and cannot accept the cautions, the users can ask for returning the device.

TABLE OF CONTENTS

CHAPTER 1	INTRODUCTION.....	8
1.1	System Overview.....	8
1.2	Contact Information.....	8
CHAPTER 2	SYSTEM SAFETY	9
2.1	Safety Overview	9
2.2	Electrical Safety.....	10
2.3	Labels.....	12
2.4	Patient Environmental Devices	13
2.5	Biological Safety.....	16
2.6	Scanning Patients and Education	17
CHAPTER 3	PREPARING THE SYSTEM FOR USE.....	23
3.1.	Site Requirements	23
3.2.	System Specifications	24
3.3.	System Positioning & Transporting	31
3.4.	Powering the System.....	32
3.5.	Probes	34
3.6	Optional installation.....	36
3.7	User Interface Control.....	39
CHAPTER 4	IMAGING	46
4.1.	General Description	46
4.2.	Beginning an Exam.....	46
4.3.	Optimizing the Image	50
4.4.	After Capturing the Image	70
CHAPTER 5	GENERAL MEASUREMENTS.....	84
5.1	Key for Measurement	84
5.2	Fast measurement.....	85
5.3	Measurement and Calculation.....	89
5.4	Edit measurement results	127
5.5	Report	127
CHAPTER 6	PRESET	130
6.1.	Recall Preset.....	130
6.2.	Save user defined preset	130
6.3.	Manage Preset	131
CHAPTER 7	SYSTEM SETTING.....	132
7.1.	General settings	132
7.2	Measurement	134
7.3	Comment:	136
7.4	Report.....	137
7.5	Network.....	140
7.6	System	144
CHAPTER 8	PROBES	154
8.1.	General Description.....	154
8.2.	Care and Maintenance	154
8.3	Probe Operation Instructions	164
CHAPTER 9	SYSTEM MAINTENANCE AND TROUBLESHOOTING	167
9.1	Back up information	167
9.2	System Care and Maintenance	167
9.3	Safety Check.....	169

9.4	Troubleshooting.....	169
9.5	Service Responsibility.....	170
APPENDIX A	THE INFORMATION OF CE REPRESENTATIVE	172
APPENDIX B	SYSTEM ONE-KEY-RECOVERY FUNCTION.....	173
APPENDIX C	MAXIMUM ACOUSTIC OUTPUT REPORT	177
APPENDIX D	TRANSDUCER MAXIMUM SURFACE TEMPERATURE	257
APPENDIX E	GUIDANCE AND MANUFACTURER'S DECLARATION	258
APPENDIX F	MEASUREMENT RESULTS SUMMARY	262

Chapter 1 Introduction

This manual contains necessary information for safe system operation.

Read and understand all instructions in this manual before operating the system. Always keep this manual with the equipment, and periodically review the procedures for operation and safety precautions.

1.1 System Overview

Indications for Use

The device is a general-purpose ultrasonic imaging instrument intended for use by a qualified physician for evaluation of Fetal/OB; Abdominal (GYN & Urology); Pediatric; Small Organ (breast, testes, thyroid); Cardiac (adult & pediatric); Peripheral Vascular, Musculo-skeletal Conventional & Superficial, Transvaginal.

Contraindication

The system is NOT intended for Ophthalmic use or any use that causes the acoustic beam to pass through the eye.

Caution: Besides necessarily medical use, this machine is not allowed to be applied into the sex check of fetus.

This machine can be sold only to qualified medical institutions and doctors. It should be used with complete apprehension and mastery of the product, observance with local laws and regulations, as well as compliance with local religion and custom.

1.2 Contact Information

For additional information or assistance, please contact your local distributor or the appropriate support resource shown below:

CHISON website	www.chison.com
Service Support	CHISON Medical Imaging Co., Ltd. Tel: 0086-400-8878-020; 0086-0510-85311707 Fax: 0086-0510-85310726 E-mail: service@chison.com.cn
Placing an Order	CHISON Medical Imaging Co., Ltd. Tel: 0086-0510-8531-0593/0937 Fax: 0086-0510-85310726 Email: export@chison.com.cn
Manufacturer	CHISON Medical Imaging Co., Ltd.. No. 8, Xiang Nan Road, Shuo Fang, New District, Wuxi, China 214142

Chapter 2 System Safety

2.1 Safety Overview

This section discusses the measures to ensure the safety of both the operator and patient. To ensure the safety of both operator and patient, please read the relevant details in this chapter carefully before operating this system. Disregarding the warnings or violation of relevant rules may result in personal injury for operator or patient. or even loss of life

Users should observe the following precautions:

- This system complies with Type BF general equipment, and the IEC standard. Please follow Chapter 2 “System Safety” in the operation manual to use this system properly.
- Please do not modify this system in any way. If modifications are necessary, please contact the manufacturer first to get more information and permission.
- This system has been fully adjusted at the factory. Do not adjust any fixed adjustable parts.
- In the event of a malfunction, turn off the system immediately and inform the manufacturer or its designated agents.
- The power cord of the system should be connected to a grounded power socket. Do not remove the ground cable for any reason.
- Only connect this system, either electronically or mechanically, with devices that comply with the IEC/EN60601-1 standard. Recheck the leakage current and other safety performance indices of the entire system to avoid potential system damage caused by leakage from a current superposition.
- The system does not incorporate any specialized protective measures in the event it is configured with high-frequency operation devices. The operator should use caution in these types of applications.
- The system should be installed only by personnel authorized by the manufacturer. Do not attempt to install the system by yourself.
- Only a CHISON's authorized service engineer can perform maintenance.
- Only a qualified operator, or someone under qualified supervision, can use the system.
- Do not use this system in the presence of flammable substances, otherwise an explosion may occur.
- Do not continuously scan the same part of a patient or expose the patient to prolonged scanning. Otherwise, it may harm the patient.
- When using the system for ultrasound testing, only use qualified ultrasound gel that complies with system standards.
- Do not unplug probe when the system is in active operation. Always go to transducer Selection screen when need to remove the probe.
- To prevent from arm or neck injury, the operator should not stay at the same position for too long during patient scanning without taking break.

- Do not put liquid on top of the main unit.

NOTE:

****To dispose of this product properly, please contact the local CHISON's Authorized Service Representative.***

2.2 Electrical Safety

Type of protection against electric shock

- Class I Equipment

CLASS I EQUIPMENT in which protection against electric shock does not rely on basic insulation only, but which includes an additional safety precaution in that accessible conductive parts are connected to the protective earthing conductor in the electrical installation in such a way that accessible parts cannot become live in the event of a failure of the basic insulation.

Degree of protection against electric shock

- Type BF Applied part (for Probes marked with BF symbol)

TYPE BF APPLIED PART providing a specified degree of protection against electric shock, with particular regard to allowable LEAKAGE CURRENT

BF: Isolation from ground; max. Patient leakage current: normal mode $\leq 100 \mu\text{A}$, single fault condition $\leq 500 \mu\text{A}$

Level of protection against harmful ingress of water

- Parts of **probe** likely to come into contact with **operator** or **patient** meet the requirements of **drip-proof equipment (IPX1)**
Parts of **probe** intended to be immersed in **normal use** meet the requirements of **watertight equipment (IPX7)**
- The IP Classification of System is Ordinary Equipment (IPX0)

The Equipment is not suitable for use in the presence of a flammable anesthetic mixed with air (with oxygen or with oxide)

Mode of operation

- Continuous Operation

For maximum safety, always follow these guidelines:

- Proper grounding of the system is critical to avoid electric shock. For protection, ground the chassis with a three-wire cable, and plug the system into three-hole outlet.
- Do not remove or circumvent the grounding wire.
- Do not remove the protective covers on the system. These covers protect users against hazardous voltages. Cabinet panels must remain in place while the system is in use. A qualified electronic technician must make all

internal replacements.

- Do not operate this system in the presence of flammable gases or anesthetics.
- All peripheral devices (unless certified as medical grade) that are connected to the system must be powered through the electrical outlet with an optional isolation transformer.
- Suggest power off the system in 30 minutes if the system continuously works in 8 hours.

Notice upon Installation of Product

Separation distance and effect from fixed radio communications equipment: field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast, and TV broadcast transmitter cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the ultrasound system is used exceeds the applicable RF compliance level as stated in the immunity declaration, the ultrasound system should be observed to verify normal operation. If abnormal operation is observed, additional measures may be necessary, such as re-orienting or relocating the ultrasound system or using an RF shielded examination room may be necessary.

- Use either power supply cords provided by or designated by CHISON. Products equipped with a power source plug should be plugged into the fixed power socket which has the protective grounding conductor. Never use any adaptor or converter to connect with a power source plug (e.g. three-prong-to-two-prong converter).
- Locate the equipment as far away as possible from other electronic equipment.
- Be sure to only use the cables provided by or designated by CHISON. Connect these cables following the installation procedures (e.g. wire power cord separately from signal cables).
- Lay out the main equipment and other peripherals following the installation procedures described in this manual.

Notice against User Modification

The user should never modify this product.

User modifications may cause degradation in Electrical Safety. Modification of the product includes changes in:

- Cables (length, material, wiring, etc.)
- System configuration/components

User modifications may cause degradation in EMC performance. Modification of the product includes changes in:

- Cables (length, material, wiring, etc.)
- System installation/layout
- System configuration/components
- Securing system parts (cover open/close, cover screwing)

2.3 Labels












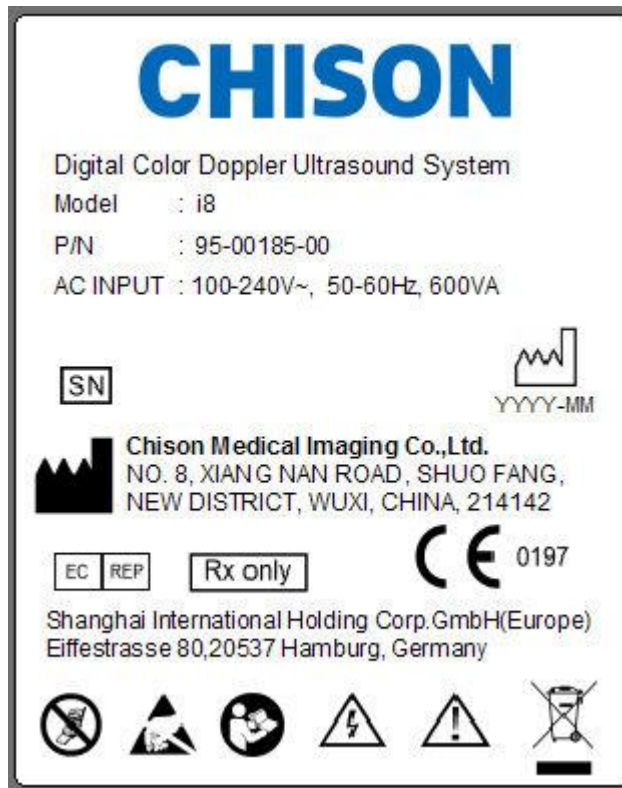








Fig.2-1 Rear panel label

2.3.1. Symbols on label

 <p>Caution, consult accompanying documents. This symbol advises the reader to consult the accompanying documents for important safety related information such as warnings and pre-cautions that Cannot be presented on the device itself.</p>	 <p>Dangerous electric voltage. Unplug the main plug before opening the system!</p>
 <p>Do not use the following devices near this equipment: cellular phone, radio receiver, and mobile radio transmitter, radio controlled toy, etc. Use of these devices near this equipment could cause this equipment to perform outside the published specifications. Keep these devices power off when near this equipment.</p>	 <p>Be careful of static.</p>
 <p>WASTE OF ELECTRICAL AND ELECTRONIC EQUIPMENT (WEEE): This symbol is used for Environment Protection, it indicates that the waste of electrical and electronic equipment must not be disposed</p>	 <p>The CE mark of Conformity indicates this equipment conforms with the Council Directive 93/42/EEC</p>

<p>as unsorted waste and must be collected separately. Please contact your local Authority or distributor of the manufacturer for information concerning the decommissioning of your equipment.</p>	
<p> AUTHORIZED REPRESENTATIVE IN THE EUROPEAN COMMUNITY: This symbol is accompanied by the name and the address of the authorized representative in the European Community.</p>	<p> Type BF Applied Part symbol is in accordance with IEC60878-02-03.</p>
<p> This symbol is followed by the serial number of the device.</p>	<p> MANUFACTURER: This symbol is accompanied by the name and the address of the manufacturer.</p>
<p> Potential equilibrium connection</p>	<p> Main power switch ON</p>
<p> Main power switch OFF</p>	<p> Power On/off. CAUTION: This Power Switch DOES NOT ISOLATE Mains Supply.</p>
<p> The “Alternating current” symbol indicates that the equipment is suitable for alternating current only.</p>	<p> This symbol signifies that the user manual must be read.</p>
<p>IPX7 Protection against the effects of immersion (probes)</p>	<p>IPX0 No protection against ingress of water (system)</p>
<p>Rx only This symbol indicates that in the united states of America, Federal law restricts the device to sale by or on the order of a licensed practitioner or therapist.</p>	<p> This symbol is followed by the manufacturing date of the device in the form YYYY-MM.</p>

2.4 Patient Environmental Devices

Front side (refer to Fig. 3-1 b in Chapter 3):

- ✧ 4 Probe ports
- ✧ 1DVD RW Driver
- ✧ 2 USB ports

Rear panel (refer to Fig.3-1c in Chapter3) :

- ✧ 2 Footswitch port: Footswitch (Steute MKF-MED recommended)

- ✧ 1 Remote port: remote cable connection to video printer
- ✧ 4 USB ports
- ✧ TV Video port: B/W or Color Printers (Sony UP-897MD, Mitsubishi CP31W recommended)
- ✧ 1 LAN port:
- ✧ 1 VGA port: External monitor (Sony LMD-1950MD recommended)

CAUTION:

- ***VIDEO, S-VIDEO and VGA are not designed to work at the same time, only one port can work at one time. Please select your most suitable port.***

Acceptable Devices

The Patient Environmental devices shown above are specified to be suitable for use within the PATIENT ENVIRONMENT.

CAUTION:

- ***DO NOT connect any probes or accessories without approval by CHISON within the PATIENT ENVIRONMENT.***
- ***DO NOT touch patient and devices without EN 60601-1 approval to avoid the leakage current risk within the PATIENT ENVIRONMENT.***

Unapproved Devices

CAUTION:

- ***DO NOT use unapproved devices.***
- ***If devices are connected without the approval of CHISON, the warranty will be INVALID.***
- ***The system can't be used with HF surgical equipment; otherwise the burns to patient may occur. Any device connected to this system must conform to one or more of the requirements listed below:***
 - ***IEC standard or equivalent standards appropriate to devices.***
 - ***The devices shall be connected to PROTECTIVE EARTH (GROUND).***

CAUTION: *Unsafe operation or malfunction may occur. Use only the accessories, options and supplies approved or recommended in these instructions for use.*

Peripheral used in the patient environment

The system has been verified for overall safety, compatibility and compliance with the following on-board image recording devices:

B/W video printer: Mitsubishi P93W; Sony UP-897MD, Sony UP-D711MD

Color video printer: Mitsubishi CP31W

The system may also be used safely while connected to devices other than those recommended above if the devices and their specifications, installation, and interconnection with the system conform to the requirements of EN 60601-1-1. The connection of equipment or transmission networks other than as specified in the user instructions can result in an electric shock hazard or equipment malfunction. Substitute or alternate equipment and connections require verification of compatibility and conformity to EN 60601-1-1 by the installer. Equipment modifications, possible resulting malfunctions and electromagnetic interference are the responsibilities of the owner.

General precautions for installing an alternate off-board, remote device or a network would include:

- The added device(s) must have appropriate safety standard conformance and CE Marking.
- There must be adequate mechanical mounting of the device and stability of the combination.
- Risk and leakage current of the combination must comply with EN 60601-1.
- Electromagnetic emissions and immunity of the combination must conform to EN 60601-1-2.

Peripheral used in the non-patient environment

The system has been verified for compatibility, and compliance for connection to a local area network (LAN) via a wire LAN. The provided LAN components are EN 60950 compliant.

General precautions for installing an alternate off-board, remote device or a network would include:

- The added device(s) must have appropriate safety standard conformance and CE Marking.

The added device(s) must be used for their intended purpose having a compatible interface.

CAUTION: Make sure using ONLY the dedicated USB disk or removable media to save or back up data. Before connecting to the ultrasound system, make sure using the latest antivirus software on the USB disk or removable media to clean any virus. It is user's responsibility to ensure the USB disk or removable media is virus-free. Improper use of USB disk or removable media may cause the virus infections of system and eventually malfunction may occur. Such malfunction may impact the stability, effectiveness and safety of the system and probes, and users should immediately stop using the system and probes until CHISON authorized engineer has checked the system and confirm the effectiveness and safety of the system and probes.

CAUTION: Use only secure Local Area Network connection. Don't connect the ultrasound system to Internet. Make sure your hospital's firewall software is configured correctly, thus blocking incoming connection requests from Internet. Improper use of network connection may cause the virus infections of system and eventually malfunction may occur.

2.5 Biological Safety

This product, as with all diagnostic ultrasound equipment, should be used only for valid reasons and should be used both for the shortest period of time and at the lowest power settings necessary (ALARA - As Low As Reasonably Achievable) to produce diagnostically acceptable images. The AIUM offers the following guidelines:

Clinical Safety Quoted from AIUM

Approved March 26, 1997

Diagnostic ultrasound has been in use since the late 1950s. Given its known benefits and recognized efficacy for medical diagnosis, including use during human pregnancy, the American Institute of Ultrasound in Medicine herein addresses the clinical safety of such use:

There are no confirmed biological effects on patients or instrument operators caused by exposures from present diagnostic ultrasound instruments. Although the possibility exists that such biological effects may be identified in the future, current data indicate that the benefits to patients of the prudent use of diagnostic ultrasound outweigh the risks, if any that may be present.

Heating: Elevating tissue temperature during obstetrical examinations creates medical concerns. At the embryo development stage, the rise in temperature and the length of time exposed to heat combine to determine potential detrimental effects. Exercise caution particularly during Doppler/Color exams. The Thermal Index (TI) provides a statistical estimate of the potential temperature elevation (in centigrade) of tissue temperature. Three forms of TI are available: Soft Tissue Thermal Index (TIS), Bone Thermal Index (TIB) and Cranial Bone Thermal Index (TIC).

Soft Tissue Thermal Index (TIS). Used when imaging soft tissue only, it provides an estimate of potential temperature increase in soft tissue.

Bone Thermal Index (TIB). Used when bone is near the focus of the image as in the third trimester OB examination, it provides an estimate of potential temperature increase in the bone or adjacent soft tissue.

Cranial Bone Thermal Index (TIC). Used when bone is near the skin surface as in transcranial examination, it provides an estimate of potential temperature increase in the bone or adjacent soft tissue.

Cavitations: Cavitations may occur when sound passes through an area that contains a cavity, such as a gas bubble or air pocket (in the lung or intestine, for example). During the process of cavitations, the sound wave may cause the bubble to contract or resonate. This oscillation may cause the bubbles to explode and damage the tissue. The Mechanical Index (MI) has been created to help users accurately evaluate the likelihood of cavitations and the related adverse effects.

MI recognizes the importance of non-thermal processes, cavitations in particular, and the Index is an attempt to indicate the probability that they might occur within the tissue.

2.6 Scanning Patients and Education

The Track-3 or IEC60601-2-37 output display standard allows users to share the responsibility for the safe use of this ultrasound system. Follow these usage guidelines for safe operation:

- In order to maintain proper cleanliness of the probes, always clean them between patients.
- Always use a disinfected sheath on all EV/ER probes during every exam.
- Continuously move the probe, rather than staying in a single spot, to avoid elevated temperatures in one part of the patient's body.
- Move probe away from the patient when not actively scanning.
- Understand the meaning of the TI, TIS, TIB, TIC and MI output display, as well as the relationship between these parameters and the thermal/cavitation bioeffect to the tissue.
- Expose the patient to only the very lowest practical transmit power levels for the shortest possible time to achieve a satisfactory diagnosis (ALARA - As Low As Reasonably Achievable).

2.6.1 Safe Scanning Guidelines

- Ultrasound should only be used for medical diagnosis and only by trained medical personnel.
- Diagnostic ultrasound procedures should be done only by personnel fully trained in the use of the equipment, in the interpretation of the results and images, and in the safe use of ultrasound (including education as to potential hazards to the patient and the operator).
- Operators should understand the likely influence of the machine controls, the operating mode (e.g. B-mode, color Doppler imaging or spectral Doppler) and probe frequency on thermal and cavitations hazards.
- Select a low setting for each new patient. Output should only be increased during the examination if penetration is still required to achieve a satisfactory result, and after the Gain control has been adjusted to its maximum value.
- Maintain the shortest examination time necessary to produce a useful diagnostic result.
- Do not hold the probe in a fixed position for any longer than is necessary. It should be removed from the patient whenever there is no need for real-time imaging or spectral Doppler acquisition. The frozen frame and Cine loop capabilities allow images to be reviewed and discussed without exposing the patient to continuous scanning.
- Do not use endo-cavitary probes if there is noticeable self heating of the probe when operating in the air. Although applicable to any probe, take particular care during trans-vaginal exams during the first eight weeks

of gestation.

- Take particular care to reduce output and minimize exposure time of an embryo or fetus when the temperature of the mother is already elevated.
- Take particular care to reduce the risk of thermal hazard during diagnostic ultrasound when exposing: an embryo less than eight weeks after gestation; or the head, brain or spine of any fetus or neonate.
- Operators should continually monitor the on-screen thermal index (TI) and mechanical index (MI) values and use control settings that keep these settings as low as possible while still achieving diagnostically useful results. In obstetric examinations, TIS (soft tissue thermal index) should be monitored during scans carried out in the first eight weeks after gestation, and TIB (bone thermal index) thereafter. In applications where the probe is very close to bone (e.g. trans-cranial applications), TIC (cranial bone thermal index) should be monitored.

MI > 0.3

There is a possibility of minor damage to neonatal lung or intestine. If such exposure is necessary, reduce the exposure time as much as possible.

MI > 0.7

There is a risk of cavitations if an ultrasound contrast agent containing gas microspheres is being used. There is a theoretical risk of cavitations without the presence of ultrasound contrast agents. The risk increases with MI values above this threshold.

TI > 0.7

The overall exposure time of an embryo or fetus should be restricted in accordance with Table 2-2 below as a reference:

TI	Maximum exposure time (minutes)
0.7	60
1.0	30
1.5	15
2.0	4
2.5	1

Table 2-2 Maximum recommended exposure times for an embryo or fetus

- Non-diagnostic use of ultrasound equipment is not generally recommended. Examples of non-diagnostic uses of ultrasound equipment include repeated scans for operator training, equipment demonstration using normal subjects, and the production of souvenir pictures or videos of a fetus. For equipment of which the safety indices are displayed over their full range of values, the TI should always be less than 0.5 and the MI should always be less than 0.3. Avoid frequent repeated exposure of any subject. Scans in the first trimester of pregnancy should not be carried out for the sole purpose of producing souvenir videos or photographs, nor should their production involve increasing the exposure levels or extending the scan times beyond those needed for clinical purposes.

- Diagnostic ultrasound has the potential for both false positive and false negative results. Misdiagnosis is far more dangerous than any effect that might result from the ultrasound exposure. Therefore, diagnostic ultrasound system should be performed only by those with sufficient training and education.

2.6.2 Understanding the MI/TI Display

Track-3 follows the Output Display Standard for systems that include fetal Doppler applications. The acoustic output will not be evaluated on an application-specific basis, but the global maximum de-rated I_{spta} must be $\leq 720 \text{ mW/cm}^2$ and either the global maximum MI must be ≤ 1.9 or the global maximum de-rated I_{sppa} must be $\leq 190 \text{ W/cm}^2$. An exception is for ophthalmic use, in which case the $TI = \max(TIS_{as}, TIC)$ is not to exceed 1.0; $I_{spta.3} \leq 50 \text{ mW/cm}^2$, and $MI \leq 0.23$. Track-3 gives the user the freedom to increase the output acoustic power for a specific exam, and still limit output acoustic power within the global maximum de-rated $I_{spta} \leq 720 \text{ mW/cm}^2$ under an Output Display Standard.

For any diagnostic ultrasonic systems, Track-3 provides an Output Indices Display Standard. The diagnostic ultrasound systems and its operation manual contain the information regarding an ALARA (As Low As Reasonably Achievable) education program for the clinical end-user and the acoustic output indices, MI and TI. The MI describes the likelihood of cavitations, and the TI offers the predicted maximum temperature rise in tissue as a result of the diagnostic examination. In general, a temperature increase of 2.5°C must be present consistently at one spot for 2 hours to cause fetal abnormalities. Avoiding a local temperature rise above 1°C should ensure that no thermally induced biologic effect occurs. When referring to the TI for potential thermal effect, a TI equal to 1 does not mean the temperature will rise 1 degree C. It only means an increased potential for thermal effects can be expected as the TI increases. A high index does not mean that bioeffects are occurring, but only that the potential exists and there is no consideration in the TI for the scan duration, so minimizing the overall scan time will reduce the potential for effects. These operator control and display features shift the safety responsibility from the manufacturer to the user. So it is very important to have the Ultrasound systems display the acoustic output indices correctly and the education of the user to interpret the value appropriately.

RF: (De-rating factor)

In Situ intensity and pressure cannot currently be measured. Therefore, the acoustic power measurement is normally done in the water tank, and when soft tissue replaces water along the ultrasound path, a decrease in intensity is expected. The fractional reduction in intensity caused by attenuation is denoted by the de-rating factor (RF),

$$RF = 10^{(-0.1 a f z)}$$

Where a is the attenuation coefficient in $\text{dB cm}^{-1} \text{ MHz}^{-1}$, f is the transducer center frequency, and z is the distance along the beam axis between the source and the point of interest.

De-rating factor RF for the various distances and frequencies with attenuation coefficient $0.3 \text{ dB cm}^{-1} \text{ MHz}^{-1}$ in

homogeneous soft tissue is listed in the following table. An example is if the user uses 7.5MHz frequency, the power will be attenuated by .0750 at 5cm, or $0.3 \times 7.5 \times 5 = -11.25\text{dB}$. The De-rated Intensity is also referred to as ‘.3’ at the end (e.g. Ispta.3).

Distance (cm)	Frequency (MHz)			
	1	3	5	7.5
1	0.9332	0.8128	0.7080	0.5957
2	0.8710	0.6607	0.5012	0.3548
3	0.8128	0.5370	0.3548	0.2113
4	0.7586	0.4365	0.2512	0.1259
5	0.7080	0.3548	0.1778	0.0750
6	0.6607	0.2884	0.1259	0.0447
7	0.6166	0.2344	0.0891	0.0266
8	0.5754	0.1903	0.0631	0.0158

$I' = I * R_F$ Where I' is the intensity in soft tissue, I is the time-averaged intensity measured in water.

Tissue Model:

Tissue temperature elevation depends on power, tissue type, beam width, and scanning mode. Six models are developed to mimic possible clinical situations.

Thermal Models	Composition	Mode	Specification	Application	
1	TIS	Soft tissue	Unscanned	Large aperture (>1cm ²)	Liver PW
2	TIS	Soft tissue	Unscanned	Small aperture (<1cm ²)	Pencil Probe
3	TIS	Soft tissue	Scanned	Evaluated at surface	Breast color
4	TIB	Soft tissue and bone	Scanned	Soft tissue at surface	Muscle color
5	TIB	Soft tissue and bone	Unscanned	Bone at focus	Fetus head PW
6	TIC	Soft tissue and bone	Unscanned/scanned	Bone at surface	Transcranial

Soft tissue:

Describes low fat content tissue that does not contain calcifications or large gas-filled spaces.

Scanned: (auto-scan)

Refers to the steering of successive burst through the field of view, e.g. B and color mode.

Unscanned:

Emission of ultrasonic pulses occurs along a single line of sight and is unchanged until the transducer is moved to a new position. For instance, the PW, and M mode.

TI:

TI is defined as the ratio of the In Situ acoustic power ($W.3$) to the acoustic power required to raise tissue temperature by 1°C (W_{deg}), $TI = W.3 / W_{deg}$.

Three TIs corresponding to soft tissue (TIS) for abdominal; bone (TIB) for fetal and neonatal cephalic; and cranial bone (TIC) for pediatric and adult cephalic, have been developed for applications in different exams.

An estimate of the acoustic power in milli-watts necessary to produce a 1°C temperature elevation in soft tissue is:

$W_{deg} = 210/f_c$, for model 1 to 4, where f_c is the center frequency in MHz.

$W_{deg} = 40 K D$ for model 5 and 6, where K (beam shape factor) is 1.0, D is the aperture diameter in cm at the depth of interest.

MI:

Cavitation is more likely to occur at high pressures and low frequencies in pulse ultrasound wave in the tissue, which contains the bubble or air pocket (for instance, the lung, intestine, or scan with gas contrast agents). The threshold under optimum conditions of pulsed ultrasound is predicted by the ration of the peak pressure to the square root of the frequency.

$$MI = Pr' / \text{sqrt}(f_c)$$

Pr' is the de-rated (0.3) peak rare-fractional pressure in Mpa at the point where PII is the maximum, and f_c is the center frequency in MHz. PII is the Pulse Intensity Integral that the total energy per unit area carried by the wave during the time duration of the pulse. The peak rare- fractional pressure is measured in hydrophone maximum negative voltage normalized by the hydrophone calibration parameter.

Display Guideline:

For different operation modes, different indices must be displayed. However, only one index needs to be shown at a time. Display is not required if maximum MI is less than 1.0 for any setting of the operating mode, or if maximum TI is less than 1.0 for any setting of the operating mode. For TI, if the TIS and TIB are both greater than 1.0, the scanners need not be capable of displaying both indices simultaneously. If the index falls below 0.4, no display is needed. The display increments are no greater than 0.2 for index value less than one and no greater than 1.0 for index values greater than one (e.g. 0.4, 0.6, 0.8, 1, 2, 3).

Display and Report in Different Mode

Located on the upper middle section of the system display monitor, the acoustic output display provides the operator with real-time indication of acoustic levels being generated by the system.

For B-Scan Mode

Only display and report MI, and start from 0.4 if maximum MI > 1.0, display in increments of 0.2.

For Color Mode

Only display and report TIS or TIB and start from 0.4 if maximum TI > 1.0, display in increments of 0.2 for values of indices of 2.0 or less, and 0.5 for values of indices greater than 2.0.

For Doppler Mode

Only display and report TIS or TIB and start from 0.4 if maximum TI > 1.0, display in increments of 0.2 for values of indices of 2.0 or less, and 0.5 for values of indices greater than 2.0.

Below is a simple guideline for the user when TI exceeds one limit exposure time to 4^(6-TI) minutes based on the

'National Council on Radiation Protection. Exposure Criteria for Medical Diagnostic Ultrasound: I. Criteria Based on Thermal Mechanisms. Report No.113 1992'.

Operator Control Features:

The user should be aware that certain operator controls may affect the acoustic output. It is recommended to use the default (or lowest) output power setting and compensate using Gain control to acquire an image. Other than the output power setting in the soft-menu, which has the most direct impact on the power; the PRF, image sector size, frame rate, depth, and focal position also slightly affect the output power. The default setting is normally around 70% of the allowable power depending on the exam application mode.

Controls Affecting Acoustic Output

The potential for producing mechanical bioeffects (MI) or thermal bioeffects (TI) can be influenced by certain controls.

Direct: The Acoustic Output control has the most significant effect on Acoustic Output.

Indirect: Indirect effects may occur when adjusting controls. Controls that can influence MI and TI are detailed under the bioeffect portion of each control in the Optimizing the Image chapter.

Always observe the Acoustic Output display for possible effects.

Best practices while scanning

HINTS: Raise the Acoustic Output only after attempting image optimization with controls that have no effect on Acoustic Output, such as Gain and TGC.

WARNING: Be sure to have read and understood control explanations for each mode used before attempting to adjust the Acoustic Output control or any control that can effect Acoustic Output.

Use the minimum necessary acoustic output to get the best diagnostic image or measurement during an examination. Begin the exam with the probe that provides an optimum focal depth and penetration.

Acoustic Output Default Levels

In order to assure that an exam does not start at a high output level, the system initiates scanning at a reduced default output level. This reduced level is preset programmable and depends upon the exam icon and probe selected. It takes effect when the system is powered on or New Patient is selected. To modify acoustic output, adjust the Power Output level on the Soft Menu.

Chapter 3 Preparing the System for Use

3.1. Site Requirements

3.1.1. Operation Environmental Requirements

The following environmental conditions are within system tolerances for operation:

Temperature:	10° C ~ 40° C
Relative Humidity:	30%~75%, non-condensing
Atmosphere Pressure:	700hPa ~ 1060hPa

Strong radiation sources or powerful electromagnetic waves (e.g. electro-magnetic waves from radio broadcasting) may result in image ghosting or noise. The system should be isolated from such radiation sources or electromagnetic waves.

To prevent damage to the system, do not use in the following locations:

- Exposed to direct sunlight
- Subject to sudden changes in temperature
- Dusty
- Subject to vibration
- Near heat generators
- High humidity

NOTE:

This equipment generates, uses and can radiate radio frequency energy. The equipment may cause radio frequency interference to other medical and non-medical devices and radio communications. To provide reasonable protection against such interference, this product complies with emissions limits for a Group 1, Class A Medical Devices Directive as stated in EN 60601-1-2. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment is found to cause interference (which may be determined by turning the equipment on and off), the user (or qualified service personnel) should attempt to correct the problem by one or more of the following measure(s):

- *reorient or relocate the affected device(s)*
- *increase the separation between the equipment and the affected device*
- *power the equipment from a source different from that of the affected device*
- *consult the point of purchase or service representative for further suggestions.*

3.1.2. Transport and Storage Environmental Requirements

The following environmental transport and storage conditions are within system tolerances:

Temperature:	-5° C ~ 40° C
Relative Humidity:	≤80% non-condensing
Atmosphere Pressure:	700hPa ~ 1060hPa

3.1.3. Electrical Requirements

Power Requirements

AC 100-240V, 50-60Hz

Fuse Requirements

Fuse specification is 250V, 10 A (time-lag), the model is 50T T10AL 250V

CAUTION:

Please use the fuse provided by manufactory if the fuse need replace. User can't buy and exchange the fuse by themselves.

Power Consumption: 600 VA

Voltage Fluctuation

WARNING:

Maintain a fluctuation range of less than $\pm 10\%$ of voltage labeling on rear panel of the system, otherwise the system may be damaged.

Grounding

Before connecting the power cord, connect the attached ground protection cable from Equipotentiality terminal on system rear panel to a specialized grounding device.

NOTE:

- *Please follow the outlined power requirements. Only use power cords that meet the system guidelines—failure to follow these procedures may result in system damage.*
- *Line power may vary in different geographic locations. Refer to the detailed ratings on the rear panel of the system for detailed information.*

3.2. System Specifications

3.2.1. Console Overview



Fig. 3-1 a: Console Overview

The following pictures show the system in different views.

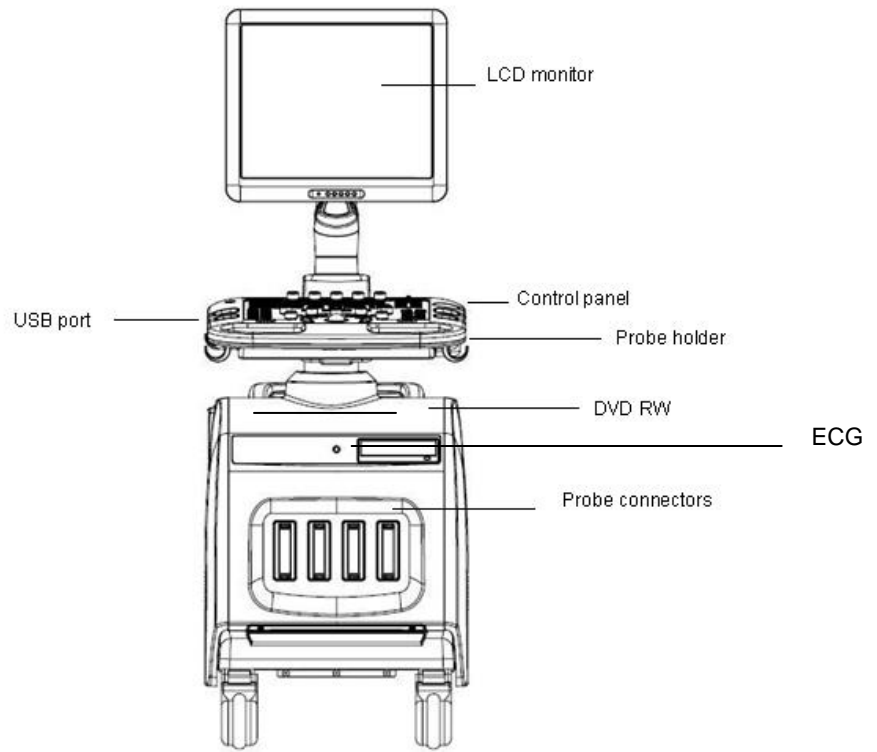


Fig. 3-1 b: front Side View

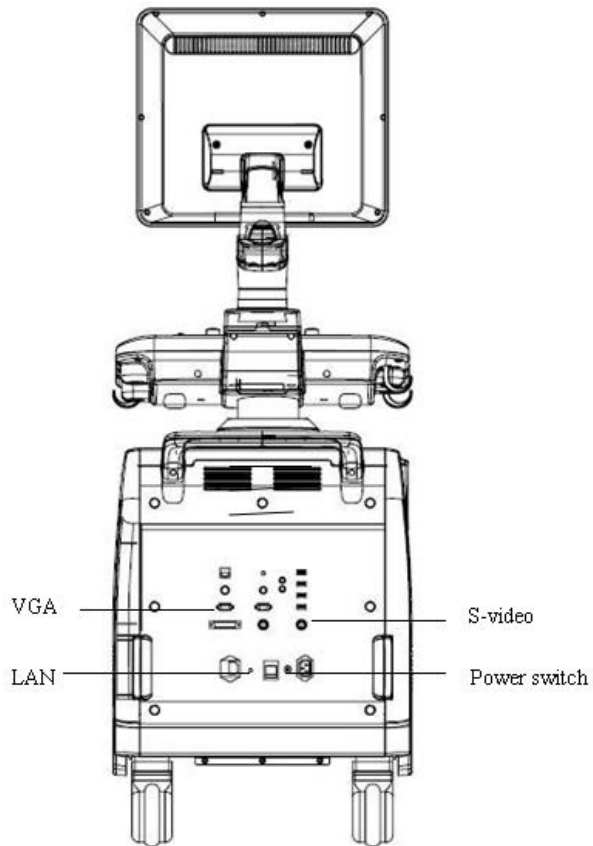


Fig. 3-1 c: System Back view

3.2.2. Physical Specification

Dimensions of main unit (approx.): 620mm(Width) x850mm (Depth) x1400mm (Height)

Net weight of main unit (approx.): 95kg (no probe included)

3.2.3. Key System Features

- Display B (2D)、B/B、4B、B/M、M、PWD、CFM、CPA、DPD、CW、 Free Steering M、Color M 、TDI、B/BC、HPRF、Triplex、Curved Panoramic Imaging 、THI、Trapezoid Imaging、4D
- ECG
- Super Needle
- 2D Steer
- Zoom and depth adjustment.
- Set the total gain, contrast, frequency band, 8 segments of TGC , dynamic range, persistence.
- Image post-processing of raw data: measurement and zoom after freezing the image
- 256 gray-scale image display technology, i-Image™ technology, stable performance, high resolution;
- Image freezing and storage function; the stored images can be recalled for analysis
- Storage file format: single and movie file formats
- Scanning direction can be changed and the image can be reversed in left/right, up/down direction.
- Distance, area, circumference, volume, fetal weight, heart rate etc. measurements are available and automatic calculation of OB, cardiology are available. direct display of gestation age and expected date of child delivery;
- Elliptical method and tracing method are provided for area/circumference measurement
- Many kinds of body marks can be displayed together with corresponding probe position indication.
- Comment function in image area of the screen, special comment terms for different exam mode can be added according to user's requirement;
- Display of Patient ID, Time and Date display according to real-time clock.
- Trackball available for operation and measurement. Characters can be input directly by keyboard.
- When one function is under operation, the corresponding key on the control panel will be brightly lit. When exiting from the function, the corresponding key on the control panel will be slightly lit.
- Measure the percentage of stenosis, blood flow velocity, velocity ratio, blood flow volume and pressure gradient. Automatically measure the values of maximum velocity, minimum velocity, time interval, pulsatility index and resistance index.

Configuration of the System

Model	i8
B mode	standard
B/M mode	standard

Digital Color Doppler Ultrasound System

M mode	standard
Dual mode	standard
Quad mode	standard
2D Steer	option
CFM mode	standard
PW mode	standard
CW mode	option
Free Steering M mode	option
HPRF	option
TDI	option
Color M mode	option
Curved Panoramic Imaging	option
Trapezoidal image	standard
compound	standard
SRA	option
i-Image	option
elastography	option
ECG	option
Human bodymark	standard
IMT	option
THI	standard
Super Needle	option
general measurement package	standard
OB measurement package	standard
GYN measurement package	standard
URO measurement package	standard
cardiac measurement package	option
vascular measurement package	standard
small parts measurement package	standard
Pediatric measurement package	standard
4D software package	option
D3C60L	option
D3C60L-B	option
D7L40L	option
D6C12L	option
D7C10L	option

D3P64L	option
D6P64L	option
D7L60L	option
D12L40L	option
D7L30L	option
V4C40L	option
V4C40L-C	option
V6C10L	option
D6C15L	option
D3C20L	option
D5C20L	option
D2D12L	option
D3C60L-C	option
D7C10L-C	option
D7L40L-C	option
D6C15L-A	option
D3C20L-A	option
probe connector	4
footswitch	option
DICOM	option

3.2.4. Accessories

Transducers:



Convex probe: D3C60L 2.5-5.3MHz



Convex probe: D3C60L-B 2.5-5.3MHz



Convex probe: D3C60L-C, 2.5-5.3MHz



Linear probe: D7L40L, 5.0-10.0MHz



Linear probe: D7L40L-C, 5.0-10.0MHz



Linear probe: D7L60L, 4.0-10.7MHz



Linear probe: D12L40L, 7.0-18.0MHz



Micro convex probe: D6C12L, 4.0-10.0MHz



Micro convex probe: D7C10L, 4.0-10.7MHz



Micro convex probe: D7C10L-C, 4.0-10.7MHz,



Volume probe: V6C10L, 4.0-10.0MHz



Micro convex probe: D5C20L, 3.5-8.0MHz



Micro convex probe: D5C20L-A, 3.5-8.0MHz



Micro convex probe: D3C20L, 2.5-6.4MHz



Micro convex probe: D6C15L, 4.0-10.7MHz



Micro convex probe: D6C15L-A, 4.0-10.7MHz



Phased array probe: D3P64L, 2.5-4.0MHz



Phased array probe : D6P64L, 4.0-10.0MHz



Volume probe: V4C40L 2.5-5.3MHz

Volume probe: V4C40L-C 2.5-5.3MHz



Linear probe: D7L30L, 4.0-10.7MHz



Pencil probe: D2D16L 1.5-2.5MHz

3.2.5. I/O ports

- VGA output for external monitor
- S-VIDEO, TV output for B&W video printer or Color video printer
- Remote port for video printer
- LAN port output for PC printer, DICOM and image review station
- 6 USB 2.0 ports for flash drive
- Foot switch port

3.3. System Positioning & Transporting

Moving the System

When moving or transporting the system, take the precautions described below to ensure maximum safety for personnel, the system and other equipments.

Before Moving the System

- Completely switch off the system. See Section 3.4.4 “Power Off” for more information.
- Unplug the power cord (if the system is plugged into wall outlet).
- Disconnect all cables from off-board peripheral devices (external printer, etc.) from the console.

NOTE:

- ***To prevent damage to the power cord, DO NOT pull excessively on the cord or sharply bend the cord while wrapping it.***
- Disconnect all probes from main unit. See Section 3.5 “Probes” for more information.
- Store all probes in their original cases or wrap them in soft cloth or foam to prevent damage.
- Replace gel and other essential accessories in the appropriate storage case.
- Ensure that no loose items are left on the main unit.

When Moving the System

- Carry the system with handle, or put the system on the cart to move it. Use extra care when crossing door or elevator thresholds.

NOTE:

- ***Always use the handle to move the system. In order to avoid possible injury or equipment damage.***
- ***Walk slowly and carefully when moving the system.***
- ***Do not let the system strike walls or doorframe.***

Transporting the System

Use extra care when transporting the system in a vehicle. After preparing the system as described above, take the following additional precautions:

- Before transporting, place the system in its original storage case.
- Ensure that the system is firmly secured while inside the vehicle.
- Load the unit aboard the vehicle carefully and over its center of gravity. Keep the storage case still and upright.
- Secure that the system firmly with straps or as directed within the vehicle to prevent movement during transport. Any movement, coupled with the weight of the system, could cause it to break loose.
- Drive carefully to prevent damage from vibration. Avoid unpaved roads, excessive speeds, and erratic stops or starts.

3.4. Powering the System

3.4.1. Acclimation Time

After being transported, the unit requires one hour for each 2.5 ° increment if its temperature is below 10 °C or above 40 °C.

NOTE:

- ***Please keep at least 20 to 30 cm spare space away from the back of the system to ensure well ventilation. Otherwise, with the increasing of the temperature inside the unit, malfunction may occur.***

3.4.2. Connecting and Using the System

To connect the system to the electrical supply:

- Check the power voltage input labeling at rear panel of the system.
- Ensure that the wall outlet is the appropriate type and well grounded.
- Ensure that the system powers off.
- Unwrap the power cord, and allow sufficient slack in the cable so that the plug will not be pulled out of the wall outlet if the system is moved slightly.
- Attach the power plug to the system and secure it in place by using the retaining clamp.

- Push the power plug securely into the wall outlet.

NOTE:

- *Only use the power cord provided by Manufacturer.*
- *Use caution to ensure that the power cord does not disconnect during system use.*
- *If the system is accidentally unplugged, data may be lost.*

WARNING:

- *To avoid risk of fire, the system power must be supplied from a separate, properly rated outlet.*
- *Under no circumstances should the AC power plug be altered, changed, or adapted to a configuration rated less than specified. Never use an extension cord or adapter plug.*
- *To help assure grounding reliability, connect to a “hospital grade” or “hospital only” grounded power outlet.*

3.4.3. Power On

NOTE:

Turn on the green power switch (main power circuit breaker switch, see Fig. 3-1 c in Section 3.2.1 Console Overview) at the back of the system, and then press the Power button on the left of control panel to turn on the system.

Power up Sequence:

The system is initialized and start-up status is reflected on the monitor:

- control panel flashing and then getting dark
- system checking BIOS data
- booting the operation system
- loading software
- entering examination status

HINTS

The power up procedure takes about approx. 180 seconds. If a problem occurs, take a picture and record the error information for service reference.

NOTE:

- *While rotating the keyboard, please hold and place the keyboard slowly and lightly on the desk.*
- *After power off the system, please wait for more than 3 minutes to power on again.*
- *When the system is powered on, for safety reason, please avoid the following*
 - *move the system*

3.4.4. Power Off

To power off the system:

- Press the Power button on the left of control panel.
- When the screen shows “Turn Off”, “Restart”, and “Cancel”, press “Turn off” to shutdown the system.

NOTE:

If the system is down or has not fully shut down, press and hold the Power button located on the left of control panel for more than 4 seconds and release it, this will force the system to shut down completely.

- *Disconnect the probes. Clean or disinfect all probes as necessary. Store them in their original cases to avoid any damage.*
- *To ensure the system is disconnected from the power source, disconnect power plug from the wall outlet.*

3.5. Probes

NOTE:

- *Only use the probes approved by Manufacturer.*

Selecting probes

- Choose the probe according to the different examination.
- Begin the scanning session by choosing the correct application and preset for the examination.

Connecting the Probe

When you connect the probes, please ensure that the probe ports are not active. Place the system in “Transducer Selection” interface by pressing PROBE-key to deactivate the probe ports.

To connect a probe:

- Place the probe’s carrying case on a stable surface and open the case.
- Carefully remove the probe and unwrap the probe cord.
- DO NOT allow head of the probe hang freely. Impact to head of the probe could result in irreparable damage.

NOTE:

Inspect the probe before and after each use for damage or degradation to the housing, strain relief, lens, seal and connector. DO NOT use a probe that appears damaged until its functional and safe performance is verified. A thorough inspection should be performed

during the cleaning process.

- Align the connector with the probe port and carefully push into place with the cable facing the back of the system.
- Turn the probe connector locking lever to “lock” status.
- Carefully position the probe cord so it is free to move and is not resting on the floor.
- When the probe is connected, the system will be automatically recognized.

CAUTION:

- ***Fault conditions can result in electric shock hazard. DO NOT touch the surface of probe connector that is exposed when the probe is removed. DO NOT touch the patient when connecting or disconnecting a probe.***
- ***Take precautions with probe cables. DO NOT bend the cable acutely.***

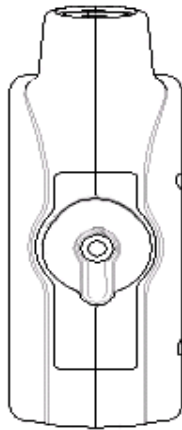


Fig.3-2 a Probe connector “Unlock” status

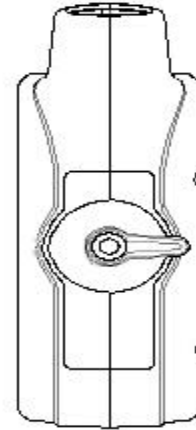


Fig.3-2 b Probe connector “Lock” status

Deactivating the Probe

When deactivating the probe, the probe is automatically placed in a standby mode.

To deactivate a probe:

- Ensure the system is in “Transducer Selection” interface. If necessary, press the PROBE-key to return. .
- Gently wipe off the excess gel from the probe surface.
- Carefully slide the probe toward the probe holder, and place the probe gently in the probe holder.

Disconnecting the Probe

Probes can be disconnected when the system is “Transducer Selection” interface.

To disconnect a probe:

- Turn the connector locking lever to an “Unlock” position.

- Pull the probe and connector straight out of the probe port.
- Carefully slide the probe and connector away from the probe port.
- Ensure that the head of the probe is clean before placing the probe in its storage box.

Transporting the Probe

When transporting a probe a long distance, store it in its original carrying case.

Storing the Probe

It is recommended that all probes should be stored in the original carrying case.

- Place the probe connector into the carrying case.
- Carefully wind the cable into the carrying case.
- Carefully place the probe head into the carrying case. DO NOT use excessive force or impact on the probe head.

3.6 Optional installation

3.6.1 Connect the printer.

1) It needs three cables: Remote cable, Video signal cable, Power cord. See picture in Fig.3-3a.

Note:

If you don't connect remote cable, you still can do the printing by pressing the key on printer.



Fig.3-3a

- 2) Connect the remote cable to remote port on the rear panel of ultrasound system.
- 3) Connect the video signal cable to the TV port of the ultrasound system

3.6.2 Set the system for Video Printer.

Caution:

Please confirm the video printer is turned on and connected well with the main unit, then you can do below setting.

- 1) Press the “setup” key; enter "system" interface, then select "Display Setting ". See picture in Fig.3-4.

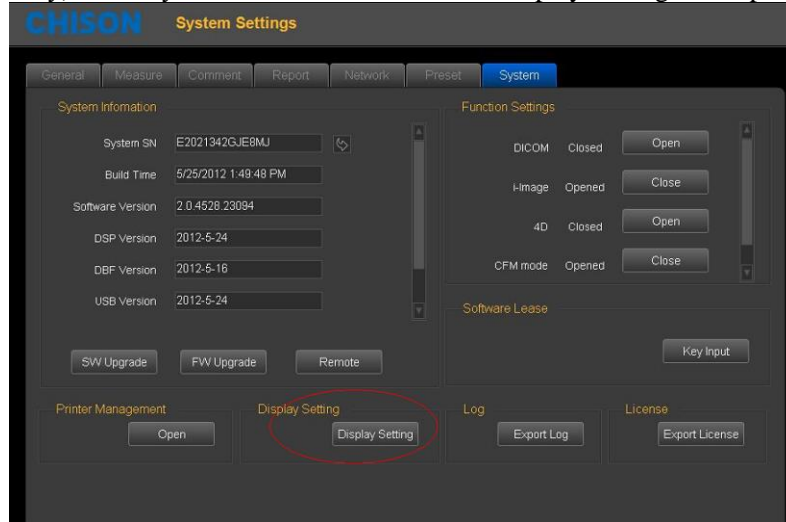


Fig.3-4

- 2) Choose "Extended Desktop" for “Operation Mode”.
- 3) Choose “Monitor” for “Primary Device”, and choose “Secondary Device” for “Television”, and then press “apply”. See picture in Fig.3-5

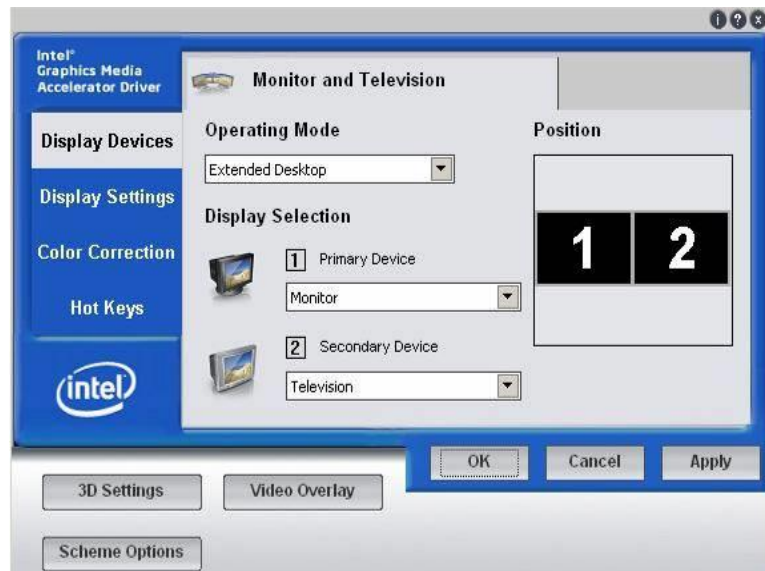


Fig.3-5

- 4) The below dialog box will appear, and press “OK”. See picture in Fig.3-6.

Note:

- *The screen will be black for a while.*

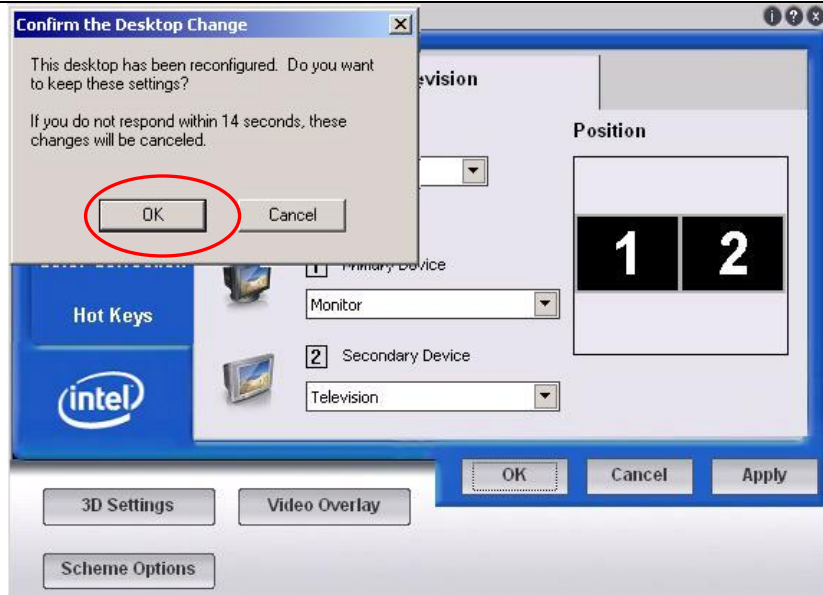


Fig.3-6

- 5) Set the video print option to choose different methods for video print. Choose the "General" setting, and select Keyboard submenu. See picture in Fig. 3-7.

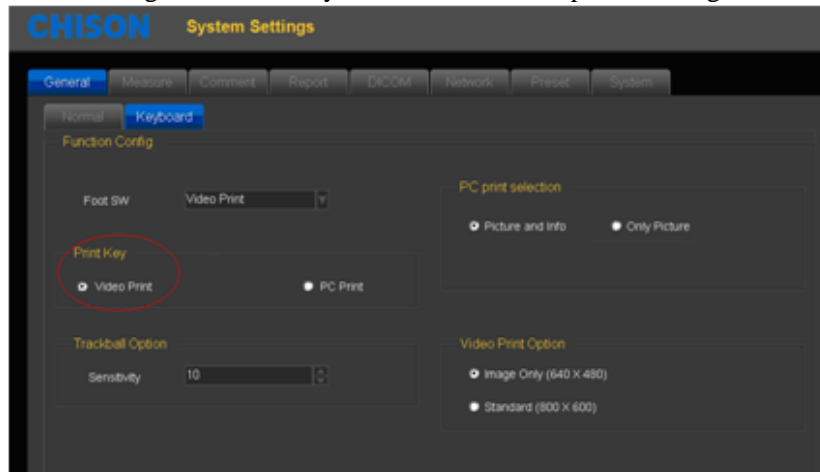


Fig.3-7

- 6) Choose "video print" under print key menu or Foot SW menu, and set the "video print option".

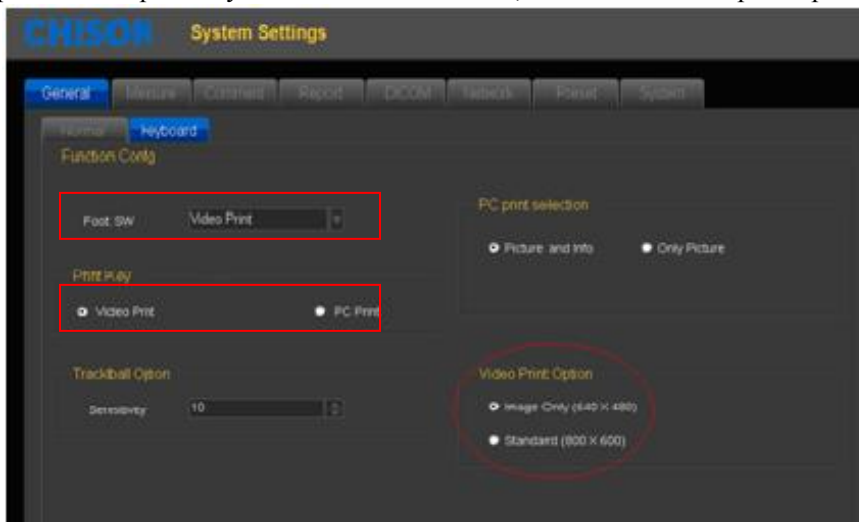


Fig.3-8

“Image only” means only print the ultrasound image.

“Standard” means print the ultrasound image with patient information.

- 7) Press the print key on keyboard or use foot switch for printing.

Note:

- *You need to restart the system after connect the cables between Video printer and the System.*
- *You can't print the system information.*

3.6.3 Connect the PC printer

1. Place the printer smoothly.
2. Connect the printer to the system.
3. Set the print manager. Please see more information in 7.6.
4. Choose “PC print” in system setting, and in the “PC print selection”, chooses “print the image with information”, or “only print image”.

3.7 User Interface Control

- B gain, Color gain and Doppler gain
- TGC
- Brightness
- Acoustic power
- Gamma
- Smooth
- Edge enhance
- Persistence
- Depth control
- Focal position/number
- Dynamic range selection
- Audio volume control
- i-Image™
- Space compound imaging
- Freeze/Unfreeze
- Image storage
- Scanning width
- Zoom
- Dual display: Dual B or color
- Quad display
- L/R inversion
- U/D inversion
- Biopsy guide
- PRF

- Wall filter
- Blood Effecton
- Steering
- Color ROI panning □
- Doppler sample volume adjustment
- Doppler angle correction
- Baseline movement
- Time base scrolling speed
- Annotation
- Patient data entry □
- Measurement and calculation package
- File management and image archiving □
- Clip image saving
- DICOM setting
- User defined preset

3.7.1 Control Panel and Alphanumeric Keyboard

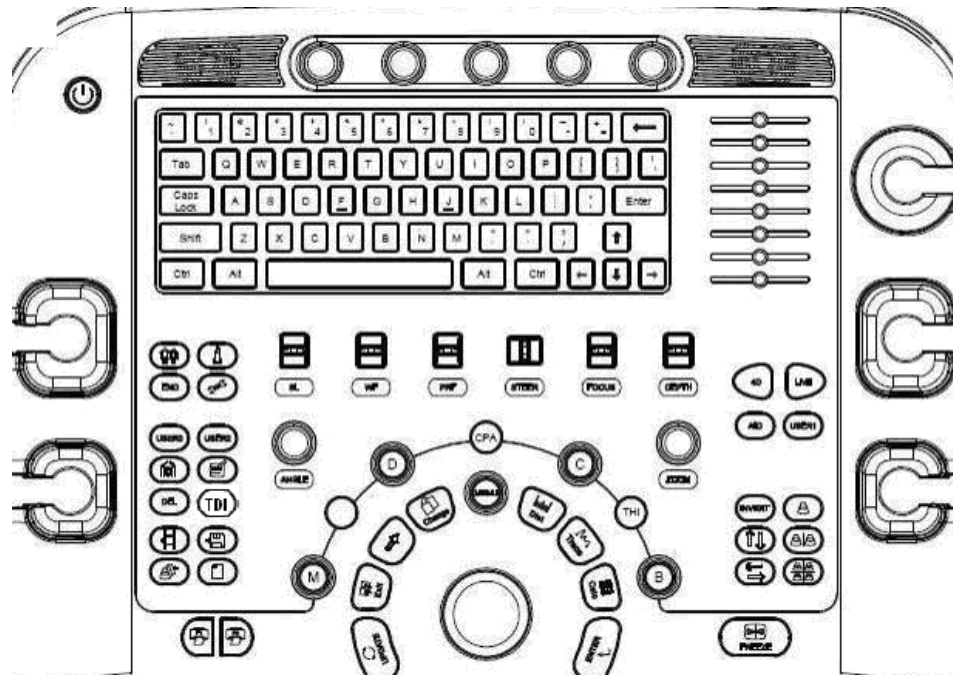







Fig. 3-3: Overview of Control Panel and Alphanumeric Keyboard

See layout of the Control Panel and Alphanumeric Keyboard in the above figure.

The main function of each key is introduced as below.






3.7.2 Exam Function Controls













This group of controls performs patient entry, exam mode/probe type selection and report production etc.



	POWER	<ul style="list-style-type: none"> ✧ Press system power key momentary on the left of Alphanumeric Keyboard to turn on the system. Press this key and choose “turn off” to turn off the system. Press this key longer than 4 seconds to force the system to shut down in case the system is down.
	PATIENT	<ul style="list-style-type: none"> ✧ Use the PATIENT-key to start a new patient record, edit a current patient’s data, or select a patient’s exam data from worklist.
	PROBE	<ul style="list-style-type: none"> ✧ Press PROBE-key to enter the “Transducer Selection” interface showing all available applications supported for the probes connected to the system
	END	<ul style="list-style-type: none"> ✧ Finish the exam of current patient; delete the information of current patient.
	SETTING	<ul style="list-style-type: none"> ✧ Do user defined setting of the system

3.7.3 Mode, Display and Record

This group of controls provides various functions related to the display mode, display orientation, image recording/saving, freezing etc.









	B mode	<ul style="list-style-type: none"> ✧ Press the B-knob to turn on B-mode imaging. The system will stay in B mode if the current state is B, or return to B-mode if the current state is not B (e.g. M, color, Duplex Doppler). ✧ Rotate this knob to change the overall B gain throughout the image.
	CFM mode	<ul style="list-style-type: none"> ✧ Press the C-knob to turn on the Color Flow Map (CFM) mode if the system’s current state is B; ✧ Press this knob can turn on Color if the system’s current state is duplex Doppler; <p>Press the C-knob for second time to turn off color and return to the previous mode (either B-mode or Duplex Doppler).</p> <ul style="list-style-type: none"> ✧ Rotate C-knob to change the overall Color gain for CFM (PD) mode.
	PD (CPA) mode	<p>Activate/turn off PD Mode (also named as Color Power Angio mode).</p> <ul style="list-style-type: none"> ✧ Press the CPA-key to turn on the PD mode if the system is in B mode; ✧ Press the CPA-key for second time to turn off PD and return to the previous mode (either B-mode or duplex Doppler)
	PW mode	<ul style="list-style-type: none"> ✧ Press the PW-knob to turn on the duplex Doppler duplex mode if the current mode is B; ✧ Rotate the knob to change the overall Doppler gain for PW mode, when activate the spectral Doppler mode.
	CW mode	<ul style="list-style-type: none"> ✧ If the probe supports CW mode, e.g. a phased array probe, press CW-key starts CW mode. The CW control operates in the same manner as the PW.



	M mode	<ul style="list-style-type: none"> ✧ Press the M-knob to enter B/M mode if the current mode is B; ✧ Press the M-knob for second time to enter M-mode without B-mode ✧ Press the M-knob for third time to go back to B-mode ✧ Rotate the knob to change the overall M gain throughout the image
	TDI	<ul style="list-style-type: none"> ✧ Press the TDI key to turn on the TDI mode if the current mode is B;
	Dual mode	<ul style="list-style-type: none"> ✧ This key splits the imaging screen for a side-by-side image comparison. ✧ Press Single-mode to quit.
	Quad mode	<ul style="list-style-type: none"> ✧ Four single B mode images can be displayed at the same time when pressing Quad- mode. One is active and the other three are frozen. Press the key again to switch the active one ✧ Press Single-mode to quit.
	Single mode	<ul style="list-style-type: none"> ✧ Enter into the single image display mode.
	UPDATE	<ul style="list-style-type: none"> ✧ Press the UPDATE-key after the sample volume gate is defined to activate the Spectral Doppler mode. Press the UPDATE-key for second time to toggle back to 2D (B or Color) update and deactivate the Spectral Doppler. ✧ In Measurement mode, it can be used to switch between start point and end point (distance), long-axis and short-axis (ellipse), and return back to last position in trace measurement before the measurement is finished.
	THI	<ul style="list-style-type: none"> ✧ Turn on/off THI (Tissue Harmonic Imaging). THI can be activated in 2D mode.
	FREEZE	<ul style="list-style-type: none"> ✧ Freeze/Un-Freeze the ultrasound image and enter/quit the Cine mode automatically.
	ZOOM	<ul style="list-style-type: none"> ✧ Press the ZOOM-knob to turn on the zoom function.
	SAVE	<ul style="list-style-type: none"> ✧ Store still images.
	SAVE CLIP	<ul style="list-style-type: none"> ✧ Store selected clips.
	Browse	<ul style="list-style-type: none"> ✧ Press the key to browse images, edit or print the stored images, etc.

	REPORT	✧ Press this key to generate a report with all the measurements
	PRINT	✧ Print the images when the printer is working.


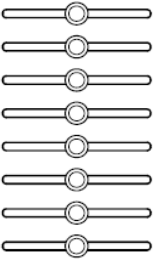






3.7.4 Measurement & Annotation








This group of controls performs various functions related to making measurements, annotating etc.

	Trackball	<ul style="list-style-type: none"> ✧ Position calipers in measurement; ✧ Position 'mouse' cursor for exam mode selection; ✧ Position the M-mode, PW cursor; ✧ Select entry in soft-menu; Select EXAM mode; ✧ Position and re-size the Color Region of Interest (CROI); ✧ Position and re-size the Doppler Sample Volume Gate; ✧ Control digital cine review frames.
	Enter	<ul style="list-style-type: none"> ✧ Confirm the command entry; ✧ Confirm EXAM mode and menu setting; ✧ Confirm caliper and measurement setting; ✧ Toggle Trackball function between Re-sizing and Re-positioning for the CROI, and Doppler Sample Volume Gate.
	DEL	✧ Press DEL-key to clear all measurement path, measurement result, comments, body marks from the imaging screen
	EXIT	✧ Exit from current operation status
	Body Mark	✧ Press the Body Mark-key in real time or cine mode to bring up the entire sets of available Body Mark icons associated with the current application.
	COMMENT	<p>Comments can be added in the image area in real time or cine mode. Manual entry or recalling the phrases from annotation library is allowed.</p> <ul style="list-style-type: none"> ✧ Press COMMENT-key to enter Comment mode. ✧ Press this key during annotation entry to confirm the annotation and quit from Comment mode.
	ARROW	✧ Press the ARROW-key and the mouse cursor will appear on the screen.
	DIST	<ul style="list-style-type: none"> ✧ In 2D (B and Color) cine mode, DIST-key is used for Distance measurement. ✧ In Doppler cine mode, press DIST-key one time to measure Flow Velocity. ✧ In M cine mode, press DIST-key for Distance measurement

	TRACE	<ul style="list-style-type: none"> ✧ In 2D mode, TRACE-key is for measurement of Area/Circumference with tracing method. ✧ In Doppler cine mode, this key can be used to calculate PI and RI ,and can envelope automatically
	CALC	<p>Use this key to activate calculation packages under different applications. This feature supports the optional OB, GYN, Vessel, Urology, Cardiac, Small parts, Pediatrics, Carotid Abdomen and General calculation packages. Refer to Measurement & Calculation section for details.</p>

3.7.5 Image Controls

	MENU	<p>This key provides multiple functions according to the active mode on the screen.</p> <ul style="list-style-type: none"> ✧ In real-time mode, it accesses the Soft- Menu that corresponds to each mode. Press MENU-knob for second time to select the item and adjust the parameters. Press MENU-knob for third time to exit from current item. Rotate the MENU- knob to select the item
	TGC Sliders	<ul style="list-style-type: none"> ✧ Manipulate the TGC (Time Gain Compensation) with 8 pairs of sliders.
	DEPTH	<ul style="list-style-type: none"> ✧ Press this paddle up/down to change the image depth of view.
	PRF	<ul style="list-style-type: none"> ✧ Press this paddle up/down to change PRF setting in the color or Spectral Doppler modes.
	BASELINE	<ul style="list-style-type: none"> ✧ Use this paddle to control the zero velocity Baseline shifting. In Color mode, the maximum detectable velocity is stretched. In Spectral Doppler mode, the spectrum is wrapped around.
	FOCUS	<ul style="list-style-type: none"> ✧ Use this paddle to move the transmitted focal position up or down in any mode while B mode is active. The transmitted focal position remains at the center of the Doppler Sample Volume Gate in spectral Doppler mode, and at the center of the CROI in color mode.
	STEER	<ul style="list-style-type: none"> ✧ In Color mode, use this paddle to change the CROI steer angle for linear probe; in Doppler mode, this paddle can be used to change PW cursor steering directions for linear probe.
	WALL FILTER	<ul style="list-style-type: none"> ✧ Use this paddle to change the setting of the Wall Filter in the color or Doppler modes.

	<p>ANGLE</p>	<ul style="list-style-type: none"> ✧ In the Spectral Doppler mode, the default angle correction feature remains active. In the real time or cine modes, rotate this knob to adjust the Doppler Angle Correction by lining up the cursor with the vessel wall for an accurate reading. The Doppler Angle Correction setting can be adjusted 5 degrees at a time. ✧ Rotate it to adjust the probe direction on the body mark status.
	<p>SK1-SK5</p>	<ul style="list-style-type: none"> ✧ Press the key to change the parameter on upper line while rotate the key to change the parameter on below line of the corresponding menu at the bottom of the screen.
	<p>INVERT</p>	<ul style="list-style-type: none"> ✧ In Color mode, the flow direction (blue and red) can be inverted by pressing the INVERT key. ✧ In PW mode, the spectrum will be reversed according to the baseline when INVERT key is pressed.
	<p>AIO</p>	<ul style="list-style-type: none"> ✧ AIO means Automatic Imaging Optimization. During image scanning, press this key will optimize the image for a better quality in resolution automatically.
	<p>U/D (Up/Down Reversion)</p>	<p>Reverse the 2D(B or Color) image orientation 180 degrees</p>
	<p>L/R (Left/Right Reversion)</p>	<p>In single image mode, use L/R-key to reverse the image between the left/right orientations;</p>
	<p>Change</p>	<p>Change menu in different mode</p>

3.7.6 Soft-Menu Controls

The Soft-Menu is activated depending on the current active mode. The Soft-Menu will provide a second level control to set the parameters in the system. The default setting depends on different applications.

Soft-Menu provides the user with an easy and flexible approach to access additional system controls. The system will display the appropriate menus for the selected Mode and functions.

Chapter 4 Imaging

4.1. General Description

- How to begin an exam
- How to select a probe and an application.
- How to optimize the image
- The operations after getting the image: adding annotation and body mark, storing and recalling the image

4.2. Beginning an Exam

Begin an exam by entering new patient information. You should enter as much information as possible, such as patient ID, patient name.

The patient's name and ID number are retained with each patient's image and transferred with each image during archiving or hard copy printing.

CAUTION:

To avoid patient identification errors, always verify the identification with the patient. Make sure the correct patient identification appears on all screens and hard copy prints.

4.2.1. Selecting a Probe and an Application

The system has four built-in probe connectors, so it can connect four probes at the same time.

Press PROBE-key to back to "Transducer Selection" interface.

1: Select the proper probe.

2: Select the proper Application.

3: Double click the preset to enter into the default preset, or click the user defined preset at the bottom of this interface.

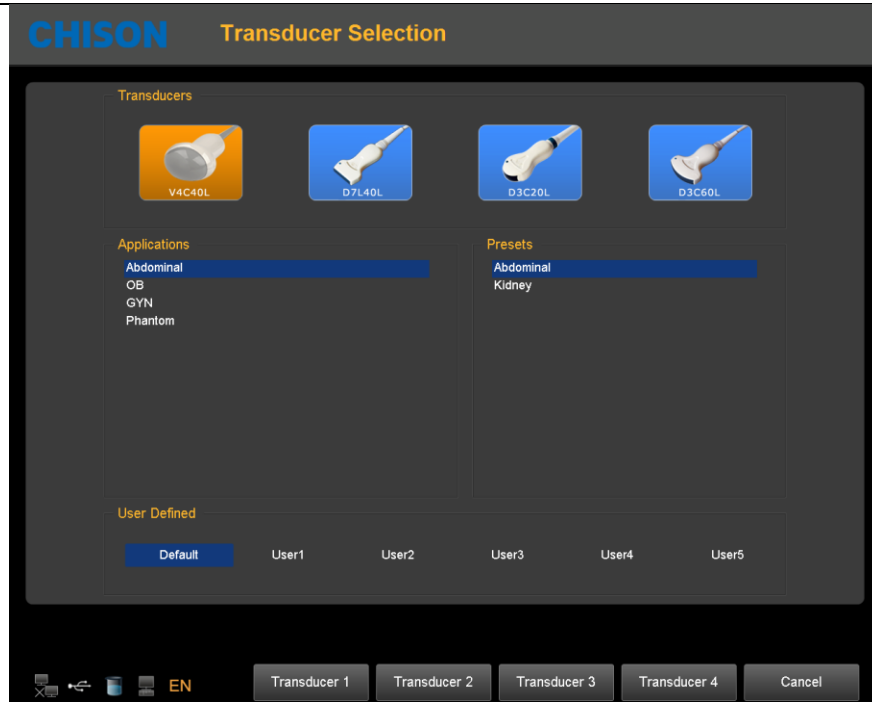


Fig. 4-1

4.2.2. Patient Data Entry

Press the PATIENT-key to display the Patient interface.

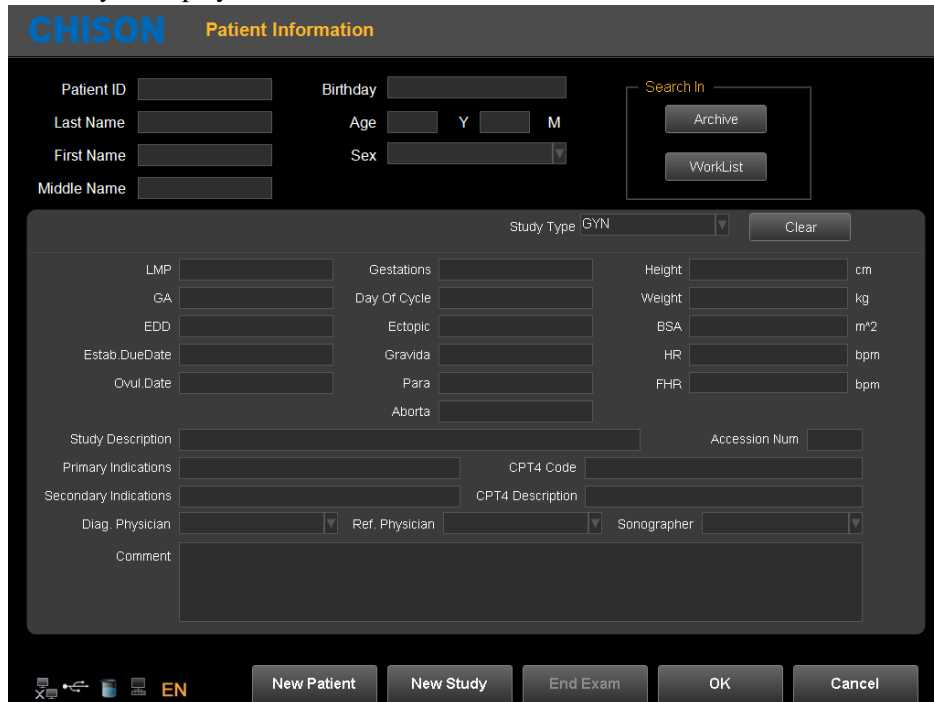


Fig. 4-2

Method of entering the patient information:

- 1) Move the cursor to input area and input the patient information through the keyboard. Move trackball to exchange between each option. E.g., patient name, birthday (birthday result will be automatic calculated

after entering the age), age (age result will be automatic calculated after entering the birthday), sex, weight, height.etc.

- 2) Click “OK” key to save the data after entering the appropriate information, then the system will go back to B mode.

4.2.3. Display Interface

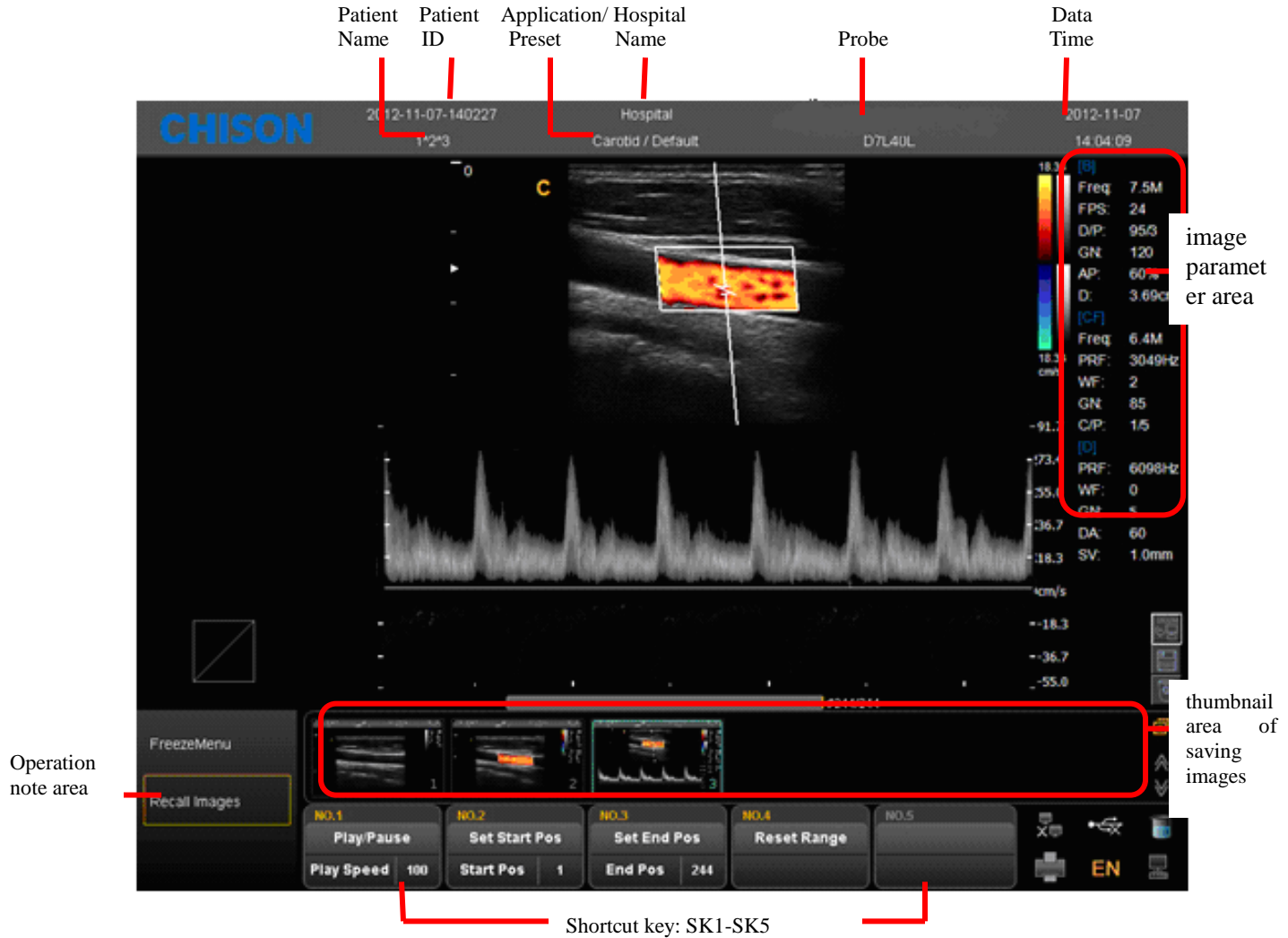


Fig. 4-3

- image parameter area: Display the parameters under current mode. The different modes have different parameters.
- thumbnail area of saving images: This area displays the thumbnail images of current patient. Selecting thumbnail images can recall this image immediately.
- shortcut key SK1—SK5: SK1—SK5 is corresponding to the menu of NO.1-NO5. Press the shortcut key to select the function in the first line. Rotate the shortcut key can adjust the parameters in the second line.
- operation note area: The first line shows the status of the current system. The second line indicates the current

function of the trackball. In the picture Fig. 4-3, the first line display “freeze menu”. It means the system is in frozen status. The second line display “recall image”. It means moving the trackball can recall the images in current status.

- system information area: Display the current system information. Moving the mouse to different icon will display detailed information accordingly.



Display current network situation. It will display the IP address after connecting to the network.



Display the status of current removable disks. If exist removable disks, click this icon can quickly enter into the storage manager interface and do the operation for disks.



Fig. 4-4



Display space size of all the drivers.



Printer management, refer to section 7.6



Task manager: display the state of current task

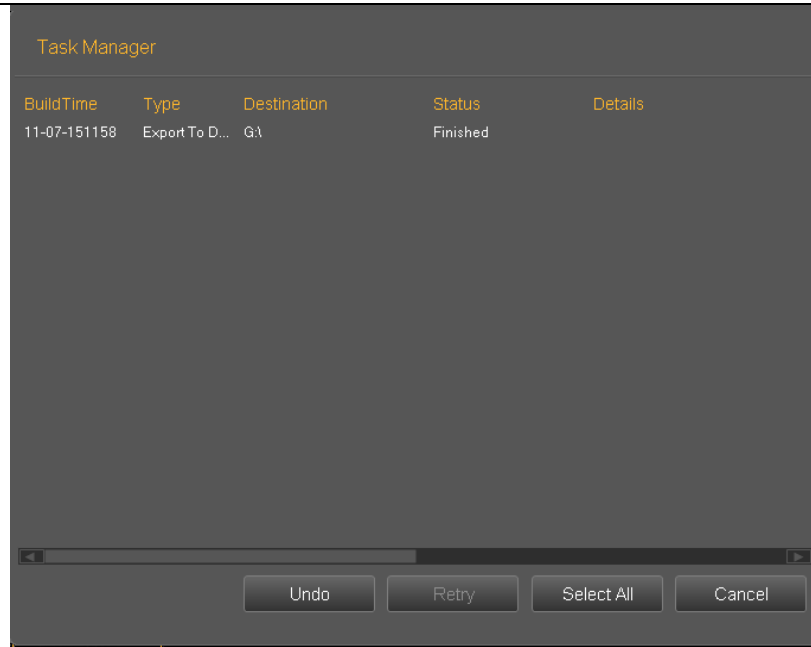


Fig. 4-5



Switch input language

4.3. Optimizing the Image

4.3.1. Image Parameters Display

B	Meaning	CFM/PD	Meaning	PW	Meaning	M	Meaning
Freq.	Frequency	Freq.	Frequency	PRF	PRF	SR	Sweep Speed
FPS	Frame rate	PRF	PRF	WF	Wall Filter		
D/P	D: Dynamic range P: Persistence	WF	Wall Filter	GN	Doppler Gain		
GN	B Gain	GN	Color Gain	DA	Doppler angle		
AP	Acoustic power	C/P	C: Color map /direction power P:Persistence	SV	Sample volume gate		
D	Display depth						

4.3.2. Scanning Modes

The system can support the following modes:

- B (2D) Mode
- 2B Mode
- 4B Mode

- B/M Mode
- M Mode
- PW Mode
- CFM Mode
- CPA Mode
- CW Mode
- B/BC Mode
- Free Steering M mode
- Color M Mode
- HPRF
- Curved Panoramic Imaging Mode
- TDI Mode

4.3.3. B Mode

Intended Use:

B-mode is intended to provide two-dimensional images and measurement capabilities concerning the anatomical structure of soft tissue. Press B -knob to enter into B mode. Rotate B-knob to adjust B gain.

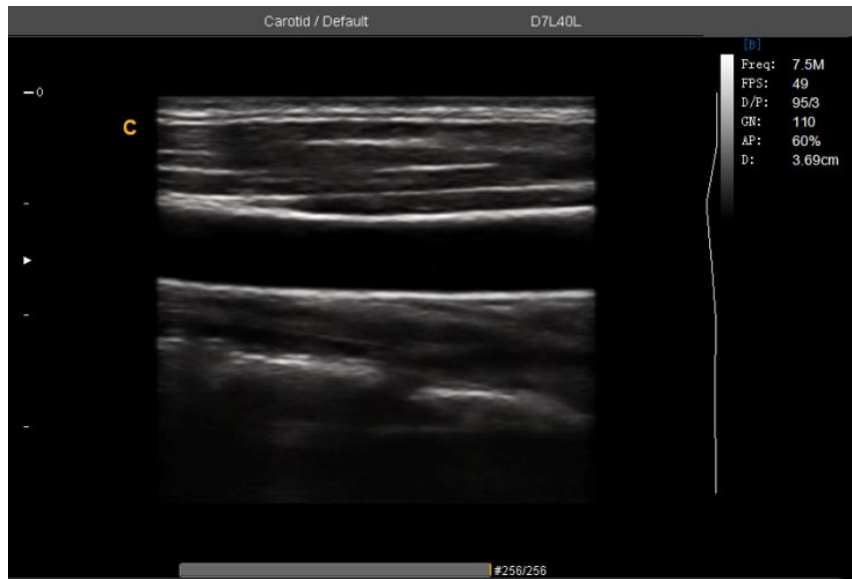




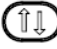



Fig. 4-6 B Mode

4.3.4. Dual Mode

In B mode, press  key .System will and display the current image at the left side of the screen, Press  key second time, system will freeze the image displayed at the left side and activate the image displayed at the right side at the same time. Press  key continuously to achieve exchange of the freeze/real status

between left side image and right side image. Use the  key to invert image which is activated in left and right direction. Use the  key to invert image which is activated in left and right direction.. Press  key to go back to B mode.

NOTE:

- *There is only one image could be activated*
- *Dual display is also available for the Color Mode*

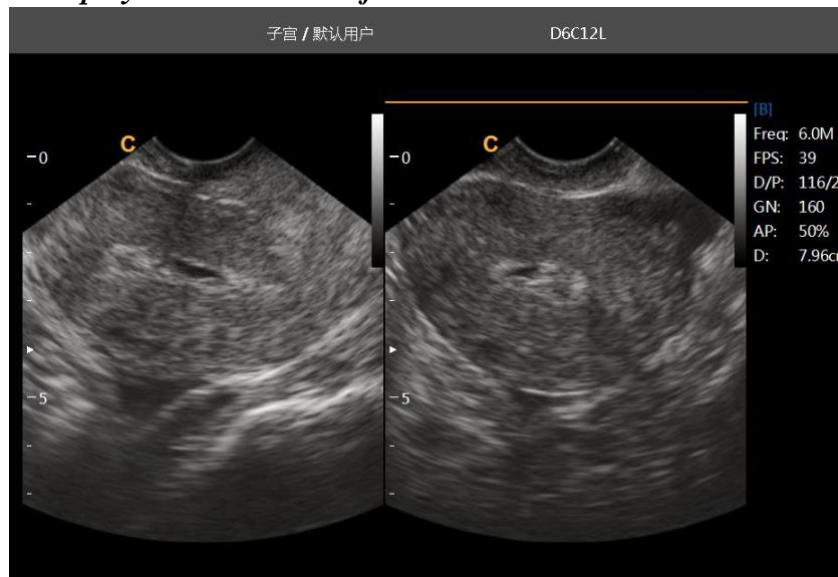






Fig. 4-7 B/B Mode

4.3.5. Quad Mode

In B Mode, press  key, the image which is activated will be displayed at the upper left side of the screen, press  key continuously will freeze and activate the upper right image, lower left image, and lower right image in order. Use  key do the Left/Right invert for current archiving image, use the  key which on the control panel can do the up/down invert for current achieving image. It will go back to B Mode if press [Single mode] key again.

NOTE:

- *There is only one image could be activated at one time*
- *Quad display is also available for the Color Mode.*

4.3.6. B/BC Mode

In active color mode, turn the [B/BC] item on to display a real B mode image at the left side of the screen and active Color mode image at the right side of the screen.



4.3.7. B/M and M Mode

Intended Use:

M-mode is used to determine patterns of motion for objects within the ultrasound beam. The most common use is for viewing motion patterns of the heart.

This system provides B mode image and M mode image at the same time. Press M knob to enter into B/M mode. Press M knob again to enter into M mode. M mode is fit for heart scanning and measurement.

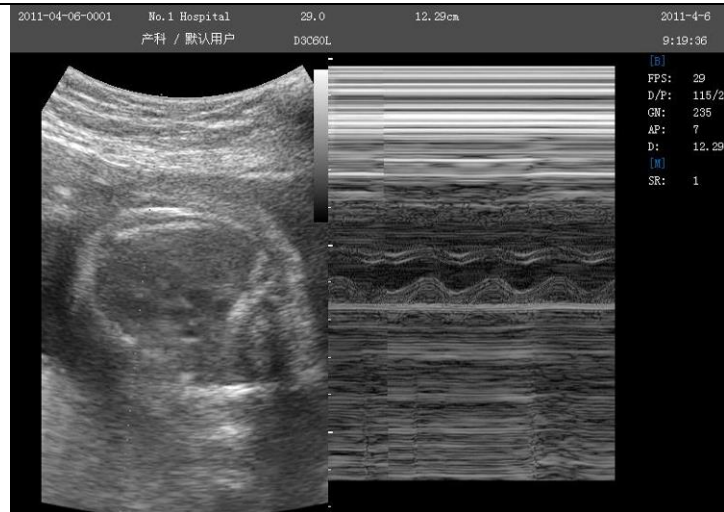


Fig. 4-8 B/M Mode

M-mode Exam Procedure:

- Get a good B-mode image. Survey the anatomy and place the area of interest near the center of the B-mode image.
- Press the M-knob, move the Trackball to position the M cursor over the area that you want to display in M-mode.
- Adjust the Sweep Speed, TGC, Gain etc, as needed.
- Press the FREEZE-key to stop the M scanning.
- Record the image to hard disk or to the printer (hard copy device).
- Press FREEZE-key to continue imaging.
- Press M-knob to enter into M-mode.
- Press M –knob again to exit M-mode.

M-mode Scanning Hints:

Sweep speed: control speed of M-mode update.

4.3.8. Free Steering M mode (Option)

Free Steering M mode is only available on phased array probe. This mode can give you the ability to manipulate the cursor at different angle and position. The M-mode display changes as per the M cursor position. User can activate the Steering M mode using Soft-Menu. Turn the item Steering M on, and turn Angle to rotate Steering M line .The system provides maximum 3 Steer M line and you can select either of them with SET-key.

- Trackball: used to move the Steer M line.
- Angle: used to rotate the Steer M line.

4.3.9. CFM mode

Intended Use:

CFM is a Doppler mode intended to add color-coded qualitative information concerning the relative velocity and direction of fluid motion within the B-mode image.

CFM is useful to see flow in a broad area. It allows visualization of flow in the CROI, whereas Doppler mode provides spectral information in a smaller area. CFM is also used as a stepping stone to Doppler mode. You can use CFM to locate flow and vessels prior to activating Doppler.

In CFM mode, move the trackball to change the position of sampling box. [STEER] button is used to adjust the angle of color sampling box (if current probe is linear probe). Press ENTER-key to fix the position of color sampling box. At this time adjust the size of color sampling box through moving trackball. Press ENTER-key again and move trackball to change the color sampling position again.

Press C knob to enter into CFM mode; after C knob light is on, rotate the knob for adjusting the gain of CFM.

CFM mode Exam Procedure:

- Follow the same procedure as described under B-mode to locate the anatomical area of interest.
- After optimizing the B-mode image, add Color Flow.
- Move the color region of interest CROI as close to the center of the image as possible.
- Optimize the color flow parameters so that a high frame rate can be achieved and appropriate flow velocity can be visualized.
- Press FREEZE-key to hold the image in cine memory.
- Record color flow image as necessary.

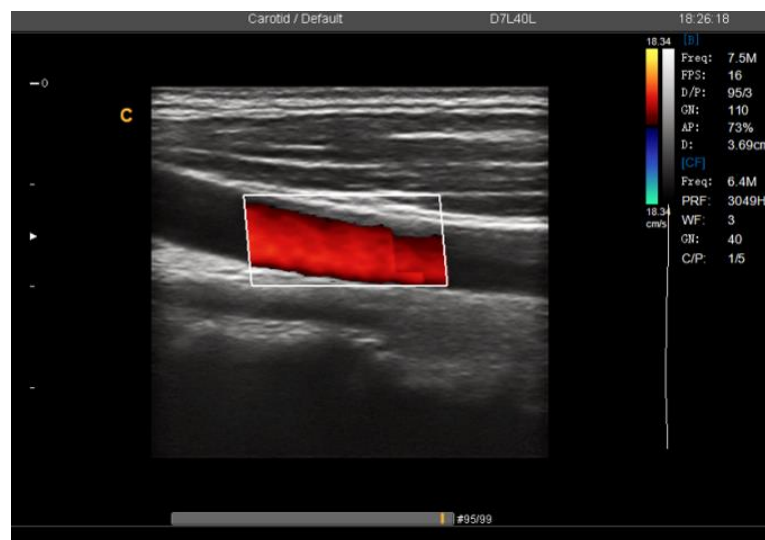


Fig. 4-9 CFM Mode

CFM Scanning Hints:

PRF: increase/decrease the PRF on the color bar. Imaging of higher velocity flow requires increased velocity scale values to avoid aliasing

Wall Filter: affect low flow sensitivity versus motion artifact

Color Map: allow you to select a specific color map. It shows the direction of the flow and highlights the higher velocity flows.

Color Gain: amplify the overall strength of echoes processed in the CROI

Persistence: affect temporal smoothing and color Doppler 'robustness'.

Line Density: trade frame rate for sensitivity and spatial resolution. If the frame rate is too slow, decrease the CROI size and the line density.

4.3.10. PD (CPA) Mode

Power Doppler Imaging (PD) is a color flow mapping technique used to map the strength of the Doppler signal coming from the flow rather than the frequency shift of the signal. Using this technique, the ultrasound system plots color flow based on the number of reflectors that are moving, regardless of their velocity. PD does not map velocity, therefore it is not subject to aliasing.

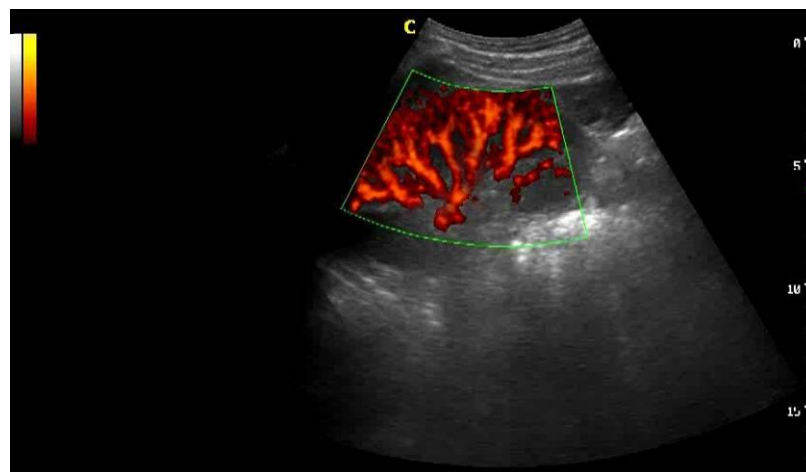


Fig. 4-10 PD Mode

Press CPA -key to enter into the CPA mode and then CPA lights is on

Direction PD mode

In Power Doppler (CPA) mode, press MENU -knob to pop up the PD Menu . Rotate MENU- knob to select the PD (CPA) Mode and press MENU-knob to enter into DPD mode.

If you need go back to PD mode from DPD mode, you could press CPA key or select the PD mode item in the DPD mode.

4.3.11. Color M mode(Option)

Color M mode is used for fetal cardiac applications. Color flow overlays color on the M-mode image using velocity and variance color maps. The color flow wedge overlays the B-mode image and M-mode timeline. The color flow maps available in M-mode are the same as in CFM mode.

Color M-mode is a Doppler mode intended to add color-coded qualitative information concerning the relative velocity and direction of fluid motion within the M-mode image.

If the system is in **color** mode and the probe supports **Color M** mode (e.g. phased array probe), press **M**-knob to enter **Color M** mode.

4.3.12. PW mode

Intended Use:

Doppler is intended to provide measurement data concerning the velocity of moving tissues and fluids. PW Doppler lets you examine blood flow data selectively from a small region called the Sample Volume.

The X axis represents time while the Y axis represents velocity in either a forward or reverse direction.

PW Doppler is typically used for displaying the speed, direction, and spectral content of blood flow at selected anatomical sites.

PW Doppler can be combined with B-mode for quick selection of the anatomical site for PW Doppler examination. The site where PW Doppler data is derived appears graphically on the B-mode image (Sample Volume Gate). The Sample Volume Gate can be moved anywhere within B-mode image.

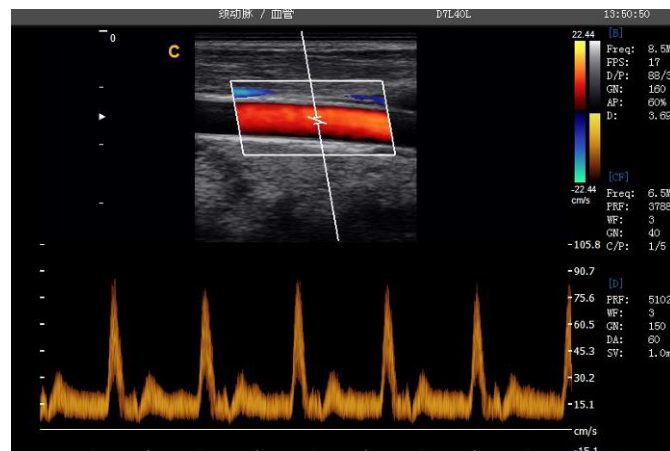


Fig. 4-11 PW Mode

PW mode Exam Procedure:

- Get a good B-mode image. Press C-knob to help locate the vessel you wish to examine.
- Press PW-knob to display the sample volume cursor and gate.
- Position the sample volume cursor by moving the Trackball left and right. Position or re-size the sample

volume gate by moving the Trackball up and down, then press Enter-key.

- Press UPDATE-key to display PW Doppler spectrum and the system will run in combined B+Doppler mode. The Doppler signal can be heard through the speakers.
- Optimize the PW Doppler spectrum, as necessary.
- Ensure that the sample line is parallel to the blood flow.
- Press FREEZE-key to hold the trace in cine memory and stop imaging.
- Perform measurements and calculations, as necessary.
- Record results with your recording devices.
- Press FREEZE-key to resume imaging.
- Repeat the above procedure until all relevant flow sites have been examined.
- Replace the probe in its respective holder.

When entering Duplex mode for the first time, the Doppler spectrum is not activated. The Doppler Sample Volume appears in the default position, and the B mode image or 2D (either B or Color or TDI) mode are active. Moving the Trackball will change the Sample Volume position. Press the Enter-key to toggle the Trackball function between Sample Volume Gate position and size. Press the UPDATE-key after the Sample Volume Gate is defined to activate the Spectral Doppler mode. Press the UPDATE-key for second time to toggle back to 2D (B or Color) update and deactivate the Spectral Doppler.

Doppler mode Scanning Hints:

The best Doppler data will be got when the scanning direction is parallel to the direction of the blood flow; when the scanning direction is perpendicular to the anatomic target, you can get the best B mode image, so you should keep the balance as you don't usually get both an ideal B-mode image and ideal Doppler data simultaneously. PRF: adjust the velocity scale to accommodate faster/slower blood flow velocity. Velocity scale determines pulse repetition frequency.

Wall Filter: remove the noise caused by vessel or heart wall motion at the expense of low flow sensitivity.

Baseline: adjust the baseline to accommodate faster or slower blood flows to eliminate aliasing.

Angle Correct: optimize the accuracy of the flow velocity. It estimates the flow velocity in a direction at an angle to the Doppler vector by computing the angle between the Doppler vector and the flow to be measured. This is special useful in vascular applications where you need to measure velocity.

Doppler Gain: allow you to control the background information of spectral.

Sweep Speed: control speed of spectral update.

Doppler Sample Volume Gate Position and Size (Trackball and SET)

Move the sample volume on the B-mode's Doppler cursor. The gate is positioned over a specific position within the vessel.

- To move Doppler cursor position, turn the trackball left or right until positioned over the vessel.

- To move sample volume gate position, move the trackball up or down until positioned inside the vessel.
- To size sample volume gate, press Enter-key to toggle trackball function from sample volume gate positioning to sizing, then move the trackball to change sample volume gate size.

Activating Triplex mode

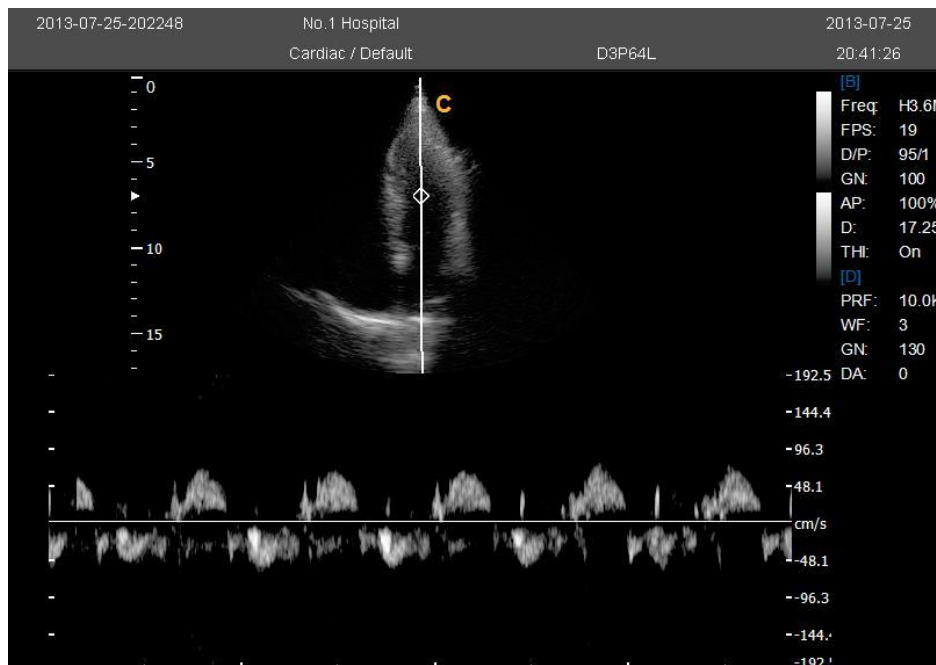
- While in **B** mode, press **C**-knob and **D**-knob to access the **B+Doppler+Color** mode, turn triplex menu on, press the **UPDATE**-key to enter the **Doppler** active mode.
- While in **B** mode, press **CPA**-key and **D**-knob access the **B+Doppler+PD** mode, turn triplex menu on, press the **UPDATE**-key to enter the **Doppler** active mode.

4.3.13. CW Mode(Optional)

Continuous Wave Doppler allows examination of blood flow data all along the Doppler cursor rather than from any specific depth. Gather samples along the entire Doppler beam for rapid scanning of the heart. Range gated CW allows information to be gathered at higher velocities.

It works with a phased array or pediatric probe.

If the velocity of the blood flow is even too high for the **HPRF** mode to detect, you have to try **CW** mode. Press **CW**-key to enter **CW** mode when the probe supports **CW** mode.



4.3.14. HPRF Mode (Option)

High Pulse Repetition Frequency (HPRF) is a special operating mode of PW Doppler. In HPRF mode,

multiple energy pulses are used. This allows higher velocities to be detected without causing aliasing artifacts. HPRF mode is used when detected velocities exceed the processing capabilities of the currently selected PW Doppler scale or when the selected anatomical site is too deep for the selected PW Doppler scale.

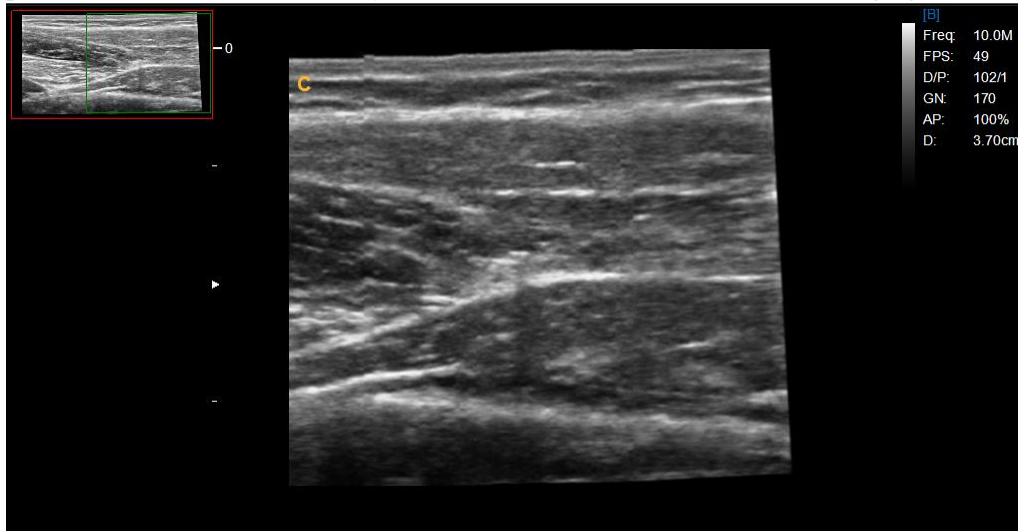
HPRF is invoked when you are operating in PW Doppler mode and conditions activate HPRF when the velocity scale factor or sample volume gate depth exceeds certain limits. When HPRF is active, multiple sample volume gates appear along the Doppler cursor. Doppler information can be received from any of the multiple sample volume gates. The Doppler signals from all the gates are added together and displayed in one spectrum. It works with a phased array or pediatric probe.

4.3.15. TDI Mode(Optional)

TDI mode is tissue Doppler mode, which is intended to provide information of low-velocity tissue motion, specifically for cardiac movement. Only phased array probe is available for TDI function.

4.3.16. Curved Panoramic Imaging (Option)


Moving through the probe to get a series of B-mode images, Press Freeze key and turn the menu on, select curved panoramic item, turn it on. The system will enter into Curved Panoramic Imaging mode.



4.3.17. Elastography mode(Optional)

Ultrasound elastography is a new ultrasound technology to study traditional ultrasound can not detect tumors and spread of disease imaging, can be applied to the breast, thyroid, prostate and other aspects.

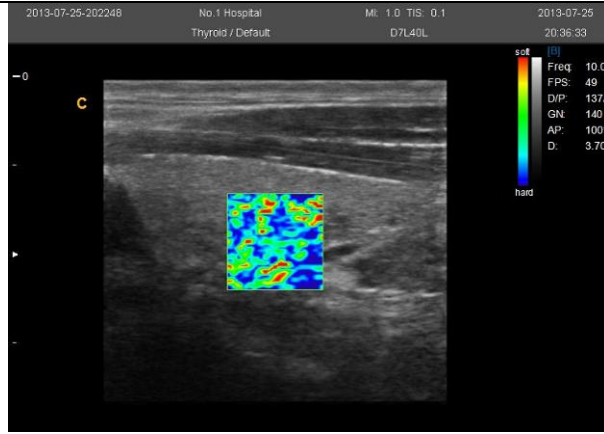In B mode, turn elastography menu on can enter into elastography mode.



4.3.18. Optimize Image

Methods for optimizing:

1. Use soft menu

Press MENU- knob under the display mode and tune out the soft menu. Press Change –key now can do the exchange of the menu to different exam mode. Rotate MENU- knob to select the menu items. Press MENU- knob to enter. Now rotate MENU -knob can change the parameter of selected item. After adjustment, press Menu-knob to exit from selected item, at this time rotate the Menu-knob can select the item again. Select the “EXIT” icon or press Menu-knob to exit from the menu.

2. Use shortcut menu

SK1—SK5 shortcut key respectively corresponding NO.1—NO5 menu. Press shortcut key will choose the menu function for the first line; rotate shortcut can adjust parameters for next line.

3. Use other keys

4.3.19. Image Optimization of B Mode

Soft menu adjustment:

Menu	Function	Menu Adjustable Range
Scan width	Adjust scan width of B image; frame rate will be fast if scan width is small.	6% to 100%
Image Rotate	Change the direction of B image. The angle will be increased at 90° interval in clockwise direction	Invert angle: 0°、90°、180°、270°.
Gamma	Adjust gamma value.	0~8 rank adjustable.
Smooth	Adjust the image smooth.	0~7 rank adjustable.
Edge enhance	Adjust edge enhance.	0~6 rank adjustable.

Digital Color Doppler Ultrasound System

A. power	Adjust acoustic power.	0-100%
Trapezoidal Mode	Turn on Trapezoidal Mode	On/Off
Brightness	Adjust brightness of LCD monitor	0-100%
Biopsy	Turn on the biopsy function	
2D Steer	Adjust the steer angle of B image	Multi-angle adjustable, only support linear probe.
Elastography	Turn on Elastography Mode	Turn on ,off
ECG	Turn on ECG	Turn on ,off
Super Needle	Turn on Super Needle function	Turn on ,off
Angle(needle)	Adjust the steer angle of Needle	Multi-angle adjustable
Utility		

Shortcut adjustment:

Shortcut (Action)	Menu	Menu function	Adjustable range of menu
SK1 (press)	frequency	Adjust frequency of the probe.	Relate to current probe.
SK1 (rotate)	dynamic range	Adjust the image dynamic range, increase or decrease, System dynamic range and contrast resolution.	multi-step adjustable
SK2 (press)	Focus Numbers	Changes focus numbers.	1~4 adjustable, relate to the probe type and depth.
SK2 (rotate)	Focus position	Changes focus position.	
SK3 (press)	Compound imaging	Open compound imaging	On, Off
SK3 (rotate)	i-Image	Open optimizing image	multi-step adjustable
SK4 (press)	SRA	Open SRA	It can be on or off while COMPOUND is off.
SK4 (rotate)	Persistence	Increase/decrease the contrast resolution of the image	0~7 grade adjustable, clockwise for increasing, anticlockwise for decreasing. Adjust range: 1.
SK5 (press)	Density	Change the image density	High and Low adjustable

SK5 (rotate)	Noise reject	Adjust the image noise suppression	Adjust range: 0-255, clockwise for increasing, anticlockwise for decreasing. Adjust range: 1.
-----------------	--------------	------------------------------------	--

Other adjustments:

1 Depth (DEPTH toggle key)

DEPTH toggle key is used for the adjustment of image depth. Depth will increase if adjust downward; Depth will decrease if adjust upward. For the best resolution of focusing and edge enhancement, it needs to adjust TGC after the depth has been adjusted.

2 Focus control (FOCUS toggle key)

Add a specific area to achieve the decomposition of image focusing optimization. Increase focusing numbers or move focusing position could increase the resolution of the special areas and strengthen scanning of special areas.

Adjust FOCUS toggle key could do up/down movements for launching the focusing position.

Note: Change focus numbers will affect the frame rate. Frame rate will be slow if focus numbers are increased.

3 Scanning direction (left/right invert key, up/down invert key)

Under the single display mode, the scanning direction will be left/right inverted if press the left/right invert key; images scanning direction will be up/down inverted if press U/D key.

4 Gain (B knob)

Adjust gain of B mode could enhance or reduce quantities of echo information in images. Adjust the main gain can adjust sensitivity of overall images (brightness). Rotate the knob of B mode could adjust B gain after the B light is on. The gain will be increased if rotate clockwise; the gain will be decreased if rotate contra. Adjust range is from 0 to 255. TGC will influence on each other between gain adjustment and TGC adjustment.

5 TGC

TGC with 8 adjustable slides: Slide the set of slider bars can change the depth gain of 2D images.

TGC returns signal amplifier to correct for the attenuation caused when adding depth. TGC balance the image and make the density of echo uniformly distributed in images. Similarly the enlargement of every sliding channel area is also different.

The range of TGC will redistribute according to the new depth while depth is changing.

Move the slide bar Left/Right can reduce/increase TGC.

6 Image magnification (ZOOM -knob)

Press ZOOM- knob, press UPDATE-key to choose the size of ROI box ,and then press UPDATE-key again to choose the position. Press ENTER- key can realize multiple enlargements. Press EXIT-key or ZOOM knob to exit operation.

4.3.20. Image Optimization of M mode

Shortcut menu adjustment:

Shortcut	Menu	Menu Function	Menu item adjustable rang
SK1 (rotate)	Color Map	Adjust the color of the M image	Adjust option:user,type1~9
SK2 (rotate)	Speed	Adjust scanning speed	1, 2, 3, 4 adjustable. adjust clockwise, the numerical increase and adjust anticlockwise,the numerical decrease. Adjust range: 1.
SK5 (press)	Layout	Adjust layout	UD, LR

4.3.21. Color Flow Map(CFM) Mode image optimization

Soft menu adjustment:

MENU	Menu Function	Menu Item adjustable Rang
Color Mode	The way of selecting color	Velocity、variance
Packet Size	Adjust the value of packet size	6~15
Wall Thre	Adjust wall threshold	0~15
Blood Efection	Choose different blood efection	Smooth, HighRes
B/BC	open B/BC Mode	turn, off

Shortcut Adjustment:

Shortcut (action)	Menu	Menu Function	Menu Item adjustable Rang
SK1 (press)	Frequency	Adjust the frequency of probe	Refer to current probe
SK1 (rotate)	Steer	Adjust angle of sampling box of blood flow under the linear probe	Adjust range from -20 to 20
SK2	Wall Filter	Change of Wall filtering	0~3 adjustable

(press)			
SK2 (rotate)	Color Map	Change of Color Map	Adjust option:User, Type:1- 9
SK3 (press)	Flow	Change sensitivity of blood	High, Middle and Low for option
SK3 (rotate)	PRF	Adjust PRF value	The maximum PRF depends on the probe and the position of sampling box.
SK4 (press)	Invert	Realize invert of blood flow	Turn on or Turn of the invert
SK4 (rotate)	Persistence	Improve the current color	0~7 grade adjustable. Adjust range: 1.
SK5 (press)	Density	Change images density	High and Low adjustable.
SK5 (rotate)	Baseline	Change the baseline of color-map	-3~3 adjustable.

Other adjustments:

1、 CFM gain control.

Rotate C- knob after C lights is on .Adjust the color gain and adjust range is from 0 to 255.

2、 Sampling box.

Moves sampling box through trackball to the area which you are interested and press the ENTER- key to confirm the position of sampling box then adjust the size of box trough trackball and press ENTER- key to confirm.

Use the STEER toggle key to change the direction of sampling box under the linear probe.

4.3.22. Power Doppler(CPA) Mode imaging optimization

Soft menu adjustment:

MENU	Menu Function	Menu Item Adjustable Rang
Packet Size	Adjust the value of packet size	6~15, adjust range: 1
Wall Thre.	Adjust wall threshold	0~15, adjust range:1
DPD Mode	Exchange PD Mode and DPD Mode	Press it can exchange between PD Mode and DPD Mode

Shortcut Adjustment:

SK1 (press)	Frequency	Adjust the probe launch frequency	Refer to current probe
SK1 (rotate)	Steer	Adjust angle of sampling box of blood flow under the linear probe	Adjust range from -20 to 20,
SK2 (press)	Wall Filter	Change of Wall filtering	0~3 adjustable
SK2 (rotate)	Color Map	Change of Color Map	type 1 ,user
SK3 (press)	Flow	Change sensitivity of blood	High, Middle and Low for option
SK3 (rotate)	PRF	Adjust PRF value	The maximum PRF depends on the probe and the position of sampling box.
SK4 (press)	Density	Change images density	High and Low adjustable
SK4 (rotate)	Persistence	Improve the current color	0~7 grade adjustable, adjust clockwise, the numerical increase and adjust anticlockwise, the numerical decrease.Adjust range: 1.

Other adjustments:

1、 CPA gain control.

Rotate C knob after CPA light is on then adjust the gain and the range is from 0 to 255.

2、 Sampling box.

Moves sampling box through trackball to the area which you are interested and press the ENTER- key to confirm the position of sampling box then adjust the size of box trough trackball and press ENTER- key to confirm.

Use the STEER toggle key to change the direction of sampling box under the linear probe.

Note: this function can only support linear probe.

4.3.23. Pulse Wave Doppler Imaging Optimization(PWD)

Soft menu adjustment:

MENU	Menu Function	Menu Item Adjustable Rang
2D Map	Adjust the color of spectrum	1~6, adjust range: 1
Spectrum Enhance	Adjust the brightness of the spectrum	0~3, adjust range:1

Digital Color Doppler Ultrasound System

DynamicRange	Adjust the dynamic range of the spectrum	46~67
Triplex	turn on/off triplex mode	turn on, turn off
Auto Cal Parameter	set the auto Calc parameters	
DTrace Smooth	Adjust the smooth of the trace of the spectrum	0~3, adjust range:1
Threshold	Adjust the threshold of the spectrum	1~5, adjust range:1
Trace Area	set the auto cal range	above,below,all

Shortcut adjustment:

Shortcut (action)	Menu	Menu Function	Menu Item Adjustable Rang
SK1 (press)	Invert	Spectrum invert	Turn on or Turn off the invert function
SK1 (rotate)	Color Map	Adjust the color map of spectrum	Adjust options:user,type1~9
SK2 (press)	Wall Filter	Change of wall filtering	0~3 adjustable
SK2 (rotate)	Audio	Adjust volume of Doppler	0~100% adjustable.
SK3 (press)	DVmean	turn on to display the trace of the Vmean	turn on, turn off
SK3 (rotate)	PRF	Adjust PRF value	The maximum PRF value depends on the probe and the position of sampling gate.
SK4 (press)	DVmax	turn on to display the trace of the Vmax	turn on, turn off
SK4 (rotate)	Speed	Adjust scanning speed	1, 2, 3 adjustable. adjust clockwise, the numerical increase and adjust anticlockwise, the numerical decrease. Adjust range: 1
SK5 (press)	Auto Cal	turn on auto cal	turn on, turn off
SK5 (rotate)	Baseline	Adjust the baseline position	0~6 adjustable.

Other adjustments:

1、 PRF

Hold faster or slower blood flow through rate adjustment. PRF toggle key is for adjusting the value of PRF and the maximum PRF value depends on the probe and the position of sampling gate.

2、 Sampling line

In PW mode, move the trackball left and right to adjust the position of sampling line

3、 Sampling gate (Sampling volume)

Move the sample volume on the B-mode's Doppler area. The gate is positioned over a specific position within the vessel.

- To move Doppler line position, move the Trackball left or right until positioned over the vessel.
- To move sample volume gate position, move the Trackball up or down until positioned inside the vessel.
- To size sample volume gate, press [Enter] key to toggle Trackball function from sample volume gate positioning to sizing, then move the Trackball to change sample volume gate size.

4、 Gain

Rotate PW knob and adjust gain of PWD mode when PW light is on. Gain will be increased if rotate clockwise; Gain will be decreased if rotate anti-clockwise. Adjustable range is from 0 to 255.

4.3.24. Continuous Wave Doppler Imaging Optimization(CW)

Soft menu adjustment:

MENU	Menu Function	Menu Item Adjustable Rang
2D Map	Adjust the color of spectrum	1~10, adjust range: 1
CWDFET	Adjust the CWDFET of the spectrum	0~15, adjust range:1
CWDEnhance	Adjust the enhance of the CW spectrum	0~7,adjust range:1
Dynamic	Adjust the dynamic range of the CW spectrum	45~80,adjust range:5

Shortcut adjustment:

Shortcut (action)	Menu	Menu Function	Menu Item Adjustable Rang
SK1 (press)	Invert	Spectrum invert	Turn on or Turn off the invert function
SK1 (rotate)	Color Map	Adjust the color map of spectrum	Adjust options:user,type1~9
SK2 (press)	Wall Filter	Change of wall filtering	0~3 adjustable
SK2 (rotate)	Audio	Adjust volume of Doppler	0~100% adjustable.
SK3 (rotate)	PRF	Adjust PRF value	The maximum PRF value depends on the probe and the position of sampling gate.
SK4 (rotate)	Speed	Adjust scanning speed	1, 2, 3 adjustable. adjust clockwise, the numerical increase and adjust anticlockwise, the numerical decrease. Adjust range: 1

SK5 (rotate)	Baseline	Adjust the baseline position	0~6 adjustable.
--------------	----------	------------------------------	-----------------

Other adjustments:

1、 PRF

Hold faster or slower blood flow through rate adjustment. PRF toggle key is for adjusting the value of PRF and the maximum PRF value depends on the probe and the position of sampling gate.

2、 Sampling line

In PW mode, move the trackball left and right to adjust the position of sampling line

3、 Gain

Rotate PW knob and adjust gain of CWD mode when CW light is on. Gain will be increased if rotate clockwise; Gain will be decreased if rotate anti-clockwise. Adjustable range is from 0 to 255.

4.3.25. ECG (option)

The ECG module is an device that provides the 3 lead ECG signal acquisition for cardiac application. It is not intent for the ECG diagnostic purpose as in the 12-lead module. In the cardiac application, the ECG trace is displayed on the bottom of the screen. For echo-stress, the R-wave triggering is used to gate or synchronize the image acquisition. The ECG has 3 leads: **LL**(left leg, RED), **LA**(left arm, BLACK), **RA**(right arm, WHITE). LA is for reference, which usually provides a bias voltage from the ECG module, and the LL, LA are the two signals from the body and going to the differential input of the ECG isolation amplifier.

The ECG control is in the soft-menu available for the cardiac probe, it allows the user to set up the following control:

ECG ON/OFF: turn on/off the ECG trace

ECG Invert: turn on/off the ECG invert

ECG GAIN: increase or decrease the ECG gain

ECG POS: set the ECG trace position

ECG Velocity: set the ECG Velocity

4.3.26. Biopsy and Super needle (option)

1、 How to enter into Biopsy

Press the MENU in B mode and choose biopsy to turn on to display biopsy line and verify the biopsy line before biopsy. Press the Biopsy again to turn off the biopsy.

2、 How to adjust the biopsy

Press Enter to edit the biopsy line and move the trackball to change the biopsy line position. Press Enter again to fix the biopsy line.

3、 Super needle

After turn on Super Needle and needle angle function will be active and user can adjust the needle angle to optimize the image for biopsy (The angle is 5 degree per step.).

4.4. After Capturing the Image

4.4.1. Adding Comments

Comment means input the words or symbols on images for making explanation. Add comments can through keyboard input directly or using the default comments.

Input characters

Operation:

1. Press [COMMENT] key will display “ | ”on the screen, and then system will go into the comment process.
2. Move the cursor to the position where need to comments.
3. Input characters at cursor position by keyboard then press [ENTER] key to confirm.
4. Press COMMENT-key or EXIT- key to exit. The COMMENT-key will light off and the comments process is finished.

Edit custom comments bar:

Operation:

- 1、 Press [COMMENT] key to enter into comments status.
- 2、 Press SK4 key to select the comment bar.
- 3、 Edit comments for text1~text6.
- 4、 Press OK key to save changes or press [CANCEL] key to cancel revises.

Note: The user can edit 6 customized comments.

Add comment note on images

Operation:

- 1、 Press [COMMENT] key to enter into the comment status.
- 2、 Rotate SK1 key to select the comment note.
- 3、 Press SK1 shortcut key to append the selected comment bar.
- 4、 Press [COMMENT] key, the [COMMENT] key will light off and the comment process is finished.

Adjust font size of comments

Operation:

- 1 Rotate SK2 in comment status.
- 2 Move the cursor to the comment note. Press [ENTER] key and choose the font size of comment note. Press [ENTER] key again to confirm revision.

Adjust position of comments

Operation:


- 1 In comments status, moves the cursor to comments note which need to be changed then press [ENTER] key.

4.4.2. Adding Body Mark

Operation:

- 1、 Press BODY- key to enter into the body status.
 - 2、 Press SK.1~ SK.5 shortcut key to add body image accordingly. Rotate SK.5 shortcut key can realize page turning.
 - 3、 Move trackball after adding the body mark image and then adjust the position of probe. Rotate [ANGLE] key can adjust the probe direction. Press Enter- key to confirm when adjustment was finished.
 - 4、 Press UPDATE-key and move the Trackball to change the position of the body mark
 - 5、 If you want to exit from the body mark function, press BODY-key or EXIT-key
- Delete comments, body marks and measure tracks.
- Press DEL- key to delete all comments、 arrows、 body marks and measure tracks in frozen status.


4.4.3. Saving Still Images

Save single image: Press  key to save the single image in the real time and frozen statue. The saved thumbnail images will be displayed under the image area for users to choose playback or post-processing quickly.

Re-call single image: Press arrow key in re-call status, then the mouse will appear on the screen. Move the mouse to thumbnail images then click.

4.4.4. Saving Clips


Save forward cine files: Press  key in the real time to save forward cine files.

Save backward cine files: Press  key in frozen statue to save backward cine files.

The saved cine images will be displayed under the image area for users to choose playback or post-processing

quickly.

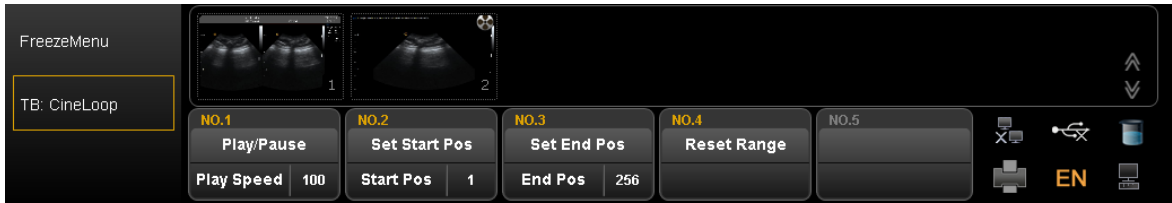Playback cine files: Press arrow key in recall status, then the mouse will appear on the screen. Move the mouse to the cine thumbnail image then click.




Shortcut / Action	SK1	SK2	SK3	SK4
Press	Play or Pause cine playback	Set the initial point of playback	Set the final point of playback	Reset the range of cine playback, the default range of playback is 1-maximum frame of current cine
Rotate	Adjust the playback speed of cine	The frame of start position	The frame of final position	

Manual playback cine: Press arrow key and back to recall status then manual playback through trackball.

Note: When you save the still images or cine images, if you do not establish a new patient, the system will establish a new ID according to the current date of the system automatically and save the data and operation under the folder of this ID.

Quick transmission of image:

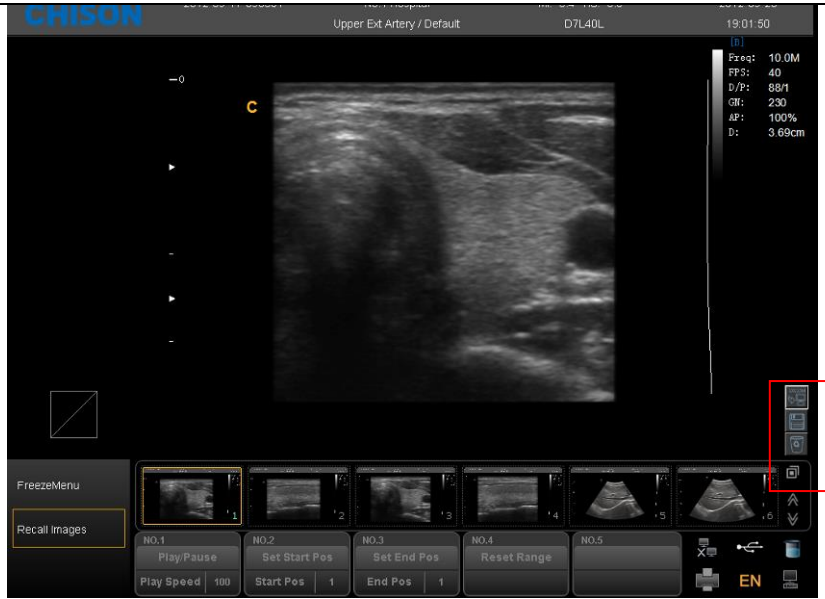Image transmission: Press arrow key. Choose the image which you want to transfer or delete, and press Update key. Three icons will display in the thumbnail area.



Transfer files to DICOM.



Transfer files to media.

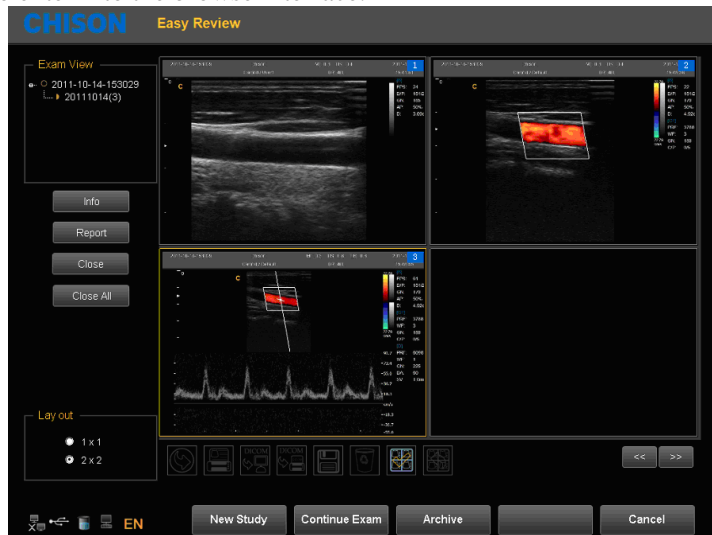


Delete the files.

4.4.5. Browse Images



Press  key to enter into the browse interface.



Info: press this key can check the information of current patient.

Report: press this key can see the report of current patient

Close: click this key and choose to close information of patient.

Close all: close all current patients.

Continue/Edit Exam: continue to check the current patient and back to B mode if press this key. If current exam is 24 hours later than last exam, the system will not allow you to continue the exam. The function will be changed to “Edit”

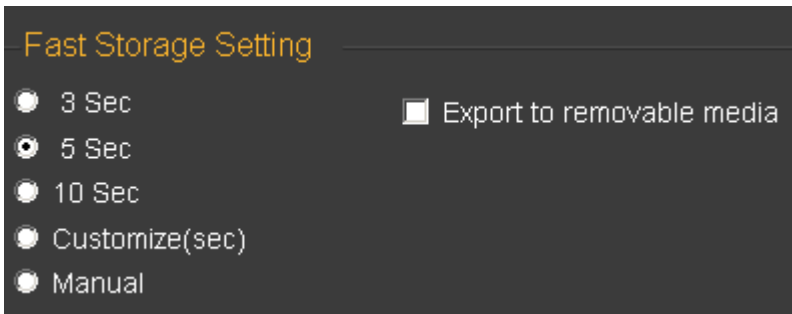
New Study: create a new patient, then the system will automatically go to this new page.

Archive: Pop up Files Management interface. Multiple patients can be compared and browsed

Cancel: Exit.

4.4.6 Fast Storage


Setting: Enter into System Settings . Press General Menu and choose Normal Sub Menu to do fast storage setting.



After confirming time, back to examination.

For example, choose 5 seconds in the fast storage setting. The system begins to save cine in the 5 seconds after

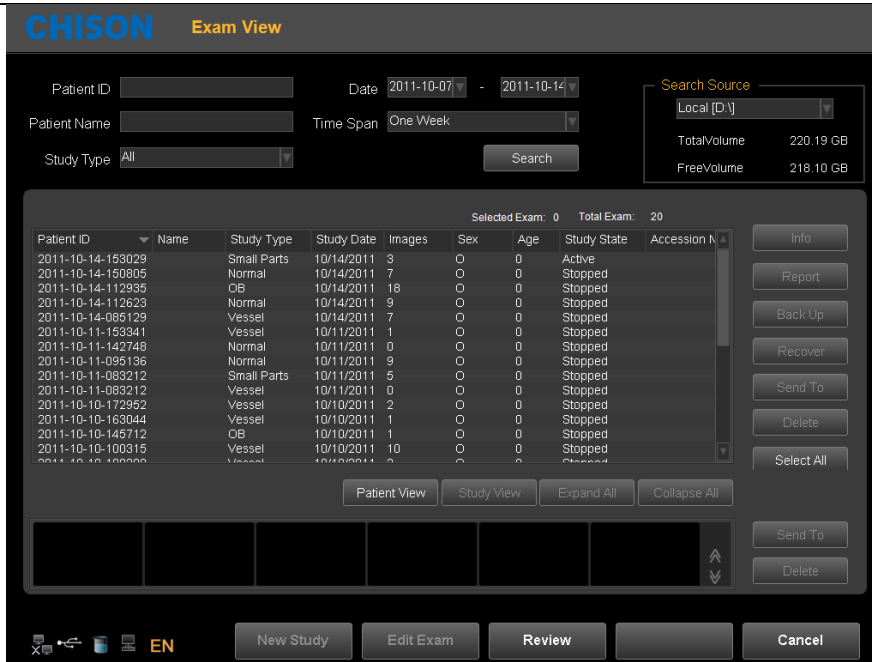
press  .

If you choose Manual in the fast storage setting, you should press  for two times. First time is to start saving and the second time is to finish .

If you choose Export to removable media, it will save the images to the removable media.

4.4.7. Files Management

In this interface you can manage patient's files. Press  key to enter into the browse interface. Then click “Archive” to enter into file management.



Search: Can search the patient files after enter a search term. Search will be more accurate when input more conditions.

Information: View the selected patient information.

Backup: Click this item will store the database of this patient to the portable storage medium

Recover: Click this item can recover the images or video from USB to the system

Send to: Select the content and click, can send files to other medium.

Delete: can delete patient's files, images etc.

Select all: Select all the patients.

Patients view: Under this view, one patient with multiple inspection data will be listed in column and also you can check each exam file in details.

Study view: This view can list exam types one by one. Different exam type of one patient will not be listed in one column.

New study: Create a new patient and recall into the patient information interface.



Continue/Edit Exam: Continue to check the current patient and back to B mode if press this key. If current exam is 24 hours later than last exam, the system will not allow you to continue the exam. The function will be changed to "Edit".

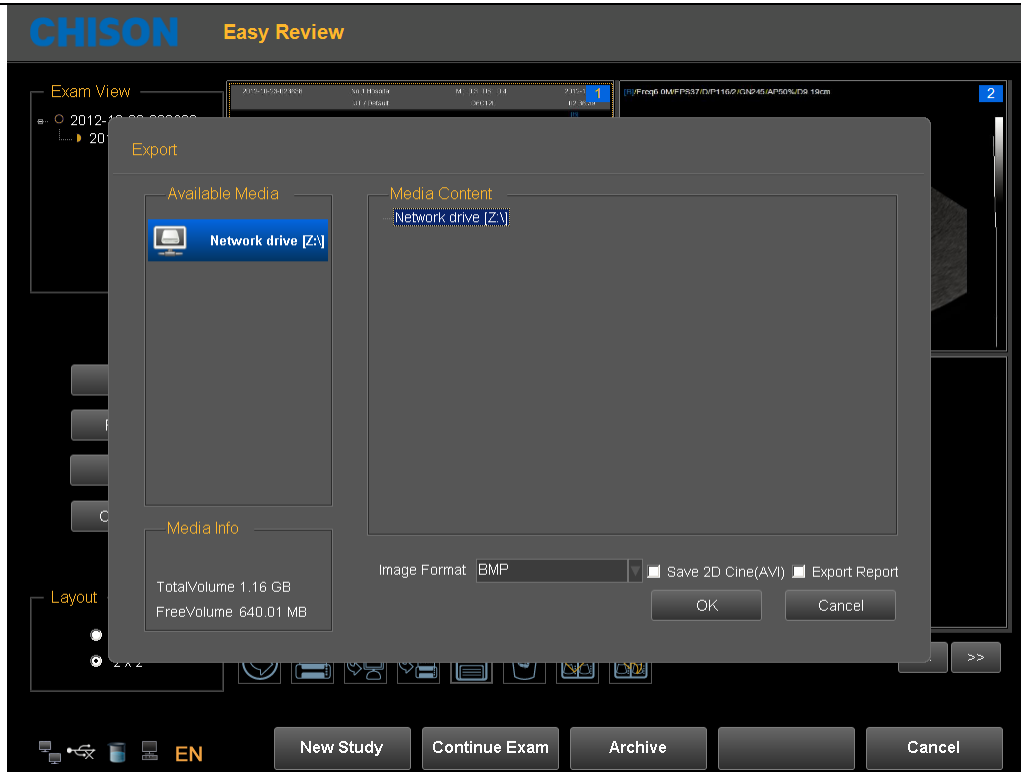
Review: Select patient and click it to enter into images browse.

Cancel: Click "Cancel" to exit.

4.4.8. Network Storage Function

Before you use this function, please refer to Section 7.5 to set network storage parameters.

Click  button to enter into the Easy Review interface, select the picture you want to send, click  to enter into the export interface, select the network drive and click "OK" to send.



After transmit into the target computer, open the shared folder, you can see the pictures you sent.

4.4.9. DICOM Function(Optional)

The system support DICOM storage, DICOM print, MPPS, WORKLIST, DICOM SR storage.

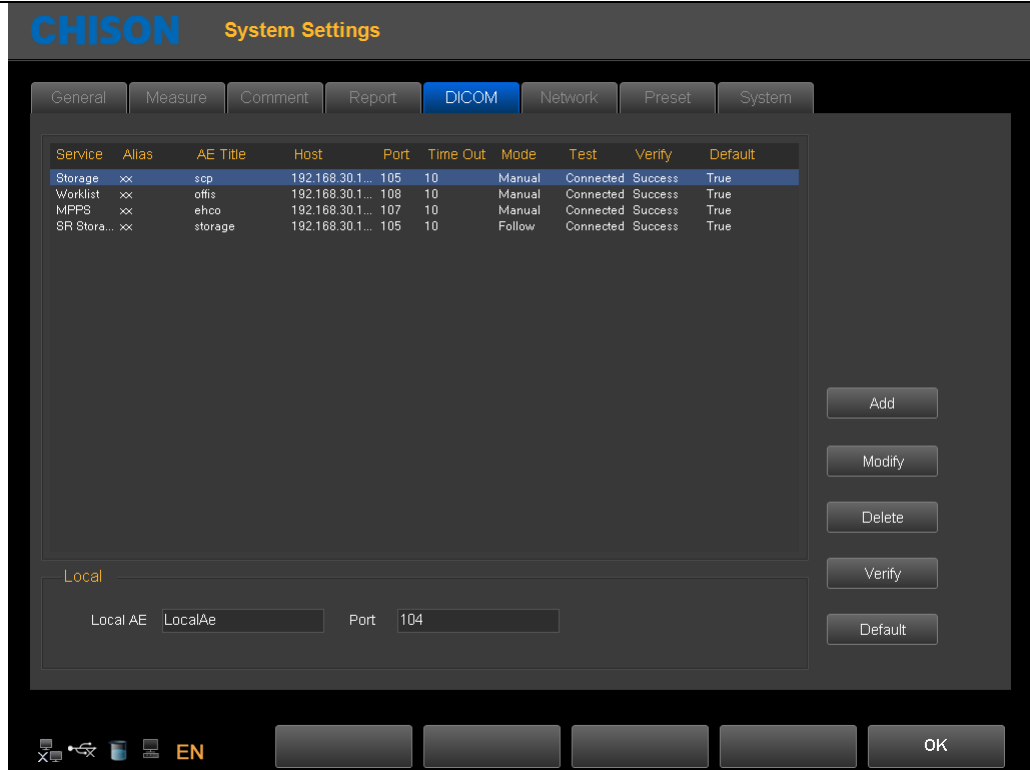
1. Connect the machine into LAN before setting the DICOM
2. Enter into DICOM Setting interface.

Press Utility key to enter into system setting interface. Press DICOM page to enter into DICOM interface.

Note:

If the DICOM function is not open, the system setting interface will not display DICOM page.

Please make sure the DICOM function is open before you use DICOM function.



Add: Press this icon to add a DICOM service link

Modify: Select the DICOM link which need to be modified. Press this icon to modify the parameters inside.

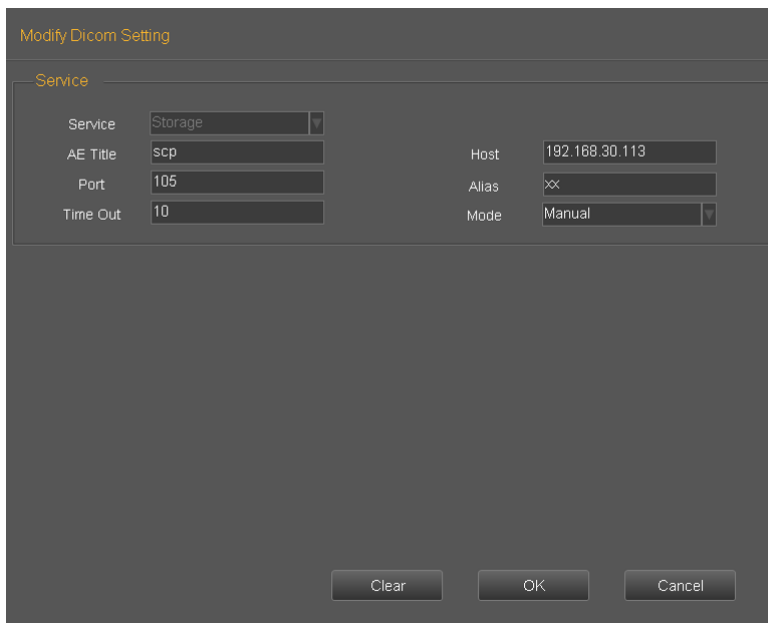
Delete: Select the DICOM link which needs to be deleted. Press this icon to delete the DICOM link.

Verify: Select the DICOM link which need to be verified. Press this icon to verify the current DICOM link status.

Default: If there are several service links of same type, you can select one of them as the default one.

3. DICOM storage

- DICOM storage parameters setting



Service: Click the drop list to select storage function.

AE title: Name the server at will

Host: Set the IP of the service host

Port: Make the port of the work station the same as the port of the server.

Alias: The name of current link

Time out: time delay

Mode: batch, follow, manual selectable

Batch: all the saved images in current exam will be sent to the server in one time

Follow: When one image is saved, the system will send this image to the server.

Manual: The user has to select the image by manual.

After all the settings are completed, please click OK to save current setting. Click clear to clear all the setting.



Clicking cancel will not save the current setting.

- DICOM storage

Batch mode: In batch mode, when the patient finishes his exam, press End key to end the current exam. All the saved images will be saved as DCM format and sent to the server automatically.

Follow: In follow mode, during the exam, press  key to save the image. The current image will be saved as DCM format and sent to the server.

Manual: In manual mode, the user has to select the image by manual and do DICOM transfer. Press

 key to enter into review interface. Select the images need to be sent, press  key to send.

4. DICOM print

- DICOM print parameters setting.

Add Dicom Setting

Service

Service: Print
AE Title: 192.168.30.222
Host: 192.168.30.60
Port: 109
Time Out: 1000
Alias: xx
Mode: Manual

Print Setting

Color: RGB
Format: 1X1
Film Orientation: Portrait
Magnification: Replicate
Border Density: BLACK
Empty Image Density: BLACK
Max Density:
Min Density:
Medium Type: Paper
Film Size: 81X101IN
Film Destination: Magazine
Smoothing Type:
Priority: High
Copies: 1
Config Info:

Clear OK Cancel

Service: Click the drop list to select print function.

AE title: Name the server at will

Host: Set the IP of the server

Port: Make the port of the work station the same as the port of the server.

Alias: The name of current link

Time out: time delay

Mode: batch, follow, manual selectable

Batch: all the saved images in current exam will be sent to the server and printed automatically in one time

Follow: When one image is saved, the system will send this image to the server and print it.

Manual: The user has to select the image by manual and print it.

After all the settings are completed, please click OK to save current setting. Click clear to clear all the setting.

Clicking cancel will not save the current setting.

- **DICOM print**

Batch mode: In batch mode, when the patient finishes his exam, press End key to end the current exam. All the saved images will be saved as DCM format and sent to the server automatically to be printed.

Follow: In follow mode, during the exam, press save key to save the image. The current image will be saved as DCM format and sent to the server to be printed.

Manual: In manual mode, the user has to select the image by manual and do DICOM transfer. Press browse

key to enter into review interface. Select the images need to be sent, click  icon to print.

5. DICOM Worklist

- DICOM Worklist parameters setting.

The screenshot shows a 'Modify Dicom Setting' dialog box with the following fields and options:

- Service:** Service (Worklist), AE Title (offis), Port (108), Time Out (10), Host (192.168.30.113), Alias (xx)
- Update Method:** Only on user request, On Startup and Every 5 minute
- Scheduled Station AE Title:** Any, This System, Another
- Start Date:** Today

Buttons: Clear, OK, Cancel

- Service: Click the drop list to select worklist function.

AE title: Name the server at will

Host: Set the IP of the server

Port: Make the port of the work station the same as the port of the server.

Alias: The name of current link

Time out: time delay

For “Update Method” parameter group, the worklist update can only started by user request for “Only on user request”, or user can set N(1~60) minutes to update worklist automatically for “On Startup and Every X minutes”.

For “Scheduled Station AE Title” parameter group, the schedule station AE title, which is one of a matching key, will be set to blank for “Any”, “LocalAE” for “This System” or user customized string for “Another”. And only the matched items will be queried for each worklist update.

For “Start Date” parameter group, it is the matching key for “Scheduled Procedure Step Start Date”, and there are five “Start Date” types to select.

- DICOM Worklist

Patient Information


Patient ID: Birthday:
 Last Name: Age: Y M
 First Name: Sex:
 Middle Name:

Search In

Patient ID: Patient Name:
 Accession #: Procedure Id:

Patient ID	Patient Name	Sex	Birthdate	Accession #	Physician	Procedure Id	Start Datetime
20120824	m m m m	Female	5/3/2012 12:00:00 ...			RP-ID1	2013/05/22 13:24:17
20120824	west	Female	5/3/2012 12:00:00 ...			RP-ID1	2013/05/22 13:24:17
CS-20120316	One*Secondary Capt...	Female	1/1/0001 12:00:00 ...			RP-ID1	2013/05/22 13:24:17
SC-3	Three*Secondary Ca...	Female	1/1/0001 12:00:00 ...				2013/05/22 13:24:17
SC-2	Two*Secondary Capt...	Female	1/1/0001 12:00:00 ...				2013/05/22 13:24:17
SC-1	One*Secondary Capt...	Female	1/1/0001 12:00:00 ...				2013/05/22 13:24:17
SC-1	One*Secondary Capt...	Female	1/1/0001 12:00:00 ...			RP-ID1	2013/05/22 13:24:17
SC-1	One*Secondary Capt...	Female	1/1/0001 12:00:00 ...			RP-ID1	2013/05/22 13:24:17
pidP645	One*Secondary Capt...	Male	7/16/1980 12:00:0...	00000187	Chief Radiologist*Fl...	0000018705	2013/05/22 13:24:17

Worklist Query Success.

Press the keyboard button shaped as “  ”, and press “Worklist”

Press “Search” to start a worklist update by the matching key specified in Worklist Configuration, all the matched items will be queried in worklist table. User also can add additional for matching keys, which are “PatientID”, “PatientName”, “Accession#” and “ProcedureId” , for a more precise query.

Select one matched item and press “Apply” for the current exam application. At this moment, a MPPS service will be started which will be introduced in MPPS section.

6. DICOM MPPS

Add Dicom Setting

Service

Service	MPPS	Host	192.168.30.113
AE Title	MPPS	Alias	LOCAL
Port	108		
Time Out	10		

Clear OK Cancel

Service: Click the drop list to select MPPS function.

AE title: Name the server at will


Host: Set the IP of the server

Port: Make the port of the work station the same as the port of the server.

Alias: The name of current link

Time out: time delay

Applying the selected worklist item will result in automated creation of an MPPS instance.

Completion of the MPPS is performed by pressing the keyboard shaped as “”.

7. DICOM SR Storage

Add Dicom Setting

Service

Service	SR Storage	Host	192.168.30.113
AE Title	SR	Alias	LOCAL
Port	104		
Time Out	10		

Clear OK Cancel

Service: Click the drop list to select SR Storage function.

AE title: Name the server at will

Host: Set the IP of the server

Port: Make the port of the work station the same as the port of the server.

Alias: The name of current link

Time out: time delay.

Only OB/GYN and Cardiac report support this server.

After finishing OB/GYN or Cardiac measurements, press report key to create report for current exam. When the patient finishes his exam, press End key to end the current exam, the SR report will be sent to the server automatically.

Chapter 5 General Measurements

Introduction

Measurements and calculations derived from ultrasound images are intended to supplement other clinical procedures available to the physician. The accuracy of measurements is not only determined by system accuracy, but also by the use of proper medical protocols by the operator. When appropriate, make sure to note any protocol associated with a particular measurement or calculation.

CAUTION: Please select the most appropriate ultrasound images, measurement tools and measurement methods for measurements according to your diagnosis needs. The final measurement results must be determined and verified by a physician. Measurement accuracies are affected by many non-technical factors, for example operator's experience, patient's status. Please do not only use the ultrasound measurement results as the sole basis for diagnosis, please always use other clinical information to do integrated diagnostics.

Overview

This section provides information about taking measurements and describing calculations available in each mode.

It includes the following topics:

- List of generic measurements
- Mode Measurements: Step-by-step instructions for taking specific measurements, organized by mode
- Measurement results display and erasing

5.1 Key for Measurement

◆ Trackball

Trackball is used to move the cursor, main functions are as follows:

Before starting a measurement, use the trackball to choose the menu options;

After starting a measurement, move the trackball to move the cursor, during the measurement, the cursor should not be moved out image area;

During the Ellipse method measurement, use trackball to change the length of short axis.

◆ 【ENTER】

During the measurement, the functions of 【ENTER】 key are as follows:

When cursor is on the menu, press the key to choose the options and start the measurement.

During the measurement, press the key to anchor the start point and end point.

◆ 【DEL】

Main functions are as follows:

In frozen status, press the DEL-key, delete all the measurement results, comments and traces.

◆ **【UPDATE】**

During the measurement, UPDATE-KEY is used to switch the start point and end point, long axis and short axis when the measurement is not finished.

During the distance measurement, press the **【ENTER】** to fix the start point, when the end point is not fixed, press the UPDATE-KEY to switch the start point and end point.

During the Ellipse measurement, when fix the long axis, but the short axis is not fixed, press the UPDATE-KEY to switch the long axis and short one.

5.2 Fast measurement

The system use Dist,Trace,SK1~SK3 to make fast measurement. In different modes, SK1~SK3 is corresponded to different measurement items. Users can rotate SK1~SK3 to choose the measurement items during the measurement.

5.2.1 Enter fast measurement

Press Dist or Trace to enter fast measurement

Dist: distance measurement

Trace: Area measurement

5.2.2 Exit fast measurement

During measurement, press the Dist key for second time, or press Trace-key or Exit-key.

5.2.3 Fast measurement in B mode

SK1-distance: distance, Distance ratio, angle

SK2-area: trace method, Ellipse method, area ratio

SK3-volume: 1 distance method, 1 Ellipse method, 1 Distance1 Ellipse method, 2 Distance method, 2 Ellipse method, 3 Distance method

SK4-Font Size: Rotate to adjust the font size, Press to reset.

SK5-Result Position: Rotate to move result position, Press to reset

Distance

- 1) In B Cine mode, press the DIST-key to bring up the first white plus sign (“+”) cursor.
- 2) Use the Trackball and the ENTER-key to anchor the start point of the desired distance to be measured.
- 3) Move the Trackball and a second white plus sign cursor will automatically appear. Move the second cursor to the specified position. With the movement of cursor the system will update the measurement distance in real time in Result Window on the right side of the screen.
- 4) Press the ENTER-key to fix the second cursor, the first measurement result will appear on the screen.
- 5) When two points have been defined, a white plus sign cursor of next distance pair will pop out. Repeat the

same steps to create other distance pairs

6) Press the DIST-key or EXIT-key to exit.

Distance ratio

In B mode, press the DIST-key to bring up the first white plus sign (“+”) cursor.

Rotate SK1, choose %stD.

Refer to the distance measurement method, measure the distance of second line.

After finishing the measurement, the distance of the two lines will appear automatically on the screen and calculate the ratio of the two lines automatically.

Area—trace

1) In B mode, press the TRACE-key to bring up the trace start cursor plus sign (“+”).

2) Use the trackball to move the cursor, press ENTER-key to set the start point.

3) Use the Trackball to trace the perimeter of the object to be measured, and press the Enter-key to set the end point. The end point and the starting point of the trace will be automatically connected with a straight line. The area and the Circumference of the shape will be calculated. With the movement of the cursor, the result will appear on the right side of the screen, and will change along with the moving of the trace.

4) To get the multiple data, repeat the same steps to create other distance pairs.

5) Press the Trace-key or EXIT-key to exit.

Area—ellipse

1) In B mode, press the TRACE-key to bring up the trace start cursor plus sign (“+”).

2) Rotate SK2, choose ellipse.

3) Move the Trackball and press ENTER-key to set start point

4) The second point marker is displayed by a line connecting the first and second points. Use the Trackball to lengthen the line in order to change the diameter and press the ENTER-key to fix the length.

5) Use the Trackball to change the length of another axis. Press the ENTER-key to fix the length. The total area and circumference will be displayed in the result window.

6) To get the multiple data, repeat the same steps to create other distance pairs.

7) Press the Trace-key or EXIT-key to exit.

Area ratio

1) In B mode, press the TRACE-key to bring up the trace start cursor plus sign (“+”).

2) Rotate SK2, choose %stA.

3) Refer to the area-ellipse method to measure two ellipses.

4) After finishing the measurement, the area of the two lines will appear on the screen, and the area ratio will be calculated automatically.

Volume—1 straight line

- 1) In B mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.

Volume—1 ellipse

- 1) In B mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.
- 3) Rotate SK3-key, choose 1 ellipse.

Volume—1 straight line 1 ellipse

- 1) In B mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.
- 3) Rotate SK3-key, choose 1 straight line 1 ellipse.

Volume—2 straight line

- 1) In B mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.
- 3) Rotate SK3-key, choose 2 straight line.

Volume—2 ellipse

- 1) In B Cine mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.
- 3) Rotate SK3-key, choose 2 ellipse.

Volume—3 straight line

- 1) In B Cine mode, press the distance-key or trace-key to enter the measurement mode.
- 2) Press SK3-key to enter volume measurement, the default measurement method is 1 straight line.
- 3) Rotate SK3-key, choose 3 straight line.

5.2.4 Fast measurement in PW mode

SK1-distance: distance, peak, HR, Angle

SK2-area: automatic envelope, manual envelope, trace, ellipse, ratio

SK3-volume: 1 straight line., 1 ellipse, 1 straight line 1 ellipse, 2 straight line, 2 ellipse, 3 straight line

SK4-Font Size: Rotate to adjust the font size, and press to reset.

SK5-Result Position: Rotate to move result position, and press to reset

Peak

- 1) In PW Cine mode, press the Dist-key to bring up the sample marker.
- 2) Move the marker to the measurement start point with the Trackball, press Enter key, velocity and pressure of the current point will appear on the screen automatically.
- 3) Go on to measure Vd, after getting the result, the system will calculate S/D, RI, heart rate automatically.

Automatic envelope measurement

- 1) In PW mode, press the Trace-key to enter automatic envelope measurement. The system will finish the envelope of spectrum automatically. The cursor“+”will appear on the screen
- 2) Move the trackball to choose a start point of one cycle, press the Enter-key to confirm
- 3) The second cursor“+”will appear on the screen automatically, move the trackball to the end point of current cycle, and press the ENTER-key to set.
- 4) The measurement result and the other calculated parameters will appear on the screen automatically.
- 5) If the spectrum automatic envelop is not accurate, rotate SK4 and reselect cycle to correct.

Manual envelope

- 1) In PW Cine mode, press the Trace-key to enter automatic envelope measurement.
- 2) Rotate SK2, choose “Manual envelope”.
- 3) Move the trackball to choose a start point of one cycle, press the Enter-key to confirm.
- 4) Move the trackball to trace the spectrum, press the ENTER-key to finish the envelope..
- 5) The measurement results and the other calculated results will appear on the screen automatically.
- 6) If the spectrum manual envelop is not accurate, rotate SK5 and restart to manual envelop.

5.2.5 Fast measurement in M mode

SK1-distance: M Distance, M Time, M Speed, Heart Rate, Distance Ratio

SK2-area: trace method, ellipse method, area ratio

SK3-Volume: 1straight line, 1ellipse, 1 straight line 1 ellipse, 2 straight line, 2 ellipse, 3 straight line

SK4-Font Size: Rotate to adjust the font size, Press to reset.

SK5-Result Position: Rotate to move result position, Press to reset

M Distance

- 1) In M mode, press the DIST-key to enter M distance measurement. The cursor“+”will appear on the screen.
- 2) Move the trackball to move the cursor, press ENTER-key to fix it.
- 3) One dotted line and second cursor will appear on the screen..
- 4) Move the cursor by moving the trackball to the end point in the dotted line vertically, press the Enter-key to confirm.

5)The measurement result will appear on the screen automatically.

M Time

- 1) In M Cine mode, press the DIST-key to enter M distance measurement. The cursor“+”will appear on the screen.
- 2) Rotate SK1, choose M time.
- 3) Move the trackball to move the cursor, press Enter-key to fix it.
- 4) One dotted line and second cursor will appear.
- 5) Move the trackball to the end point in the dotted line horizontally, press the Enter-key to fix the cursor.
- 6) The measurement result will appear on the screen automatically.

5.3 Measurement and Calculation

There are Corresponding measurement menu in different modes. Press Calc-key to recall the measurement menu. Press Change-key to do exchange during the measurements menus of different modes. Press Exit key to close the measurement menu.

Move the trackball to select the measurement item on the measurement menu and do the measurement in the image.

After finishing the measurement, press Report-key to generate the report template to view the measurement results or print the report.

5.3.1 Measurement in B mode

Press Calc-Key to enter into measurement package menu. Press Change-Key to change measurement package .

5.3.1.1. General Measurement in B mode



1. Distance

- 1) Move the trackball to move the cursor to the “distance” item. Press Enter-key to select it and a mark “+” will appear on the screen
- 2) Move the trackball to anchor the first mark and press Enter-key to set it.
- 3) The second mark will appear on the screen. Move the trackball to place the mark to the aimed place. The value of the measurement will be displayed on the right side of the screen and will be changed along with the trace of the measurement.
- 4) Press Enter-key to fix the second mark and the measurement results will be fixed and displayed on the screen.
- 5) To get the multiple data, repeat the same steps to create other distance pairs.
- 6) Press Calc-key or Exit-key to exit..

2. Area(Ellipse)

- 1) Rotate the trackball to move the cursor to the “Area (ellipse)” item. Press Enter-key to select it. At this time, a mark “+” will appear on the screen.
- 2) Rotate the trackball to anchor the first mark and press Enter-key to fix it.
- 3) The second mark will appear on the screen. Rotate the trackball to change the length of the long axis of the ellipse. Press Enter-key to confirm the length.
- 4) Rotate the trackball to change the length of another axis of the ellipse and press Enter-key to fix it. The value of the area and perimeter will be displayed on the right side of the screen
- 5) To get the multiple data, repeat the same steps to create other distance pairs.
- 6) Press Calc-key or Exit key to exit.

3. Area(Trace)

- 1) Rotate the trackball to move the cursor to the “Area (trace)” item. Press Enter-key to select it. At this time, a mark “+” will appear on the screen
- 2) Move the trackball to move the mark to the start point of the measurement. Press Enter-key to confirm..
- 3) Rotate the trackball to trace the edge of the aimed object and press Enter-key to fix the end point. The second mark will appear on the screen. Rotate the trackball to place the mark to the aimed place. The value of the measurement will be displayed on the right side of the screen and will be changed along with the moving of the measurement
- 4) To get the multiple data, repeat the same steps to create other distance pairs.
- 5) Press Calc-key or Exit key to exit.

4. Volume(1distance)

Calculate the volume of the object by one line.

$$\text{Volume}=\text{distance} \times \text{distance} \times \text{distance} \times 3.1415926/6$$

5. Volume (1ellipse)

Through 1 ellipse measurement, calculate the volume of the object.

$$\text{Volume}=\text{axis A} \times \text{Axis B} \times \text{axis B} \times 3.1415926/6$$

6. Volume(2distance)

Through 2 distance measurement, calculate the volume object.

Volume =distance1×distance2×distance2×3.1415926/6 (distance1 > distance2)

Volume =distance2×distance1×distance1×3.1415926/6 (distance2 > distance1)

7. Volume(3 distance)

Through 3 distance measurement, calculate the volume of object

Volume=distance1×distance2×distance3×3.1415926/6

8. Volume(1distance1ellipse)

Through 1distance measurement and 1 ellipse measurement, calculate the volume of object.

Volume=distance ×axis A × axis B×3.1415926/6

9. Volume(2 ellipse)

Through 2 ellipse measurement, calculate the volume of the object.

Volume=d1×d2×d4×3.1415926/6

d1 and d2 are axis of ellipse 1;d3 and d4 are axis of ellipse 2: | d2-d3 | ≤ | d1-d4 | ,d2 > d3

10. Ratio

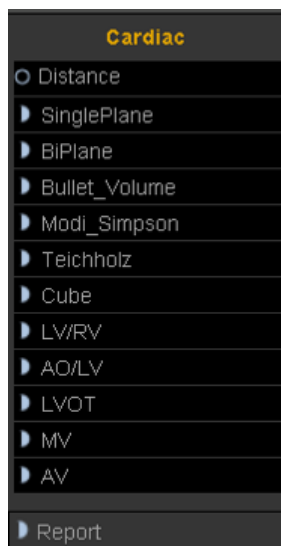
Through 2 distance measurement, calculate the ratio of the 2 distance

Ratio=distance1/distance2

11. Angle

- 1) Rotate the trackball to move the cursor to the “Angle” item. Press Enter-key to select it and d1 line will appear on the screen
- 2) Rotate the trackball to move d1 and rotate the Angle button to adjust the direction of the line ,then press Enter-key to fix it.
- 3) Fix other 2 lines of d2 and d3 as step2)
- 4) After finishing the measurement, the result of the angles named α and β will be displayed on the left side of the screen. α is the angle between line d1 and line d2. β is the angle between line D1 and D3.
- 5) Press Calc-key or Exit key to exit.

5.3.1.2. Cardiology Measurement in B mode



1. Single Plane

This method calculates the left ventricular volumes by using the 2D-mode long-axis image. When the left ventricle is traced and its long axis is specified on a 2D-mode cross-sectional image, this method allows the system to automatically divide the long axis into 20 segments to calculate the volume of the left ventricle as the 2D-mode cross-sectional image is rotated.

Calculation formula for volume

$$EDV = \pi \frac{LVLd}{20} \times \sum_{i=1}^{20} r_i^2$$

$$ESV = \pi \frac{LVLs}{20} \times \sum_{i=1}^{20} r_i^2$$

ri: Radius of the i-th circle

LVLd: Left ventricular long-axis length at end diastole

LVLs: Left ventricular long-axis length at end diastole

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. method
EDV (A4C)	End-diastolic volume (A4C)[ml]	Measurement (trace)
ESV (A4C)	End-systolic volume (A4C)[ml]	Measurement (trace)

<< Items to be calculated >>

Calc. item name	Description [Unit]	Calc. formula
SV	Stroke Volume[mL]	SV=EDV – ESV
EF	ejection fraction	EF=SV/EDV

2. BiPlane

This method allows the volume of the left ventricle to be calculated by repeating “Simpson SP method” on two 2D-mode cross-sectional images (two-chamber and four-chamber cross-sectional images).

Calculation is also possible for only two-chamber cross-sectional images or only four-chamber cross-sectional images.

Calculation formula for volume

$$(1)EDV = \pi \frac{LVLd}{20} \times \sum_{i=1}^{20} (r_{2i} \times r_{4i})$$

$$(2)ESV=\pi\frac{LVLs}{20}\times\sum_{i=1}^{20}(r_{2i}\times r_{4i})$$

r_{2i}:Radius of i-th ellipse(2CH)

r_{4i}:Radius of i-th ellipse(4CH)

LVLd: Length of the left-ventricular long axis at end diastole for 2CH or 4CH, whichever is longer.

LVLs: Length of the left-ventricular long axis at end systole for 2CH or 4CH, whichever is longer.

$$(3)EDV=\pi\frac{LVLd}{20}\times\sum_{i=1}^{20}r_{2i}^2$$

$$(4)ESV=\pi\frac{LVLs}{20}\times\sum_{i=1}^{20}r_{2i}^2$$

r_i: Radius of the i-th circle(2CH)

LVLd: Left ventricular long-axis length at end diastole

LVLs: Left ventricular long-axis length at end diastole

$$(5)EDV=\pi\frac{LVLd}{20}\times\sum_{i=1}^{20}r_{4i}^2$$

$$(6)ESV=\pi\frac{LVLs}{20}\times\sum_{i=1}^{20}r_{4i}^2$$

r_i: Radius of the i-th circle(4CH)

LVLd: Left ventricular long-axis length at end diastole

LVLs: Left ventricular long-axis length at end diastole

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. method
EDV (A2C)	End-diastolic volume (A2C) [ml]	Measurement (trace)
ESV (A2C)	End-systolic volume (A2C) [ml]	Measurement (trace)
EDV (A4C)	End-diastolic volume (A4C) [ml]	Measurement (trace)
ESV (A4C)	End-systolic volume (A4C) [ml]	Measurement (trace)

<< Items to be calculated >>

Calc. item name	Description [Unit]	Calc. formula
SV (SimpBP)	Stroke Volume[mL]	SV=EDV – ESV
EF (SimpBP)	ejection fraction	EF=SV/EDV

3. Bullet volume

This method calculate the left ventricular volume using the 2D-mode long-axis image and the short-axis image at the level of the mitral valve.

Calculation formula for volume

$$EDV=\frac{5}{6}\times LVLd\times LVAMd$$

$$ESV=\frac{5}{6}\times LVLs\times LVAMs$$

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. method
LVAMd	Left ventricular short-axis area at the level of the mitral valve at end diastole.	Refer to “Area-trace measurement”

LVLd	Left ventricular long-axis length at end diastole.	Refer to "Distance Measurement (Distance)".
LVAMs	Left ventricular short-axis area at the level of the mitral valve at end systole.	Refer to "Area-trace measurement"
LVLs	Left ventricular long-axis length at end systole.	Refer to "Distance Measurement (Distance)".
HR	Heart rate	Refer to "Distance Measurement (Distance)".

<<Items to be calculate>>

Calc. item name	Description [Unit]	Calc. formula
EDV	End-diastolic volume [mL]	$EDV = (5/6.0) * LVLd * LVAMd$
ESV	End-systolic volume [mL]	$ESV = (5/6.0) * LVLs * LVAMs$
SV	Stroke Volume [mL]	$SV = EDV - ESV$
CO	Cardiac output [L/min]	$CO = SV * HR / 1000$
EF	ejection fraction	$EF = SV / EDV$

4. Modi_Simpson

This method calculates the left ventricular using the 2-D mode long-axis image, the short-axis image at the level of the mitral valve, and the short-axis image at the level of the papillary muscle.

Calculation formula for volume:

$$EDV = \frac{LVLd}{9} \times (4 \times LVAMd + 2 \times LVAPd + \sqrt{LVAMd \times LVAPd})$$

$$ESV = \frac{LVLs}{9} \times (4 \times LVAMs + 2 \times LVAPs + \sqrt{LVAMs \times LVAPs})$$

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. method
LVAMd	Left ventricular short-axis area at the level of the mitral valve at end diastole	Refer to "Area/Circumference Measurement (Area)"
LVLd	Left ventricular long-axis length at end diastole	Refer to "Distance Measurement (Distance)"
LVAPd	Left ventricular long-axis area at the level of the papillary muscle at end diastole	Refer to "Area/Circumference Measurement (Area)".
LVAMs	Left ventricular short-axis area at the level of the mitral valve at end systole	Refer to "Area/Circumference Measurement (Area)"
LVLs	Left ventricular long-axis length at end systole	Refer to "Distance Measurement (Distance)"
LVAPs	Left ventricular long-axis area at the level of the papillary muscle at end systole	Refer to "Area/Circumference Measurement (Area)".

HR	Heart rate	Refer to “Heart Rate Measurement”
----	------------	-----------------------------------

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
EDV	End-diastolic left ventricular volume [mL]	Refer to << Calculation formula for volume>>
ESV	End-systolic left ventricular volume [mL]	Refer to << Calculation formula for volume>>
SV	Stroke volume [mL]	SV=EDV – ESV
CO	Cardiac output [L/min]	CO=SV x HR /1000
EF	Ejection fraction [no unit]	EF=SV/ EDV

5. Cube Method

This method calculates the left ventricular volume by approximating the given region to a cube.

Calculation formula for volume

$$EDV = LVIDd^3$$

$$ESV = LVIDs^3$$

<< Items to be measured>>

Meas. item name	Description	Meas. method
Diastole	End-diastolic left ventricular measurements	Refer to “Measurement position”
IVSTd	Interventricular septal thickness at end diastole	Refer to "Distance Measurement (Distance)".
LVIDd	Left ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".
LVPWd	Left ventricular posterior wall thickness at end diastole	Refer to "Distance Measurement (Distance)".
IVSTs	Interventricular septal thickness at end systole	Refer to "Distance Measurement (Distance)".
LVIDs	Left ventricular short-axis diameter at end systole	Refer to "Distance Measurement (Distance)".
LVPWs	Left ventricular posterior wall thickness at end systole	Refer to "Distance Measurement (Distance)".

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
EDV	End-diastolic left ventricular volume [mL]	Refer to << Calculation formula for volume>>
ESV	End-systolic left ventricular volume [mL]	Refer to << Calculation formula for volume>>
SV	Stroke volume [mL]	SV=EDV – ESV
CO	Cardiac output [L/min]	CO=SV x HR /1000
FS	Fractional shortening	FS=(LVIDd-LVIDs)/LVIDd

6. Teichholz Method

Calculation formula for volume

$$EDV = (7 * LVIDd^3) / (2.4 + LVIDd)$$

$$ESV = (7 * LVIDs^3) / (2.4 + LVIDs)$$

The items to be measured, the measurement procedures, and the items to be calculated are identical to those in subsection "CUBE method".

7. LV/RV

<<Item to be measured>>

Meas. item name	Description	Meas. method
RVIDd	Right ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".
IVSTd	Interventricular septal thickness at end diastole	Refer to "Distance Measurement (Distance)".
LVIDd	Light ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".
LVPWd	Left ventricular posterior wall thickness at end diastole	Refer to "Distance Measurement (Distance)".
RVIDs	Right ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".
IVSTs	Interventricular septal thickness at end diastole	Refer to "Distance Measurement (Distance)".
LVIDs	Light ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".
LVPWs	Left ventricular posterior wall thickness at end diastole	Refer to "Distance Measurement (Distance)".

<<Item to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
EDV	End-diastolic left ventricular volume [mL]	EDV= LVIDd* LVIDd* LVIDd
ESV	End-systolic left ventricular volume	ESV= LVIDs* LVIDs* LVIDs

	[mL]	
SV	Stroke volume [mL]	$SV=EDV - ESV$
CO	Cardiac output [L/min]	$CO=SV * HR /1000$
EF	Ejection fraction [no unit]	$EF=SV/EDV$
FS	Fractional shortening	$FS = (LVIDd -LVIDs)/LVIDd$

8. AO/LV

<<Item to be measured>>

Meas. item name	Description	Meas. method
Root of aorta	Root of aorta diameter	Refer to "Distance Measurement (Distance)".
LVIDd	Light ventricular short-axis diameter at end diastole	Refer to "Distance Measurement (Distance)".

<<Item to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
AO/LV	AO/LVIDd	AO/LVIDd

9. LVOT

<<Item to be measured>>

Meas. item name	Description	Meas. method
LA Diam	left ventricular outflow tract diameter	Refer to "Distance Measurement (Distance)"

<<Item to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
Area	left ventricular outflow tract area	Refer to "area"

10. MV

<<Item to be measured>>

Meas. item name	Description	Meas. method
MV Diam	Mitral valve diameter	Refer to "Distance


		Measurement (Distance)"
MVArea	Mitral valve area	Refer to "area Measurement

11. AV

<<Item to be measured>>

Meas. item name	Description	Meas. method
AV Diam	Tricuspid valve diameter	Refer to "Distance Measurement (Distance)"
AVArea	Tricuspid valve area	Refer to "area Measurement

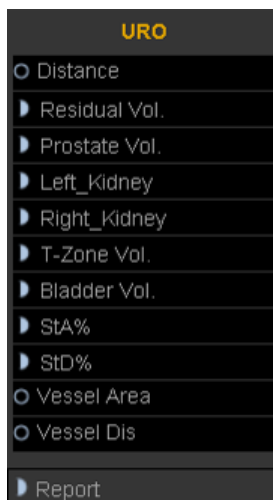
5.3.1.3. Urology Measurement in B mode



Meas. item name	mark	unit	Meas. method
Residual urine	RUV length RUV width RUV height RUV	cm cm cm ml	Refer to "Distance Measurement (Distance)". Refer to "Distance Measurement (Distance)". Refer to "Distance Measurement (Distance)". $RUV = (\pi/6) * RUV \text{ Length} * RUV \text{ Width} * RUV \text{ Height}$
PV	PV Length PV Width	cm cm	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)"

	PV Height PV PPSA PSAD	cm ml ml ng/ ml ²	Refer to "Distance Measurement (Distance)" $PV = (\pi/6) * PV \text{ Length} * PV \text{ Width} * PV \text{ Height}$ $PPSA = 0.12 * PV$ $PSAD = SPSA[ng]/PV[ml]$ Note: SPSA must be input by manual in patient interface when you select the urology exam type
Left_Kidney	Length Width Height VOL	cm cm cm ml	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $V = (\pi/6) * \text{Length} * \text{Width} * \text{Height}$
Right_Kidney	Length Width Height VOL	cm cm cm ml	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $V = (\pi/6) * \text{length} * \text{Width} * \text{Height}$
T-Zone	Length Width Height VOL	cm cm cm ml	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $V = (\pi/6) * \text{length} * \text{Width} * \text{Height}$
Bladder Vol	Length Width Height VOL	cm cm cm ml	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $V = (\pi/6) * \text{length} * \text{Width} * \text{Height}$
StA%	A Out A In StA%	cm ² cm ² %	Refer to "Area Measurement (Area)"
StD%	D Out D In StD%	cm ² cm ² %	Refer to "Area Measurement (Area)"
Vessel Area	Vessel Area	cm ²	Refer to "Area Measurement (Area)"
Vessel Dis	Vessel Dis	cm	Refer to "Distance Measurement (Distance)"

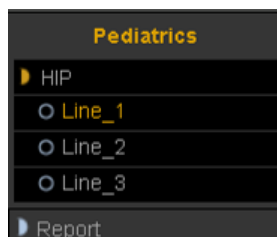5.3.1.4 Pediatric Measurement in B mode




Hip

Hip is used for evaluating the abnormality of the cotyle. In order to make measurement, user has to draw three lines on the image responding to the anatomy. The system will calculate the angles automatically.

Operation Method:

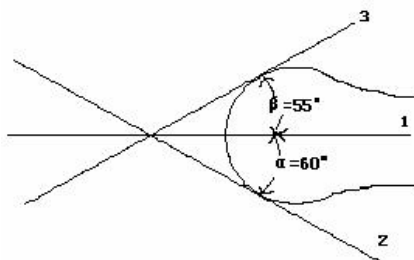- Select HIP from Pediatric submenu on CALC menu. Press Enter key to select it.
- The first line D1 will be displayed on the screen and use the Trackball to move the line.
- The second line D2 will be displayed on the screen, repeat the above procedure to fix the second line and third line D3. The angles between these three lines will be displayed on the screen.



Note: α is the angle between the D1 and D2 (acute angle). β is the angle between D1 and D3 (acute angle)

5.3.1.5 Small Parts Measurement in B mode

Small Parts Measurement is the same as the normal Measurement.

5.3.1.6 Vascular Measurement in B mode



Meas. item name	mark	unit	Meas. Method and calc. formular
Prox CCA (Rt) / (Lt)	A Out A In StA% D Out D In StD% Vessel Area Vessel Dis	cm ² cm ² % cm cm % cm ² cm	Refer to "Area/Circumference Measurement (Area)" Refer to "Area/Circumference Measurement (Area)" StA% = (A Out - A In) / A Out * 100% Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" StD% = (D Out - D In) / D Out * 100% Refer to Area(Ellipse) Refer to "Distance Measurement (Distance)"
Mid CCA (Rt) / (Lt)	The same as above	The same as above	The same as above
Distal CCA (Rt) / (Lt)	The same as above	The same as above	The same as above

Digital Color Doppler Ultrasound System

Prox ICA (Rt) / (Lt)	The same as above	The same as above	The same as above
Mid ICA (Rt) / (Lt)	The same as above	The same as above	The same as above
Distal ICA (Rt) / (Lt)	The same as above	The same as above	The same as above
ECA (Rt) / (Lt)	The same as above	The same as above	The same as above
Vertebral A (Rt) / (Lt)	The same as above	The same as above	The same as above
INT IIL (Rt) / (Lt)	The same as above	The same as above	The same as above
EXT IL (Rt) / (Lt)	The same as above	The same as above	The same as above
ILIAC (Rt) / (Lt)	The same as above	The same as above	The same as above
CFA (Rt) / (Lt)	The same as above	The same as above	The same as above
ProFun (Rt) / (Lt)	The same as above	The same as above	The same as above
LTCIR (Rt) / (Lt)	The same as above	The same as above	The same as above
SFA (Rt) / (Lt)	The same as above	The same as above	The same as above
Pop A (Rt) / (Lt)	The same as above	The same as above	The same as above

ATA (Rt) / (Lt)	The same as above	The same as above	The same as above
PTA (Rt) / (Lt)	The same as above	The same as above	The same as above
PERON (Rt) / (Lt)	The same as above	The same as above	The same as above
DRPED (Rt) / (Lt)	The same as above	The same as above	The same as above

5.3.1.7 GYN Measurement in B mode





Meas. item name	Mark	Unit	Meas. Method and calc. formular
UT	UT_L	cm	Refer to "Distance Measurement (Distance)
	UT_W	cm	Refer to "Distance Measurement (Distance)
	UT_H	cm	Refer to "Distance Measurement (Distance)
	UT_D	cm	UT_D= UT_L +UT_H + UT_W
	CX_L	cm	Refer to "Distance Measurement (Distance)
Cervix Vol.	Length	cm	Refer to "Distance Measurement (Distance)
	Height	cm	Refer to "Distance Measurement (Distance)
	Width	cm	Refer to "Distance Measurement (Distance)
	Volume	cm	$PV = (\pi/6) * L * W * H$
ENDO	endometrium	cm	Refer to "Distance Measurement (Distance)
Left_OV_Volume	Left_OV_L	cm	Refer to "Distance Measurement (Distance)
	Left_OV_W	cm	Refer to "Distance Measurement (Distance)
	Left_OV_H	cm	Refer to "Distance Measurement (Distance)
	Left_OV_Volume	ml	$Left_OV_Volume = (\pi/6) * L * W * H$

Right_OV_Volume	Right_OV_L Right_OV_W Right_OV_H Right_OV_Volume	cm cm cm ml	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $PV = (\pi/6) * L * W * H$
Left_FO_D	Length Width	cm cm	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)"
RightFO_D	Length Width	cm cm	Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)"
Uterine Artery	A Out A In STA% D Out D In STD% Vessel Area Vessel Dis	cm ² cm ² % cm cm % cm ² cm	Refer to "Refer to Area(Ellipse)" Refer to "Refer to Area(Ellipse)" $StA\% = (A \text{ Out} - A \text{ In}) / A \text{ Out} * 100\%$ Refer to "Distance Measurement (Distance)" Refer to "Distance Measurement (Distance)" $StD\% = (D \text{ Out} - D \text{ In}) / D \text{ Out} * 100\%$ Refer to "Refer to Area(Ellipse)" Refer to "Distance Measurement (Distance)"


For CCA and ICA, user also can measure the thickness of intimae.

IMT: measure the thickness of intimae manual.

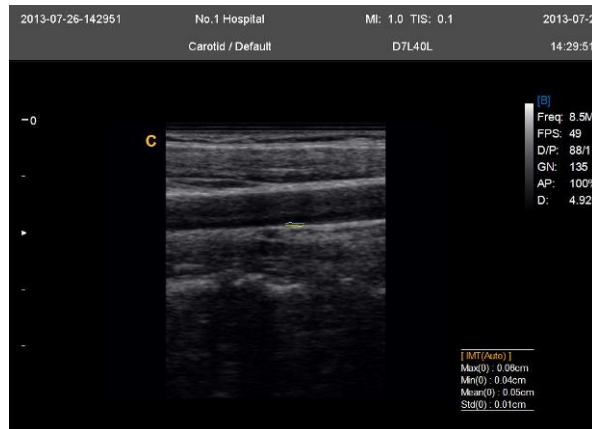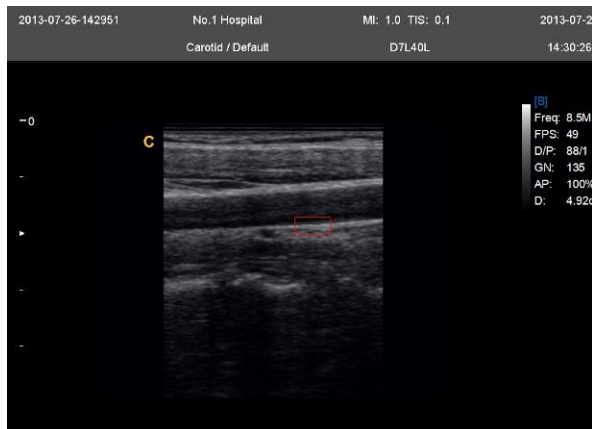IMT Auto: get a good B image for intimae, and then click this measurement. Set start position and end position, be sure to contain the intimae in the box, the system will calculate the thickness of intimae.



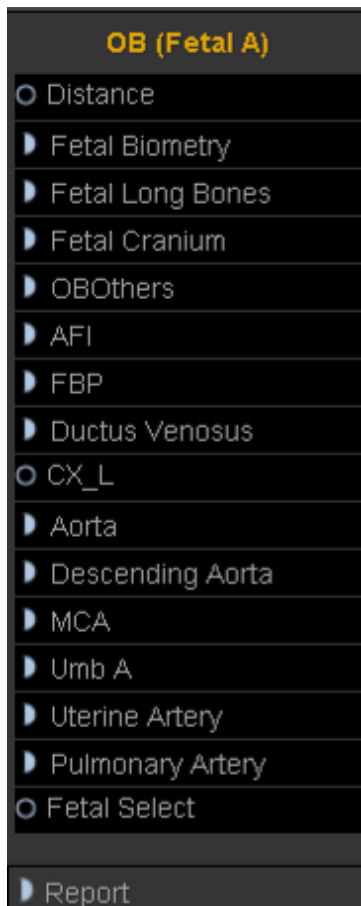5.3.1.8 OB Measurement in B mode




Meas. item name	Mark	Unit	Meas. Method and calc. formular
GS	GS GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)" EDD = current date + (280days – average UGA)
CRL	CRL GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)" EDD = current date + (280days – average UGA)
YS	YS	cm	Refer to "Distance Measurement (Distance)"

BPD	BPD GA EDD EFW	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
OFD	OFD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
HC (ellipse)	HC GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
APD	APD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
TAD	TAD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
AC (ellipse)	AC GA EDD EFW	cm week/day mm/dd/yyyy g	Refer to "Distance Measurement (Distance)"
FTA	FTA GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
FL	FL GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
SL	SL	cm	Refer to "Distance Measurement (Distance)"
APTD	APTD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
TTD	TTD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
ThC	ThC	cm	Refer to "Distance Measurement (Distance)"
Humerus	Humerus	cm	Refer to "Distance Measurement (Distance)"

	GA EDD	week/day mm/dd/yyyy	(Distance)"
ULNA	ULNA GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
Tibian	Tibian GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
RAD	RAD	cm	Refer to "Distance Measurement (Distance)"
FIB	FIB	cm	Refer to "Distance Measurement (Distance)"
CLAV	CLAV GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
CER	CER GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
CM	CM	cm	Refer to "Distance Measurement (Distance)"
NF	NF	cm	Refer to "Distance Measurement (Distance)"
OOD	OOD GA EDD	cm week/day mm/dd/yyyy	Refer to "Distance Measurement (Distance)"
IOD	IOD	cm	Refer to "Distance Measurement (Distance)"
NB	NB	cm	Refer to "Distance Measurement (Distance)"
Lvent	Lvent GA EDD	cm	Refer to "Distance Measurement (Distance)"
HW	HW	cm	Refer to "Distance Measurement (Distance)"
LtKid	LtKid	cm	Refer to "Distance Measurement (Distance)"
RtKid	RtKid	cm	Refer to "Distance Measurement (Distance)"


			(Distance)”
LtRenalAP	LtRenalAP	cm	Refer to "Distance Measurement (Distance)”
RtRenalAP	RtRenalAP	cm	Refer to "Distance Measurement (Distance)”
LVWrHEM	LVWrHEM	cm	Refer to "Distance Measurement (Distance)”
MAD	MAD	cm	Refer to "Distance Measurement (Distance)”
AFI	AF1	cm	Refer to "Distance Measurement (Distance)”
	AF2	cm	Refer to "Distance Measurement (Distance)”
	AF3	cm	Refer to "Distance Measurement (Distance)”
	AF4	cm	Refer to "Distance Measurement (Distance)”
	AFI	cm	Refer to "Distance Measurement (Distance)” AFI=AF1+AF2+AF3+AF4
FBP	AF Result	cm Scores	Refer to "Distance Measurement (Distance)” Give result according to AF

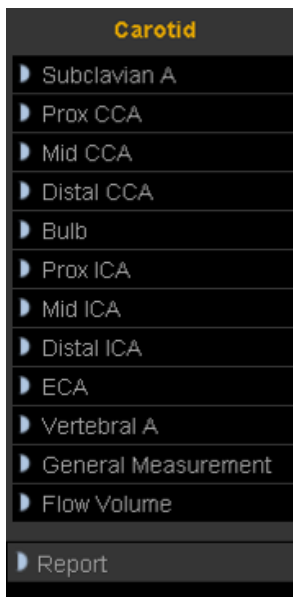5.3.1.9 Carotid Measurement in B mode



Meas. item name	Mark	Unit	Meas. Method and calc. formular
Subclavian A (Lt/Rt)	A Out	cm ²	Refer to "Area Measurement (Ellipse)"
	A In	cm ²	Refer to "Area Measurement (Ellipse)"
	StA%	%	StA %= (A Out-A In) / A Out*100%
	D Out	cm	Refer to "Distance Measurement (Distance)"
	D In	cm	Refer to "Distance Measurement (Distance)"
	StD%	%	StD %= (D Out-D In) / D Out*100%
	Vessel Area	cm ²	Refer to "Area Measurement (Ellipse)"
	Vessel Dis	cm	Refer to "Distance Measurement (Distance)"
Prox CCA (Lt/Rt)	The same as above	The same as above	The same as above
Mid CCA (Lt/Rt)	The same as above	The same as above	The same as above
Distal CCA (Lt/Rt)	The same as above	The same as above	The same as above
Bulb (Lt/Rt)	The same as above	The same as above	The same as above
Prox ICA (Lt/Rt)	The same as above	The same as above	The same as above
Mid ICA (Lt/Rt)	The same as above	The same as above	The same as above
Distal ICA (Lt/Rt)	The same as above	The same as above	The same as above
ECA (Lt/Rt)	The same as above	The same as above	The same as above
Vertebral (Lt/Rt)	The same as above	The same as above	The same as above
General Measurement	The same as above	The same as above	The same as above
Flow Volume	Vessel Dis	cm	Refer to "Distance Measurement (Distance)"
	Vessel Area	cm ²	Refer to "Area Measurement (Ellipse)"

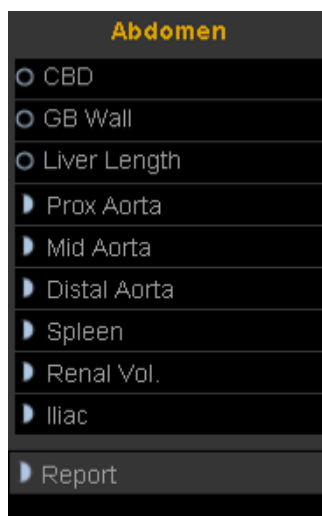5.3.1.10 Abdomen Measurement in B mode




Meas. item name	Mark	Unit	Meas. Method and calc. formular
CBD	CBD	cm	Refer to "Distance Measurement (Distance)"
GB Wall	GB Wall	cm	Refer to "Distance Measurement (Distance)"
Liver Length	Liver Length	cm	Refer to "Distance Measurement (Distance)"
Prox Aorta	Height	cm	Refer to "Distance Measurement (Distance)"
	Width	cm	Refer to "Distance Measurement (Distance)"
	A Out	cm ²	Refer to "Area Measurement (Trace)"
	A In	cm ²	Refer to "Area Measurement (Trace)"
	StA%	%	$StA\% = (A\ Out - A\ In) / A\ Out * 100\%$
	D Out	cm	Refer to "Distance Measurement (Distance)"
	D In	cm	Refer to "Distance Measurement (Distance)"
	StD%	%	$StD\% = (D\ Out - D\ In) / D\ Out * 100\%$
	Vessel Area	cm ²	Refer to "Area Measurement (Trace)"
Vessel Dis	cm	Refer to "Distance Measurement (Distance)"	
Mid Aorta	The same as above	The same as above	The same as above
Distal Aorta	The same as above	The same as above	The same as above
Spleen	Length	cm	Refer to "Distance Measurement (Distance)"
	Height	cm	Refer to "Distance Measurement (Distance)"
	Width	cm	Refer to "Distance Measurement (Distance)"

	Volume	ml	Volume = $(\pi/6) * \text{Length} * \text{Width} * \text{Height}$
Renal Vol.(Rt/Lt)	Length	cm	Refer to "Distance Measurement (Distance)"
	Height	cm	Refer to "Distance Measurement (Distance)"
	Width	cm	Refer to "Distance Measurement (Distance)"
Lliac(Rt/Lt)	Height	cm	Refer to "Distance Measurement (Distance)"
	Width	cm	Refer to "Distance Measurement (Distance)"
	A Out	cm ²	Refer to "Area Measurement (Trace)"
	A In	cm ²	Refer to "Area Measurement (Trace)"
	StA%	%	StA% = $(A \text{ Out} - A \text{ In}) / A \text{ Out} * 100\%$
	D Out	cm	Refer to "Distance Measurement (Distance)"
	D In	cm	Refer to "Distance Measurement (Distance)"
	StD%	%	StD% = $(D \text{ Out} - D \text{ In}) / D \text{ Out} * 100\%$
	Vessel Area	cm ²	Refer to "Area Measurement (Trace)"
	Vessel Dis	cm	Refer to "Distance Measurement (Distance)"

5.3.2 Measurement in M mode

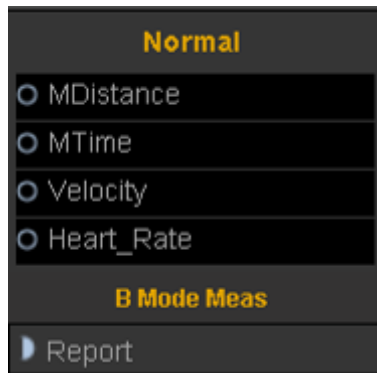5.3.2.1 General Measurement in M mode



1. M distance

This feature allows the measurement of the distance between two points. It is a measurement between the two horizontal lines that lean on the two cursors. The position of the vertical time line does not affect the distance measurement

Operation:

Rotate the trackball to select the “M distance” item in the menu. A start cursor “+” will appear on the screen. Move the cursor through rotating the trackball and press Enter-key to fix the first point. The second cursor will appear. Move the second cursor to the end point and press Enter-key to fix it. The measurement result will appear on the right side of the screen.

2. M time

Time is the measurement between the two vertical time lines created by two cursors. The position of the horizontal distance line does not affect time measurements.

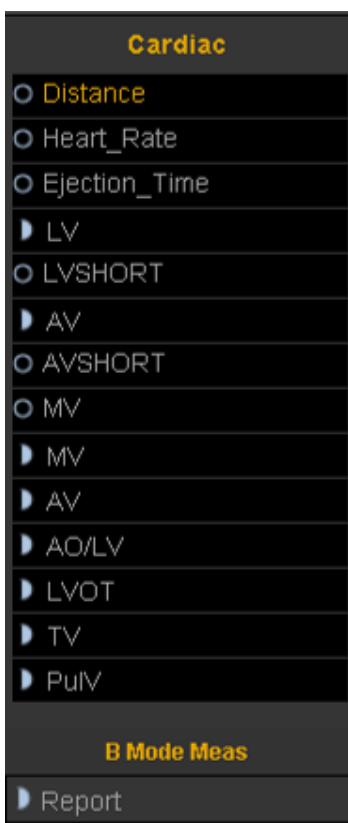
3. Velocity

Velocity is the measurement between the intersections of the two cursors. Velocity can be positive or negative, and is measured as the rate of change between the two points defined by the intersections of the cursors in cm/sec.

4. HR

HR is the measurement between the two vertical lines that are created by two cursors in beat per minute (BPM). The position of the horizontal distance line does not affect HR.

5.3.2.2 Cardiology Measurement in M mode



1. Distance

Refer to distance measurement in general measurement in M mode

2. HR

Refer to HR measurement in general measurement in M mode

3. Ejection_Time

Refer to Time measurement in general measurement in M mode

4. LV

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
IVSd	Interventricular septal thickness at end diastole [mm]	Refer to distance measurement (Distance) in B mode
LVIDd	Left ventricular short-axis diameter at end diastole [mm]	Refer to distance measurement (distance) in B mode
LVIDs	Left ventricular short-axis diameter at end systole [mm]	Refer to distance measurement (Distance) in B mode
LVPWd	Left ventricular posterior wall thickness at end diastole [mm]	Refer to distance measurement (Distance) in B mode

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
EDV	End-diastolic left ventricular volume [mL]	$EDV = 7.0 / (2.4 + LVIDd) * LVIDd * LVIDd$
ESV	End-systolic left ventricular volume [mL]	$ESV = 7.0 / (2.4 + LVIDs) * LVIDs * LVIDs$
SV	Stroke volume [mL]	EDV - ESV
CO	Cardiac output [L/min]	SV x HR /1000
CI	Cardiac index	CO/BSA
EF	Ejection fraction [no unit]	SV / EDV
FS	Fractional shortening [no unit]	$(LVIDd - LVIDs) / LVIDd$

5. LVSHORT

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
AIVSD	Anterior wall of interventricular septum at end diastole	Refer to distance measurement in M mode
PIVSD	Posterior interventricular septum distance at end diastole	Refer to distance measurement in M mode
ENDOD	Endocardium of posterior wall of left ventricular at diastole	Refer to distance measurement in M mode
EPID	Epicardium at diastole	Refer to distance measurement in M mode
AIVSS	Anterior wall of interventricular septum at end systole	Refer to distance measurement in M mode
PIVSS	Posterior wall of interventricular septum at end systole	Refer to distance measurement in M mode

ENDOS	Endocardium of posterior wall of left ventricular at systole	Refer to distance measurement in M mode
EPIS	Epicardium at systole	Refer to distance measurement in M mode

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
IVSd	interventricular septum at end diastole[mm]	
LVIDd	Left ventricular short-axis diameter at end diastole [mm]	
LVIDs	Left ventricular short-axis diameter at end systole [mm]	
LVPWd	Left ventricular posterior wall thickness at end diastole [mm]	
EDV	End-diastolic left ventricular volume [mL]	$EDV = 7.0 / (2.4 + LVIDd) * LVIDd * LVIDd$
ESV	End-systolic left ventricular volume [mL]	$ESV = 7.0 / (2.4 + LVIDs) * LVIDs * LVIDs$
SV	Stroke volume [mL]	$EDV - ESV$
CO	Cardiac output [L/min]	$SV * HR / 1000$
CI	Cardiac index	CO / BSA
EF	Ejection fraction [no unit]	SV / EDV
FS	Fractional shortening [no unit]	$(LVIDd - LVIDs) / LVIDd$

6. AV

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
AOD	Aorta diameter at diastole[mm]	Refer to distance measurement in M mode
LAS	Posterior interventricular septum distance at end diastole[mm]	Refer to distance measurement in M mode
AVO	AVO[mm]	Refer to distance measurement in M mode
LVET	Left ventricular ejection time[s]	Refer to time measurement in M mode
PEP	Pre-ejection period[s]	Refer to time measurement in M

		mode
--	--	------

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
LAR	Ratio between LA and aorta	$LAR=LAS/AOD$
ETR	Ratio between PEP and LVET	$ETR=PEP/LVET$

7. AVSHORT

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
AAW	Posterior wall of aorta at diastole	Refer to distance measurement in M mode
PAWD	Posterior wall of aorta at diastole	Refer to distance measurement in M mode
AAL	Anterior leaflet at AVO	Refer to distance measurement in M mode
PAL	Posterior leaflet at AVO	Refer to distance measurement in M mode
PAWS	Posterior wall of aorta at leaflet closed point	Refer to distance measurement in M mode
PLA	Posterior wall of posterior leaflet of aorta at closed point	Refer to distance measurement in M mode

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
AO	Ratio between LA and aorta[mm]	
LAS	Ratio between PEP and LVET	
AVO	AVO[mm]	
LVET	Left ventricular ejection time[s]	
PEP	Pre-ejection period[s]	
LAR	Ratio between LA and aorta	$LAR=LAS/AOD$
ETR	Ratio between PEP and LVET	$ETR=PEP/LVET$

8. MV

<<Items to be measured>>

Meas. item name	Description [Unit]
D	D wave
E	E wave
F	F wave
EPSS	The distance between mitral valve and interventricular septum

<<Items to be calculated>>

Calc. item name	Description [Unit]
EPSS	Distance between the mitral valve and interventricular septum[cm]
DEex	D-E shift of mitral valve[s]
DEsl	D-E slope of mitral valve[deg]
EFsl	E-F slope of mitral valve[deg]

9. MV Expert

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
EFSLP	Closed velocity of mitral valve[cm/s]	
EPSS	The distance between E point and interventricular septum[mm]	
CEAMP	The amplitude of E wave[mm]	
CAAMP	The amplitude of A wave[mm]	
DEAMP	The amplitude of DE wave[mm]	
DESLP	The open velocity of mitral valve[cm/s]	

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
CA/CE	The ratio between A wave and E wave	CAAMP/CEAMP

10. AV (EXPERT)

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
AOD	Aorta diameter[mm]	M distance
LAD	Left ventricular diameter[mm]	M distance
AVD	Diameter of mitral valve[mm]	M distance

ET	Ejection time[ms]	M Time
RVOTD	Diameter of right ventricular outlet[mm]	M distance

11. AO/LV

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
AO Root	Aorta diameter[mm]	M distance
LA Diam	Left ventricular diameter[mm]	M distance
LVOT Diam	Diameter of mitral valve[mm]	M distance

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
LA/AO	Ratio between left atrium and aorta	LAD/AOD

12. LVOT

LVOT Diam:According to the left ventricular outflow tract diameter , automatically calculated left ventricular outflow area.

Left ventricular outflow tract diameter measurement method refer to" M distance "

13. TV

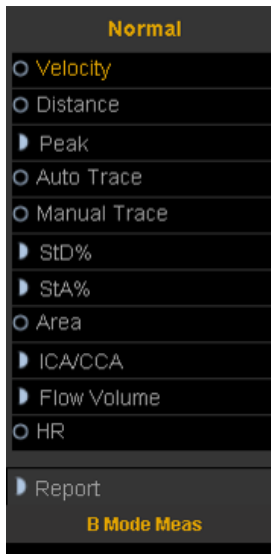
R-R interval: Select one or two cycles, automatically calculate the R-R interval, the measurement method refer to" M Time "

14. PulV

R-R interval: Select one or two cycles, automatically calculate the R-R interval, the measurement method refer to" M Time "

5.3.3 Measurement in PW mode

5.3.3.1 General Measurement in PW mode



1. Velocity

- 1) Rotate the trackball to move the cursor to the velocity item in the menu and press Enter-key to select it. A cursor "+" will appear on the screen.
- 2) Move the cursor "+" to the place where need to be measured and press Enter-key to fix it.
- 3) The value of velocity and pressure will appear on the screen.
- 4) Repeat step 1) to 3) to measure the next point.

2. Distance

The measurement method is the same as distance in general measurement in B mode.

3. Peak


Calculate the velocity over one cardiac cycle. The velocity, slope, RI and SD ratio are calculated.

- 1) Scan the object area in PW mode and freeze the image
- 2) Rotate the trackball to move the cursor to the Peak item in the menu and press Enter-key to select.
- 3) A cursor "+" will appear on the screen. Move the cursor "+" to the peak point where the cardiac systole and press Enter-key to fix it.
- 4) A second cursor "+" will appear on the screen again. Fix the second cursor to the end point where the cardiac diastole
- 5) When the two points are all fixed, the value of $V_s, V_d, RI, SD(V_s/V_d)$ will appear on the right side of the screen.

4. Full Trace

- 1) Rotate the trackball to move the cursor to the auto trace item in the menu.
- 2) Press Enter-key to select it. A cursor "+" will appear on the screen.

- 3) Move the cursor by rotating the trackball to the start point of the one cycle and press Enter-key to fix it. A second cursor “|” will appear.
- 4) Rotate the trackball the end point of the cycle and press Enter-key to fix it.
- 5) After the measurement, the value of Vs,Vd,RI,SD ratio,PI will appear on the screen.

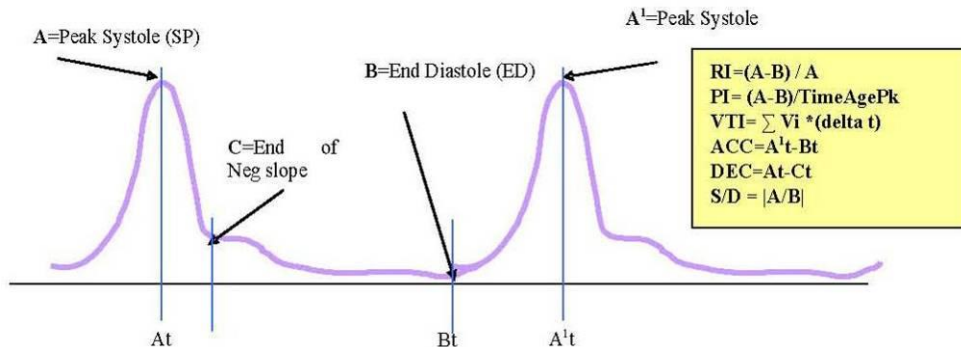

- Peak Velocity or Doppler frequency
Peak velocity over one cardiac cycle (Vpk)
- Doppler Time Distance
Time distance between two cursors in ms. The invert is the heart beat rate if the two cursors are at one cardiac cycle period. (T)
- Spectral Velocity Time Integral (VTI)
 $VTI = \sum Vpk * (\text{delta } t)$. Where delta t is T/N, N is the data point over one cardiac cycle.
- Pulsatility Index (PI)
It can be used to represent the degree of pulse-wave damping at different arterial sites; the smaller the PI, the greater the degree of damping. Typical value for CCA is 1.90+/-0.5.
 $PI = |(A-B)/\text{TimeAvgPk}|$, $\text{TimeAvgPk} = \sum Vpk / N$
- Resistive Index (RI)
Vary from 0 to 1. It is an indicator of the circulatory resistance. Typical value for CCA is 0.75+/- 0.05.
 $RI = |(A-B)/A|$

Note:

- a) In order to get accurate result, the PW image must be clear and high quality.
- b) Insure you fix the cursor at the exact place of cardiac systole and diastole

Manual editing of the envelope curve:

- 1:After you get the envelope curve press “Update”, the start point will be displayed in orange.
- 2:Move trackball to the start point, where you want to edit, press “Enter” key to enter into manual status.
- 3:Trace the curve by trackball, and press ”Enter” key to finish.
- 4:The envelope will be updated after being edited.
- 5:Select the cycle, the result the result will be displayed on the screen.

5. Manual trace

Automatically measure the Peak Systolic Velocity, VTI, end diastolic velocity, HR, Time, minimum velocity, PI, and RI after manually tracing of the curve is finished.

Note: Manual trace requires the user to trace two peak point of two cycles

6. STD

Measure the diameter reduction ratio of vessel in B mode

- 1) Move the cursor to the “STD” item and press Enter-key to select it .A cursor “+” will appear on the screen.
- 2) Move the cursor to the point of outside wall of the vessel and press Enter key to fix it. The method to measure the diameter of the outside wall of the vessel is the same as “distance” in genera measurement in B mode
- 3) When the diameter of outside wall of the vessel is finished,the cursor “+”will appear on the screen again. Measure the diameter of the stenosis area;
- 4) The value of every diameter and STD will be displayed in the result window.

7. STA

Measure the area reduction ratio of vessel in B mode

- 1) Move the cursor to the “STA” item and press Enter-key to select it .A cursor “+”will appear on the screen.
- 2) Move the cursor to the point of outside wall of the vessel and press Enter-key to fix it. The method to measure the area of the outside of the vessel is the same as “area-ellipse” in genera measurement in B mode
- 3) When the area of outside of the vessel is finished , the cursor “+”will appear on the screen again. Measure the area of the stenosis area;
- 4) The value of every area and STD will be displayed in the result window.

8. Area

The same as “area-trace” in general measurement in B mode.

9. ICA/CCA

Measure ICA and CCA separately and get the ratio value of ICA/CCA

10. Flow volume

<<Items to be measured>>

Meas. item name	Description [Unit]	Meas. Method
Diam	Diameter of vessel [mm]	Refer to “distance” measurement
VTI	Velocity time integral	Refer to “area-trace” measurement
Time	Time	Refer to “time” measurement in M mode

<<Items to be calculated>>

Calc. item name	Description [Unit]	Calc. formula
SV	Stroke volume[ml]	$SV = 0.785 * \text{diameter} * \text{diameter} * \text{VTI} $
CO	Cardiac output[L/min]	$CO = SV * \text{heart rate} / 1000$
HR	Heart rate	$\text{Heart rate} = 60 / \text{time}$

11. Heart rate

The measurement is the same as “heart rate” in general measurement in M mode

5.3.3.2 Cardiology Measurement in PW mode

1. MV

Measurement of Mitral valve function is performed in frozen Doppler spectral trace.

<<Measurement and calculation items>>


Meas. item name	Description [Unit]	Meas. procedure / Calc. formula
E Vel	E-wave flow velocity [cm/s]	Refer to "Velocity Measurement".
A Vel	A-wave flow velocity [cm/s]	Refer to "Velocity Measurement".
PHT	Pressure half time [ms]	Refer to "Velocity Measurement".
E Dur	E-wave duration [ms]	Refer to "Time Measurement (Time)".
A Dur	A-wave duration [ms]	Refer to "Time Measurement (Time)".
IRT	Isovelocity relaxation time [ms]	Refer to "Time Measurement (Time)".
MV VTI	Mitral valve VTI [m]	Refer to "Full trace".
MV VM	Mitral valve mean velocity [cm/s]	Refer to "Full trace"
MV VP	Mitral valve maximum velocity [cm/s]	Refer to "Full trace"
MV MPG	Mitral valve mean pressure gradient [mmHg]	Refer to "Full trace"
MV PPG	Mitral valve maximum pressure gradient [mmHg]	Refer to "Full trace"
MV Diam	Mitral valve diameter [mm]	Refer to "Distance Measurement in B mode"
HR	Heart rate [bpm]	Refer to "Heart Rate Measurement (HR)"
E/A	[no unit]	E Vel / A Vel
A/E	[no unit]	A Vel / E Vel
SV	Stroke volume [mL]	$0.785 \times (\text{MVDiam})^2 * \text{abs VTI }$
CO	Cardiac output [L/min]	SV x HR / 1000
MV Area	Mitral valve area [mm ²]	220/ PHT

Note:


[MV Diam] can only be performed in 2-D mode.

<<Measurement points>>

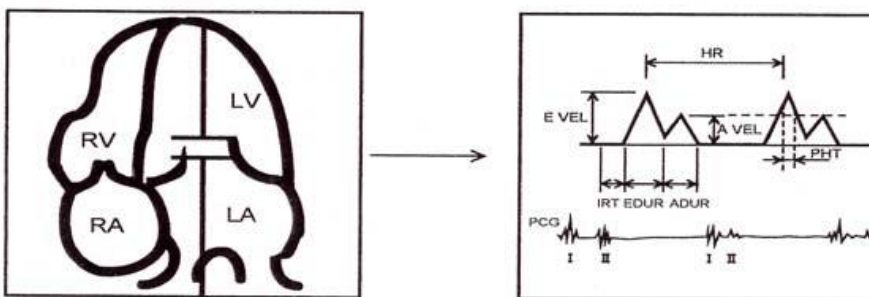
1) Measure [E Vel] and [A Vel]. Measure [PHT]. Measure [E-Dur], [A-Dur], and [IR] Measure [HR].



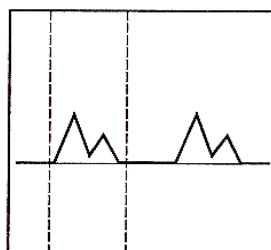2) Measure [MV V Trace].



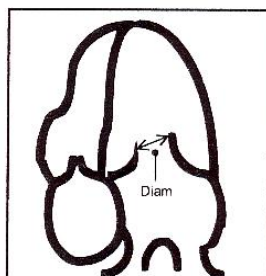3) Measure [MV Diameter



2. AV

Measurement of aortic valve function is performed by using 2-D-mode and Doppler images.

<<Measurement items>>

Meas. Item name	Description [Unit]	Meas. method
LVOT Vel	PW-mode LV out tract flow velocity [cm/s]	Refer to "Velocity Measurement".
LVOT PG	PW-mode LV out tract pressure gradient [mmHg]	Refer to "Velocity Measurement".

LVOT V General

Meas. Item name	Description [Unit]	Meas. method
LVOT VTI	Aortic valve VTI [m]	Refer to "Full trace"
LVOT VM	Aortic valve mean velocity [cm/s]	Refer to "Full trace"
LVOT VP	Aortic valve maximum velocity [cm/s]	Refer to "Full trace"
LVOT MPG	Aortic valve mean pressure gradient [mmHg]	Refer to "Full trace"
LVOT PPG	Aortic valve maximum pressure gradient [mmHg]	Refer to "Full trace"

AoV VM


Meas. Item name	Description [Unit]	Meas. method
AoV MPG	Aortic valve mean pressure gradient [mmHg]	Refer to "Full trace"
AoV PPG	Aortic valve maximum pressure gradient [mmHg]	Refer to "Full trace"
LVOT Diam	Left ventricular outflow tract diameter [mm]	Refer to "Distance Measurement in B mode".
HR	Heart rate [bpm]	Refer to "Heart Rate Measurement (HR)".
SV	Stroke volume [mL]	$0.785 \times (\text{LVOT Diam})^2 \times \text{abs VTI }$
CO	Cardiac output [L/min]	$\text{SV} \times \text{HR} / 1000$
AV Area	AoV area [cm ²]	$\text{SV} / \text{AoV VTI} / 100$

Note:


[LVOT Diam] can only be performed in 2D mode

<<Measurement points>>

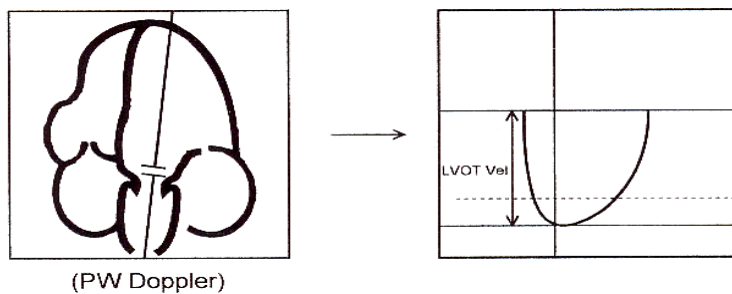1) Measure [LVOT Vel]. Measure [HR].



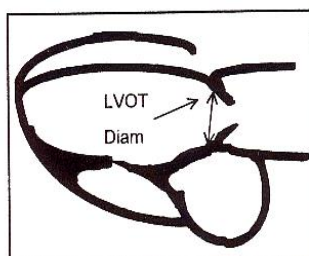2) Measure [LVOT Diam]



3. TV

Measurement of tricuspid valve function is performed by using Doppler images.


<<Measurement items>>

Meas. Item name	Contents	Meas. Method
TV VTI	Tricuspid valve VTI [m]	Refer to "Manual Trace Measurement"..
TV VM	Tricuspid valve mean velocity [cm/s]	Refer to " Manual Trace Measurement "..
TV VP	Tricuspid valve maximum velocity [cm/s]	Refer to " Manual Trace Measurement ".


TV MPG	Tricuspid valve mean pressure gradient [mmHg]	Refer to " Manual Trace Measurement ".
TV PPG	Tricuspid valve maximum pressure gradient [mmHg]	Refer to "Manual Trace Measurement ".
TV Vel	Tricuspid valve velocity [cm/s]	Refer to "Velocity Measurement".
TV PG	Tricuspid valve pressure gradient [mmHg]	Refer to " Velocity Measurement'.

<<Measurement points>>

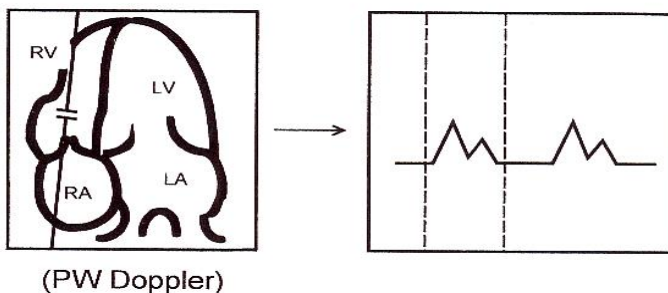1)Measure [TV VEL Trace]



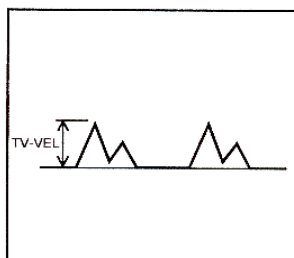2)Measure [TV VEL]



4. PV

Measurement of pulmonary valve function is performed by using a Doppler image.

<<Measurement items>

Meas. item name	Description [Unit]	Meas. method
PV VTI	Pulmonary valve VTI [m]	Refer to "Full Trace Measurement ".
PV VM	Pulmonary valve mean velocity [cm/s]	Refer to "Full Trace Measurement ".
PV VP	Pulmonary valve maximum velocity [cm/s]	Refer to "Full Trace Measurement ".
PV MPG	Pulmonary valve mean pressure gradient [mmHg]	Refer to "Full Trace Measurement ".
PV PPG	Pulmonary valve maximum pressure gradient [mmHg]	Refer to " Velocity Measurement"
PV Max Vel	Maximum velocity [cm/s]	Refer to " Velocity Measurement"

PV Max PG	Maximum pressure gradient [mmHg]	Refer to "Velocity Measurement"
PV Diam	Pulmonary artery diameter [mm]	Refer to "Distance Measurement in B mode".
HR	Heart rate [bpm]	Refer to "Heart Rate Measurement (HR)"
RV ET	Ejection time [ms]	Refer to "Time Measurement (Time)"
RV AcT	Acceleration time [ms]	Refer to "Time Measurement (Time)"
RV PEP	Pre-ejection period [ms]	Refer to "Time Measurement (Time)"

Note:


[PV Diam] can only be performed in 2D mode

<<Calculation items>>


Calc. item name	Description [Unit]	Calc. formula
SV	Stroke volume [mL]	$0.785 \times (\text{PV Diam})^2 * \text{abs VTI }$
CO	Cardiac output [L/min]	$\text{SV} \times \text{HR} / 1000$
RV AcT/ET	Ratio of AcT to ET	$\text{RV ACT} / \text{RV ET}$
RV STI	Systolic time interval	$\text{RV REP} / \text{RV ET}$

<<Measurement points>>

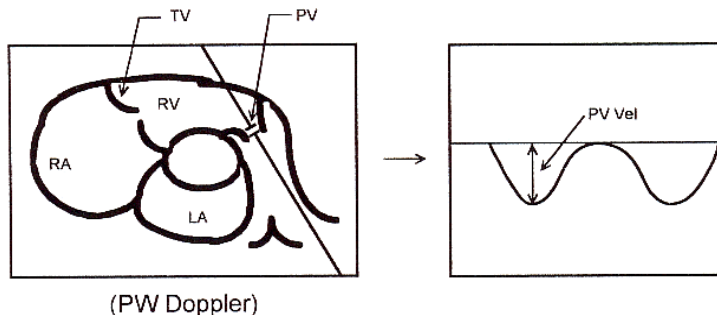
1) Measure [PV Vel]



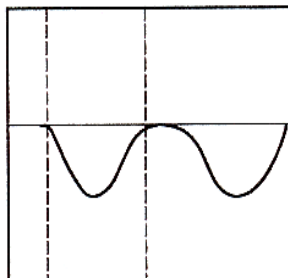
2) Measure [PVV Trace]



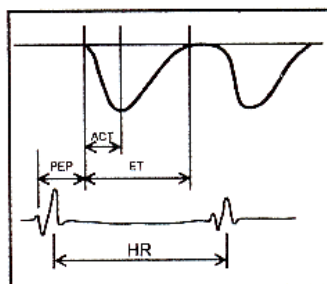3) Measure [ET],[ACT], and [PEP]



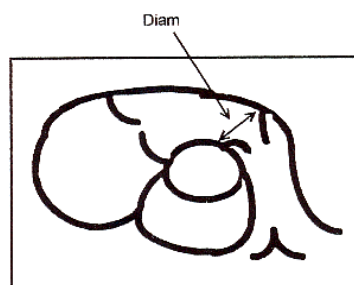4) Measure [PV Diam]



5.3.3.3 Urology Measurement in PW mode

The measurement method and calculation items are the same as general measurement in PW mode

5.3.3.4 Pediatric Measurement in PW mode

The measurement method and calculation items are the same as general measurement in PW mode.

5.3.3.5 Small parts Measurement in PW mode

The measurement method and calculation items are the same as general measurement in PW mode.

5.3.3.6 Vascular Measurement in PW mode

The measurement method and calculation items are the same as “manual trace”

5.3.3.7 GYN Measurement in PW mode

The measurement method and calculation items are the same as “manual trace”

5.3.3.8 OB Measurement in PW mode

The measurement method and calculation items are the same as GYN measurement in PW mode.

5.4 Edit measurement results

After the user finishes the measurements, this system allows users to move the position of measurement results, or change the font size of measurement results.

To move result position, operations are as follows:

In the measurement status, rotate SK2-key to choose “Move result position”.

Press SK2-key, and use trackball to move the position of measurement results .

Press Enter-key to confirm it

To reset result position, operations are as follows:

In the measurement status, rotate SK3-key to choose “Reset result position”

Press SK3-key, the position of current measurement result will be reset back to default.

To change the font size of measurement results, operations are as follows:

In the measurement status, rotate SK1-key to adjust the font size of measurement results.

To reset the font size of measurement results, operations are as follows:

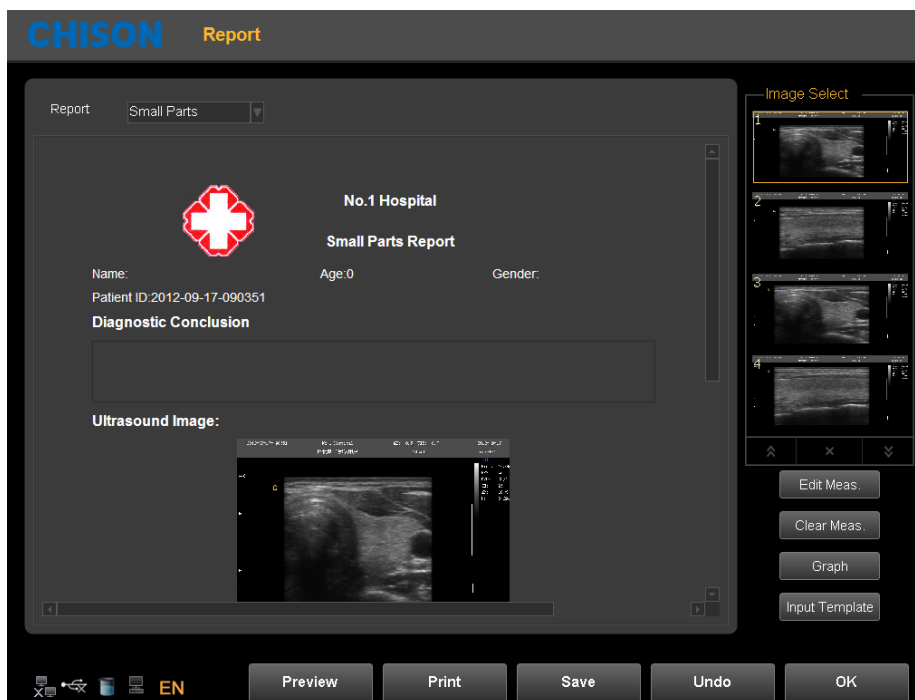
In the measurement status, press SK1-key, the font size of measurement results will be reset back to default.

The system default font size is 11.

5.5 Report

After finishing the exam, press the Report-key to pop up report interface, edit and print the report.

Report Interface:



Choose the image: Click the image can add the image to the image area in the report, and click the image in the

report to remove it.

Preview: preview the whole report and its format before printing

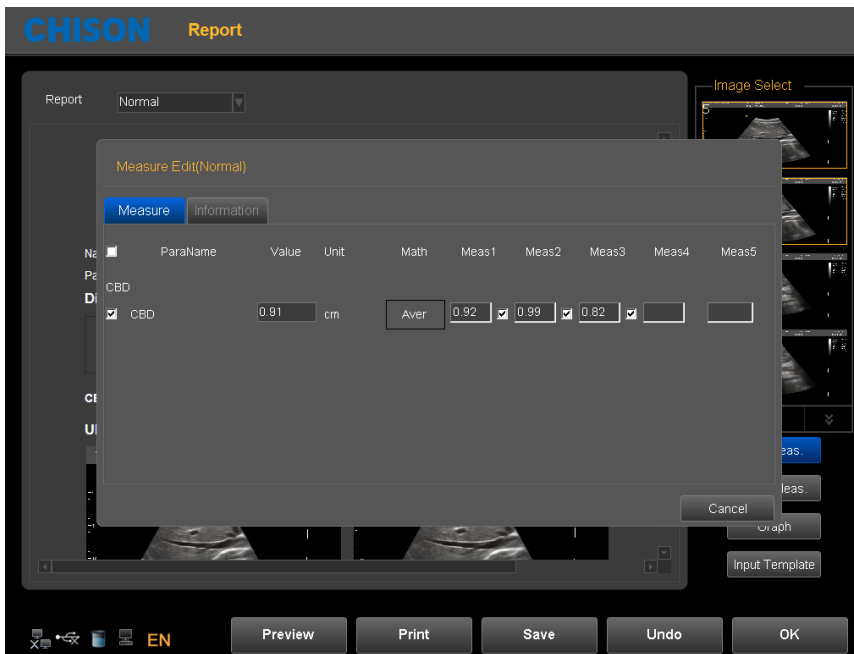
Print: print the current report, please make sure that the printer is normal working.

Save: store the report on the disk.


OK: confirm the operation and exit the interface

Undo: cancel the operation and exit the interface

Edit Meas: Press this icon to enter into Measurement Edit Item. Choose the measurement result, which you want to display in the report. Customer also can edit the result.



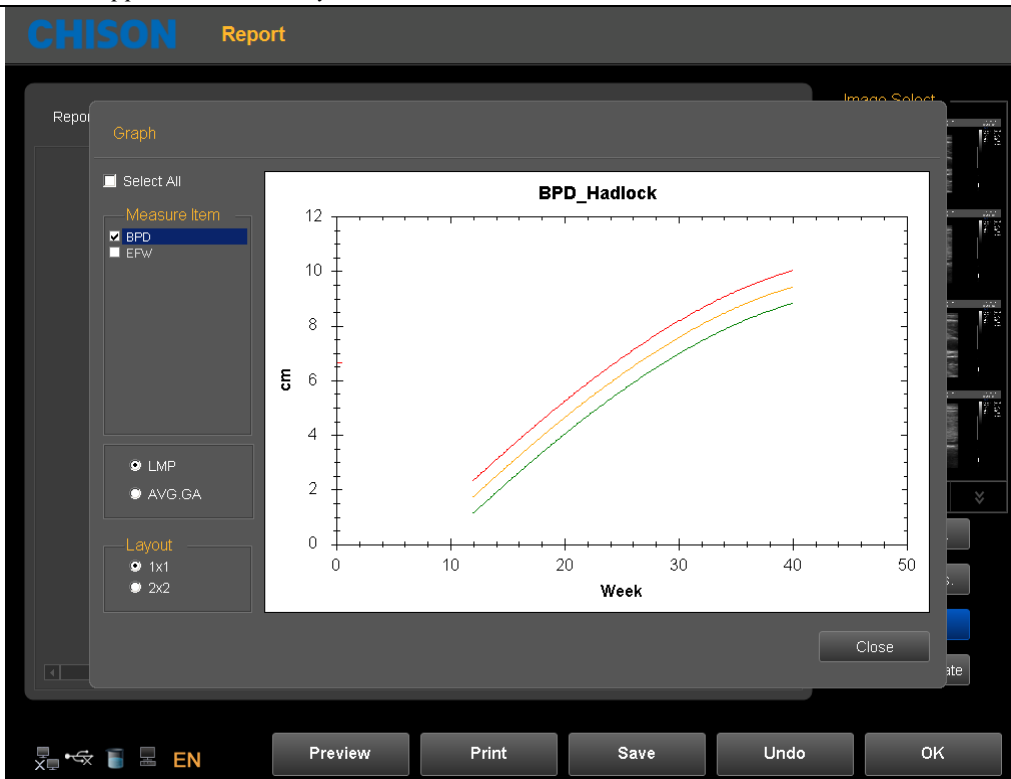Graph : Under OB report you can press this icon to enter in to see the fetal production graph.




Input Template: Input annotation from templates

The figure shows a 'Template Select' dialog box in the CHISON Report software. The dialog is titled 'Template Select' and has a dropdown menu set to 'Pediatrics'. On the left, there is a tree view showing a 'Template' folder containing 'Normal', 'Liver', 'Kidney', and 'Heart'. The 'Normal' folder is expanded, and 'Normal' is selected. On the right, there is a 'Name' field containing 'Normal' and a list of 'Systemic Normal' templates. A 'Cancel' button is located at the bottom right of the dialog. An 'Input Template' button is visible at the bottom right of the main report window.

Chapter 6 Preset

6.1. Recall Preset



- a) Press  button to choose the probe type.
- b) Choose a clinical application.
- c) Choose a preset.
- d) Choose a user defined preset and double-click it.

The default choice of the system is DEFAULT preset.

After entering into the system, if you want to change preset and do not want to quit the current interface, operate as follows:

- 1、 Press MENU- knob.
- 2、 Rotate MENU- knob and select [Utility] item.
- 3、 Press MENU-knob to enter into the item.
- 4、 Now SK1—SK5 Shortcut keys are corresponding with following operation:
 - SK1—rotate SK1 to choose the clinical application.
 - SK2—rotate SK2 to choose the preset.
 - SK3—rotate SK3 to choose the user-defined.
 - SK3—press SK3 edit user name (just user 1 to user 5 can be edited).
 - SK4—press SK4 to load preset.
 - SK5—press SK5 to save preset. (Just can save into user 1 to user 5 can be edited)

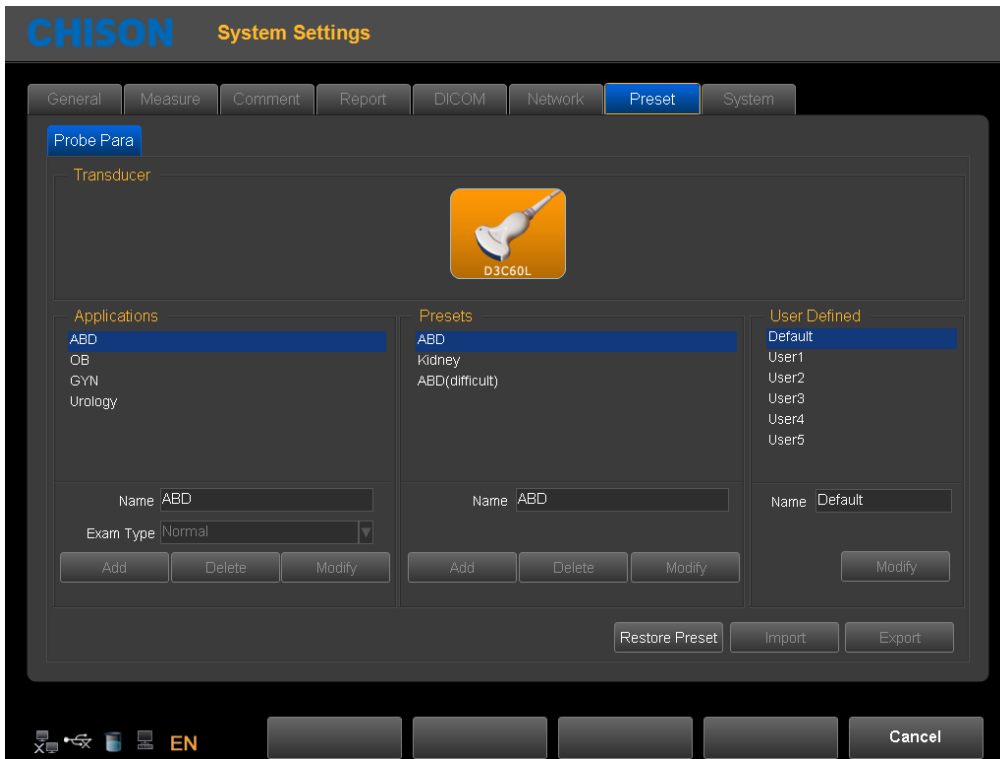
6.2. Save user defined preset

- 1) Choose and recall a preset first.
- 2) Adjust parameters to current preset.
- 3) Press MENU- knob when adjustment is finished. Rotate MENU- knob and select [utility] item. Press MENU- knob to enter.
- 4) Rotate SK1 to choose clinical application name (skip this step if do not need adjustment).
- 5) Rotate SK2 to choose the name of preset(skip this step if do not need adjustment)
- 6) Rotate SK3 to choose user defined. Press SK3 for editing user name. (Just user 1 to user 5 can be edited).
- 7) Press SK5 to save preset.

6.3. Manage Preset



Press key to enter into system setting interface. Select preset page. Add or delete the current preset.



Name: input Application/Preset/User Defined name.

Exam Type: select clinical application.

Add: add Application/Preset.

Delete: delete a Application/Preset .

Modify: revise Application/Preset/User Defined name

Restore Preset: restore the preset to factory setting.


Import: import preset to system. Plug in U disk which has preset. Press Import key ,preset can be imported into the system .

Export: export system preset. Plug in USB, Press the Export key ,System automatically derive the preset to U disk.

Cancel: exit system setting interface

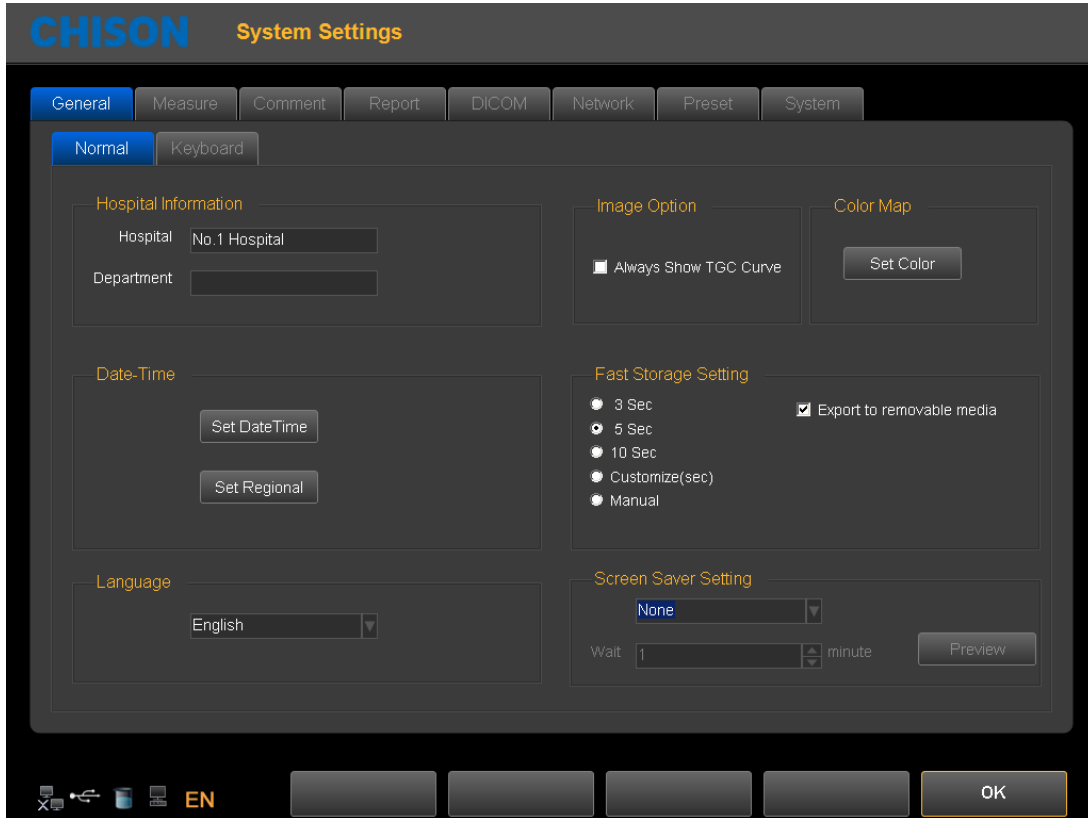
Chapter 7 System Setting



Press the  key to enter system setting interface. User can do user-defined setting.

7.1. General settings


Normal Settings: set the hospital information, date-time, language etc.



Hospital Information	hospital; Department;	Input hospital name, department name
Date-Time	Set Date Time Set Regional	Set Date and time (need password) Set Time Zone, Data Format, Time Format
Language	Chinese; English; French; Spanish; Russian; Polish; Portuguese	Switch language interface
Image Option	Always show TGC Curve or not	Select it ,it will always show TGC Curve .
Fast storage	3 seconds/5 seconds/10	Please refer 4.4.6

	seconds fast storage User-defined setting the storage seconds Fast storage manually Storage derive to mobile medium	
Color Map		Click the Configure key to set the color map

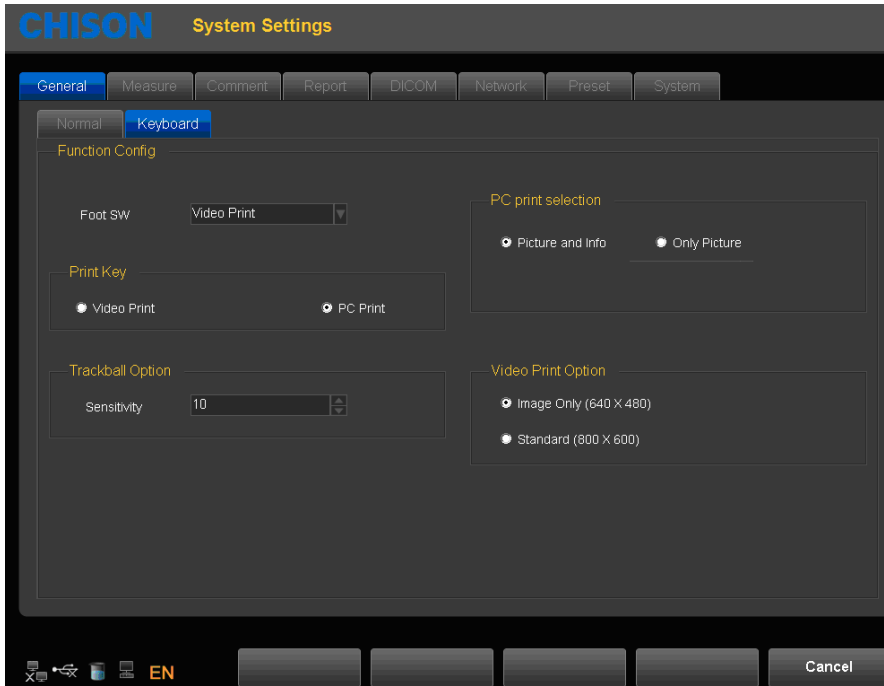Keyboard Settings,




Footswitch	Video Print; PC Print; Freeze/UnFreeze; Cine Save; Still Save	Choose the footswitch function
Print-key	Video printing; PC print	Set the print-key function
Trackball Option	Sensitivity(1~20)	Set the sensitivity of the trackball option.
PC print selection	Picture and info or Only Picture	Please refer to 3.6.3
Video Print Option	Image Only(640*480) Standard(800*600)	Please refer to 3.6.1

7.2 Measurement


Set the measurement formula of measurement.

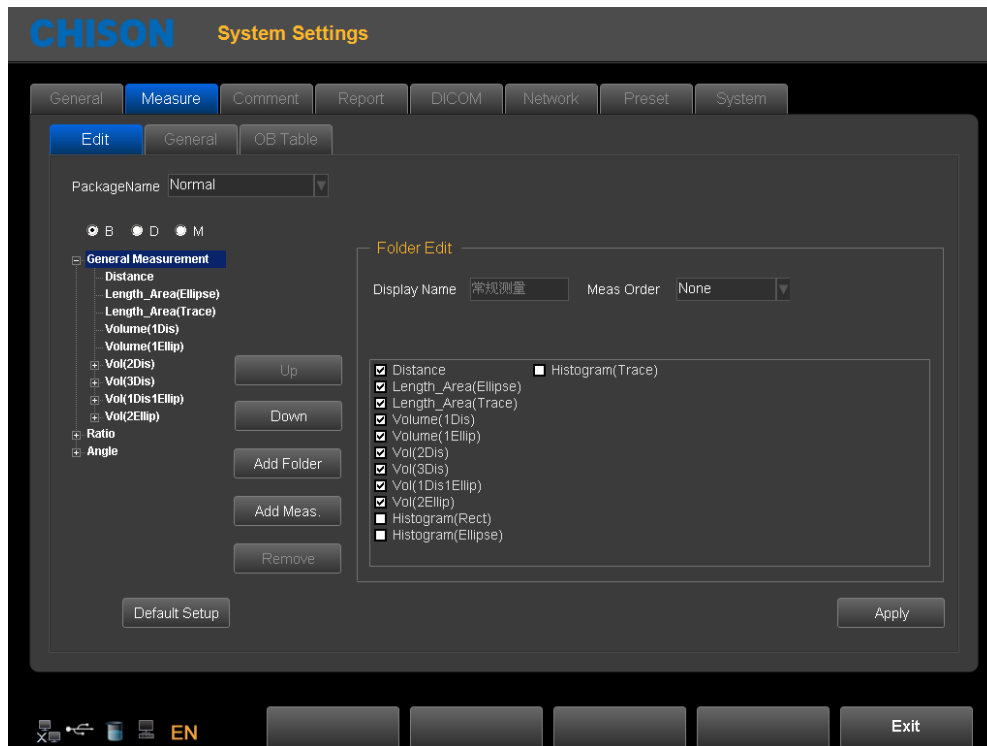Edit Settings: Click the measurement page, and click on the Edit page, then users can edit custom configuration measurement.



- 1、 Package Name: Choose the different examinations.
- 2、 Choose the different exam modes.
- 3、 Choose the measurement item and click "UP" or "Down" button to move it`s position
- 4、 Add Folder: Click this button to add a new item.
- 5、 Clicking “Add Meas.” can add a new measurement.

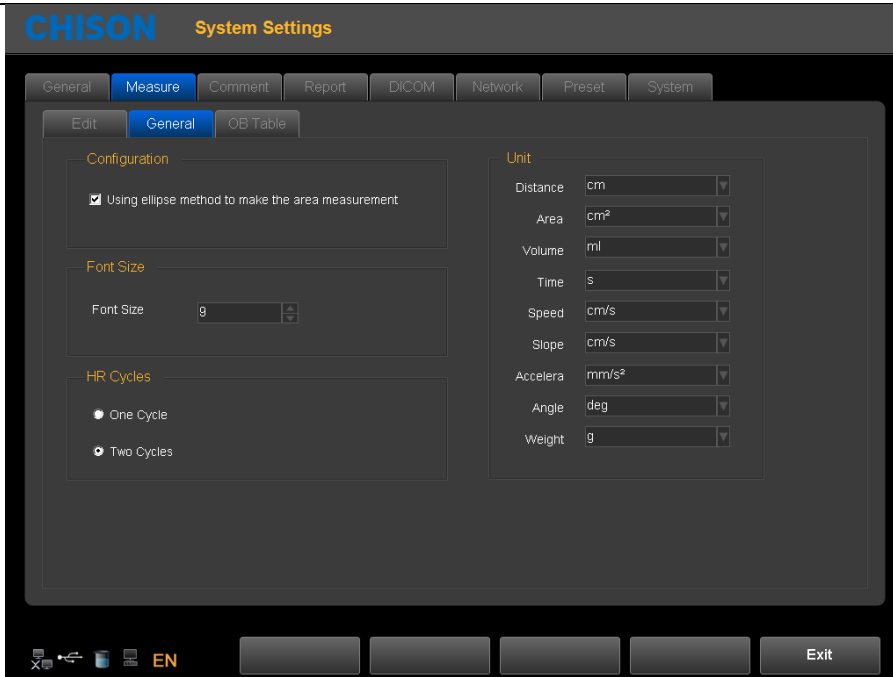General Settings: Click the General page, and you can do some general settings of measurement items



Font size: Set to default font size of the measurement results

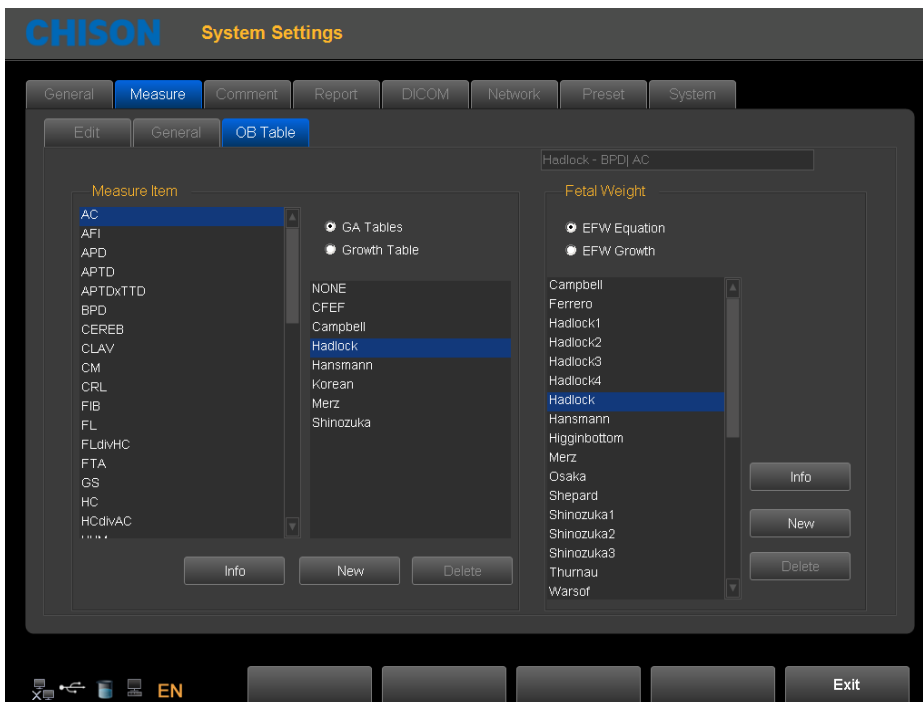
HR cycle options:

One cycle: one cycle to measure heart rate in the Doppler envelope calculation

Two cycles: two cycles to measure heart rate in the Doppler envelope calculated, the result will be more accurate.

Unit: You can set the unit of measurement results

OB Table Setting: Click OB Table page, you can set the formula of the OB measurement items for measuring fetal weight formula.




Set measurement Item: choose a measurement project, gestational age table or developmental table, and then choose a formula, click the information button, you can view the current formula. Click New, you can create a new formula in the current directory.

Fetal Weight: Select the EFW equation, or EFW development table, and then choose a formula, click on the information button, you can view the information of current fetal weight formula. Click New, you can create a new fetal weight formula in the current directory.

7.3 Comment:


Set the comment function, manage the comment Library

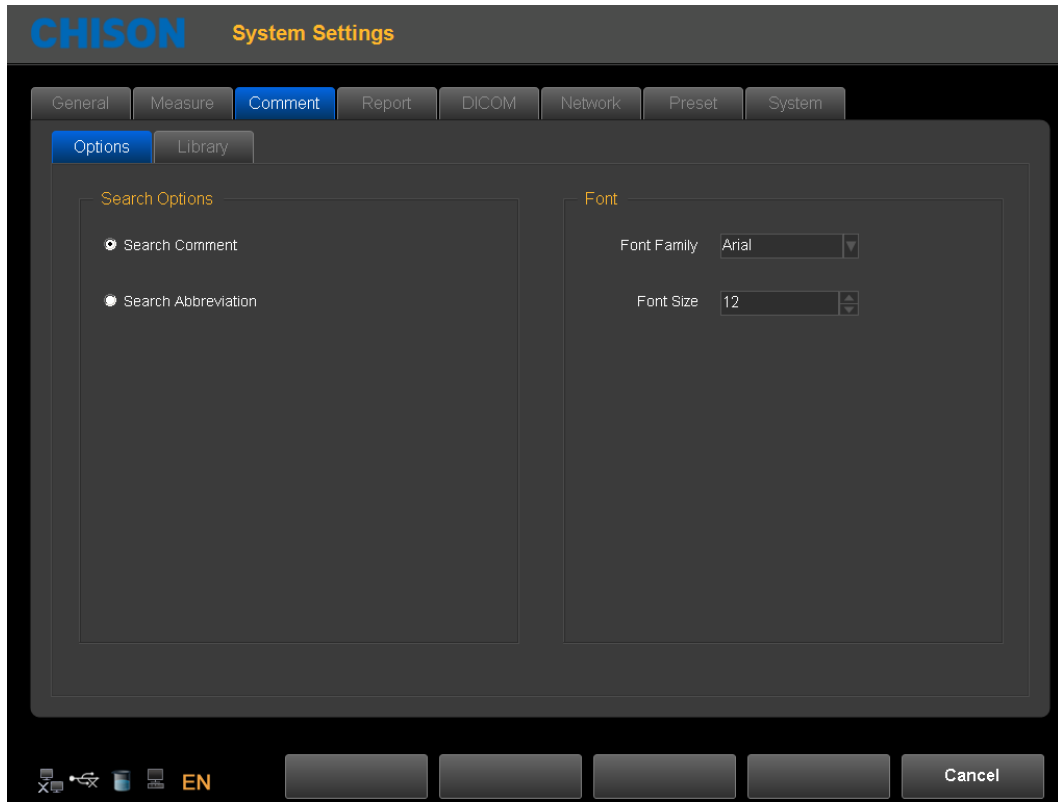Options Page:



Search option	Search comment or Abbreviations	Input search function Select the comment item, it will search notes when searching Select abbreviations, it will search abbreviations when searching
Font size	Font family	Set the font family of notes
	Font size	Set the font size of notes

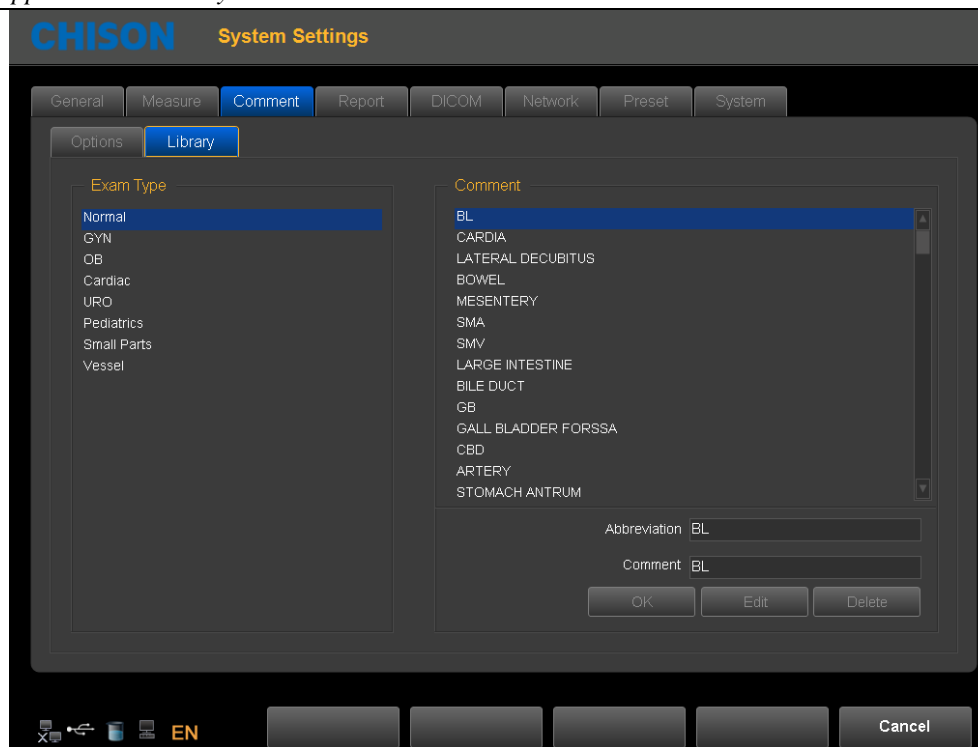Library Page: Edit and delete the annotation in the comment library.



Edit notes:

1. Select exam type, select annotation that needs to be edited
2. Input the comments in the lower right corner of the comment box. Input the abbreviations in the abbreviation box.
3. Press the edit-key, finish the edition of current comment.

Add notes:

1. Select the exam mode which the added notes belong to.
2. Input the comments in the lower right corner of the comment box. Input the abbreviations in the abbreviation box.
3. Press the SET-key, current comment will be added to the comment library.

Delete notes:

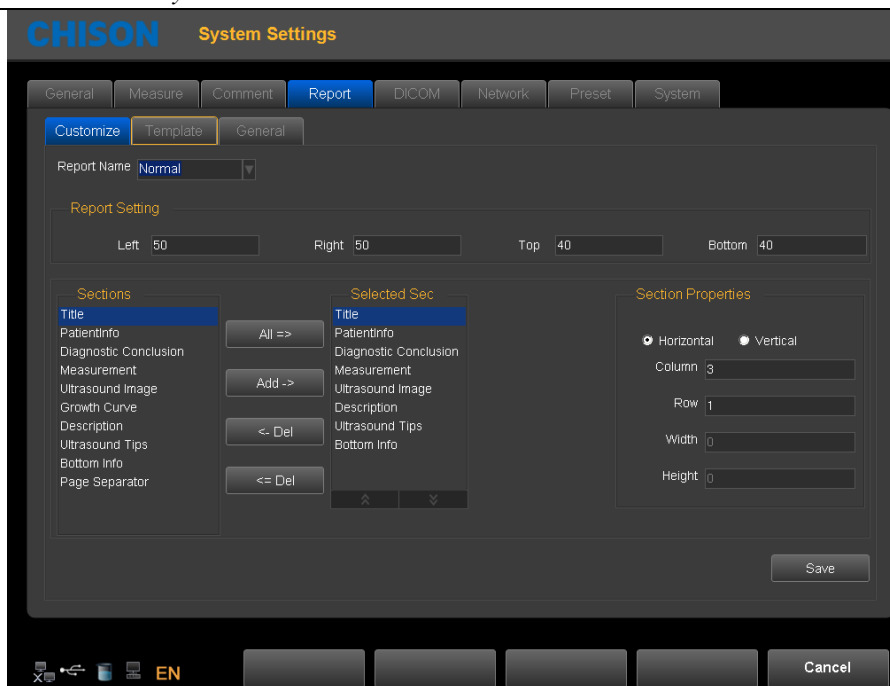
1. Select the exam mode, and select the notes that need to be deleted
2. Press delete-key to delete current comment.

Exit: exit current page

7.4 Report

Including the report design and report template

Customize settings: set the design of the report



Report Name: select report type

Left: Set left margins

Right: Set right margins

Top: Set top margins

Bottom: Set bottom margins

Sections: list all the sections which can be added

Selected sections: Section used in current report

All: Click this button to add " available section " to " selected section

Add: Select the section that need to be added, then click this button, add selected section "selected section"

Delete: Select the section that need to be deleted, then click this button, delete the selected section from the "selected section"

Delete all: Click this button to empty the content of "selected section"

Section attribute: Set the display of section, you can select arrange horizontally or vertically

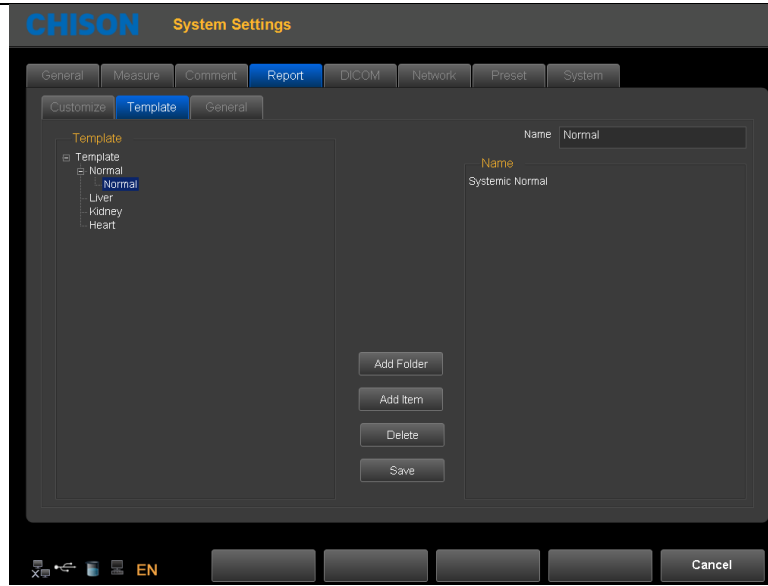
Set rows and columns of the arrangement.

Select the "Ultrasound image", set the width and height of the image

Save: After setting the current report, click the button, save the settings

Exit: Exit the page

Template Page:



Modify template note: Select the template note which you want to change, input modification content in the right box

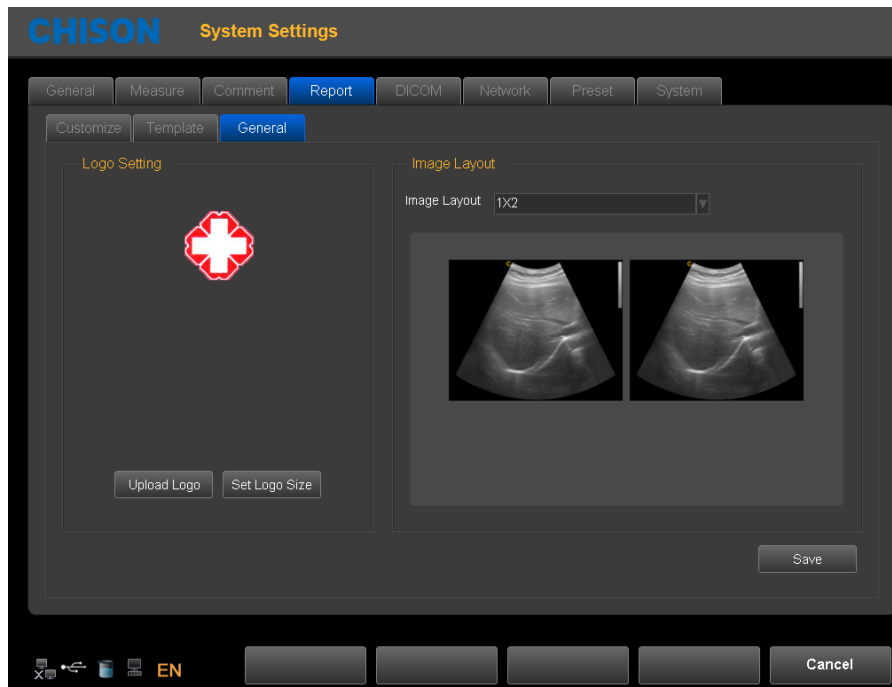
Add Folder: Press this icon to add a new Folder and name it.

Add Item: Press this icon to a new Item under current folder, and name it.

Delete: Press this icon to delete the template, which you selected.

Save: Save the changes.

General Page:



Logo Setting: Upload Logo and set logo size.


Image Layout: Choose different styles of image layout.

Save: Save the changes.

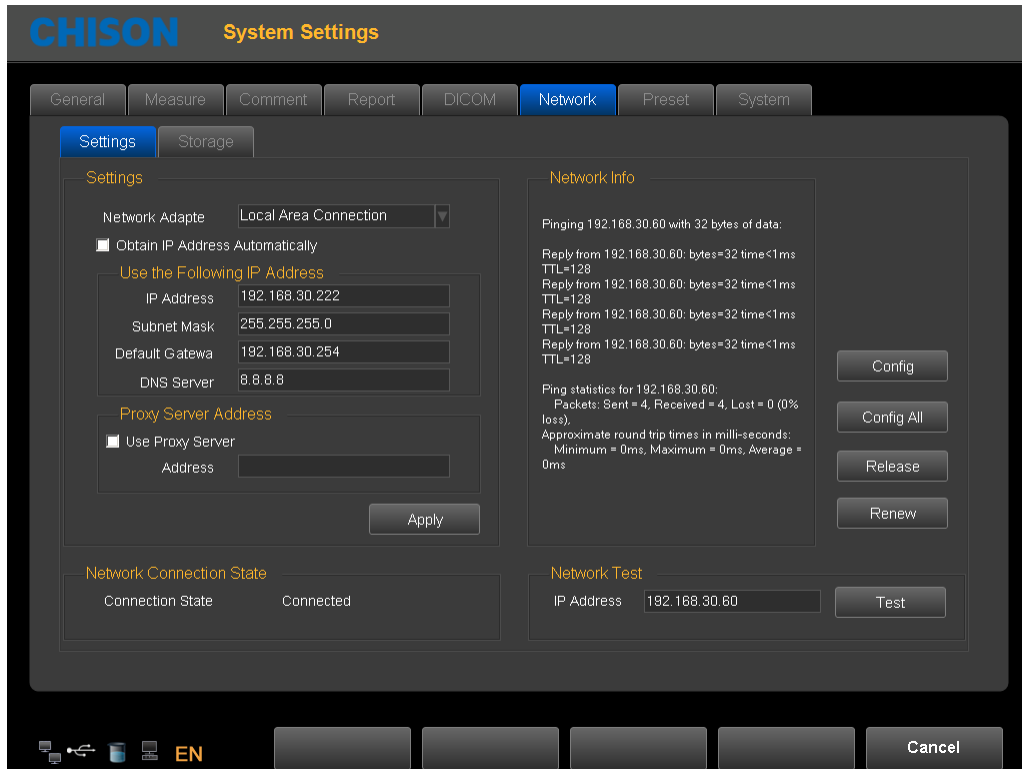
Note: the LOGO should be named with 'hLogo' in PNG format, which resolution is 168x169. And put LOGO file into the folder named 'hLogo'.

7.5 Network



Press  key to enter into the system setting unit. Click the page of Network Settings to enter into the network settings interface.

Before setting up the network settings, plug the cable into the LAN port of the system.



Network Settings

Network Adapter: Display the existing network cards on this system. The user can select the network card for current use.

Obtain an IP address automatically: When this option is selected, the system will obtain an IP address automatically

Use the following IP address (recommended): Set the IP address of the system manually. After setting, click Apply to confirm the application.

IP address: Input the IP address of the system manually, and make sure the IP address of the network printer is in the same network segment.

Subnet Mask: Input the subnet mask manually.

Default Gateway: Input the gateway manually.

DNS server: Input DNS server manually.

Network Connection Status: Check the network connection status.

Connected: The system is connected to the network successfully.

Unconnected: The system is not connected to the network.

Network Information: Display the current network information

Configuration information: Click this button to display the network configuration information

For more information: Click this button to display network details

Release of the connection: Click this button to disconnect the current connection

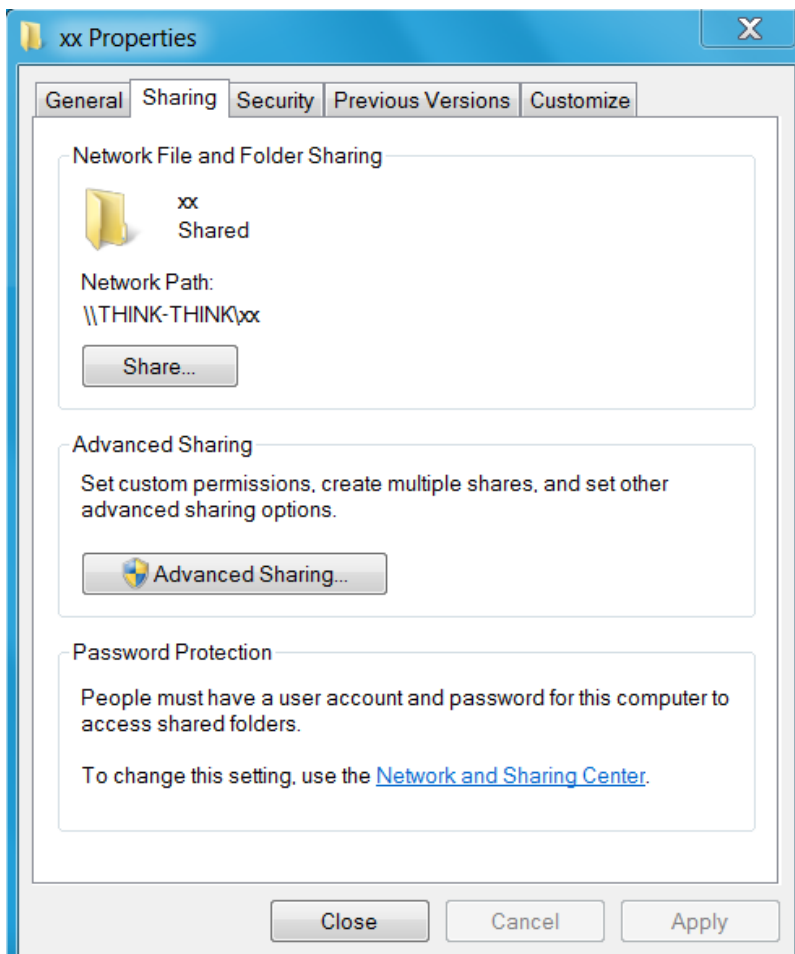
Update Connection: Click this button to update the current connection status

Network test: in the case of network connection status is connected, enter the IP address of the test, click the test, you can test the network the connection status between the IP of current system and the IP that needs to be tested.


Network Storage: With this feature, you can save the image to PC by network.

To use the network storage function, the user should connect the target computer to the network first, and build a new folder on the computer, right-click it to open the attribute, select shared page, set to share this folder on the network and allow network users to change the file (as shown below).


Note: the folder built on the computer is not allowed to sit on the desktop, otherwise create an error 67.



Click "Advanced Sharing"

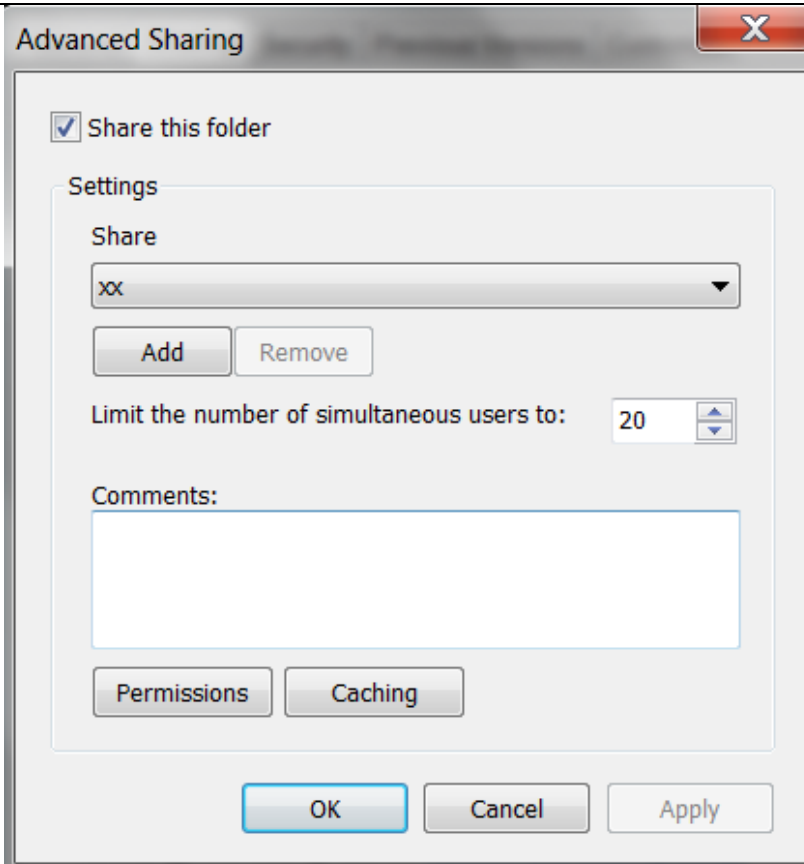


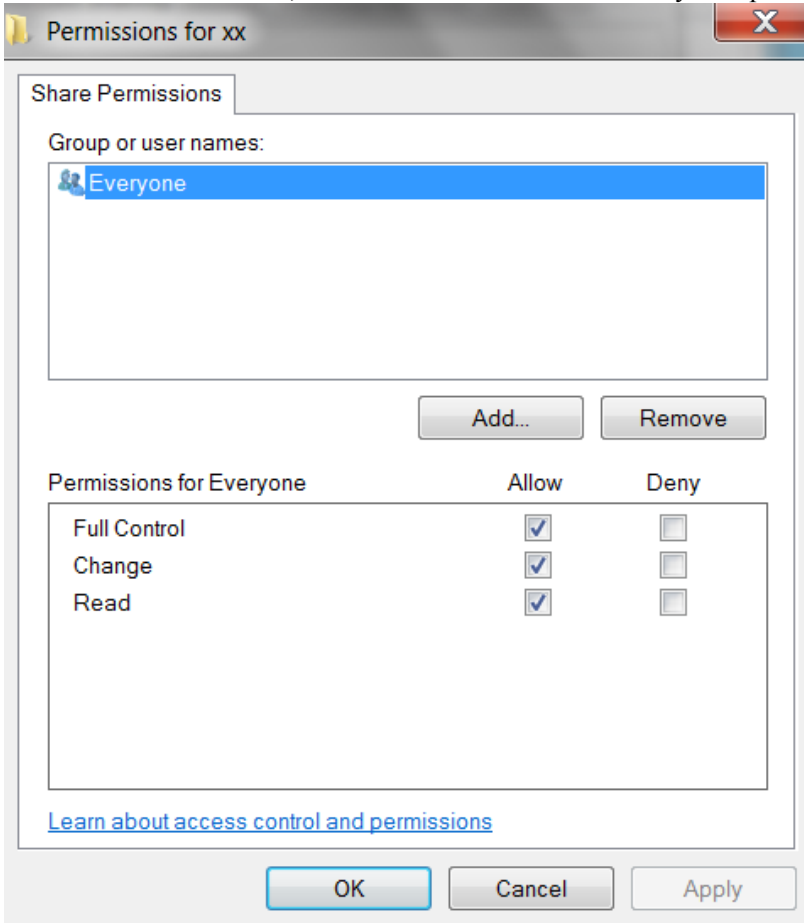Select “Share this folder”, and click “Permissions” to modify user permissions.



Select “Full Control”, “Change”, “Read” for “Allow”, click “OK” to complete modification.

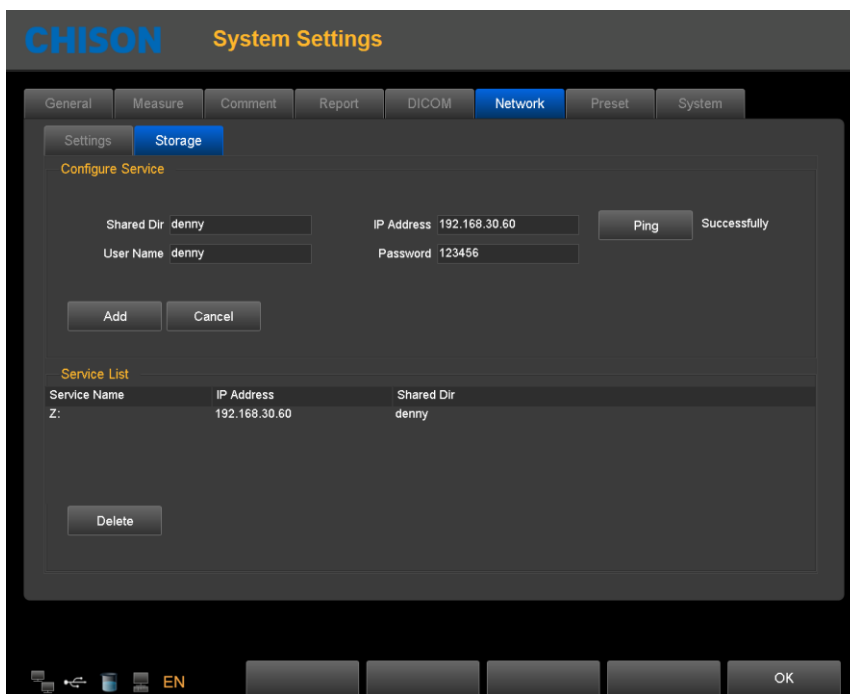Click Network Storage page, enter the network storage settings page. Then input shared directory name and IP address, and click Add.



Shared Dir: The shared directory name should be same as the shared folder name in target computer.

IP address: the IP address of the computer with the shared directory

User name: input the name of computer account

Password: input the password of computer account

Add: add a network storage service

Delete: can delete the selected network storage service

Note: you can add multi-numbers of network storage service to realize the transmission among multi systems.

7.6 System



Display the settings of system configuration information and functions

System information:

System SN: display the unit SN

Build time: The build time of software

Version: display the current software version and hardware version

Upgrade: When the U-disk with the upgrading software is inserted to the system, click this button to upgrade software.

Operating System: Display the operating system used by the current system

Function settings: DICOM, i-Image™, 4D, CFM mode, Pulse Doppler

Display the status of current function, and click the “turn on” button to turn on this function.

Note:

DICOM is option. If need to turn on this function, the user need to input the password. Please contact the CHISON's authorized service engineer to get the corresponding password.

Software Lease: Key Input

It is time to the data time, i8 cannot be entered for use. User can decode by pressing Key Input key and input the code which gets from CHISON's authorized service engineer.

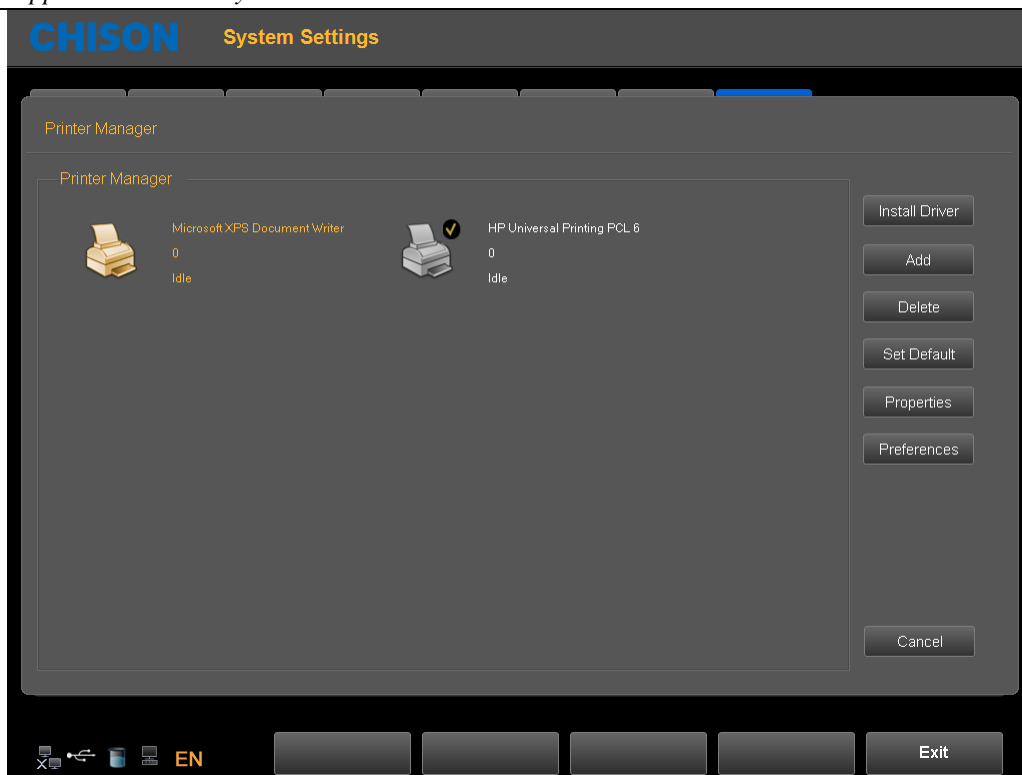
Log: Export Log

Plug in U disk to export log files which saved on system automatically for engineers to analyze.


License: Export License

Plug in U disk to export license to U Disk.

Printer management: Users can manage the printers. Click “Open” to into the printer management screen.




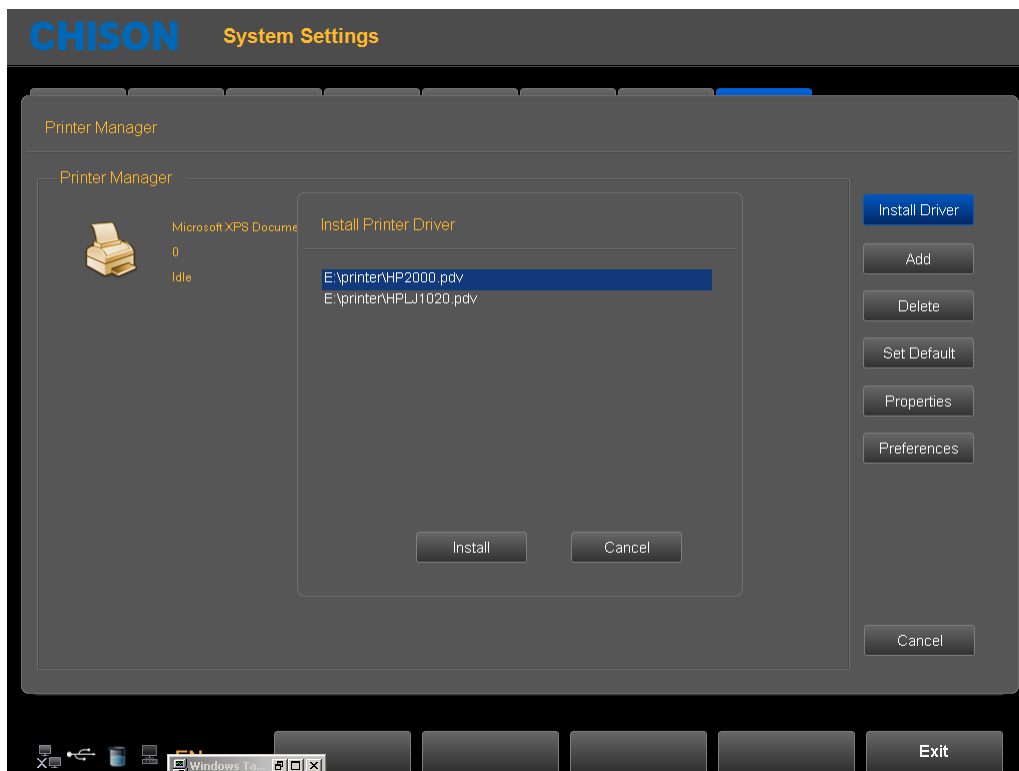Operating methods: insert U disk with installation driver to the system, click “Install Driver”,




Select the driver, click “Install”, the system will jump out installation guide. Complete the installation by following up the guide.

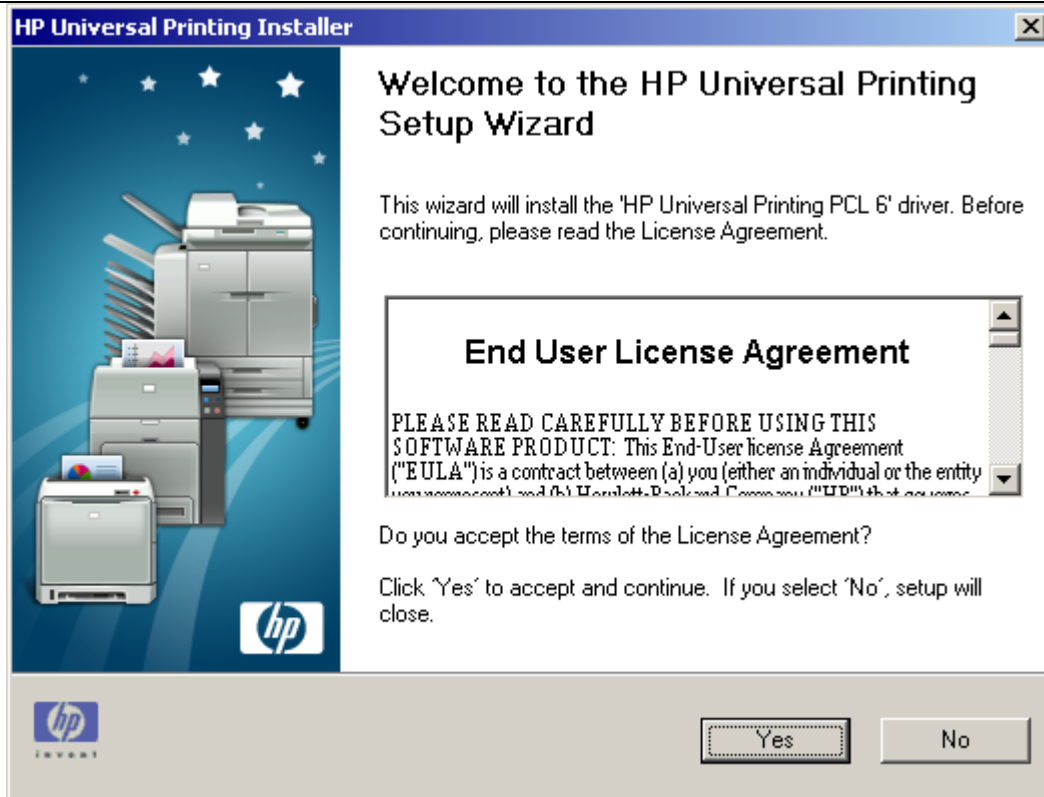
The steps are as below




Click “Yes”

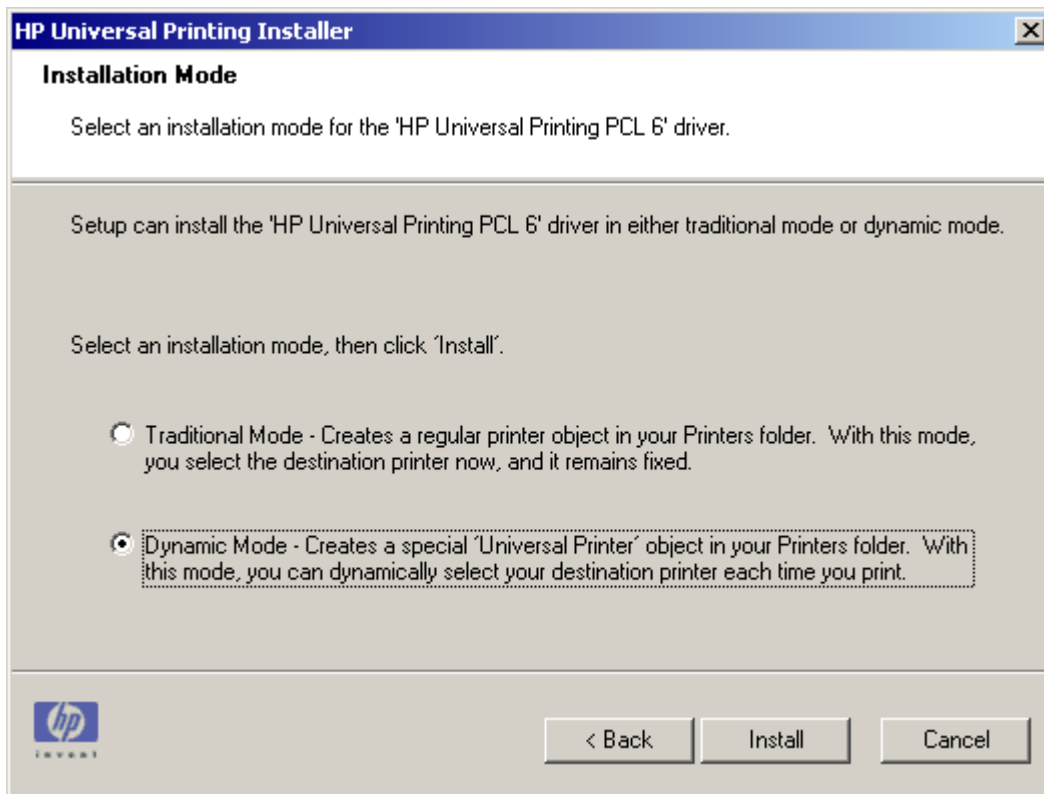

Select “Dynamic Mode”, click “Install”

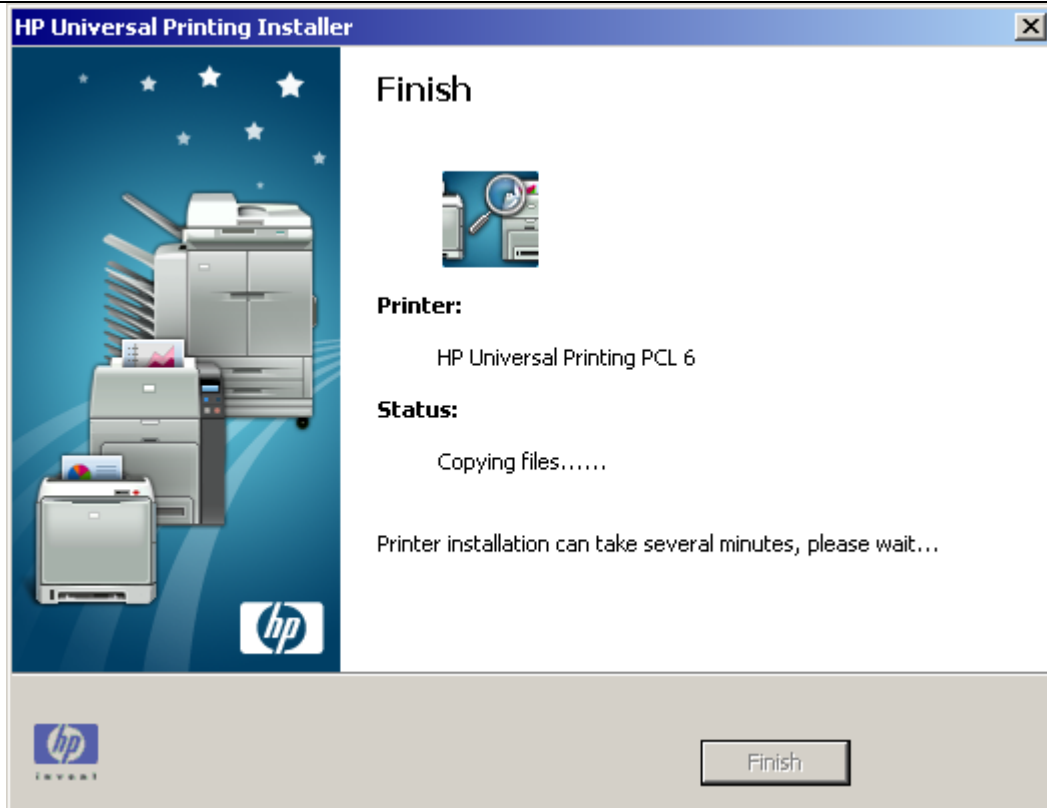

Being installed automatically

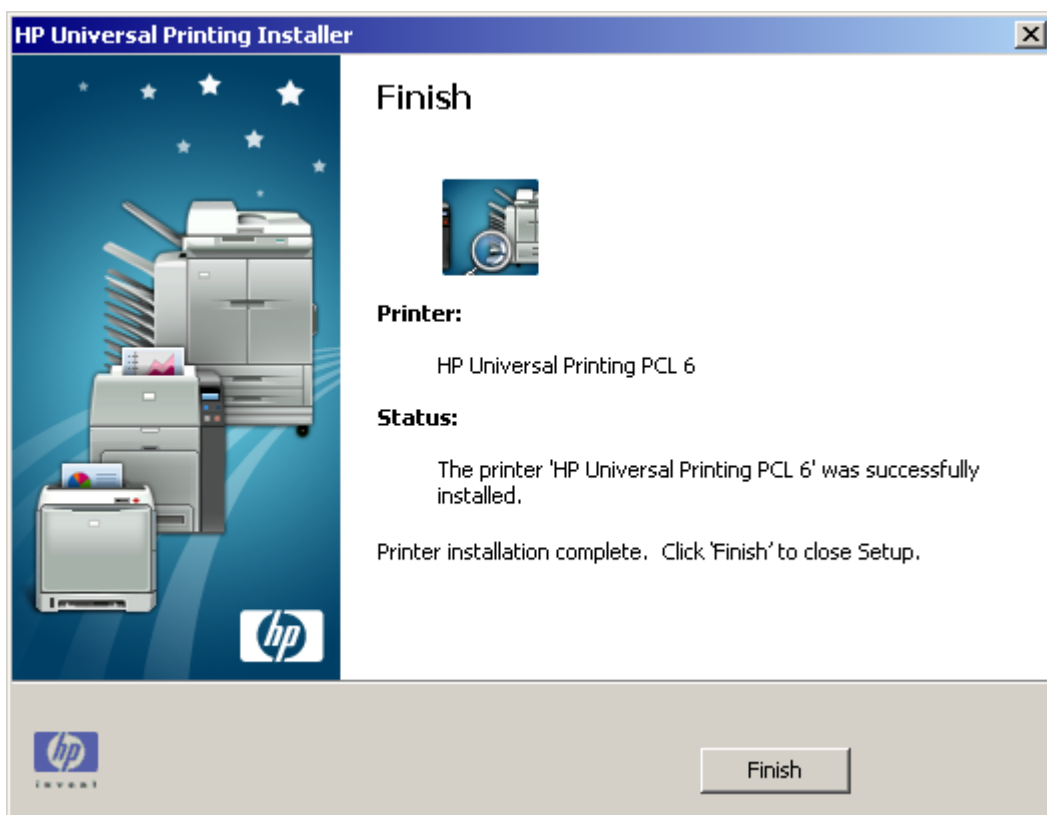


Click "Finish" to finish the printer installation.


Add: Click "Add", complete adding printer according to operation guide.



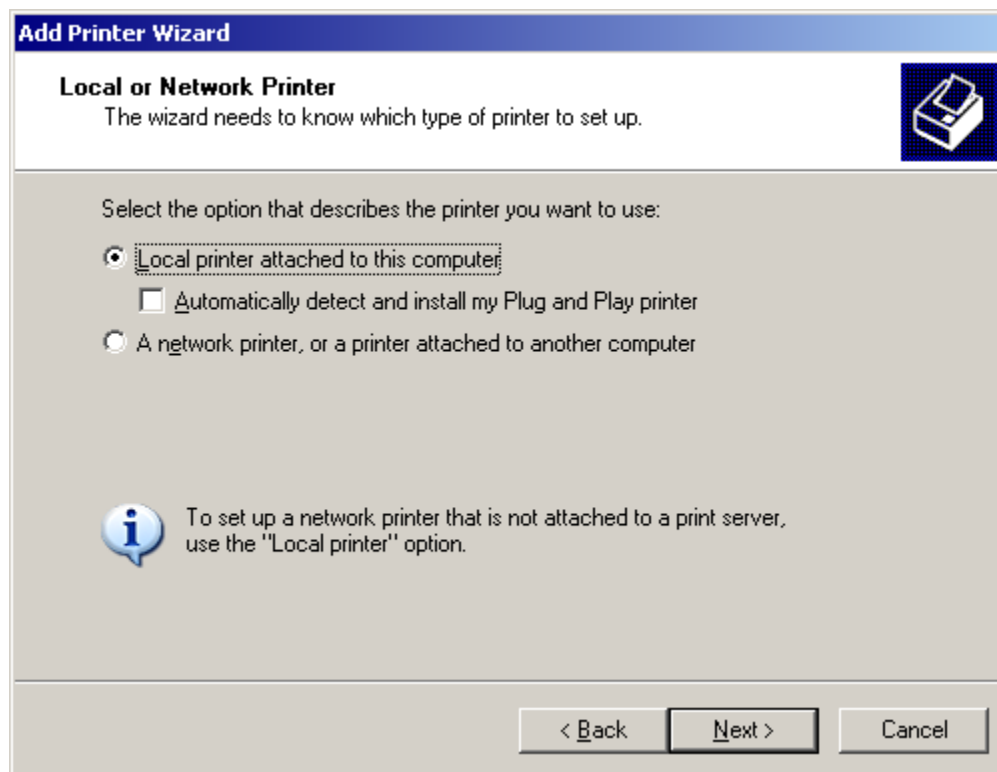
Click "NEXT"



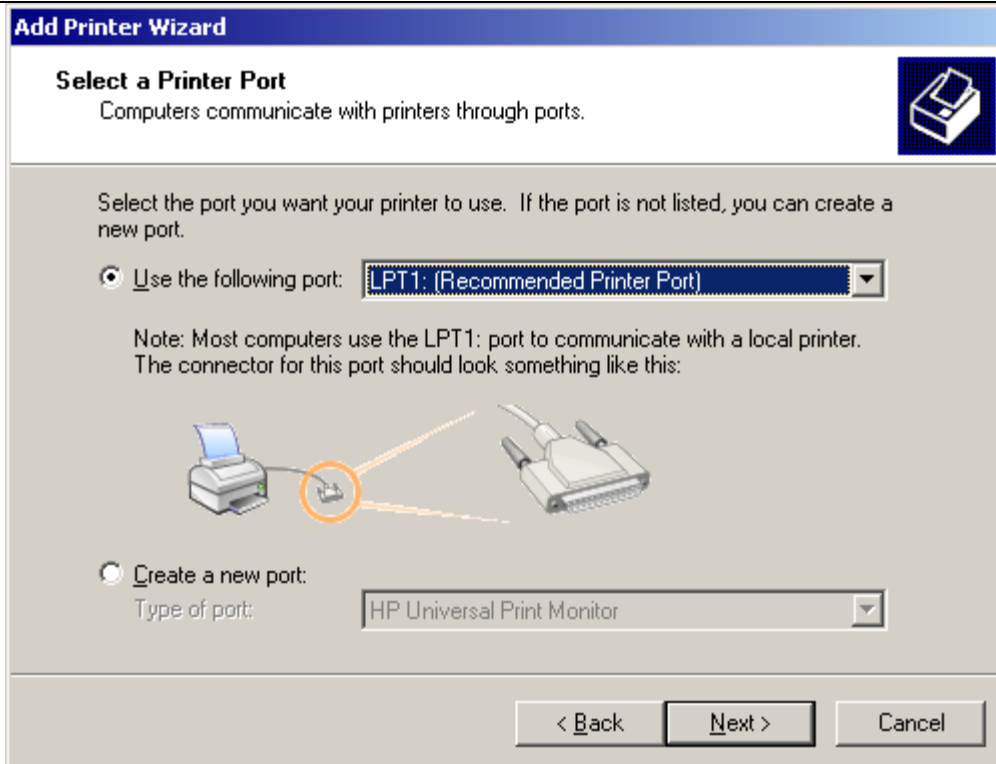
Select "Local printer attached to this computer", and Click "Next".



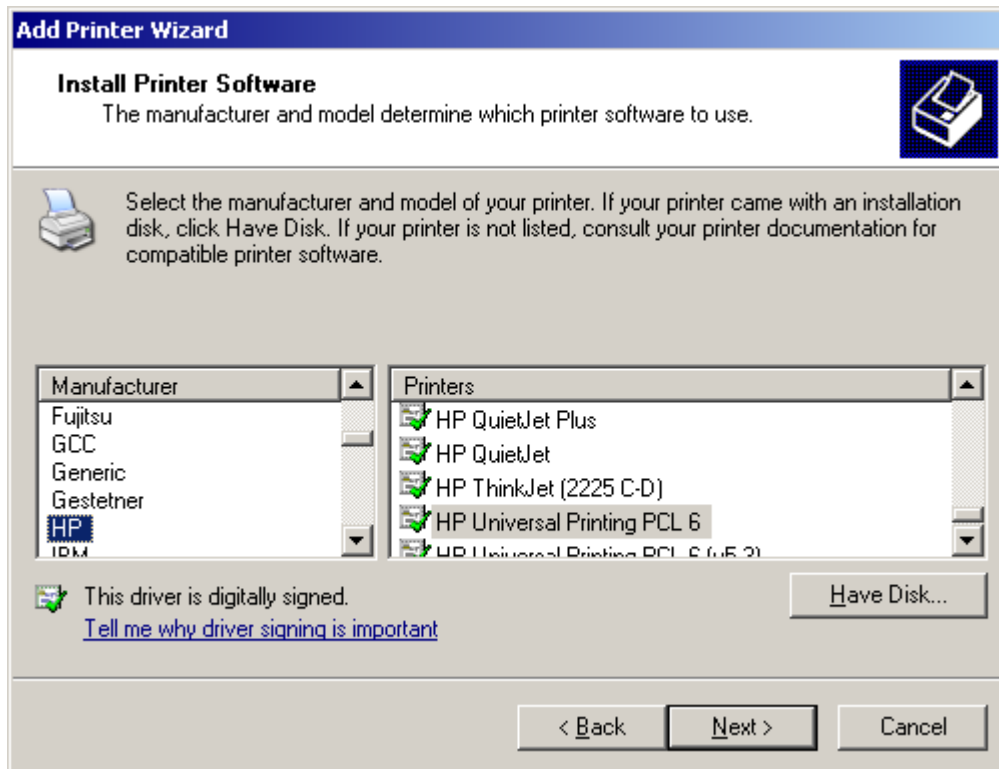
Select "Use the following port", and click "Next".




Select the printer, and click "Next".

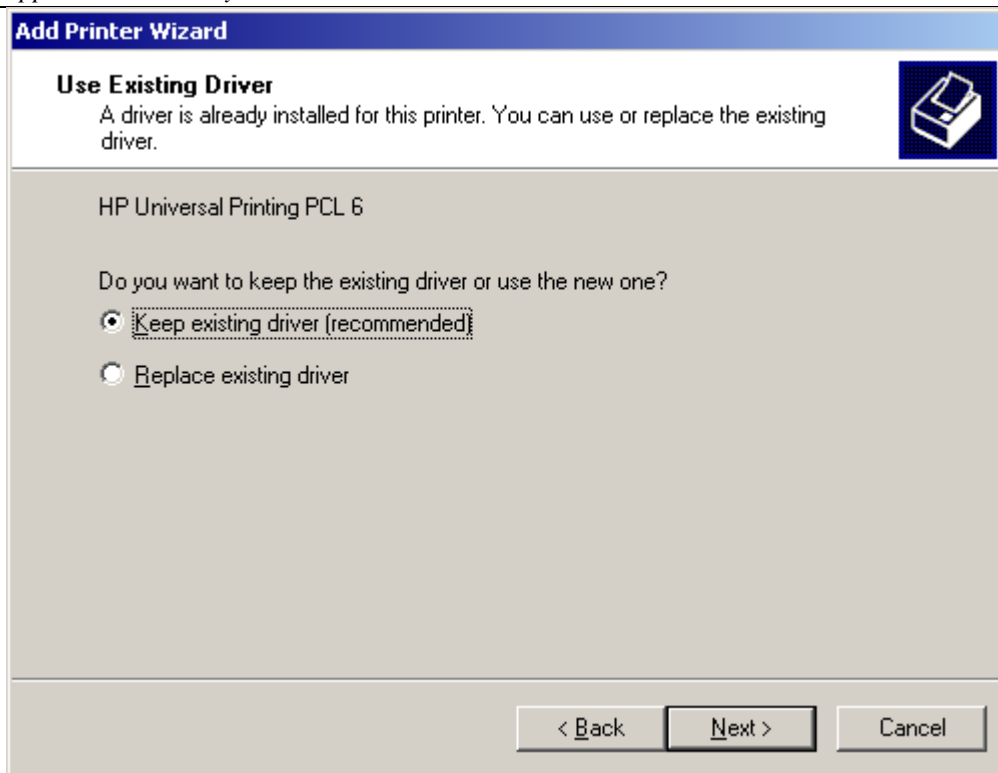

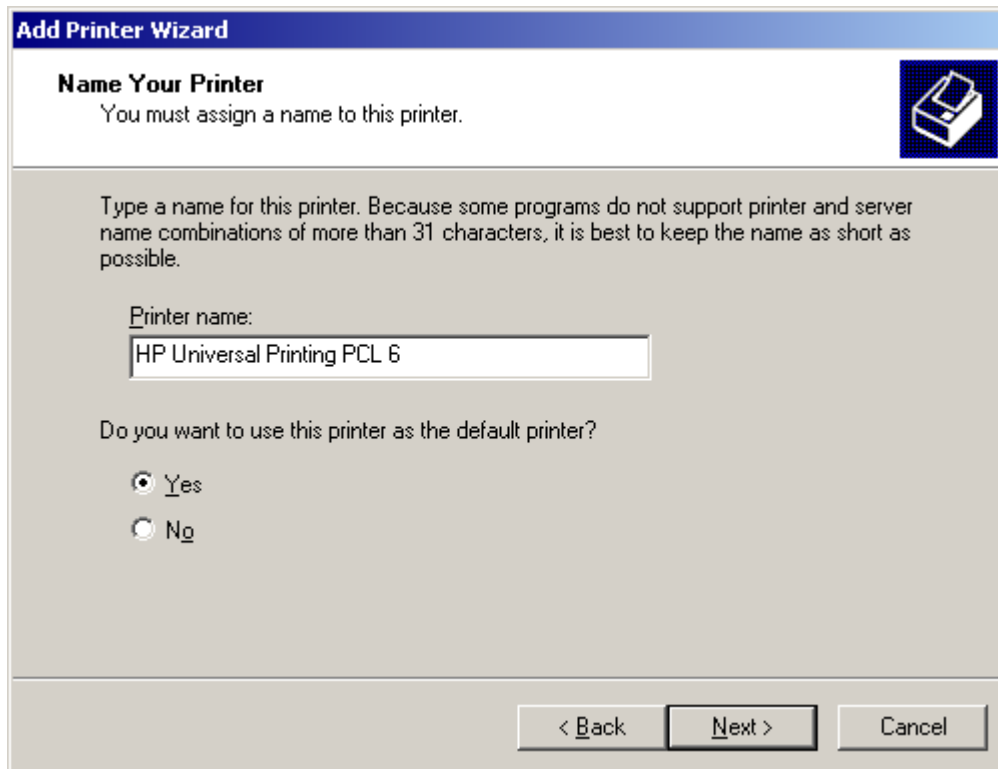
Select "Keep existing driver (recommended)", and Click "Next".



Select "Yes", and Click "Next".

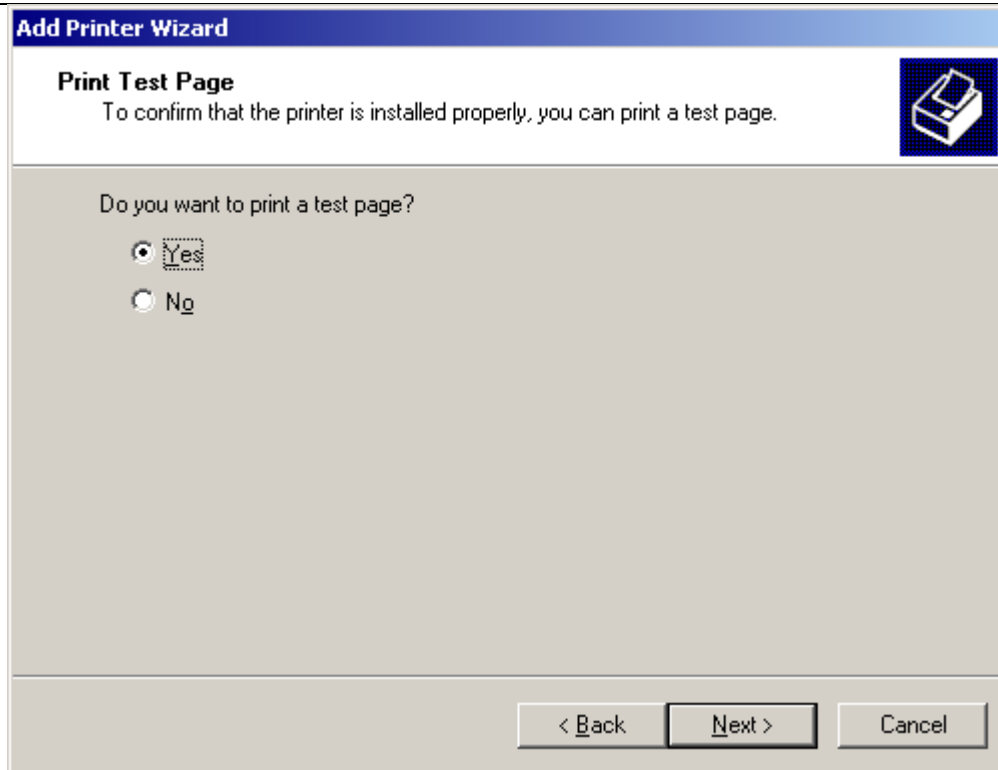


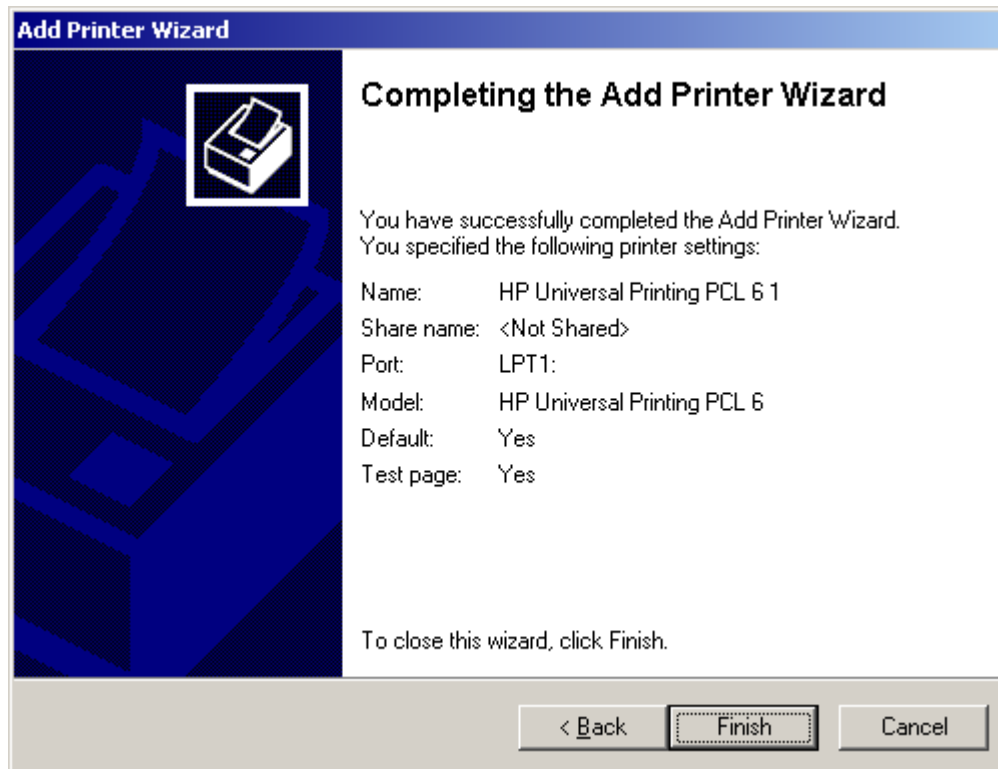Select "Yes" to print a test page, and click "Next" to continue.



Click "Finish".

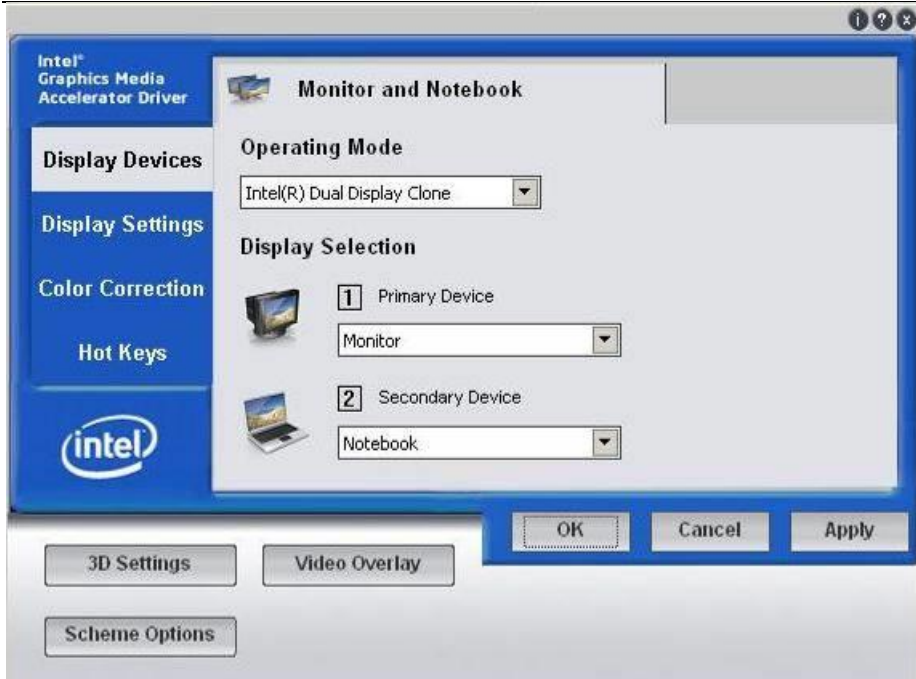Display Settings: The settings include display device, display Settings, color correction, short Keys etc..



Display Devices: set display device

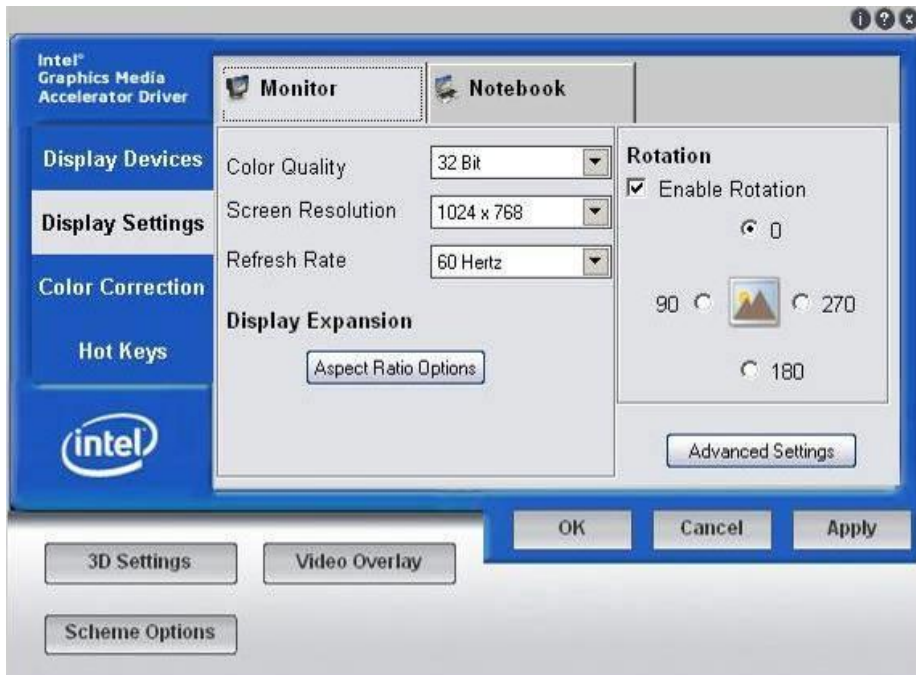
Operating Mode: Select display Mode. There are 3 options:

Single Display: Single display

Intel (R) Dual Display Clone: Intel (R) Dual Display copy, the display content of two screens are the same.

Extended Desktop: Expanded display

Display Selection: Select display device.




Display Settings: Settings of display.

Color Quality: 32Bit can be chosen.


Screen Resolution: Resolution of the Screen, it should be 1024*768.

Refresh Rate: Refresh Rate of screen, it should be 60Hertz.

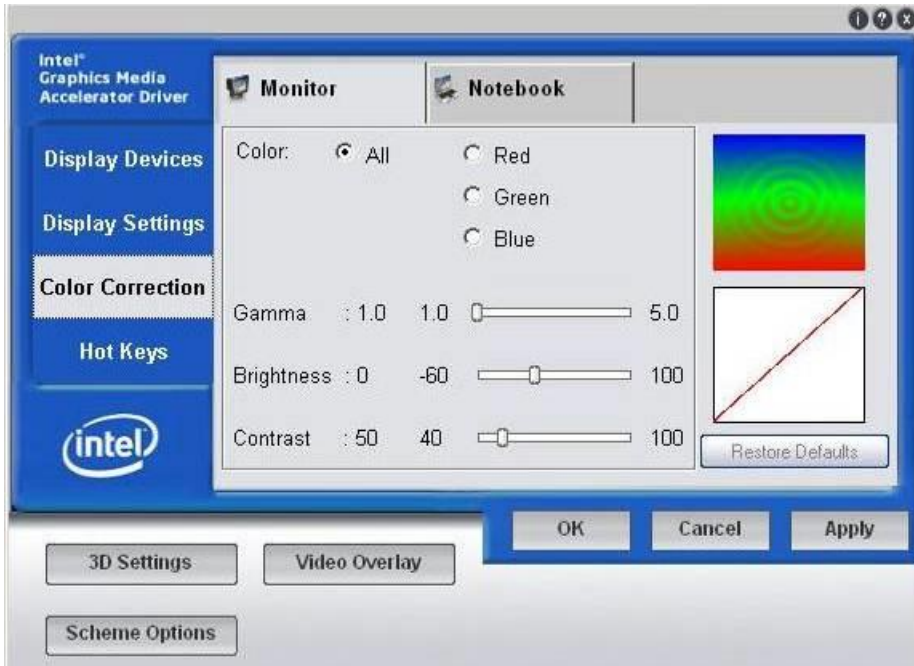Rotation: Rotation of the screen



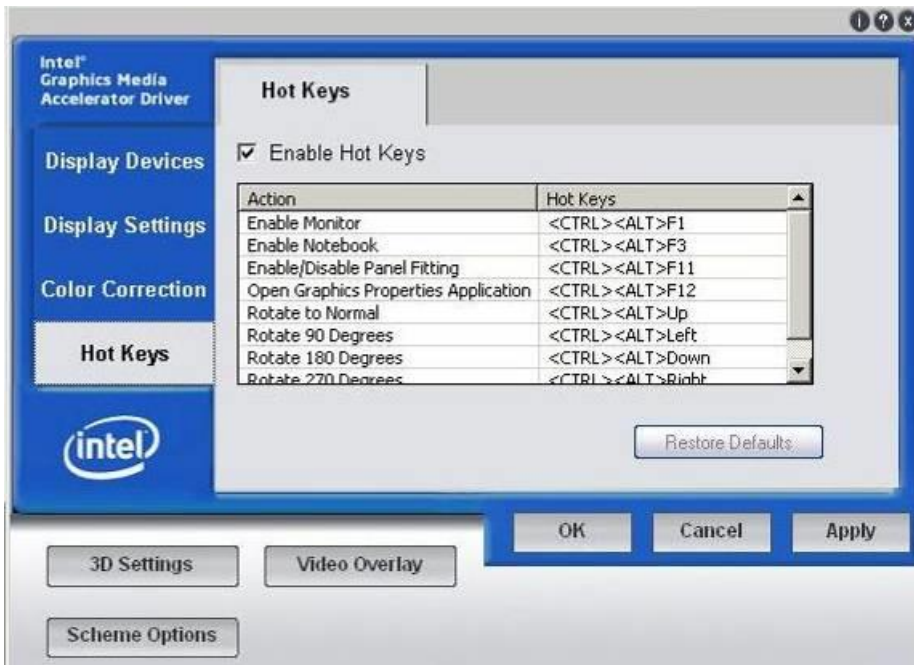Color Correction: In this interface, users can correct the color. Adjust the color of monitor.



Hot Keys: The setting of hot keys. In this interface, users can set the shortcut key.

CAUTION:

Please follow the instructions above strictly when changing the display setting or installing the printer driver. Any other changes are not allowed. If user changes to wrong settings, it may cause the system abnormal or malfunction. Any doubt or question, please contact CHISON's authorized service engineer.

Chapter 8 Probes

8.1. General Description

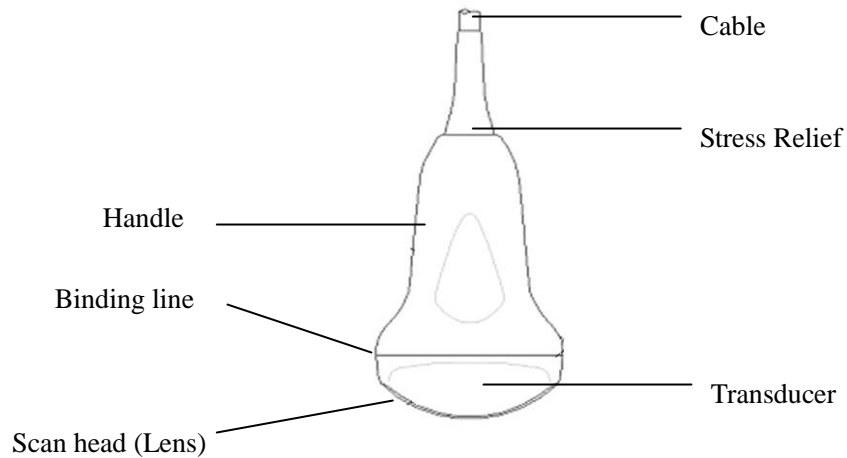


Fig.8-1: Convex Probe Overview

The probes provide high spatial and contrast ultrasound imaging of frequencies from 2.5MHz to 12.0MHz. These probes operate by pulsing sound waves into the body and listening to the returning echoes to produce high-resolution brightness mode, and a real time display.

8.2. Care and Maintenance

The probes that come with the system are designed to be durable and dependable. These precision instruments should be inspected daily and handled with care. Please observe the following precautions:

- Do not drop the transducer on hard surface. This can damage the transducer elements and compromise the electrical safety of the transducer.
- Avoid kinking or pinching the transducer cable.
- Use only approved ultrasonic coupling gels.

8.2.1 Inspecting Probes

Before and after each use, inspect carefully the probe's lens, cable, casing, and connector. Look for any damage that would allow liquid to enter the probe. If any damage is suspected, do not use the probe until it has been inspected and repaired/replaced by a CHISON's authorized service engineer.

NOTE:

Keep a log of all probe maintenance, along with a picture of any probe malfunction.

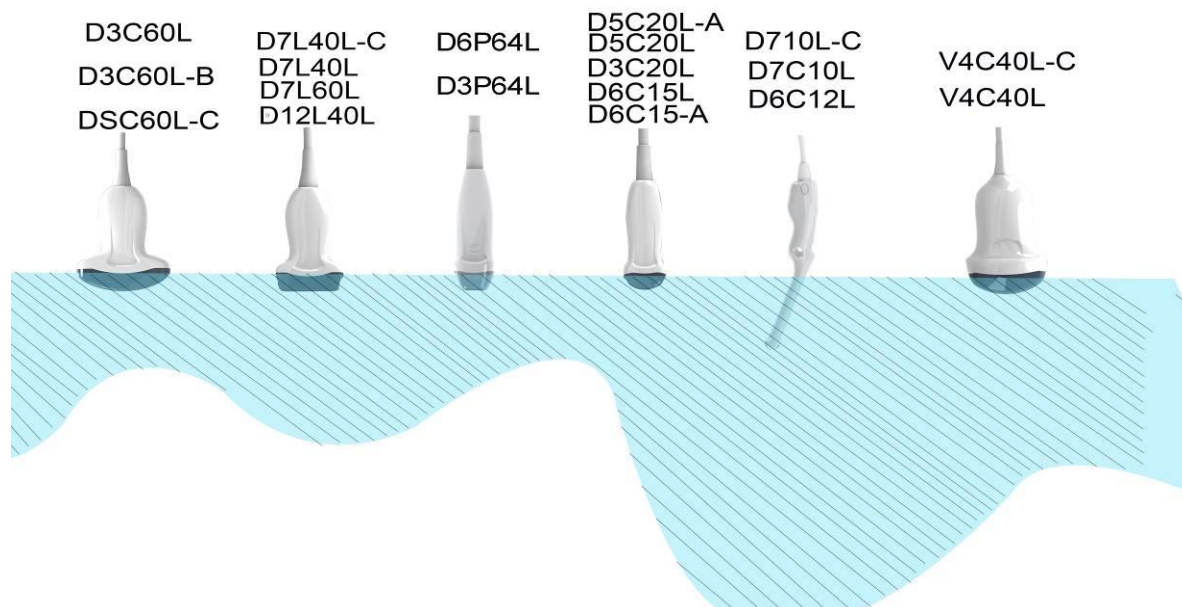
WARNING:

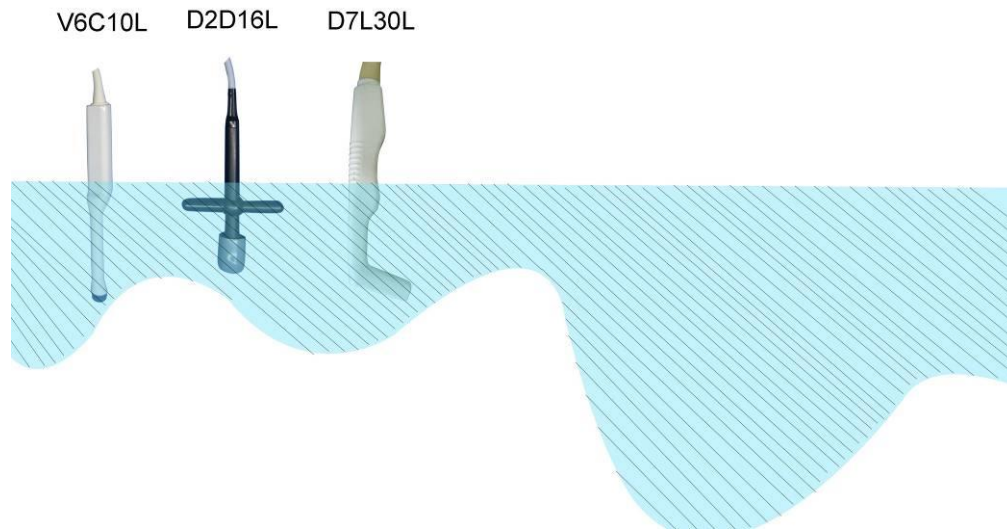
The probes are designed to be used only with this ultrasound system. Use of these probes on any other system or a non-qualified probe may cause electric shock or damage on the system/transducer.

8.2.2 Cleaning and Disinfecting

- Place the probe into the solution of cleaning-disinfectant. Make sure not to immerse the probe into the liquid beyond the immersion level given in the pictures below. Make sure that the probe is covered with the cleaning-disinfectant up to the immersion level during the complete disinfection time.
- For the recommended cleaning and disinfection time, please see your Operating Manual .
- Scrub the probe as needed using a soft sponge, gauze, or cloth to remove all visible residue from the probe surface.
- Rinse the probe with enough clean, potable water to remove all disinfectant residues.
- Use a soft cloth to clean the cable and the user section of the probe with the cleaning disinfectant liquid. Make sure that the surface of the probe and cable is wetted thoroughly with the cleaning-disinfectant.
- Allow probe to air dry completely.
- Reconnect the probe to the ultrasound console and place the probe into it's holder.

Probe Immersion Levels





CAUTION:

These transducers are not designed to withstand heat sterilization methods. Exposure to temperatures in excess of 60 ° C will cause permanent damage. The transducers are not designed to be totally submerged in fluid, as permanent damage will result if the entire transducer is submerged.

Probe Safety

Handling precautions

Ultrasound probes are highly sensitive medical instruments that can easily be damaged by improper handling. Use care when handling and protect from damage when not in use. DO NOT use a damaged or defective probe. Failure to follow these precautions can result in serious injury and equipment damage.

Electric shock hazard:

The probe is driven with electrical energy that can injure the patient or user if live internal parts are contacted by conductive solution:

- DO NOT immerse the probe into any liquid beyond the level indicated by the immersion level diagram. Never immerse the probe connector into any liquid.
- Prior to each use, visually inspect the probe lens and case area for cracks, cuts, tears, and other signs of physical damage. DO NOT use a probe that appears to be damaged until you verify functional and safe performance. You need to perform a more thorough inspection, including the cable, strain relief, and connector, each time you clean the probe.
- Before inserting the connector into the probe port, inspect the probe connector pins. If a pin is bent, DO NOT

use the probe until it has been inspected and repaired/replaced by a CHISON's authorized service engineer.

- Electrical leakage checks should be performed on a routine basis by CHISON's authorized service engineer.

Mechanical hazard:

A defective probe or excess force can cause patient injury or probe damage:

- Observe depth markings and do not apply excessive force when inserting or manipulating endocavitary probe.
- Inspect probes for sharp edges or rough surfaces that may injure sensitive tissue.
- DO NOT apply excessive force to the probe connector when inserting into the probe port. The pin of a probe connector may bend.

Special handling instructions

Using protective sheaths

The use of market cleared probe sheaths is recommended for clinical applications. Reference FDA March 29, 1991 "Medical Alert on Latex Products".

Protective sheaths may be required to minimize disease transmission. Probe sheaths are available for use with all clinical situations where infection is a concern. Use of legally marketed, sterile probe sheaths is strongly recommended for endo-cavitary procedures.

DO NOT use pre-lubricated condoms as a sheath. In some cases, they can damage the probe. Lubricants in these condoms may not be compatible with probe construction.

Devices containing latex may cause severe allergic reaction in latex sensitive individuals. Refer to FDA's March 29, 1991 Medical Alert on latex products.

DO NOT use an expired probe sheath. Before using a sheath, verify if it has expired.

Endocavitary Probe Handling Precautions

If the sterilization solution comes out of the endocavitary probe, please follow the cautions below:

Sterilant Exposure to Patient (e.g., Cidex): Contact with a sterilant to the patient's skin for mucous membrane may cause an inflammation. If this happens, refer to instruction manual of the sterilant.

Sterilant Exposure from Probe handle to Patient (e.g. Cidex): DO NOT allow the sterilant to contact the patient. Only immerse the probe to its specified level. Ensure that no solution has entered the probe's handle before scanning the patient. If sterilant comes into contact with the patient, refer to the sterilant's instruction manual.

Sterilant Exposure from Probe connector to Patient (e.g. Cidex): DO NOT allow the sterilant to contact the patient. Only immerse the probe to its specified level. Ensure that no solution has entered the probe's connector before scanning the patient. If sterilant comes into contact with the patient, refer to the sterilant's instruction manual.

Endocavitary Probe Point of Contact: Refer to the sterilant's instruction manual.

Probe handling and infection control:

This information is intended to increase user awareness of the risks of disease transmission associated with using this equipment and provide guidance in making decisions directly affecting the safety of the patient as well as the equipment user.

Diagnostic ultrasound systems utilize ultrasound energy that must be coupled to the patient by direct physical contact.

Depending on the type of examination, this contact occurs with a variety of tissues ranging from intact skin in a routine exam to recirculating blood in a surgical procedure. The level of risk of infection varies greatly with the type of contact.

One of the most effective ways to prevent transmission between patients is with single use or disposable devices. However, ultrasound transducers are complex and expensive devices that must be reused between patients. It is very important, therefore, to minimize the risk of disease transmission by using barriers and through proper processing between patients.

Risk of Infection

ALWAYS clean and disinfect the probe between patients to the level appropriate for the type of examination and use FDA-cleared probe sheaths where appropriate.

Adequate cleaning and disinfection are necessary to prevent disease transmission. It is the responsibility of the equipment user to verify and maintain the effectiveness of the infection control procedures in use. Always use sterile, legally marketed probe sheaths for intra-cavitary procedures.

Probe Cleaning process:

DO disconnect the probe from the system prior to cleaning/disinfecting the probe. Failure to do so could damage the system.

Perform Cleaning probe after each use

- Disconnect the probe from the ultrasound console and remove all coupling gel from the probe by wiping with a soft cloth and rinsing with flowing water.
- Wash the probe with mild soap in lukewarm water. Scrub the probe as needed using a soft sponge, gauze, or cloth to remove all visible residue from the probe surface. Prolonged soaking or scrubbing with a soft bristle brush (such as a toothbrush) may be necessary if material has dried onto the probe surface.

WARNING:

To avoid electric shock, always turn off the system and disconnect the probe before cleaning the probe.

CAUTION:

Take extra care when handling the lens face of the Ultrasound transducer. The lens face is especially sensitive and can easily be damaged by rough handling. NEVER use excessive force when cleaning the lens face.

- Rinse the probe with enough clean potable water to remove all visible soap residue.
- Air dry or dry with a soft cloth.

CAUTION:

To minimize the risk of infection from blood-borne pathogens, you must handle the probe and all disposables that have contacted blood, other potentially infectious materials, mucous membranes, and non-intact skin in accordance with infection control procedures. You must wear protective gloves when handling potentially infectious material. Use a face shield and gown if there is a risk of splashing or splatter.

Disinfecting the probes:

After each use, please disinfect the probes. Ultrasound probes can be disinfected using liquid chemical germicides. The level of disinfection is directly related to the duration of contact with the germicide. Increased contact time produces a higher level of disinfection.

In order for liquid chemical germicides to be effective, all visible residue must be removed during the cleaning process. Thoroughly clean the probe, as described earlier before attempting disinfection.

You MUST disconnect the probe from the system prior to cleaning/disinfecting the probe. Failure to do so could damage the system.

DO NOT soak probes in liquid chemical germicide for longer than is stated by the germicide instructions for use. Extended soaking may cause probe damage and early failure of the enclosure, resulting in possible electric shock hazard.

- Prepare the germicide solution according to the manufacturer's instructions. Be sure to follow all precautions for storage, use and disposal. The transducer is not designed to be totally submerged in fluid. Permanent damage will result if the entire transducer is submerged. The immersed part shall not exceed the transducer binding line.
- Place the cleaned and dried probe in contact with the germicide for the time specified by the germicide manufacturer. High-level disinfection is recommended for surface probes and is required for endocavitary probes (follow the germicide manufacturer's recommended time).
- After removing from the germicide, rinse the probe following the germicide manufacturer's rinsing instructions. Flush all visible germicide residue from the probe and allow to air dry.

Ultrasound transducers can easily be damaged by improper handling and by contact with certain chemicals.

Failure to follow these precautions can result in serious injury and equipment damage

- Do not immerse the probe into any liquid beyond the level specified for that probe. Never immerse the transducer connector or probe adapters into any liquid.
- Avoid mechanical shock or impact to the transducer and do not apply excessive bending or pulling force to the cable.
- Transducer damage can result from contact with inappropriate coupling or cleaning agents:
 - Do not soak or saturate transducers with solutions containing alcohol, bleach, ammonium chloride compounds or hydrogen peroxide
 - Avoid contact with solutions or coupling gels containing mineral oil or lanolin
 - Avoid temperatures above 60°C. Under no circumstances should the transducer be subjected to heat sterilization method. Exposure to temperatures above 60° C will cause permanent damage to the transducer.
- Inspect the probe prior to use for damage or degeneration to the housing, strain relief, lens and seal. Do not use a damaged or defective probe.

Coupling gels

AQUASONIC Gel made by R. P. Kincheloe Company in USA is recommended.

In order to assure optimal transmission of energy between the patient and probe, a conductive gel must be applied liberally to the patient where scanning will be performed.

CAUTION:

Please do not use any gel or other materials which are not provided by CHISON. Un-authorized gel, lubricants and other materials may corrode probes and other parts of the device, for example the keyboard. This may reduce the safety and effectiveness of the system and probes, and may also reduce the life time of the systems and probes. Damages caused by such reason will not be covered by the warranty.

DO NOT apply gel to the eyes. If there is gel contact to the eye, flush eye thoroughly with water.

Coupling gels should not contain the following ingredients as they are known to cause probe damage:

- Methanol, ethanol, isopropanol, or any other alcohol-based product.
- Mineral oil
- Iodine
- Lotions
- Lanolin
- Aloe Vera
- Olive Oil

- Methyl or Ethyl Parabens (para hydroxybenzoic acid)
- Dimethylsilicone

Planned maintenance

The following maintenance plan is suggested for the system and probes to ensure optimum operation and safety.

Daily: inspect the probes

After each use: clean the probes disinfect the probes.

As necessary: inspect the probes, clean the probes, and disinfect the probes.

Returning/Shipping Probes and Repair Parts

Transportation dept. and our policy require that equipment returned for service **MUST** be clean and free of blood and other infectious substances.

When you return a probe or part for service, you need to clean and disinfect the probe or part prior to packing and shipping the equipment.

Ensure that you follow probe cleaning and disinfection instructions provided in this Manual.

This ensures that employees in the transportation industry as well as the people who receive the package are protected from any risk.

AIUM outlines cleaning the endocavitary transducer:

Guidelines for Cleaning and Preparing Endocavitary Ultrasound Transducers Between Patients From AIUM

Approved June 4, 2003

The purpose of this document is to provide guidance regarding the cleaning and disinfection of transvaginal and transrectal ultrasound probes.

All sterilization/disinfection represents a statistical reduction in the number of microbes present on a surface. Meticulous cleaning of the instrument is the essential key to an initial reduction of the microbial/organic load by at least 99%. This cleaning is followed by a disinfecting procedure to ensure a high degree of protection from infectious disease transmission, even if a disposable barrier covers the instrument during use.

Medical instruments fall into different categories with respect to potential for infection transmission. The most critical level of instruments are those that are intended to penetrate skin or mucous membranes. These require sterilization. Less critical instruments (often called "semi-critical" instruments) that simply come into contact with mucous membranes such as fiber optic endoscopes require high-level disinfection rather than sterilization.

Although endocavitary ultrasound probes might be considered even less critical instruments because they are routinely protected by single use disposable probe covers, leakage rates of 0.9% - 2% for condoms and 8%-81% for commercial probe covers have been observed in recent studies. For maximum safety, one should therefore perform high-level disinfection of the probe between each use and use a probe cover or condom as an aid in keeping the probe clean.

There are four generally recognized categories of disinfection and sterilization. Sterilization is the complete elimination of all forms of microbial life including spores and viruses.

Disinfection, the selective removal of microbial life, is divided into three classes:

High-Level Disinfection - Destruction/removal of all microorganisms except bacterial spores.

Mid-Level Disinfection - Inactivation of Mycobacterium Tuberculosis, bacteria, most viruses, fungi, and some bacterial spores.

Low-Level Disinfection - Destruction of most bacteria, some viruses and some fungi. Low-level disinfection will not necessarily inactivate Mycobacterium Tuberculosis or bacterial spores.

The following specific recommendations are made for the use of Endocavitary ultrasound transducers. Users should also review the Centers for Disease Control and Prevention document on sterilization and disinfection of medical devices to be certain that their procedures conform to the CDC principles for disinfection of patient care equipment.

1. CLEANING

After removal of the probe cover, use running water to remove any residual gel or debris from the probe. Use a damp gauze pad or other soft cloth and a small amount of mild non-abrasive liquid soap (household dishwashing liquid is ideal) to thoroughly cleanse the transducer. Consider the use of a small brush especially for crevices and areas of angulation depending on the design of your particular transducer. Rinse the transducer thoroughly with running water, and then dry the transducer with a soft cloth or paper towel.

2. DISINFECTION

Cleaning with a detergent/water solution as described above is important as the first step in proper disinfection since chemical disinfectants act more rapidly on clean surfaces. However, the additional use of a high level liquid disinfectant will ensure further statistical reduction in microbial load. Because of the potential disruption of the barrier sheath, additional high level disinfection with chemical agents is necessary. Examples of such high level disinfectants include but are not limited to:

- 2-3.2% glutaraldehyde products (a variety of available proprietary products including "Cidex," "Metricide," or "Procide").
- Non-glutaraldehyde agents including Cidex OPA (o-phthalaldehyde), Cidex PA (hydrogen peroxide & peroxyacetic acid).
- 7.5% Hydrogen Peroxide solution.

- Common household bleach (5.25% sodium hypochlorite) diluted to yield 500 parts per million chlorine (10 cc in one liter of tap water). This agent is effective, but generally not recommended by probe manufacturers because it can damage metal and plastic parts.

Other agents such as quaternary ammonium compounds are not considered high level disinfectants and should not be used. Isopropanol is not a high level disinfectant when used as a wipe and probe manufacturers generally do not recommend soaking probes in the liquid.

The FDA has published a list of approved sterilants and high level disinfectants for use in processing reusable medical and dental devices. That list can be consulted to find agents that may be useful for probe disinfection.

Practitioners should consult the labels of proprietary products for specific instructions. They should also consult instrument manufacturers regarding compatibility of these agents with probes. Many of the chemical disinfectants are potentially toxic and many require adequate precautions such as proper ventilation, personal protective devices (gloves, face/eye protection, etc.) and thorough rinsing before reuse of the probe.

3. PROBE COVERS

The transducer should be covered with a barrier. If the barriers used are condoms, these should be non-lubricated and non-medicated. Practitioners should be aware that condoms have been shown to be less prone to leakage than commercial probe covers, and have a six-fold enhanced AQL (acceptable quality level) when compared to standard examination gloves. They have an AQL equal to that of surgical gloves. Users should be aware of latex-sensitivity issues and have available non-latex-containing barriers.

4. ASEPTIC TECHNIQUE

For the protection of the patient and the health care worker, all endocavitary examinations should be performed with the operator properly gloved throughout the procedure. Gloves should be used to remove the condom or other barrier from the transducer and to wash the transducer as outlined above. As the barrier (condom) is removed, care should be taken not to contaminate the probe with secretions from the patient. At the completion of the procedure, hands should be thoroughly washed with soap and water.

Note: *Obvious disruption in condom integrity does NOT require modification of this protocol. These guidelines take into account possible probe contamination due to a disruption in the barrier sheath.*

In summary, routine high-level disinfection of the endocavitary probe between patients, plus the use of a probe cover or condom during each examination is required to properly protect patients from infection during endocavitary examinations. For all chemical disinfectants, precautions must be taken to protect workers and patients from the toxicity of the disinfectant.

Amis S, Ruddy M, Kibbler CC, Economides DL, MacLean AB. Assessment of condoms as probe covers for transvaginal sonography. *J Clin Ultrasound* 2000;28:295-8.

Rooks VJ, Yancey MK, Elg SA, Brueske L. Comparison of probe sheaths for endovaginal sonography. *Obstet. Gynecol* 1996;87:27-9.

Milki AA, Fisch JD. Vaginal ultrasound probe cover leakage: implications for patient care. *Fertil Steril* 1998;69:409-11.

Hignett M, Claman P. High rates of perforation are found in endovaginal ultrasound probe covers before and after oocyte retrieval for in vitro fertilization-embryo transfer. *J Assist Reprod Genet* 1995;12:606-9.

Sterilization and Disinfection of Medical Devices: General Principles. Centers for Disease Control, Division of Healthcare Quality Promotion. <http://www.cdc.gov/ncidod/hip/sterile/sterilgp.htm> (5-2003).

ODE Device Evaluation Information--FDA Cleared Sterilants and High Level Disinfectants with General Claims for Processing Reusable Medical and Dental Devices, March 2003. <http://www.fda.gov/cdrh/ode/germlab.html> (5-2003).

8.3 Probe Operation Instructions

For details on connecting, activating, deactivating, disconnecting, transporting and storing the probes, see Section 3.7 “Probes” in Chapter 3.

8.3.1 Scanning the Patient

In order to assure optimal transmission of energy between the patient and probe, a conductive gel must be applied liberally to the patient where scanning will be performed.

After the examination is complete, follow the cleaning and disinfecting, or sterilizing procedures as appropriate.

8.3.2 Operating Transvaginal probe

The transvaginal probe is an endo-cavity probe, for the operation safety, please refer to “Care and Maintenance” for cleaning and disinfection.

Transvaginal probe should be used with FDA approved condom or probe cover. See the following instructions to put the probe into the condom:

CAUTION:

- *Some patients may be allergic to natural rubber or medical device with rubber contains. FDA suggests that the user to identify these patients and be prepared to treat allergic reactions promptly before scanning.*
- *Only water-solvable solutions or gel can be used. Petroleum or mineral oil-based materials may harm the cover.*
- *When the transvaginal probe is activated outside patient's body, its acoustic output level should be decreased to avoid any harmful interference with other equipment.*

Operation Procedure:

- Put on medical sterile glove
- Get the condom for the package.
- Unfold the condom.
- Load some ultrasound gel into condom.
- Take the condom with one hand, and put the probe head into the condom.
- Fasten the condom on the end of the probe handle.
- Confirm the integrity of the condom, and repeat the above steps to the condom if any damage to the condom is found.

8.3.3 Cleaning and Disinfecting TV Probe

We strongly recommend wearing gloves when cleaning and disinfecting any endocavitary probe.

- Every time before and after each exam, please clean the probe handle and disinfect the transvaginal probe using liquid chemical germicides
- If the probe is contaminated with body fluids, you should disinfect the probe after cleaning.
- Regard any exam waste as potentially infectious and dispose of it accordingly.

CAUTION:

- *Since the probe is not waterproof, you should disconnect it from the system before cleaning or disinfecting.*

Before and after each exam, please clean the probe handle and disinfect the transvaginal probe using liquid chemical germicides.

Cleaning

- You can clean the transvaginal probe to remove all coupling gel by wiping with a soft cloth and rinsing with flowing water. Then wash the probe with mild soap in lukewarm water. Scrub the probe as needed and use a soft cloth to remove all visible residues from the transvaginal probe surface. Rinse the probe with enough clean

potable water to remove all visible soap residues, and let the probe air dry.

CAUTION:

- *Please remove the cover (if any) before cleaning the probe. (The cover like condom is one time usable).*
- *When cleaning the TV probe, it is important to be sure that all surfaces are thoroughly cleaned.*
- Disinfecting
- 2% Glutaraldehyde-based solutions have been shown to be very effective for this purpose. Cidex is the only germicide that has been evaluated for compatibility with the material used to construct the probes.

To keep the effectiveness of the disinfection solutions, a thoroughly cleaning must be done to the probe before the disinfecting, make sure no residues remain on the probe.

Disinfecting Procedure:

- Following all precautions for storage, use and disposal, prepare the germicide solution according to the manufacturer's instructions.
- Place the cleaned and dried probe to contact with the germicide, being careful not to let the probe drop to the bottom of the container and thus damage the probe.
- After placing/immersing, rotate and shake the probe while it is below the surface of the germicide to eliminate air pockets. Allow the germicide to remain in contact with the fully immersed probe. For high level disinfection, follow the manufacturer's recommended time.
- Following all precautions for storage, use and disposal, prepare the germicide solution according to the manufacturer's instructions.
- After removing from the germicide, rinse the probe according to the germicide manufacturer's rinsing instructions.
- Flush all visible germicide residues from the probe and allow to air dry.

8.3.4 Thermal and Tip Angle Monitoring

The Transvaginal probe is equipped with a thermal sensor providing continuous feed-back to the system on the temperature of the probe tip. The tip temperature is displayed in Celsius(° C). Whenever the probe is activated , the temperature and tip angle are displayed on the screen. An optional patient body temperature is entered in the system menu, and the default is 37 degree C.

For maximum safety, the temperature will flash when the tip temperature reach 41° C, and the system has a thermal limit of 43° C; when the probe tip reaches this limit, the system will automatically deactivates the probe and back to the EXAM screen.

You should remove the probe from the patient and wait the probe cool down. As soon as the probe has cooled, the exam can be restored. At the thermal limit, all system controls are inhibited under normal operating conditions, the probe will not reach the thermal limit except scanning febrile patients or the thermal sensor of the probe fails.

See following suggestions that may help to keep the probe tip temperature in safe range:

In 2D, scan with the maximum available angle and deeper depth available.

CFM and Doppler are the modes with the possibility of temperature increasing; in febrile patients, use these modes as little and as short time possible.

Chapter 9 System Maintenance and Troubleshooting

9.1 Back up information

Caution:

All patient data created is NOT backed-up! It is highly recommended to create a full system backup of patient data regularly and empty the hard disk (HDD), to ensure the hard disk (HDD) has never reached its maximum capacity.

NOTE:

To Backup exams to DVD/CD+(R)W disk, please confirm that the DVD/CD+(R)W storage medium used is clean and not scratched!

Warning:

Do not disconnect an external USB pen drive without stopping it. Disconnecting without stopping can lead to data loss on the external device.

9.2 System Care and Maintenance

The system is a precise electrical device. To ensure the best performance and operation of the system, observe proper maintenance procedures. Contact the local CHISON's Authorized Service Representative for parts or periodic maintenance inspections.

Inspecting the System

Examine the following on a monthly basis:

- Connectors on cables for any mechanical defects.
- Entire length of electrical and power cords for cuts or abrasions.
- Equipment for loose or missing hardware.
- Control panel and keyboard for defects.

To avoid electric shock hazard, do not remove panels or covers from console. This servicing must be performed by CHISON's authorized service engineer. Failure to do so could serious injury.

If any defect is observed or malfunctions occur, do not operate the equipment but inform CHISON's authorized

service engineer for information.

Weekly Maintenance

The system requires weekly care and maintenance to function safely and properly. Clean the following:

- LCD monitor
- Operator control panel
- Footswitch
- Printer

Cleaning the System

Prior to cleaning any part of the system, turn off the system power and disconnect the power cord. See Section 3.4.4“Power Off” in Chapter 3 for more information.

Cleaning Method

- Moisten a soft, non-abrasive folded cloth.
- Wipe down the top, front, back, and both sides of the system.

NOTE:

- *Do not spray any liquid directly into the unit.*
- *Do not use acetone/alcohol or abrasives on painted or plastic surfaces.*

Cleaning LCD Monitor

To clean the monitor face:

- *Use a soft, folded cloth. Gently wipe the monitor face.*
- *Do NOT use a glass cleaner that has a hydrocarbon base (such as Benzene, Methyl Alcohol or Methyl Ethyl Ketone) on monitors with the filter (anti-glare shield). Hard rubbing will also damage the filter.*

NOTE:

When cleaning the screen, make sure not to scratch the LCD.

Cleaning Control Panel

- Moisten a soft, non-abrasive folded cloth with a mild, general purpose, non-abrasive soap and water solution.
- Wipe down operator control panel.
- Use a cotton swab to clean around keys or controls. Use a toothpick to remove solids from between keys and controls.

NOTE:

- ***When cleaning the operator control panel, make sure not to spill or spray any liquid on the controls, into the system cabinet, or in the probe connection receptacle.***
- ***DO NOT use T-spray or Sani Wipes on the control panel.***

Cleaning Footswitch

- Moisten a soft, non-abrasive folded cloth with a mild, general purpose, non-abrasive soap and water solution.
- Wipe the external surfaces of the unit then dry with a soft, clean cloth.

Cleaning Printer

- Turn off the power. If possible, disconnect the power cord.
- Wipe the external surfaces of the unit with a soft, clean, dry cloth.
- Remove stubborn stains with a cloth lightly dampened with a mild detergent solution.

NOTE:

- ***Never use strong solvents, such as thinner or benzine, or abrasive cleansers because they will damage the cabinet.***
- ***No further maintenance, such as lubrication, is required.***
- ***For more information, see the Printer's Operation Manual.***

9.3 Safety Check

To ensure the system work normally, please make a maintenance plan, check the safety of the system periodically. If there is any abnormal phenomenon with the machine, please contact our authorized agent in your country as soon as possible.

If there is no image or menu on the screen or other phenomenon appears after switching on the machine, please do troubleshooting first according to the following check list. If the trouble is still not solved, please contact our authorized agent in your country as soon as possible.

9.4 Troubleshooting

It is necessary to maintain the system regularly, as it can ensure the system being operated under safe state by eliminating possible trouble, and it can shorten the checking and repair period, lower the service costs and reduce the operation danger.

If you have any difficulty with the system, use the following information for your reference to help correct the problem. For a problem not covered here, contact your local distributor or Manufacturer.

Symptom	Solution
The system can't power on	1) Check power connections, e.g. power cord connection on the rear panel; 2) Check the fuse: if it is burnt due to mains fluctuation, use spare fuse for replacement.
When starting the system, the monitor has signal but no ultrasound image	Switch off the system, and check probe connection.
System image quality is not good	1) Adjust the LCD monitor position for a better viewing angle; 2) Adjust the brightness and contrast of LCD monitor; 3) Adjust the image parameters, e.g. Gain, Dynamic range.
No OB calculation package menu	Select the OB application before scanning.
PRINT-key doesn't work	1) Check if the approved printer is connected; 2) Check if the printer power is on; 3) Check printer connection; 4) Check printer setting in system setup.
External monitor doesn't work	1) Check the monitor connections; 2) Check if the monitor power is on and is set up correctly.
CFM or PW Doppler image has noise	1) Adjust CFM or PW gain value properly; 2) Check if there is appliance or equipment resulting in strong electromagnetic interference
Image has interference	1) Move or avoid interference source; 2) Use separate power outlet; 3) Perform good ground protection
The gray scale is S- twisted in the image area	Adjust the power supply to normal voltage or use a voltage stabilizer
The date and time on the screen is not correct	Press Setup-key to display General Setting screen, and correct time and date.
The Video printer is not work	1) Please confirm the signal cable, Remote cable are connected well. 2) Please make sure you have finished the setting for Video printer at system interface. 3) If you can't change the setting at system interface, please check whether the video printer is turned on and connected well with the main unit. 4) Please make sure the switch on the rear panel of printer is "on" status.

9.5 Service Responsibility

If users install, use and maintain the system fully according to CHISON's installation manual, operation manual and service manual, then CHISON main unit has a life time of 5 years and probes have life time of 5 years after ex-work (except that the 4D probe life time is 18 months).

The warranty of the system and probes after ex-work is as the time in the warranty card.

The system is a precise electronic system. Only the CHISON's authorized service engineer could replace the defective parts. Any assembly, disassembly, handling, repair, or replacement by any other people may have adverse impact on the safety and effectiveness of the systems and probes, and thus will reduce the life time of the system and probes, and such systems and probes will not be covered by CHISON warranty after the above improper handling. Standard maintenance must be performed by CHISON's authorized service engineer during the life time of the product.



CAUTION: When the above life time is expired, the effectiveness and safety of system and probes maybe greatly affected, so it's NOT suggested to continue using the system and probes even the system and probes seem work properly. But if user still wants to continue using the system and probes, user should first contact CHISON service center at CHISON headquarter to arrange the necessary safety check and calibration by CHISON's authorized service engineer. If CHISON headquarter service center provides the calibration certificate for the related system or probe, then user could continue use the system or probes according to the calibration certificate. However, if CHISON headquarter service center concludes that the system or probe is no longer complied to the safety and effectiveness standard, then user should immediately stop using the system or probe. User understands that such check and calibration cost will be born by the user. Systems and probes keep on using after the life time may also be difficult to repair and maintain, so it's suggested to renew the product after the life time.

Appendix A The Information of CE Representative

Shanghai International Holding Corp.GmbH(Europe)

Add: Eiffestrasse 80,20537 Hamburg,Germany

Tel: 0049-40-2513175



Fax:0049-40-255726

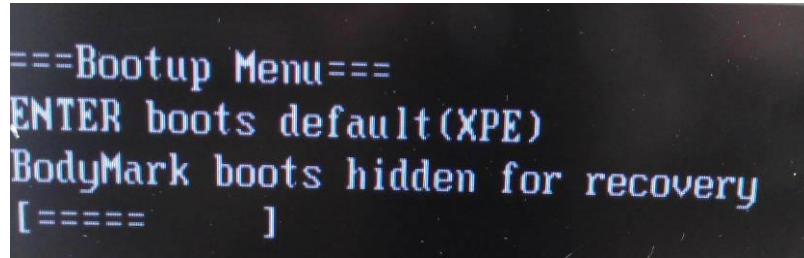
E-mail: antonsissi@hotmail.com shholding@hotmail.com

Appendix B System one-key-recovery Function


This system has one-key-recovery function. User can use this function to recovery system when the system has problems.

Detailed operations are as follows:

1. Press  button to boot the system.
2. Press  button when the system enter into the following boot screen.



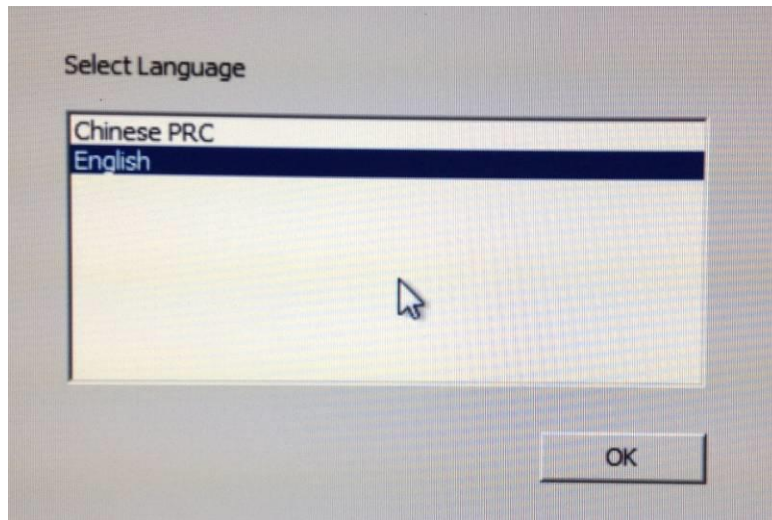
3. When the recovery system starts, the following interface will pop up, users can select language,



Choose to whether to recovery the system.

System Recovery: Recovery the system

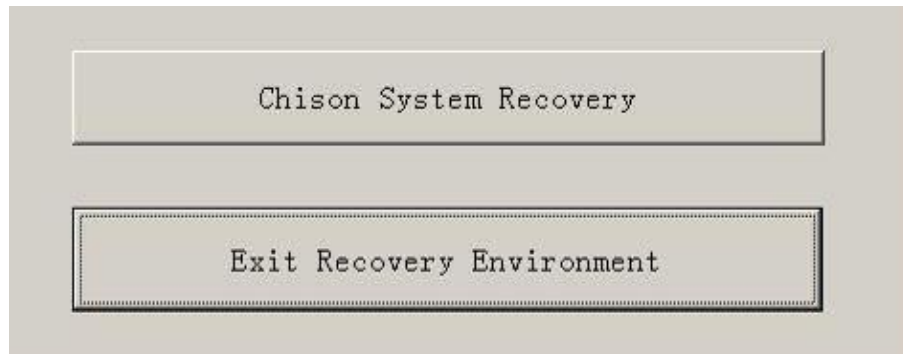Exit : Exit the system recovery



4. After click “Chison System Recovery”, the system will start to backup ultrasound system data, after finishing the backup, the system will transfer to the recovery procedure automatically, and the following interface will pop up.



Click “OK” to confirm and start the recovery.

Click “Cancel” to cancel the system recovery, and go back to the CMD interface.



5. After finishing the recover, the system will pop the CMD interface. Input EXIT in the command line to reboot the system will reboot.

Note:

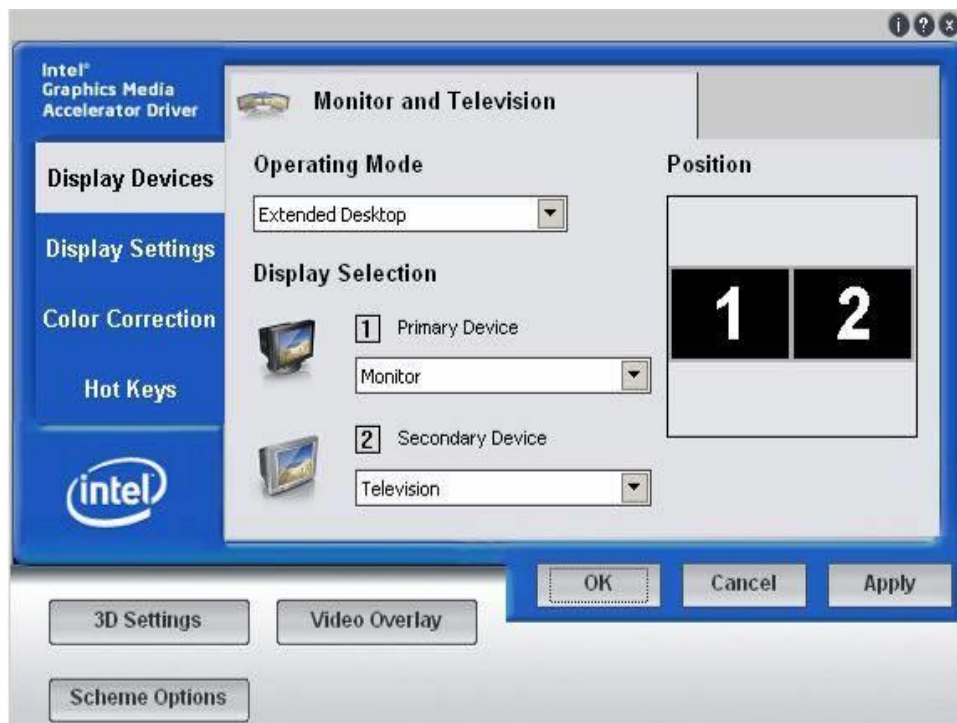
After recovering the system successfully, users need to set "display Settings" manually to ensure video printer work properly

Setting procedures are as follows:


Press “Set-up” key to enter into the setting interface, select the system interface, and click “display setting”



Callout the display setting interface, and set up Operating Mode and Display Selection respectively.



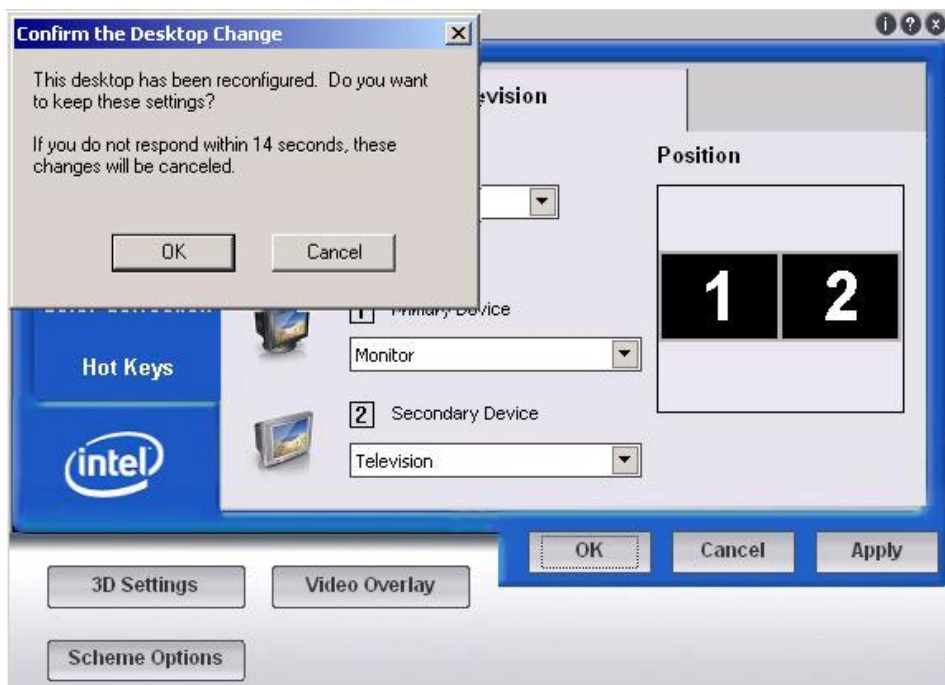Click “OK”, the following interface will pop up, and click “OK” again to confirm it.



Appendix C MAXIMUM ACOUSTIC OUTPUT REPORT

Acoustic Output Table

System: i8

Transducer Model: D3C20L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.82	0.009				
Associated Acoustic Parameter	Pra	Mpa	3.852					
	P	mW		4				
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a		cm	3.7				
	deq(Zb)		cm					
	fawf		MHz	3.486	3.512			
	Dim of Aaprt	X	cm		0.966			
Y		cm		0.3301				
Other Information	td		μs	0.583				
	pr		Hz	2456				
	pr at max.Ipi		MPa	5.854				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	124				
Operating Control Conditions	Focus position		cm	4	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C20L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.747	0.029				
Associated Acoustic Parameter	Pra		Mpa	1.34				
	P		mW		8			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.9				
	deq(Zb)		cm					
	fawf		MHz	3.475	3.764			
	Dim of Aaprt	X	cm		0.8121			
Y		cm		0.4475				
Other Information	td		μs	0.597				
	pr		Hz	4970				
	pr at max.Ipi		MPa	2.079				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	0.117				
Operating Control Conditions	Focus position		cm	14	4			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C20L

Operating Mode: M

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.774		0.087	0.487		
Associated Acoustic Parameter	Pra		Mpa	1.477				
	P		mW				12	
	Min of [Pa(Zs), Ita.a(Zs)]		mW			4.834		
	Zs		cm			2.05		
	Zbp		cm			1.95		
	Zb		cm				4.45	
	Z at max.Ipi.a		cm	4.4				
	deq(Zb)		cm				0.124	
	fawf		MHz	3.533		3.555	3.555	
	Dim of Aaprt	X	cm			0.3457	0.3457	
Y		cm			0.2808	0.2808		
Other Information	td		μs	0.7				
	prr		Hz	2347.4				
	pr at max.Ipi		MPa	2.222				
	deq at max.Ipi		cm				0.125	
	Ipa.a at max.MI		W/cm ²	147.968				
Operating Control Conditions	Focus position		cm	4		7	7	
	A Power		%	100		100	100	

Acoustic Output Table

System: i8

Transducer Model: D3C20L

Operating Mode: PW

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.161		0.05	0.121		
Associated Acoustic Parameter	Pra		Mpa	0.281				
	P		mW				6	
	Min of [Pa(Zs), Ita.a(Zs)]		mW			3.947		
	Zs		cm			2		
	Zbp		cm			1.95		
	Zb		cm				4	
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm				0.727	
	fawf		MHz	2.937		2.939	2.939	
	Dim of Aaprt	X	cm			1.128	1.128	
Y		cm			0.3458	0.3458		
Other Information	td		μs	0.164				
	pr		Hz	10400				
	pr at max.Ipi		MPa	0.344				
	deq at max.Ipi		cm				0.596	
	Ipa.a at max.MI		W/cm ²	2.564				
Operating Control Conditions	Focus position		cm	5		1	1	
	A Power		%	100		100	100	

Acoustic Output Table

System: i8

Transducer Model: D3C60L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.84	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.47				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.89				
	deq(Zb)		cm					
	fawf		MHz	3.24	3.58			
	Dim of Aaprt	X	cm		3.52			
Y		cm		1.1				
Other Information	td		μs	0.78				
	pr		Hz	235				
	pr at max.Ipi		MPa	2.39				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	71.45				
Operating Control Conditions	Focus position		cm	6	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C60L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.56	0.14				
Associated Acoustic Parameter	Pra	Mpa	1.07					
	P	mW		49.6				
	Min of [Pa(Zs), Ita.a(Zs)]	mW						
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a	cm	3.34					
	deq(Zb)	cm						
	fawf	MHz	3.92	3.92				
	Dim of Aaprt	X	cm		1.71			
Y		cm		1.1				
Other Information	td	μs	1.27					
	pr	Hz	1194					
	pr at max.Ipi	MPa	1.52					
	deq at max.Ipi	cm						
	Ipa.a at max.MI	W/cm ²	31.24					
Operating Control Conditions	Focus position	cm	5	7				
	A Power	%	100	100				

Acoustic Output Table

System: i8

Transducer Model: D3C60L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.774		0.07		0.16		
Associated Acoustic Parameter	Pra		Mpa	1.477					
	P		mW		21.32		32		
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm				4.45		
	Z at max.Ipi.a		cm	3.6					
	deq(Zb)		cm				1.99		
	fawf		MHz	3.52		3.52		3.51	
	Dim of Aaprt	X	cm			6.84		6.84	
Y		cm			1.1		1.1		
Other Information	td		μs	0.42					
	pr		Hz	164					
	pr at max.Ipi		MPa	2.33					
	deq at max.Ipi		cm				1.95		
	Ipa.a at max.MI		W/cm ²	52.65					
Operating Control Conditions	Focus position		cm	5		5		7	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D3C60L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.56		0.33		1.08	
Associated Acoustic Parameter	Pra		Mpa	1.00				
	P		mW			40		36
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					4.97
	Z at max.Ipi.a		cm	5.0				
	deq(Zb)		cm					0.23
	fawf		MHz	2.89		2.86		2.89
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td		μs	1.23				
	pr		Hz	4376				
	pr at max.Ipi		MPa	1.62				
	deq at max.Ipi		cm					0.24
	Ipa.a at max.MI		W/cm ²	36.64				
Operating Control Conditions	Focus position		cm	3		7		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D3C60L-B

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.84	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.47				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.89				
	deq(Zb)		cm					
	fawf		MHz	3.24	3.58			
	Dim of Aaprt	X	cm		3.52			
Y		cm		1.1				
Other Information	td		μs	0.78				
	prr		Hz	235				
	pr at max.Ipi		MPa	2.39				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	71.45				
Operating Control Conditions	Focus position		cm	6	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C60L-B

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.56	0.14				
Associated Acoustic Parameter	Pra		Mpa	1.07				
	P		mW		49.6			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.34				
	deq(Zb)		cm					
	fawf		MHz	3.92	3.92			
	Dim of Aaprt	X	cm		1.71			
Y		cm		1.1				
Other Information	td		μs	1.27				
	pr		Hz	1194				
	pr at max.Ipi		MPa	1.52				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	31.24				
Operating Control Conditions	Focus position		cm	5	7			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C60L-B

Operating Mode: M

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.774		0.07		0.16		
Associated Acoustic Parameter	Pra		Mpa	1.477					
	P		mW		21.32		32		
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm				4.45		
	Z at max.Ipi.a		cm	3.6					
	deq(Zb)		cm				1.99		
	fawf		MHz	3.52		3.52		3.51	
	Dim of Aaprt	X	cm			6.84		6.84	
Y		cm			1.1		1.1		
Other Information	td		μs	0.42					
	pr		Hz	164					
	pr at max.Ipi		MPa	2.33					
	deq at max.Ipi		cm				1.95		
	Ipa.a at max.MI		W/cm ²	52.65					
Operating Control Conditions	Focus position		cm	5		5		7	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D3C60L-B

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.56		0.33		1.08	
Associated Acoustic Parameter	Pra		Mpa	1.00				
	P		mW			40		36
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					4.97
	Z at max.Ipi.a		cm	5.0				
	deq(Zb)		cm					0.23
	fawf		MHz	2.89		2.86		2.89
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td		μs	1.23				
	pr		Hz	4376				
	pr at max.Ipi		MPa	1.62				
	deq at max.Ipi		cm					0.24
	Ipa.a at max.MI		W/cm ²	36.64				
Operating Control Conditions	Focus position		cm	3		7		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D3C60L-C

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.84	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.47				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.89				
	deq(Zb)		cm					
	fawf		MHz	3.24	3.58			
	Dim of Aaprt	X	cm		3.52			
Y		cm		1.1				
Other Information	td		μs	0.78				
	prr		Hz	235				
	pr at max.Ipi		MPa	2.39				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	71.45				
Operating Control Conditions	Focus position		cm	6	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3C60L-C

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.56	0.14				
Associated Acoustic Parameter	Pra	Mpa	1.07					
	P	mW		49.6				
	Min of [Pa(Zs), Ita.a(Zs)]	mW						
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a	cm	3.34					
	deq(Zb)	cm						
	fawf	MHz	3.92	3.92				
	Dim of Aaprt	X	cm		1.71			
Y		cm		1.1				
Other Information	td	μs	1.27					
	pr	Hz	1194					
	pr at max.Ipi	MPa	1.52					
	deq at max.Ipi	cm						
	Ipa.a at max.MI	W/cm ²	31.24					
Operating Control Conditions	Focus position	cm	5	7				
	A Power	%	100	100				

Acoustic Output Table

System: i8

Transducer Model: D3C60L-C

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.774		0.07		0.16	
Associated Acoustic Parameter	Pra		Mpa	1.477				
	P		mW			21.32		32
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					4.45
	Z at max.Ipi.a		cm	3.6				
	deq(Zb)		cm					1.99
	fawf		MHz	3.52		3.52		3.51
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td		μs	0.42				
	pr		Hz	164				
	pr at max.Ipi		MPa	2.33				
	deq at max.Ipi		cm					1.95
	Ipa.a at max.MI		W/cm ²	52.65				
Operating Control Conditions	Focus position		cm	5		5		7
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D3C60L-C

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.56		0.33		1.08		
Associated Acoustic Parameter	Pra		Mpa	1.00					
	P		mW			40		36	
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm					4.97	
	Z at max.Ipi.a		cm	5.0					
	deq(Zb)		cm					0.23	
	fawf		MHz	2.89		2.86		2.89	
	Dim of Aaprt	X	cm			6.84		6.84	
Y		cm			1.1		1.1		
Other Information	td		μs	1.23					
	pr		Hz	4376					
	pr at max.Ipi		MPa	1.62					
	deq at max.Ipi		cm					0.24	
	Ipa.a at max.MI		W/cm ²	36.64					
Operating Control Conditions	Focus position		cm	3		7		3	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D3P64L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.82	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.47				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.88				
	deq(Zb)		cm					
	fawf		MHz	3.24	3.58			
	Dim of Aaprt	X	cm		3.52			
Y		cm		1.1				
Other Information	td		μs	0.78				
	pr		Hz	2376				
	pr at max.Ipi		MPa	2.39				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	71.44				
Operating Control Conditions	Focus position		cm	6	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3P64L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.53	0.14				
Associated Acoustic Parameter	Pra	Mpa	1.07					
	P	mW		49.6				
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a		cm	3.36				
	deq(Zb)		Cm					
	fawf		MHz	3.92	3.92			
	Dim of Aaprt	X	cm		1.71			
Y		cm		1.1				
Other Information	td		μs	1.27				
	pr		Hz	1194				
	pr at max.Ipi		MPa	1.52				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	31.45				
Operating Control Conditions	Focus position		cm	5	7			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D3P64L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.84		0.07		0.16	
Associated Acoustic Parameter	Pra	Mpa	1.51					
	P	mW			21.32		32.54	
	Min of [Pa(Zs), Ita.a(Zs)]	mW						
	Zs	cm						
	Zbp	cm						
	Zb	cm					4.23	
	Z at max.Ipi.a	cm	3.6					
	deq(Zb)	cm					1.99	
	fawf	MHz	3.54		3.52		3.51	
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td	μs	0.42					
	pr	Hz	164					
	pr at max.Ipi	MPa	2.33					
	deq at max.Ipi	Cm					1.95	
	Ipa.a at max.MI	W/cm ²	52.87					
Operating Control Conditions	Focus position	cm	5		5		7	
	A Power	%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D3P64L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.54		0.33		1.08	
Associated Acoustic Parameter	Pra		Mpa	1.00				
	P		mW			40		36
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					4.54
	Z at max.Ipi.a		cm	5.0				
	deq(Zb)		cm					0.23
	fawf		MHz	2.89		2.86		2.89
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td		μs	1.23				
	pr		Hz	4376				
	pr at max.Ipi		MPa	1.62				
	deq at max.Ipi		cm					0.24
	Ipa.a at max.MI		W/cm ²	36.63				
Operating Control Conditions	Focus position		cm	3		7		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D5C20L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			1.02	0.09				
Associated Acoustic Parameter	Pra		Mpa	2.19				
	P		mW		12			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	1.85				
	deq(Zb)		cm					
	fawf		MHz	4.83	4.71			
	Dim of Aaprt	X	cm		0.76			
Y		cm		0.8				
Other Information	td		μs	0.49				
	prr		Hz	3846.2				
	pr at max.Ipi		MPa	2.93				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	195.3				
Operating Control Conditions	Focus position		cm	3	6			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D5C20L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.51	0.09				
Associated Acoustic Parameter	Pra		Mpa	1.14				
	P		mW		12			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.4				
	deq(Zb)		cm					
	fawf		MHz	4.89	4.89			
	Dim of Aaprt	X	cm		0.76			
Y		cm		0.8				
Other Information	td		μs	0.45				
	pr		Hz	3846.2				
	pr at max.Ipi		MPa	1.9				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	60.11				
Operating Control Conditions	Focus position		cm	5	5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D5C20L

Operating Mode: M

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.78		0.05		0.09	
Associated Acoustic Parameter	Pra		Mpa	1.70				
	P		mW			2		2
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.5
	Z at max.Ipi.a		cm	3.3				
	deq(Zb)		cm					0.32
	fawf		MHz	4.71		4.8		4.8
	Dim of Aaprt	X	cm			3.02		3.02
Y		cm			0.8		0.8	
Other Information	td		μs	0.5				
	pr		Hz	320.52				
	pr at max.Ipi		MPa	2.86				
	deq at max.Ipi		cm					0.32
	Ipa.a at max.MI		W/cm ²	95.96				
Operating Control Conditions	Focus position		cm	4		2		2
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D5C20L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.36		0.25		0.47	
Associated Acoustic Parameter	Pra		Mpa	0.82				
	P		mW			10		10
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					3.65
	Z at max.Ipi.a		cm	3.7				
	deq(Zb)		cm					0.13
	fawf		MHz	5.22			5.22	5.22
	Dim of Aaprt	X	cm				3.02	3.02
Y		cm				0.8	0.8	
Other Information	td		μs	0.72				
	pr		Hz	6098				
	pr at max.Ipi		MPa	1.5				
	deq at max.Ipi		cm					0.18
	Ipa.a at max.MI		W/cm ²	41.013				
Operating Control Conditions	Focus position		cm	3		3		3
	A Power		%	100		100		

Acoustic Output Table

System: i8

Transducer Model: D5C20L-A

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			1.02	0.09				
Associated Acoustic Parameter	Pra		Mpa	2.19				
	P		mW		12			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	1.85				
	deq(Zb)		cm					
	fawf		MHz	4.83	4.71			
	Dim of Aaprt	X	cm		0.76			
Y		cm		0.8				
Other Information	td		μs	0.49				
	prr		Hz	3846.2				
	pr at max.Ipi		MPa	2.93				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	195.3				
Operating Control Conditions	Focus position		cm	3	6			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D5C20L-A

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.51	0.09				
Associated Acoustic Parameter	Pra		Mpa	1.14				
	P		mW		12			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.4				
	deq(Zb)		cm					
	fawf		MHz	4.89	4.89			
	Dim of Aaprt	X	cm		0.76			
Y		cm		0.8				
Other Information	td		μs	0.45				
	pr		Hz	3846.2				
	pr at max.Ipi		MPa	1.9				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	60.11				
Operating Control Conditions	Focus position		cm	5	5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D5C20L-A

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.78		0.05		0.09	
Associated Acoustic Parameter	Pra		Mpa	1.70				
	P		mW			2		2
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.5
	Z at max.Ipi.a		cm	3.3				
	deq(Zb)		cm					0.32
	fawf		MHz	4.71		4.8		4.8
	Dim of Aaprt	X	cm			3.02		3.02
Y		cm			0.8		0.8	
Other Information	td		μs	0.5				
	pr		Hz	320.52				
	pr at max.Ipi		MPa	2.86				
	deq at max.Ipi		cm					0.32
	Ipa.a at max.MI		W/cm ²	95.96				
Operating Control Conditions	Focus position		cm	4		2		2
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D5C20L-A

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.36		0.25		0.47	
Associated Acoustic Parameter	Pra		Mpa	0.82				
	P		mW			10		10
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					3.65
	Z at max.Ipi.a		cm	3.7				
	deq(Zb)		cm					0.13
	fawf		MHz	5.22			5.22	5.22
	Dim of Aaprt	X	cm				3.02	3.02
Y		cm				0.8	0.8	
Other Information	td		μs	0.72				
	pr		Hz	6098				
	pr at max.Ipi		MPa	1.5				
	deq at max.Ipi		cm					0.18
	Ipa.a at max.MI		W/cm ²	41.013				
Operating Control Conditions	Focus position		cm	3		3		3
	A Power		%	100		100		

Acoustic Output Table

System: i8

Transducer Model: D6P64L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.62	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.47				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.88				
	deq(Zb)		cm					
	fawf		MHz	6.24	6.58			
	Dim of Aaprt	X	cm		3.52			
Y		cm		1.1				
Other Information	td		μs	0.68				
	prr		Hz	2476				
	pr at max.Ipi		MPa	2.49				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	51.44				
Operating Control Conditions	Focus position		cm	6	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6P64L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.53	0.14				
Associated Acoustic Parameter	Pra	Mpa	1.07					
	P	mW		49.6				
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a		cm	3.36				
	deq(Zb)		Cm					
	fawf		MHz	5.92	5.92			
	Dim of Aaprt	X	cm		1.71			
Y		cm		1.1				
Other Information	td		μs	1.27				
	pr		Hz	1194				
	pr at max.Ipi		MPa	1.52				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	31.45				
Operating Control Conditions	Focus position		cm	5	7			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6P64L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.84		0.07		0.16		
Associated Acoustic Parameter	Pra		Mpa	1.51					
	P		mW		21.32		32.54		
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm				4.23		
	Z at max.Ipi.a		cm	3.6					
	deq(Zb)		cm				1.99		
	fawf		MHz	5.54		5.52		5.51	
	Dim of Aaprt	X	cm			6.84		6.84	
Y		cm			1.1		1.1		
Other Information	td		μs	0.42					
	pr		Hz	164					
	pr at max.Ipi		MPa	2.33					
	deq at max.Ipi		Cm				1.95		
	Ipa.a at max.MI		W/cm ²	52.87					
Operating Control Conditions	Focus position		cm	5		5		7	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D6P64L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.54		0.33		1.08	
Associated Acoustic Parameter	Pra		Mpa	1.00				
	P		mW			40		36
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					4.54
	Z at max.Ipi.a		cm	5.0				
	deq(Zb)		cm					0.23
	fawf		MHz	5.89		5.86		5.89
	Dim of Aaprt	X	cm			6.84		6.84
Y		cm			1.1		1.1	
Other Information	td		μs	1.23				
	pr		Hz	4376				
	pr at max.Ipi		MPa	1.62				
	deq at max.Ipi		cm					0.24
	Ipa.a at max.MI		W/cm ²	36.63				
Operating Control Conditions	Focus position		cm	3		7		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L60L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.82	0.009				
Associated Acoustic Parameter	Pra		Mpa	3.852				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.7				
	deq(Zb)		cm					
	fawf		MHz	3.486	3.512			
	Dim of Aaprt	X	cm		0.966			
Y		cm		0.3301				
Other Information	td		μs	0.583				
	pr		Hz	2456				
	pr at max.Ipi		MPa	5.854				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	124				
Operating Control Conditions	Focus position		cm	4	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L60L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.42	0.01				
Associated Acoustic Parameter	Pra	Mpa	0.97					
	P	mW		2				
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a		cm	1.8				
	deq(Zb)		cm					
	fawf		MHz	5.41	5.41			
	Dim of Aaprt	X	cm		1.54			
Y		cm		0.5				
Other Information	td		μs	0.94				
	pr		Hz	3049				
	pr at max.Ipi		MPa	1.35				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	36.29				
Operating Control Conditions	Focus position		cm	2.5	2.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L60L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.30		0.08		0.03	
Associated Acoustic Parameter	Pra		Mpa	0.87				
	P		mW			2		12
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.5
	Z at max.Ipi.a		cm	1.5				
	deq(Zb)		cm					0.124
	fawf		MHz	8.65		8.65		8.65
	Dim of Aaprt	X	cm			6.14		6.14
Y		cm			0.5		0.5	
Other Information	td		μs	0.21				
	pr		Hz	270.58				
	pr at max.Ipi		MPa	1.36				
	deq at max.Ipi		cm					0.6
	Ipa.a at max.MI		W/cm ²	50.43				
Operating Control Conditions	Focus position		cm	2		2		2
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L60L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.32		0.05		0.15		
Associated Acoustic Parameter	Pra	Mpa	0.74						
	P	mW			2		2		
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs	cm							
	Zbp	cm							
	Zb	cm					3.05		
	Z at max.Ipi.a		cm	3.05					
	deq(Zb)		cm					0.05	
	fawf		MHz	5.29		5.29		5.29	
	Dim of Aaprt	X	cm			6.14		6.14	
Y		cm			0.5		0.5		
Other Information	td		μs	0.71					
	pr		Hz	6098					
	pr at max.Ipi		MPa	1.30					
	deq at max.Ipi		cm					0.08	
	Ipa.a at max.MI		W/cm ²	29.64					
Operating Control Conditions	Focus position		cm	3.5		3.5		3.5	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: V4C40L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			1.192	0.007				
Associated Acoustic Parameter	Pra		Mpa	2.198				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2.5				
	deq(Zb)		cm					
	fawf		MHz	3.4	3.53			
	Dim of Aaprt	X	cm		1			
Y		cm		0.6439				
Other Information	td		μs	0.578				
	pr		Hz	3623.3				
	pr at max.Ipi		MPa	2.948				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	140.286				
Operating Control Conditions	Focus position		cm	7	15			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: V4C40L-C

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			1.198	0.006				
Associated Acoustic Parameter	Pra		Mpa	2.158				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2.5				
	deq(Zb)		cm					
	fawf		MHz	3.4	3.53			
	Dim of Aaprt	X	cm		1			
Y		cm		0.6439				
Other Information	td		μs	0.574				
	pr		Hz	3623.3				
	pr at max.Ipi		MPa	2.748				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	142.286				
Operating Control Conditions	Focus position		cm	6	14			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D2D16L

Operating Mode: CW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.212		1.787		0.695	
Associated Acoustic Parameter	Pra		Mpa	0.596				
	P		mW			46		46
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.209
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.484
	fawf		MHz	1.922		2.158		2.158
	Dim of Aaprt	X	cm			0.3613		0.3613
Y		cm			0.1062		0.1062	
Other Information	td		μs	0.471				
	pr		Hz	7168				
	pr at max.Ipi		MPa	1.031				
	deq at max.Ipi		cm					0.484
	Ipa.a at max.MI		W/cm ²	11.52				
Operating Control Conditions	Focus position		cm	5.5		0.5		0.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D6C12L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.514	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.147				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	4.649	5.147			
	Dim of Aaprt	X	cm		0.3479			
Y		cm		0.4447				
Other Information	td		μs	0.257				
	prr		Hz	2614				
	pr at max.Ipi		MPa	5489				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	44.912				
Operating Control Conditions	Focus position		cm	4	1			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C12L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.643	0.084				
Associated Acoustic Parameter	Pra		Mpa	1.44				
	P		mW		14			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	5.486	5.486			
	Dim of Aaprt	X	cm		0.2895			
Y		cm		0.2679				
Other Information	td		μs	0.264				
	pr		Hz	10400				
	pr at max.Ipi		MPa	2.061				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	0.346				
Operating Control Conditions	Focus position		cm	4	4			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C12L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.854		0.14		0.419	
Associated Acoustic Parameter	Pra		Mpa	1.84				
	P		mW			8		8
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.177
	fawf		MHz	4.515		4.54		4.54
	Dim of Aaprt	X	cm			0.1964		0.1964
Y		cm			0.3193		0.3193	
Other Information	td		μs	0.297				
	pr		Hz	2379				
	pr at max.Ipi		MPa	3.074				
	deq at max.Ipi		cm					0.99
	Ipa.a at max.MI		W/cm ²	97.144				
Operating Control Conditions	Focus position		cm	7		3		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D6C12L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.485		3.139		1.662		
Associated Acoustic Parameter	Pra		Mpa	1.208					
	P		mW			72		102	
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm					2.05	
	Z at max.Ipi.a		cm	2					
	deq(Zb)		cm					0.588	
	fawf		MHz	6.211		6.24		6.105	
	Dim of Aaprt	X	cm			0.4378		0.386	
Y		cm			0.2959		0.3095		
Other Information	td		μs	0.648					
	pr		Hz	6098					
	pr at max.Ipi		MPa	1.856					
	deq at max.Ipi		cm					0.587	
	Ipa.a at max.MI		W/cm ²	47.381					
Operating Control Conditions	Focus position		cm	4		6		2	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D6C15L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.941	0.024				
Associated Acoustic Parameter	Pra		Mpa	2.089				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.1				
	deq(Zb)		cm					
	fawf		MHz	5.147	5.784			
	Dim of Aaprt	X	cm		0.434			
Y		cm		0.5489				
Other Information	td		μs	0.246				
	pr		Hz	4346				
	pr at max.Ipi		MPa	3.048				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	188.364				
Operating Control Conditions	Focus position		cm	7	4			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C15L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.357	0.077				
Associated Acoustic Parameter	Pra		Mpa	0.876				
	P		mW		8			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.9				
	deq(Zb)		cm					
	fawf		MHz	5.662	5.673			
	Dim of Aaprt	X	cm		0.3142			
Y		cm		0.2546				
Other Information	td		μs	0.225				
	pr		Hz	8779				
	pr at max.Ipi		MPa	1.791				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	47.363				
Operating Control Conditions	Focus position		cm	6	8			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C15L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.877		0.211		0.447		
Associated Acoustic Parameter	Pra	Mpa	1.927						
	P	mW			10		8		
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs	cm							
	Zbp	cm							
	Zb	cm					2.1		
	Z at max.Ipi.a		cm	2.1					
	deq(Zb)		cm					0.15	
	fawf		MHz	4.824		5.273		4.824	
	Dim of Aaprt	X	cm			0.3607		0.5356	
Y		cm			0.2917		0.5072		
Other Information	td		μs	0.247					
	pr		Hz	4476					
	pr at max.Ipi		MPa	2.72					
	deq at max.Ipi		cm					0.15	
	Ipa.a at max.MI		W/cm ²	175.565					
Operating Control Conditions	Focus position		cm	8		4		8	
	A Power		%	100		100		100	

Acoustic Output Table

System: i8

Transducer Model: D6C15L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.349		2.27		0.947	
Associated Acoustic Parameter	Pra		Mpa	0.874				
	P		mW			76		76
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.796
	fawf		MHz	6.178		6.249		6.178
	Dim of Aaprt	X	cm			0.4905		0.5125
Y		cm			0.2259		0.2537	
Other Information	td		μs	0.57				
	pr		Hz	12700				
	pr at max.Ipi		MPa	1.493				
	deq at max.Ipi		cm					0.753
	Ipa.a at max.MI		W/cm ²	29.126				
Operating Control Conditions	Focus position		cm	1		2		1
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D6C15L-A

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.941	0.024				
Associated Acoustic Parameter	Pra		Mpa	2.089				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	4.1				
	deq(Zb)		cm					
	fawf		MHz	5.147	5.784			
	Dim of Aaprt	X	cm		0.434			
Y		cm		0.5489				
Other Information	td		μs	0.246				
	prr		Hz	4346				
	pr at max.Ipi		MPa	3.048				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	188.364				
Operating Control Conditions	Focus position		cm	7	4			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C15L-A

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.357	0.077				
Associated Acoustic Parameter	Pra		Mpa	0.876				
	P		mW		8			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	3.9				
	deq(Zb)		cm					
	fawf		MHz	5.662	5.673			
	Dim of Aaprt	X	cm		0.3142			
Y		cm		0.2546				
Other Information	td		μs	0.225				
	pr		Hz	8779				
	pr at max.Ipi		MPa	1.791				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	47.363				
Operating Control Conditions	Focus position		cm	6	8			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D6C15L-A

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.877		0.211		0.447	
Associated Acoustic Parameter	Pra		Mpa	1.927				
	P		mW			10		8
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.1
	Z at max.Ipi.a		cm	2.1				
	deq(Zb)		cm					0.15
	fawf		MHz	4.824		5.273		4.824
	Dim of Aaprt	X	cm			0.3607		0.5356
Y		cm			0.2917		0.5072	
Other Information	td		μs	0.247				
	pr		Hz	4476				
	pr at max.Ipi		MPa	2.72				
	deq at max.Ipi		cm					0.15
	Ipa.a at max.MI		W/cm ²	175.565				
Operating Control Conditions	Focus position		cm	8		4		8
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D6C15L-A

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.349		2.27		0.947	
Associated Acoustic Parameter	Pra		Mpa	0.874				
	P		mW			76		76
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.796
	fawf		MHz	6.178		6.249		6.178
	Dim of Aaprt	X	cm			0.4905		0.5125
Y		cm			0.2259		0.2537	
Other Information	td		μs	0.57				
	pr		Hz	12700				
	pr at max.Ipi		MPa	1.493				
	deq at max.Ipi		cm					0.753
	Ipa.a at max.MI		W/cm ²	29.126				
Operating Control Conditions	Focus position		cm	1		2		1
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7C10L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.972	0.021				
Associated Acoustic Parameter	Pra		Mpa	2.225				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2.35				
	deq(Zb)		cm					
	fawf		MHz	5.241	6.455			
	Dim of Aaprt	X	cm		0.2592			
Y		cm		0.2868				
Other Information	td		μs	0.194				
	pr		Hz	5556				
	pr at max.Ipi		MPa	3.405				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	153.965				
Operating Control Conditions	Focus position		cm	4	1			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7C10L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.456	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.066				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2.85				
	deq(Zb)		cm					
	fawf		MHz	5.467	6.353			
	Dim of Aaprt	X	cm		0.7756			
Y		cm		0.333				
Other Information	td		μs	0.195				
	pr		Hz	12900				
	pr at max.Ipi		MPa	1.826				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	32.932				
Operating Control Conditions	Focus position		cm	3	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7C10L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.778		0.061		0.206	
Associated Acoustic Parameter	Pra		Mpa	1.734				
	P		mW			2		2
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.05
	Z at max.Ipi.a		cm	2.1				
	deq(Zb)		cm					0.108
	fawf		MHz	4.96		6.411		4.96
	Dim of Aaprt	X	cm			0.1832		0.4741
Y		cm			0.3151		0.2093	
Other Information	td		μs	0.166				
	pr		Hz	2386.6				
	pr at max.Ipi		MPa	2.463				
	deq at max.Ipi		cm					0.108
	Ipa.a at max.MI		W/cm ²	116.448				
Operating Control Conditions	Focus position		cm	5		1		5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7C10L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.227		1.21		0.439	
Associated Acoustic Parameter	Pra		Mpa	0.64				
	P		mW			32		32
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.15
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.517
	fawf		MHz	7.932		7.943		7.942
	Dim of Aaprt	X	cm			0.6991		0.7096
Y		cm			0.2338		0.1788	
Other Information	td		μs	0.461				
	pr		Hz	6352				
	pr at max.Ipi		MPa	1.107				
	deq at max.Ipi		cm					0.517
	Ipa.a at max.MI		W/cm ²	20.424				
Operating Control Conditions	Focus position		cm	7		1		2
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7C10L-C

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.972	0.021				
Associated Acoustic Parameter	Pra		Mpa	2.225				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2.35				
	deq(Zb)		cm					
	fawf		MHz	5.241	6.455			
	Dim of Aaprt	X	cm		0.2592			
Y		cm		0.2868				
Other Information	td		μs	0.194				
	prr		Hz	5556				
	pr at max.Ipi		MPa	3.405				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	153.965				
Operating Control Conditions	Focus position		cm	4	1			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7C10L-C

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.456	0.02				
Associated Acoustic Parameter	Pra	Mpa	1.066					
	P	mW		2				
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs	cm						
	Zbp	cm						
	Zb	cm						
	Z at max.Ipi.a		cm	2.85				
	deq(Zb)		cm					
	fawf		MHz	5.467	6.353			
	Dim of Aaprt	X	cm		0.7756			
Y		cm		0.333				
Other Information	td		μs	0.195				
	pr		Hz	12900				
	pr at max.Ipi		MPa	1.826				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	32.932				
Operating Control Conditions	Focus position		cm	3	2			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7C10L-C

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.778		0.061		0.206	
Associated Acoustic Parameter	Pra		Mpa	1.734				
	P		mW			2		2
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.05
	Z at max.Ipi.a		cm	2.1				
	deq(Zb)		cm					0.108
	fawf		MHz	4.96		6.411		4.96
	Dim of Aaprt	X	cm			0.1832		0.4741
Y		cm			0.3151		0.2093	
Other Information	td		μs	0.166				
	pr		Hz	2386.6				
	pr at max.Ipi		MPa	2.463				
	deq at max.Ipi		cm					0.108
	Ipa.a at max.MI		W/cm ²	116.448				
Operating Control Conditions	Focus position		cm	5		1		5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7C10L-C

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.227		1.21		0.439	
Associated Acoustic Parameter	Pra		Mpa	0.64				
	P		mW			32		32
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.15
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.517
	fawf		MHz	7.932		7.943		7.942
	Dim of Aaprt	X	cm			0.6991		0.7096
Y		cm			0.2338		0.1788	
Other Information	td		μs	0.461				
	pr		Hz	6352				
	pr at max.Ipi		MPa	1.107				
	deq at max.Ipi		cm					0.517
	Ipa.a at max.MI		W/cm ²	20.424				
Operating Control Conditions	Focus position		cm	7		1		2
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L30L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.62	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.68				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	1.35				
	deq(Zb)		cm					
	fawf		MHz	7.25	7.25			
	Dim of Aaprt	X	cm		0.70			
Y		cm		0.60				
Other Information	td		μs	0.16				
	pr		Hz	4082				
	pr at max.Ipi		MPa	2.46				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	135.26				
Operating Control Conditions	Focus position		cm	3	3			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L30L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.83	0.025				
Associated Acoustic Parameter	Pra		Mpa	2.32				
	P		mW		28			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	1.55				
	deq(Zb)		cm					
	fawf		MHz	7.76	7.84			
	Dim of Aaprt	X	cm		0.70			
Y		cm		0.60				
Other Information	td		μs	0.63				
	pr		Hz	7575.8				
	pr at max.Ipi		MPa	3.37				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	136.25				
Operating Control Conditions	Focus position		cm	2.5	7			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L30L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.46		0.07		0.04	
Associated Acoustic Parameter	Pra		Mpa	1.15				
	P		mW			2		2
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.6
	Z at max.Ipi.a		cm	1.65				
	deq(Zb)		cm					0.177
	fawf		MHz	6.26		7.58		6.36
	Dim of Aaprt	X	cm			2.82		2.82
Y		cm			0.6		0.6	
Other Information	td		μs	0.18				
	pr		Hz	341				
	pr at max.Ipi		MPa	1.47				
	deq at max.Ipi		cm					0.57
	Ipa.a at max.MI		W/cm ²	42.36				
Operating Control Conditions	Focus position		cm	6		1.5		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L30L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.75		1.01		1.15	
Associated Acoustic Parameter	Pra		Mpa	1.85				
	P		mW			34		34
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.5
	Z at max.Ipi.a		cm	3.35				
	deq(Zb)		cm					0.27
	fawf		MHz	6.11		6.25		6.25
	Dim of Aaprt	X	cm			2.82		2.82
Y		cm			0.6		0.6	
Other Information	td		μs	0.59				
	pr		Hz	6970				
	pr at max.Ipi		MPa	3.17				
	deq at max.Ipi		cm					0.27
	Ipa.a at max.MI		W/cm ²	142.36				
Operating Control Conditions	Focus position		cm	8.5		3.5		3.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L40L

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.445	0.148				
Associated Acoustic Parameter	Pra		Mpa	1.095				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	6.049	7.714			
	Dim of Aaprt	X	cm		0.4229			
Y		cm		0.1475				
Other Information	td		μs	0.195				
	pr		Hz	6024				
	pr at max.Ipi		MPa	1.664				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	29.638				
Operating Control Conditions	Focus position		cm	2.5	0.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L40L

Operating Mode: C

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.492	0.425				
Associated Acoustic Parameter	Pra		Mpa	1.19				
	P		mW		48			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	5.863	7.61			
	Dim of Aaprt	X	cm		0.177			
Y		cm		0.2549				
Other Information	td		μs	0.196				
	pr		Hz	12100				
	pr at max.Ipi		MPa	1.976				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	51.49				
Operating Control Conditions	Focus position		cm	3.5	6.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L40L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.517		0.532		0.476	
Associated Acoustic Parameter	Pra		Mpa	1.289				
	P		mW			16		16
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.25
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.258
	fawf		MHz	6.219		6.982		6.982
	Dim of Aaprt	X	cm			0.2936		0.2396
Y		cm			0.1686		0.1686	
Other Information	td		μs	0.166				
	pr		Hz	2336.4				
	pr at max.Ipi		MPa	1.981				
	deq at max.Ipi		cm					0.258
	Ipa.a at max.MI		W/cm ²	54.798				
Operating Control Conditions	Focus position		cm	4.5		3.5		3.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L40L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.212		1.787		0.695	
Associated Acoustic Parameter	Pra		Mpa	0.596				
	P		mW			46		46
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.209
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.484
	fawf		MHz	7.922		8.158		8.158
	Dim of Aaprt	X	cm			0.3613		0.3613
Y		cm			0.1062		0.1062	
Other Information	td		μs	0.471				
	pr		Hz	7168				
	pr at max.Ipi		MPa	1.031				
	deq at max.Ipi		cm					0.484
	Ipa.a at max.MI		W/cm ²	11.52				
Operating Control Conditions	Focus position		cm	5.5		0.5		0.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L40L-C

Operating Mode: B

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.445	0.148				
Associated Acoustic Parameter	Pra		Mpa	1.095				
	P		mW		2			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	6.049	7.714			
	Dim of Aaprt	X	cm		0.4229			
Y		cm		0.1475				
Other Information	td		μs	0.195				
	pr		Hz	6024				
	pr at max.Ipi		MPa	1.664				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	29.638				
Operating Control Conditions	Focus position		cm	2.5	0.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L40L-C

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.492	0.425				
Associated Acoustic Parameter	Pra		Mpa	1.19				
	P		mW		48			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	5.863	7.61			
	Dim of Aaprt	X	cm		0.177			
Y		cm		0.2549				
Other Information	td		μs	0.196				
	pr		Hz	12100				
	pr at max.Ipi		MPa	1.976				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	51.49				
Operating Control Conditions	Focus position		cm	3.5	6.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D7L40L-C

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.517		0.532		0.476	
Associated Acoustic Parameter	Pra		Mpa	1.289				
	P		mW			16		16
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.25
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.258
	fawf		MHz	6.219		6.982		6.982
	Dim of Aaprt	X	cm			0.2936		0.2396
Y		cm			0.1686		0.1686	
Other Information	td		μs	0.166				
	pr		Hz	2336.4				
	pr at max.Ipi		MPa	1.981				
	deq at max.Ipi		cm					0.258
	Ipa.a at max.MI		W/cm ²	54.798				
Operating Control Conditions	Focus position		cm	4.5		3.5		3.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D7L40L-C

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.212		1.787		0.695	
Associated Acoustic Parameter	Pra		Mpa	0.596				
	P		mW			46		46
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					1.209
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.484
	fawf		MHz	7.922		8.158		8.158
	Dim of Aaprt	X	cm			0.3613		0.3613
Y		cm			0.1062		0.1062	
Other Information	td		μs	0.471				
	pr		Hz	7168				
	pr at max.Ipi		MPa	1.031				
	deq at max.Ipi		cm					0.484
	Ipa.a at max.MI		W/cm ²	11.52				
Operating Control Conditions	Focus position		cm	5.5		0.5		0.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D12L40L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.495	0.177				
Associated Acoustic Parameter	Pra		Mpa	1.495				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	11.049	11.714			
	Dim of Aaprt	X	cm		0.4229			
Y		cm		0.1475				
Other Information	td		μs	0.145				
	pr		Hz	6024				
	pr at max.Ipi		MPa	1.644				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	49.638				
Operating Control Conditions	Focus position		cm	2.5	0.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D12L40L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.292	0.425				
Associated Acoustic Parameter	Pra		Mpa	1.19				
	P		mW		48			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	11.863	11.61			
	Dim of Aaprt	X	cm		0.177			
Y		cm		0.2549				
Other Information	td		μs	0.146				
	pr		Hz	12400				
	pr at max.Ipi		MPa	1.476				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	41.49				
Operating Control Conditions	Focus position		cm	3.5	6.5			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: D12L40L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.517		0.532		0.476	
Associated Acoustic Parameter	Pra		Mpa	1.289				
	P		mW			16		16
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2.25
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.258
	fawf		MHz	11.219		11.982		11.982
	Dim of Aaprt	X	cm			0.2936		0.2396
Y		cm			0.1686		0.1686	
Other Information	td		μs	0.146				
	prr		Hz	2306.4				
	pr at max.Ipi		MPa	1.281				
	deq at max.Ipi		cm					0.258
	Ipa.a at max.MI		W/cm ²	34.798				
Operating Control Conditions	Focus position		cm	4.5		3.5		3.5
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: D12L40L

Operating Mode: PW

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.202		1.747	0.695		
Associated Acoustic Parameter	Pra		Mpa	0.546				
	P		mW		46		46	
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm				1.209	
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm				0.484	
	fawf		MHz	11.922		11.158		11.158
	Dim of Aaprt	X	cm			0.3613		0.3613
Y		cm			0.1062		0.1062	
Other Information	td		μs	0.471				
	pr		Hz	6168				
	pr at max.Ipi		MPa	1.031				
	deq at max.Ipi		cm				0.484	
	Ipa.a at max.MI		W/cm ²	10.52				
Operating Control Conditions	Focus position		cm	5.5		0.5		
	A Power		%	100		100		

Acoustic Output Table

System: i8

Transducer Model: V6C10L

Operating Mode: B

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.514	0.02				
Associated Acoustic Parameter	Pra		Mpa	1.147				
	P		mW		4			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	4.649	5.147			
	Dim of Aaprt	X	cm		0.3479			
Y		cm		0.4447				
Other Information	td		μs	0.257				
	pr		Hz	2614				
	pr at max.Ipi		MPa	5489				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	44.912				
Operating Control Conditions	Focus position		cm	4	1			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: V6C10L

Operating Mode: C

Index Label			MI	TIS		TIB	TIC	
				Scan	Non-scan			Non-scan
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.643	0.084				
Associated Acoustic Parameter	Pra		Mpa	1.44				
	P		mW		14			
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					
	fawf		MHz	5.486	5.486			
	Dim of Aaprt	X	cm		0.2895			
Y		cm		0.2679				
Other Information	td		μs	0.264				
	pr		Hz	10400				
	pr at max.Ipi		MPa	2.061				
	deq at max.Ipi		cm					
	Ipa.a at max.MI		W/cm ²	0.346				
Operating Control Conditions	Focus position		cm	4	4			
	A Power		%	100	100			

Acoustic Output Table

System: i8

Transducer Model: V6C10L

Operating Mode: M

Index Label			MI	TIS			TIB	TIC
				Scan	Non-scan		Non-scan	
					Aaprt ≤1cm ²	Aaprt >1cm ²		
Global Maximum Index Value			0.854		0.14		0.419	
Associated Acoustic Parameter	Pra		Mpa	1.84				
	P		mW			8		8
	Min of [Pa(Zs), Ita.a(Zs)]		mW					
	Zs		cm					
	Zbp		cm					
	Zb		cm					2
	Z at max.Ipi.a		cm	2				
	deq(Zb)		cm					0.177
	fawf		MHz	4.515		4.54		4.54
	Dim of Aaprt	X	cm			0.1964		0.1964
Y		cm			0.3193		0.3193	
Other Information	td		μs	0.297				
	pr		Hz	2379				
	pr at max.Ipi		MPa	3.074				
	deq at max.Ipi		cm					0.99
	Ipa.a at max.MI		W/cm ²	97.144				
Operating Control Conditions	Focus position		cm	7		3		3
	A Power		%	100		100		100

Acoustic Output Table

System: i8

Transducer Model: V6C10L

Operating Mode: PW

Index Label			MI	TIS			TIB	TIC	
				Scan	Non-scan		Non-scan		
					Aaprt ≤1cm ²	Aaprt >1cm ²			
Global Maximum Index Value			0.485		3.139		1.662		
Associated Acoustic Parameter	Pra		Mpa	1.208					
	P		mW			72		102	
	Min of [Pa(Zs), Ita.a(Zs)]		mW						
	Zs		cm						
	Zbp		cm						
	Zb		cm					2.05	
	Z at max.Ipi.a		cm	2					
	deq(Zb)		cm					0.588	
	fawf		MHz	6.211		6.24		6.105	
	Dim of Aaprt	X	cm			0.4378		0.386	
Y		cm			0.2959		0.3095		
Other Information	td		μs	0.648					
	pr		Hz	6098					
	pr at max.Ipi		MPa	1.856					
	deq at max.Ipi		cm					0.587	
	Ipa.a at max.MI		W/cm ²	47.381					
Operating Control Conditions	Focus position		cm	4		6		2	
	A Power		%	100		100		100	

Display Accuracy and Acoustic Measurement Uncertainties

According to IEC60601-2-37 and NEMA UD-3 2004, the display accuracy and acoustic measurement uncertainties are summarized in the table below.

Display accuracy of MI is $\pm 20\%$, and TI is $\pm 40\%$ or < 0.1 , if MI, TI below 0.5.

Item	Measurement Uncertainty (Percentage, 95% Confidence Value)
Center Frequency	$\pm 5\%$
Acoustic Power	$\pm 30\%$
Acoustic Intensity	$\pm 30\%$
Peak Rarefactional Pressure	$\pm 15\%$

APPENDIX D Transducer Maximum Surface Temperature

According to the requirements of the section 42.3 in the standard IEC 60601-2-37:2007, the transducer surface temperature has been tested in two kinds of conditions: the transducer suspended in still air or transducer contacting human-tissue mimicking material. The calculation of the expanded uncertainty is based on the ISO Guide tout ye Expression of uncertainty in measurement. Three transducer samples have been tested and the confidence coefficient is at 95%, the value of t.975 is 4.30.

The measurement data were obtained under the test conditions employed at CHISON.

Transducer model	Ambient (°C)	Maximum surface temperature(°C) Contacting human-tissue mimicking material	Maximum surface temperature(°C) Suspending in air
D3C60L-B	25	41±1	49±1
D7L40L	25	38±1	49±1
D6C12L	25	38±1	43±1
D7C10L	25	38±1	48±1
D3P64L	25	41±2	49±1
D12L40L	25	41±2	42±1
V4C40L	25	41±2	41±1
V4C40L-C	25	41±2	41±1
D6C15L	25	41±1	40±1
D5C20L	25	40±1	46±1
D3C60L	25	41±1	49±1
D3C60L-C	25	41±1	49±1
D6P64L	25	41±1	49±1
D7L60L	25	41±1	48±1
D7L30L	25	39±2	47±2
D3C20L	25	40±2	44±2
D5C20L-A	25	40±2	45±2
D6C15L-A	25	41±1	47±2
D2D16L	25	37±1	39±1
D7L40L-C	25	38±1	49±1
V6C10L	25	38±1	48±1
D7C10L-C	25	38±1	48±1

Note: Values following the “±”mark indicate the expanded uncertainty with a confidence lever of 95%,t.975=4.30.


APPENDIX E GUIDANCE AND MANUFACTURER’S DECLARATION

1. Guidance and manufacturer’s declaration – electromagnetic emissions		
The system is intended for use in the electromagnetic environment specified below. The customer or the user of the system should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The i8 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class A	The system is suitable for use in all establishments, including domestic stablishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Class A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

2. Guidance and manufacturer’s declaration – electromagnetic immunity			
The i8 is intended for use in the electromagnetic environment the system should assure that it is used in such an environment.			
Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	±6 kV contact ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	±2 kV for power supply lines ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.

Digital Color Doppler Ultrasound System

Surge IEC 61000-4-5	±1 kV line(s) to line(s) ±2 kV line(s) to earth	±1 kV line(s) to line(s) ±2 kV line(s) to earth	Mains power quality should be that of a typical commercial or hospital environment.
interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 0,5 cycle 40 % <i>UT</i> (60 % dip in <i>UT</i>) for 5 cycles 70 % <i>UT</i> (30 % dip in <i>UT</i>) for 25 cycles <5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 5 sec	<5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 0,5 cycle 40 % <i>UT</i> (60 % dip in <i>UT</i>) for 5 cycles 70 % <i>UT</i> (30 % dip in <i>UT</i>) for 25 cycles <5 % <i>UT</i> (>95 % dip in <i>UT</i>) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the system requires continued operation during power mains interruptions, it is recommended that the system be powered from an uninterruptible power supply or a battery.
Power frequency frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE <i>UT</i> is the a.c. mains voltage prior to application of the test level.			

3 Guidance and manufacturer's declaration – electromagnetic immunity			
The system is intended for use in the electromagnetic environment specified below. The customer or the user of the system should assure that it is used in such an environment.			
3.1. Immunity Test	IEC 60601 test level	IEC 60601 test level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6 Radiated RF IEC 61000-4-3	3 Vrms 150 kHz to 80 MHz 3 V/m 80 MHz to 2,5 GHz	3 Vrms 3 V/m	<p>Portable and mobile RF communications equipment should be used no closer to any part of the system, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended separation distance</p> $d = 1,2 \sqrt{P}$ $d = 1,2 \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = 2,3 \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m)</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than the compliance level in each frequency range.^a</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 
<p>NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p> <p>^a Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the system is used exceeds the applicable RF compliance level above, the system should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as reorienting or relocating the system.</p> <p>^b Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.</p>			

Recommended separation distances between portable and mobile RF communications equipment and the system			
<p>The system is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the system can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the system as recommended below, according to the maximum output power of the communications equipment.</p>			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = 1,2 \sqrt{P}$	80 MHz to 800 MHz $d = 1,2 \sqrt{P}$	800 MHz to 2,5 GHz $d = 2,3 \sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,38	0,38	0,73
1	1,2	1,2	2,3
10	3,8	3,8	7,3
100	12	12	23
<p>For transmitters rated at a maximum output power not listed above, the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.</p> <p>NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.</p> <p>NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.</p>			

Appendix F: Measurement Results Summary

Measurement	Useful Range	Accuracy
Distance	Full Screen	<±5%
Circumference: trace method,ellipse method	Full Screen	<±5%
Area: trace method,ellipse method	Full Screen	<±10%
Volume	Full screen	<±10%
Angle	Full screen	<±5%
Time	Full Screen	<±5%
Heart rate	Full Screen	<±5%
Velocity	Full Screen	<±10%