

# CINTURE IMMOBILIZZAZIONE

## IMMOBILISATION BELTS

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

### INTRODUCTION

This product is used with stretcher when transporting patients in emergency,

It has the function of fixation.

The tape is made of polpropelene

The material of the button is POM

The length of the belt can be adjusted, and the plug can be separated quickly

The load bearing capacity of the whole product is >60kg

### OPERATION

Unfasten the safety strap, and fix the patient on a stretcher, then fasten the buckle, adjust the length of strap for securing the patient properly.

### WARNING

During transportation, pay attention to whether the fixed belt is damaged due to limp. If the belt is loose, adjust the tightness of the belt properly.











### PRODUCT STORAGE AND MAINTENANCE

After each use, please use clear water or neutral detergent to clean the dirt on the surface. Store it for subsequence use when it is dry. If there are bloodstains or body fluids left, please disinfect the product with disinfectant, and clean up.

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

### SYMBOLS

	Product code
	Caution: read instructions (warnings) carefully
	Medical Device
	Consult instructions for use
	Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745
	Keep in a cool, dry place
	GB Keep away from sunlight
	Lot number
	Date of manufacture

**REF** 34010 - 34011

 Gima S.p.A.  
 Via Marconi, 1 - 20060 Gessate (MI) Italy  
 gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)  
 Made in China

**MD** **CE**

