

SET DA 3 CINTURE - TIPO B - arancioni

SET OF 3 BELTS - B - orange

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

INTRODUCTION

Dimension: 5CM*274CM

Products include: 3 pieces seat belt (color optional in the picture) and 1 piece outsourcing.

PRODUCT FEATURES

1. This product is used with stretcher when transporting patients in emergency.
2. The material of the belt is PP.
3. The material of the button is POM (middle) + steel (both ends)
4. The length of the belt can be adjusted, and the plug can be separated quickly
5. The tensile force that the whole product can bear is more than 60kg.










OPERATION STEPS

Unfasten the safety strap, and fix the patient on a stretcher, then fasten the buckle, adjust the length of strap for securing the patient properly.

PRODUCT STORAGE AND MAINTENANCE

After each use, please use clear water or neutral detergent to clean the dirt on the surface. Store it for subsequence use when it is dry.

SYMBOLS

	Product code
	Caution: read instructions (warnings) carefully
	Medical Device
	Consult instructions for use
	Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745
	Keep in a cool, dry place
	GB Keep away from sunlight
	Lot number

REF 34014



Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
 Made in China

