

ADJUSTABLE SPIDER STRAP

34018 - Misura/Size: 203x51x5 cm
 34082 - Misura/Size: 183x46x5 cm

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

PRODUCT INTRODUCTION

This product is designed to be used for spinal board with the use of spider strap for fixing patient. It can adjust the space between two straps vertically to better fit patients of different sizes.

The product is reticular, strap ① for fixing shoulder and chest, strap ② for fixing the waist, strap ③ for fixing the thigh, strap ④ for fixing the crus. These straps are interrelated and adjustable vertically, and easy to operate.

OPERATION STEPS



Fig. 1



Fig. 2

1. Place the patient in the spinal board which operated by professionals.
2. Fix the patient's body parts, and make adjustments for a better fit (adjust the space between two straps vertically). Check to ensure it is suitable after fixing. During the process of patient transfer, pay attention to the tightness of straps, and make appropriate adjustments if necessary.

Notice:

The operators must be first-aid professionals, or trained and qualified personnel.

Read the user manual carefully before use.











Check to see if the product is intact before use.

Know about the patient's condition before use, and pay attention to the injured part during fixing.

Warning:

This product must be operated by first-aid professionals, or it may cause other harm to patients.

SYMBOLS

 REF	Product code		Medical Device compliant with Regulation (EU) 2017/745
	Caution: read instructions (warnings) carefully		Keep in a cool, dry place
	Medical Device		GB Keep away from sunlight
	Consult instructions for use		Lot number
	Manufacturer		Date of manufacture

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

REF 34018 - 34082

 **Gima S.p.A.**
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
 Made in China

MD 

