



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. – Via Marconi, 1 – 20060 Gessate (MI) Italy

Italia: tel. 199 400 401 – fax. 199 400 403

Export: tel. +39 02 953854209/221/225 – fax. +39 02 95380056

gima@gimaitaly.com – export@gimaitaly.com

www.gimaitaly.com

SPINAL IMMOBILIZER – TYPE B



User manual

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

REF 34028



GIMA S.p.A.
Via Marconi, 1
20060 Gessate (MI)
Italy
Made in China



DISCRIPTION:

- Used in first aid to fix spine;
- Made of waterproof nylon;
- Internal support of Bamboo;
- X-ray transparent;
- Can be put into a Simple bag for portability.

STEPS

When using this product, need at least three operators

1. Unfold it, an operator stand behind the patient and put the product with no straps side towards to the back of the patient.
2. The other two operators should stand on both sides of the patient, wrap the products around the patient's armpit and adjust it to alignment the patient's spine.
3. Pull out and restrict the strap of the chest. Raise the product and make the upper edge just in the patient's armpit, restrict the strap.
4. Bypass the straps between the patient's legs and button the fastener.
5. If there is interspace between patient's head and product, a rectangle accessory should be used to fulfill it.
6. Use two head straps to fix both forehead and chap with the smooth side towards to the head of patient.

WARNING

1. This product should be operated by trained people or will harm to patient.
2. When using the product, determine the patient has no other injuries on body (chest fractures, leg fractures). Otherwise, a recommendation of health care worker should be taken.
3. Any needs of transpose should be operated by trained people.

STORAGE AND MAINTENANCE:

1. Use clean soft wiper to clear the product after using it.
2. Blood and body fluid should be disinfected then reserve.