GIMA

PROFESSIONAL MEDICAL PRODUCTS

BARELLA A SOLLEVAMENTO AUTOMATICO
AUTOMATIC STRETCHER
BRANCARD À LEVAGE AUTOMATIQUE
KRANKENBETT MIT AUTOMATISCHER HEBEVORRICHTUNG
CAMILLA DE ELEVACIÓN AUTOMÁTICA
AMACA DE RODAS DE ELEVAÇÃO AUTOMÁTICA
ΦΟΡΕΙΟ ΜΕ ΑΥΤΟΜΑΤΗ ΑΝΥΨΩΣΗ
نقالة الرفع الأوتوماتيكية

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd

No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made In China

GIMA 34055

REF

TD01013A

EC REP

SUNGO Europe B.V.

Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle
aan den IJssel, The Netherlands

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

Technical parameters

- Product Size: (L x W x H)
High position 193 x 60 x 88 cm
Low position 190 x 60 x 21 cm
- Maximum Angle of the Back 60°
- Carton Size 1pc 195 x 61 x 26 cm
- N.W. 40kg , G.W. 45kg
- Load Bearing 159kg

II. How to use

- The stretcher surface is made of soft sponge mattress, which make patient feel comfortable. One medical staff can push the trolley on to the ambulance or send to ward.
- When need low position, pull out the safety handle & then press it, the stretcher's height will automatically lower down.
- When destination arrived, pulling out the safety handle, the stretcher's legs will unfold automatically.
- There are the return elastic lock device at both side of the stretcher & the leg parts. Lock tightly when turn up for patient safety, turn down for patients out.
- Pressing the button under the stretcher, the head part will be raised or lowered.

III. Applied for

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, etc. to carry the wounded & patients. It also can be used in general operation.

IV. Cautions

- When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
- In general operation, be careful not to out torn the stretcher surface.

V. Maintenance

- Keep clean as routine (incl. sterilization).
- Often check if any parts loose or not.

VI. Storage & transportation

- Store this product in the place of damp proof & non corrosion.
- Common transporting vehicle can carry this product.
- Upside down is prohibited during transportation & storage. Stack the goods as per the instructions printed on the package.

| | | | | | |
|--|---------------------------|---|--|---|---|
|  REF | Product code |  | Manufacturer |  | Consult instructions for use |
|  LOT | Lot number |  | Medical Device compliant with Regulation (EU) 2017/745 |  | Date of manufacture |
|  | Keep in a cool, dry place |  | Caution: read instructions (warnings) carefully |  | Authorized representative in the European community |
|  | Keep away from sunlight |  | Medical Device |  | Unique identifier |
|  | Imported by | | | | |

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.