# GIMA

PROFESSIONAL MEDICAL PRODUCTS

# GIMA

**BARELLA COMPLETAMENTE AUTOMATICA  
CON POSIZIONI VARIABILI  
FULL AUTOMATIC STRETCHER WITH VARIED POSITIONS  
CIVIÈRE ENTIÈREMENT AUTOMATIQUE À POSITIONS VARIÉES  
VOLLAUTOMATISCHE VERSTELLBARE KRANKENTRAGE  
CAMILLA TOTALMENTE AUTOMÁTICA  
CON POSICIONES VARIADAS  
MACA TOTALMENTE AUTOMÁTICA COM VÁRIAS POSIÇÕES  
ΠΛ'ΗΡΩΣ ΑΥΤΟΜΑΤΟ ΦΟΡΕΪΟ ΠΟΛΛΑΠΛΩΝ Θ'ΕΞΕΩΝ ΠΛ'ΗΡΩΣ  
Θ'ΕΞΕΩΝ**

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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## GIMA 34057

**Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd**

No.999 Hongqi Road, Changyinsha  
Modern Agricultural Demonstration Area,  
Zhangjiagang City, Jiangsu, 215623, China  
Made In China

**TD010131F**

**SUNGO Europe B.V.**

Fascinatio Boulevard 522, Unit 2909 ,1.7VA  
Capelle aan den IJssel, The Netherlands

**Gima S.p.A.**

Via Marconi, 20060 - 1 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

## I. Technical parameters

- Size (L \*W\*H):  
Maximum Size: 89 6 5\*198 cm  
Minimum Size: 33 6 5\*198 cm
- The height scope of touching down ( 4 grades):  
3 9 cm 2 8 cm 70 cm /5 3 cm
- Loading Bearing : 159kg
- Net weight: 40kgs  
Gross Weight: 46 kgs
- Maximum angle of leaning forward: 11 degrees
- Maximum angle of leaning back forward: 25degrees

## II. Operating Method

- Put the sponge mat on the stretcher and fix the straps. Which would make the patient feel safety and comfortable.
- Fixing or unfix the stretcher from loading stretcher: when fixing the stretcher to the loading stretcher, carry the stretcher onto the loading stretcher, push the stretcher forward, the front wheels would enter the fixation, meanwhile the back wheels would also enter the fixation. The fixation would make the stretcher locked automatically. When unfixing the stretcher from the loading stretcher, press the red ball button, pull the stretcher out of the fixation, and then the stretcher can be unfix. Stretcher can become wheelchair.
- Fixing or unfix the stretcher from the ambulance: ① when the loading stretcher enter the carriage board, pull the handle, free the handle after the front legs and back legs folded. Push the loading stretcher to the fixation to make it fixed. (this loading stretcher are blessed with fixation). ② When unfixing the loading stretcher from the ambulance, unlock the fixation, pull the stretcher, when the legs of the loading stretchers out of the ambulance, just pull the handle, which would make the back legs become replacement, when the loading stretcher out of the ambulance, the front legs would become replacement automatically.
- Handrail: automatic inserting fixation, when let it down, pinch the handle and pull it out of the fixation.
- IV stand: ① Lift up the pole, and push it forward to make it fixed. Just do the opposite to make it down. ② The height of the IV stands can be adjusted, anticlockwise circumgyrate it, free the nut, the pole of the IV stands can be pulled out. When fixing the IV stands, just adjusting the IV stands to a higher or lower height, and desail circumgyrate the nut.
- The adjustment for lathe bed: the loading stretcher

are blessed with handles on two sides, pull the handle, the loading stretcher can be adjusted to 4 levels and fold back and forth according to practical needs. Meanwhile, the height of the lathe bed can also be adjusted, which can be achieved by adjusting the handles on each side at the same time.

## III. Fitting scope

It can be separate, suitable for emergency system. Can used for Ambulance car, mainly used in some. Areas that entering difficulty, such as and elevator narrow halls and the above stretcher can become wheelchair.

## IV. Maintenance

- Keeps it clean, antirust included.
- Make sure all the parts tighten together firmly.

## V. Attention in Particular

When transfer patient, make sure close the insurance device, upgrade and lock Fix the belts.

## VI. Storage and transportation

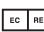
- This product must be kept in the condition of dry, antirust.
- Can be transported by traditional vehicles.
- When storing or transporting, stack as the guide on the package, stand upside down is prohibited.

## VII. Packing and Accessory

- This stretcher is packed within carton as a whole.
- Accessory: eligibility certificate as well as illumination paper.

## GIMA WARRANTY TERMS

The Gima -12month standard B2B warranty applies.

 REF	Product code	 LOT	Lot number		Consult instructions for use
	Manufacturer		Medical Device compliant with Regulation (EU) 745/2017		Keep away from sunlight
	Date of manufacture		Caution: read instructions (warnings) carefully		Keep in a cool, dry place
 MD	Medical Device		Authorized representative in the European community		Unique Device Identifier
	Imported by:				