



PROFESSIONAL MEDICAL PRODUCTS

---

## ***CARROZZINA BARELLA WHEELCHAIR STRETCHER***

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

### I. Technical Features:

- Product Size (LxWxH):  
High Position: 196x60x88 cm  
Low Position: 196x60x27 cm
- Packing Size (1pc): 199x60x28 cm
- Net Weight: 34 kg - Gross Weight: 40 kg
- Maximum angle of Back: 85°
- Loading Bearing: less than 159 kg

### II. How to use:

The stretcher trolley can be converted into a chair; the angle of the stretcher can be adjusted according to your different requirements. It is quite suitable to transfer the patients in the limited place, such as lift in hospital, ambulance, city road etc. It is made of Aluminum Alloy materials. It is characterized by its being anti-corrosion, use-safely and easy for sterilization and cleaning.

### III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

### IV. Cautions:














- When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
- in general operation, be careful not to out torn the stretcher surface.

### V. Maintenance:

- Keep clean as routine (incl. sterilization).
- Often check if any parts loose or not.

### VI. Storage & transportation:

- Store this product in the place of damp proof & non-corrosion.
- Common transporting vehicle can carry this product.
- Upside down is prohibited during transportation & storage. Stack the goods as per the instructions printed on the package.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		Unique device identifier
	Imported by				

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

### GIMA 34070



**Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd**  
No.999 Hongqi Road, Changyinsha  
Modern Agricultural Demonstration Area,  
Zhangjiagang City, Jiangsu, 215623, China  
Made in China

**REF**
**TD010132**


SUNGO Europe B.V.  
Fascinatio Boulevard 522, Unit 1.7, 2909VA  
Capelle aan den IJssel, The Netherlands



**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

