

BARELLA MULTIPOSIZIONE AUTOMATICA

FULL AUTOMATIC MULTIPOSITION STRETCHER

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

I. Technical Features:

- Product Size (LxWxH):
High Position: 198x54x94 cm
Low Position: 198x54x35 cm
- Packing Size (1pc): 200x60x35 cm
- Net Weight: 39 kg - Gross Weight: 45 kg
- Maximum angle of Back: 85°
- Loading Bearing: 159 kg

II. How to use:

The stretcher trolley can be used with one person push inside ambulance and no need press any button, just push enough, it's full automatic; the angle of the stretcher can be adjusted according to your different requirements. It is quite suitable to transfer the patients in the limited place, such as lift in hospital, ambulance, city road etc. Structure in high-strength aluminium alloy. It is characterized by its being anti-corrosion, safe for use and easy for sterilization and cleaning.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

IV. Cautions:












- When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
- In general operation, be careful not to out torn the stretcher surface.

V. Maintenance:

- Keep clean as routine (incl. sterilization).
- Often check if any parts loose or not.


VI. Storage & transportation:


- Store this product in the place of damp proof & non-corrosion.
- Common transporting vehicle can carry this product.
- Upside down is prohibited during transportation & storage. Stack the goods as per the instructions printed on the package.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

 TD010131G (Gima34072)

 Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd
No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made in China

 SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherland

Importato da / Imported by:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

