



GIMA

PROFESSIONAL MEDICAL PRODUCTS

BARELLA CARROZZINA VALUE - 4 RUOTE
VALUE WHEELCHAIR STRETCHER - 4 WHEELS

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

I. Technical Parameter:

- 1.fold Size: 92 x 52 x 16 cm
- 2.fold into wheelchair: *backrest height*: 91 cm - width: 50 cm
seat: height: 49 cm - width: 50 cm
- 3.Net weight: 8 kgs

II. Operating Method:

- 1.This model is equipped with high quality aluminum alloy as well as Oxford leather. To make the carrying of patient more safety, belts are equipped.
- 2.Open the wheelchair stretcher and pull part 4 and press it down,
- 3.after settled, then press down part 2, to the standard point, the wheelchair will become locked automatically.
- 4.Press the button on part 2, the length of the handle can be pulling out and pushing back at will. Press part 3, the handle at the back part can be folded.
- 5.When folding the wheelchair stretcher, lift part 1, step on one of the rails in front of the castors, meanwhile, lift part 4, and the stretcher can be folded.

III. Fitting Scope:

This model is mainly used in areas with difficult traffic or entry such as narrow halls, gateways, cloisters or several turnings.

IV. Maintenance:

- 1.Keep it clean, antiseptis included.
- 2.Check whether any part of the stretcher become loosens.

V. Attention:













When carrying the patient, insure every part works well and fasten the belt.

VI. Storage & transportation:

- 1.This product must be stored in the environment of dry and no erode.
- 2.This product can be transported by every day transporting vehicle.

III. Packing and accessories:

- 1.This product is packed in a carton as a whole.
- 2.Accessories: eligibility certificate as well as illumination paper.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		Unique identifier
	Imported by				

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.



Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd
No.999 Hongqi Road, Changyinsha
Modern Agricultural Demonstration Area,
Zhangjiagang City, Jiangsu, 215623, China
Made in China

Gima 34073



TD010114A



SUNGO Europe B.V.
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

