



**GIMA**

PROFESSIONAL MEDICAL PRODUCTS

---

***BARELLA PIEGHEVOLE IN 2  
STRETCHER - FOLDABLE IN 2***

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

## II. Technical Features

1. Product size (L x W x H): 189 x 48 x 4 cm
2. Folded size (L x W x H): 95 x 48 x 9 cm
3. Packing size (2pcs/carton): 96 x 50 x 18 cm
4. Net Weight: 4 kg Gross Weight: 9 kg
5. Load bearing: less than 159 kg

## II. How to use

This separable first-aid folding stretcher is mainly used for transporting patients with fractures. Both ends of the stretcher have hinge-type clutches to separate the stretcher into a right and left part. This allows for the patient to be quickly and gently placed onto the stretcher, operation table or bed. The stretcher is then withdrawn from beneath the patient.

## III. Applied for

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

## IV. Cautions












1. When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
2. In general operation, be careful not to out torn the stretcher surface.

## V. Maintenance

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

## VI. Storage & transportation


1. Store this product in the place of damp proof & non-corrosion.
2. Common transporting vehicle can carry this product.

	Manufacturer		Keep in a cool, dry place		Medical Device compliant with Regulation (EU) 2017/745
	Product code		Keep away from sunlight		Caution: read instructions (warnings) carefully
	Lot number		Medical Device		Authorized representative in the European community
	Consult instructions for use		Date of manufacture		

## GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

### TD01011 (GIMA 34076)

 Zhangjiagang Tengda Machinery Manufacturer Co.,Ltd  
No.999 Hongqi Road, Changyinsha  
Modern Agricultural Demonstration Area,  
Zhangjiagang City, Jiangsu, 215623, China  
Made in China

 SUNGO Europe B.V.  
Fascinatio Boulevard 522, Unit 1.7, 2909VA Capelle aan den IJssel, The Netherland

Importato da / Imported by:  
**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

