


# COPERTA ISOTERMICA

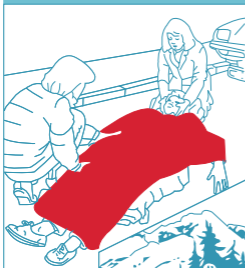

# COUVERTURE DE SURVIE



# RESCUE BLANKET

# RETTUNGSDECKE MANTA DE RESCATE

2100 x 1600 mm.

Film di poliestere metallizzato

Metallized polyester film

Film polyester métallisé

Metallisierte Polyesterfolie

Film de poliester metalizado

Filme de poliéster metalizado

Folie de poliester metalizată

Metalliserad polyesterfilm

Metallisoitu polyesterikalvo

Fémbevonatú poliészter fólia

Metalizirana poliesterska folija

Επιμεταλλωμένη μεμβράνη πολυεστέρα

طبقة من البوليستر المععدن

REF 34085

LOT 000053



02-2025



03-2030

UDI



(01)08023279340853  
(10)000053  
(11)250200  
(17)300300



Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
www.gimaitaly.com - Made in China

**GIMA**

www.gimaitaly.com



8 023279 340853 >

Rev. 3-04.25

**GB - APPLICATION** The reflecting material permits the body to be protected against the sun rays and undercooling. Use in the case of **ACCIDENT, WIND, RAIN OR HEAT. INSTRUCTIONS FOR USE** Wrap injured person in rescue sheet. **Golden** side up to protect against undercooling; **silver** side up to protect against heat.

**CAUTIONS** • The damp or musty products shall not be used. • For single use only, re-use is not allowed. • Don't store at extreme temperatures. • Dispose of safely after use to avoid risk of infection. • Report any serious incident in relation to the device to the manufacturer and the competent authority of the State member in which the user and / or patient is established. **STORAGE AND TRANSPORTATION** • The device shall be stored at a relative humidity of less than 80%, the storage environment shall be kept non-corrosive gases, dry, avoiding sunlight, well-ventilated conditions, and clean. • The device shall be