

## INSTRUCTION OF USE

### 1. General warnings •

For correct use of the device, refer to this manual • The product must be used only by qualified personnel • Check that the product is delivered to you intact and keep it packaged away from heat sources • The useful life of the device is determined by the wear of non-repairable and / or replaceable parts • Store in dry places • Always pay attention to the presence of children

### 2. Before each use •

Always check the state of wear of the parts in order to guarantee the use of the device in total safety for people and things

### 3. Warnings for use •

Use the seat belt when necessary • Periodically check the stability and sturdiness of the product. Replace it immediately if it is broken. • Do not use with patients whose weight exceeds that indicated; guaranteed capacity 120 kg • If in doubt about the use or maintenance of the product, contact the retailer. • To avoid any risks, keep the device out of the reach of children.

### 4. Instructions •

Opening the stretcher • Correctly move the patient onto the stretcher • The presence of handles allows the patient to be lifted by grasping the stretcher from several sides.

### 5. Maintenance .

The devices at the time of placing on the market are carefully checked and bear the CE mark. In case of repairs, only original spare parts and accessories must be used.

6. It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities

We declare that the device complies with MDR 745/2017 "Medical Devices"



PVS SPA VIA LEONARDO DA VINCI N.18  
20051 CASSINA DE PECCHI (MI) ITALY

---

### PVS SpA

20060 CASSINA DE PECCHI (MI)  
Via Leonardo da Vinci, 18

+39 029160011  
+39 0291600310  
+39 0291600149

Email: [info@pvs-spa.it](mailto:info@pvs-spa.it) Web: [www.pvs-spa.it](http://www.pvs-spa.it)

cod.fiscale part.IVA/VAT IT 06532250153  
cap.soc. Euro 619.748,28 i.v.  
Reg.imprese MI n. 215184  
R.E.A. MI n. 1103165

---

**PVS SpA**

20060 CASSINA DE PECCHI (MI)  
Via Leonardo da Vinci, 18

 +39 029160011  
 +39 0291600310  
 +39 0291600149

Email: [info@pvs-spa.it](mailto:info@pvs-spa.it) Web: [www.pvs-spa.it](http://www.pvs-spa.it)

cod.fiscale part.IVA/VAT IT 06532250153  
cap.soc. Euro 619.748,28 i.v.  
Reg.imprese MI n. 215184  
R.E.A. MI n. 1103165