

**SCOOP STRETCHER**  
Users' Manual



1. Product features

The scoop stretcher is made of aluminum alloy etc., which is light, beautiful, and easy to operate. The stretcher can be separated into left and right parts when in use, and the length of the stretcher can be adjusted according to the length of the wounded person's body.

2. Main purpose and scope

This product is mainly used to transfer fractured and paralyzed patients with ambulances, and can also be used when transferring patients in other places such as hospitals. This product is used by emergency personnel.

3. Structure composition

The scoop stretcher is composed of the stretcher main body, guard plate, small guard plate, hinge joint, telescopic rod, lock buckle, safety belt and so on. see Fig. 1)

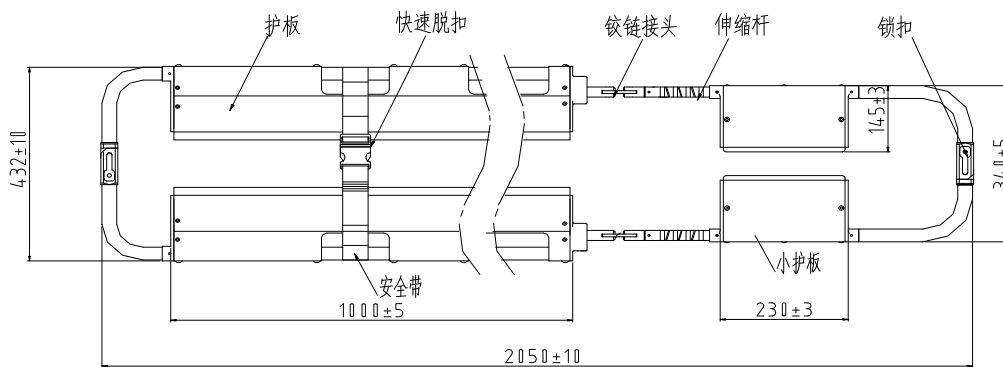


Fig. (1)


4. The basic parameters (See Table 1)

Tabel 1 Basic parameters of scoop stretcher

Unit: mm

| No. | Model  | Product Size |        |        | Folded Size |        |        | Four stage adjustment length |
|-----|--------|--------------|--------|--------|-------------|--------|--------|------------------------------|
|     |        | Length       | Width  | Height | Length      | Width  | Height |                              |
| 1   | WSX-E3 | 1710±10      | 432±10 | 85±5   | 1210±10     | 432±10 | 95±5   | 1710、1820、1935、2050          |

5. Composition and representative significance of model

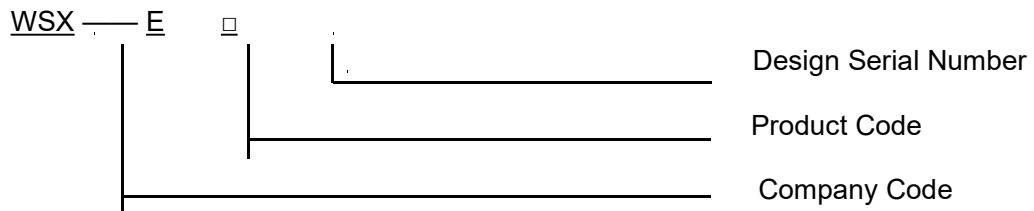

- 6. Capacity: 159kg
- 7. Instructions for use
- 7.1 separation operation

Turn the small guard board to the horizontal shape, push the telescopic rod into the tube sleeve of the guard board, adjust the required length, and then press the locking pin handle (positioning block) to make the positioning pin enter the corresponding positioning hole; Press down the lock catch at both ends to separate the stretcher from the left and right. Shovel the stretcher from both sides of the patient lying flat to the bottom of the hip to ensure that the patient's head is at the wide end of the stretcher.

Steps: Fig. (2) turn over the small guard board and push forward the telescopic rod;  
 Fig. (3) pull up the locking pin handle (locating block) upward;  
 Fig. (4) press the lock catch at both ends to separate the stretcher.



Fig. (2)



Fig. (3)



Fig. (4)

### 7.2 folding operation

Lift up the positioning pin; pull the telescopic rod outward to the longest; longitudinally fold (also can be separated from left and right and then folded) for easy storage.

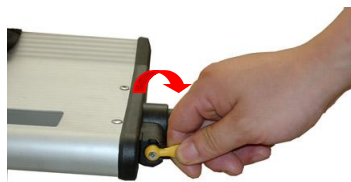


Fig. (5)



Fig. (6)

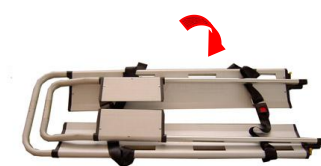


Fig. (7)

Operation steps: I Pull up the lock pin handle II Pull the telescopic rod outward to the longest III Fold longitudinally

### 8. Maintenance and disinfection

This product is a reusable medical device. See 8.1 and 8.2 for cleaning and disinfection methods.

8.1 Use a face cloth or soft brush, dipped in a little clean water and neutral detergent, and wipe it with a dry face cloth.

8.2 Use neutral disinfectant spray or wipe disinfection, and iodine and phenol disinfectants cannot be used.

8.3 Check every week, tighten the screws and rivets of all parts.

8.4 Lubricate the rotating parts every month.

#### 9. Matters needing attention

9.1 At least two people in front and back must operate the stretcher. Safety belts must be worn when rescuing the wounded. It is strictly forbidden to operate this stretcher brutally or illegally, otherwise the stretcher may be damaged!

9.2 It is very necessary to use unmanned stretcher for operation drill. Personnel who have not studied and drilled above are not allowed to operate this stretcher!

9.3 When the stretcher is folded, be sure to pull the small end outward (pull out the hinge joint completely) before folding, otherwise the stretcher may be damaged.

9.4 When lying on the stretcher, the wounded must keep their body on the wide side of the stretcher, and it is strictly forbidden to turn them upside down.

9.5 During first aid, make sure that the positioning pin is stuck in the positioning hole of the telescopic tube, otherwise the small end of the stretcher may slip out of the big end when carrying the injured, which may injure the patient and damage the stretcher.

9.6 This product is only suitable for transporting the wounded and sick. It is strictly forbidden to load or transport other objects with overweight, and it is strictly forbidden to collide with ambulances or other objects violently.

9.7 It is strictly forbidden to flush with high-pressure water gun.

9.8 Regularly check whether the bolts of each part are loose or not, and add lubricating oil to each movable part.

9.9 If any parts are damaged, please contact our company. Using non-our company parts will bring safety hazards to the stretcher!

9.10 Modifying this stretcher privately will bring potential safety hazards to stretcher use!

9.11 The damage to the stretcher parts, the operation method and the use environment may cause the safety hazard of the stretcher. Once the stretcher is in a situation that may cause the patient to fall or be injured, the operator before and after the stretcher must support the stretcher forcefully to ensure the safety of the patient.

9.12 If accident occur during use of the stretcher, please report to the manufacturer and the competent authority of the Member State where the user and/or patient is located.

10. Contraindications: None.

11. The warranty period of this product is one year. Damage caused by improper use by the user or force majeure is not covered. During the warranty period, the company provides repair parts free of charge. After the warranty period, the company will also be responsible for product repair/parts replacement and charge corresponding fees. It is not responsible for other quality assurance responsibilities beyond the warranty.

12. General transportation can be used when shipping or according to the order contract. During the transportation, avoid violent collision, sun and rain; the packaged scoop stretcher should be stored in a well-ventilated room without corrosive gas. , Avoid mixing with toxic and harmful substances.

#### 13. Contact information and after-sales service

##### **Manufacturer**

Name: Jiangyin Everise Medical Equipment Co.,Ltd

Add: 1001 Chengyang Road, Jiangyin City, Jiangsu, China

Tel: 0510-86906151

Fax: 0510-86900111

E-mail: [gdj@everisegroup.com](mailto:gdj@everisegroup.com)

[www.Everisemedical.com](http://www.Everisemedical.com)

|    |     |
|----|-----|
| EC | REP |
|----|-----|

**Authorised representative in the European Community**

Name: Share Info GmbH

Address: Heerdter Lohweg 83, 40549 DÜsseldorf

Tel:0049 1795 6665 08

Fax: 0049 1795 6665 08

E-mail: Info@share-info.com












Packing list

| Name        | Quantity       |
|-------------|----------------|
| Stretcher   | 1              |
| Safety belt | At least 2 pcs |

Drafting date: 2021.03.27

Release date: 2021.04.02

Version No: A/1

|  |  |   |   |
|--|--|---|---|
|   | " CE " marking                         |    | Caution   |
|    | Keep away from sunlight                |   | Keep dry  |
|   | Batch code                             |  | Consult instructions for use                        |
|   | Date of manufacture                    |  | Authorized representative in the European Community |
| <br> | Manufacturer<br><br><b>item number</b> |  | <b>Medical Device</b>                               |

Production date: see product label

**Jiangyin Everise Medical Equipment Co. Ltd.**