

# QUICK SPLINT

SET

REF PNE244



**I** ISTRUZIONI PER L'USO

**GB** INSTRUCTIONS

**F** MODE DE EMPLOI

**E** MODO DE EMPLEO

**P** INSTRUÇÕES

 PVS SPA VIA LEONARDO DA VINCI 18 20051 CASSINA DE PECCHI (MI) ITALY

2021-7-30(A/1)  
EU Regulation 2017/745(MDR)





## ENGLISH

**INDICATIONS :** Inflatable QUICK SPLINT are used to immobilise broken or injured limbs. This system is specially indicated to limit all movements during transportation and therefore enhances patient comfort and exerts a uniform pressure around the injury to control bleeding. QUICK SPLINT material permit the X ray procedures and therefore can remain in position throughout the X ray hence minimising the patient discomfort.

**SPECIFICATIONS :** QUICK SPLINT are produced in PVC , with zip for closed and open the splint . They should be inflated through the pump  
Inflatable QUICK SPLINT are available in four sizes :


1 - LONG LEG

2 - ANCKLE-FOOT

3 - LONG ARM

4 - SHORT ARM - HAND

5 - PUMP

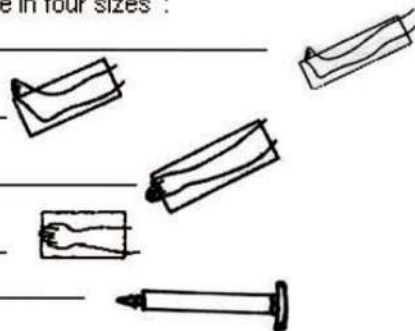

**IISTRUCTIONS : How to fit the inflatable QUICK SPLIINT:** put and wrap it around the injured limb and close the zipper. Blow the air through the pump  
**How to liberate the inflatable QUICK SPLIINT :** open the zipper , take out the air from it pressing the valve and take out the injured limb.



**CAUTION :** should be applied by skilled staff.  
Please stop using if the material damage or air leakage is found.

**CLEANNING IISTRUCTIONS :** after using the inflatable QUICK SPLINT , it should be accurately washed with water and soap. It can also be disinfected by a bactericide solution better if manufactured in according to new MDR 745/2017.

**STORAGE :** keep in a cool and dust free side

It is necessary to report any serious accident occurring in relation to the DM provided by PVS SPA to the Manufacturer and the competent authorities