

TELO SALMA PVC - NERO - PORTATA 150 Kg

BODY BAG PVC - BLACK - LOAD 150 Kg

ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

PRODUCT INTRODUCTION

Size: 210 x 90 cm

- this product is used for bags containing corpses.
- it is made of PVC, which is firm and sealed.
- the load bearing capacity of the product is 150 kg.
- this product is disposable.

USE STEPS OF THE PRODUCT

1. First open the zipper on the outer layer of the bag, and then open the zipper on the inner layer.
2. Load the body and zip up the inner and outer layers.
3. Carry away.

WARNING

When carrying the corpse with infection source, the carrying personnel should have appropriate protection.










STORAGE AND MAINTENANCE OF PRODUCTS

Before the product is used, the storage environment temperature is between - 20°C and 60°C.

USE SAFETY WARNING

1. Be sure to read the manual carefully before use.
2. The product needs professional personnel or qualified personnel after first aid training.

SYMBOLS

	Product code
	Caution: read instructions (warnings) carefully
	Medical Device
	Consult instructions for use
	Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745
	Keep in a cool, dry place
	GB Keep away from sunlight
	Lot number

REF 34116



Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
 Made in China

