



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. – Via Marconi, 1 – 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 – fax. 199 400 403
Export: tel. +39 02 953854209/221/225 – fax. +39 02 95380056
gima@gimaitaly.com – export@gimaitaly.com
www.gimaitaly.com

Padded strip stick for fingers

OPERATOR'S MANUAL



ATTENTION: The operators must carefully read and completely understand the present manual before using the product



REF 04SD000A (Gima code 34120) – 04SD000B (Gima code 34121) - 04SD000C (Gima code 34122)



Megapharma Ospedaliera s.r.l.
Via Asolana, 26/b · I-31017 Crespano del Grappa (TV)
Tel. +39 0423 538746 · Fax. +39 0423 538748

USE:

- 1 – Determine the necessary length, then cut the stick. (the size can be found by doubling the length between the metacarpus and the tip of the finger to immobilize it);
- 2 – Position an end of the stick on the back of the hand, follow the length of the finger then bend it so that the other end reaches the palm of the hand;
- 3 – If necessary bend finger to obtain the correct position of the articulation; 4 – Fix the stick with a bandage.

WARNING: Use under medical control.

MATERIALS:

Aluminum, styrolbutadiene 100% adhesive.

MEASUREMENTS:

A 50 x 1,5 cm, B 50 x 1,9 cm, C 50 x 2,4 cm. Thickness 1.2 mm.

INDICATIONS:

Immobilization and protection of fingers and inter-phalanx articulations, in case of fracture.

HOW TO KEEP:

Keep in a dry place, far from direct sun light and atmospheric.