# **GIMA**

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

---

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### CAUTION

1. Read the instructions for use before use.
2. Federal Law (USA) restricts this device to sale by or on the order of a physician.
3. Patient should be monitored constantly whenever this device is in use.

### DIRECTIONS FOR USE

1. Check for clear airflow passage prior to use.
2. Connect to breathing circuit or resuscitation device with appropriate fitting.
3. Make sure all connections are compatible and fit firmly.
4. Hand-hold or secure with head strap.

### CLEANING AND STERILIZATION

#### 1. Cleaning (Manual)

- a. Rinse neutral detergent to all surface of mask.
- b. Use brush to clean mask.
- c. Wash mask under the cleaning running water until mask is cleaned completely.
- d. Dry it in the room temperature or 50°C oven.

#### 2. Sterilize the components using one of the following methods:

- a. Autoclave (max 121°C following sterilizer's instructions)
- b. Most common disinfecting solutions which are used for equipment coming into contact with the patient. Rinse thoroughly with water after sterilization.

### STORAGE

1. Store in clean dry conditions. Away from heat and light.
2. The temperature for storage and transport is between -20°C and 60°C.

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REF

34223 - 34224 - 34225

Gima S.p.A.  
Via Marconi, 1  
20060 Gessate (MI) Italy  
Made in Taiwan

0476