

## INSTRUCTION SHEET

### GUEDEL AIRWAYS

REF 34431 Guedel airway 40mm

REF 34432 Guedel airway 50mm

REF 34433 Guedel airway 60mm

REF 34434 Guedel airway 70mm

REF 34435 Guedel airway 80mm

REF 34436 Guedel airway 90mm

REF 34437 Guedel airway 100mm

REF 34438 Guedel airway 110mm

REF 34383 Guedel airway 120mm

REF 34439 Mixed Guedel Airways (1pc REF 34431 + 1pc REF 34432 + 1pc REF 34433 + 2pc REF 34434 + 2pc REF 34435 + 1pc REF 34436 + 1pc REF 34437 + 1pc REF 34438)

\*FOR SINGLE PATIENT USE \* DISPOSABLE  
Directions for Use:



STERILE EO

1. Choice the correct size (the correct size for a patient is determined by the distance between the corner of the mouth and the tragus).
2. Lubricated and inserted with the curvature initially facing upwards and then rotated halfway through so as to follow the curvature of the tongue.
3. In edentulous patients, it may also be inserted right from the outset with its curvature along the curvature of the tongue.
4. Once inserted, it is very important to support the jaw to maintain patency of the airway.

#### Caution:

- Read this instruction manual carefully before use.
- This product should be used by trained personnel.

Storage and transport temperature: -20°C to +60°C (-4°F to 140°F).

LOT



GIMA S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italia - [www.gimaitaly.com](http://www.gimaitaly.com)

MADE IN TAIWAN

CE 0476