GIMA

PROFESSIONAL MEDICAL PRODUCTS

COLLARI COLLARS COLLIERS COLLARES COLARES ΠΕΡΙΛΑΙΜΙΑ ZERVIKALSTÜTZEN KRAGEN KRAGAR

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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Made in Italy

FEATURES

First-aid collars GIMA are conceived to guarantee the utmost efficiency, comfort and safety as far as support and correction in first-aid conditions concerns. They are also equipped with an anatomical chin rest and a Velcro fastener, which makes the patient be more comfortable with the correct regulation. The product or its components cannot be used for purposes different from the one specified in the present manual.

PRESCRIPTIONS

The collar GIMA can be used only upon medical prescription.

Before starting the therapy, it is advisable to identify the most suitable collar to meet each individual requirement and in case of necessity to customise it by means of adjustments in compliance with the doctor's instructions.

In presence of plagues and infectious states, apply to a doctor for a suitable treatment and ask for indications on the product cleaning!

Do not use the product if it is damaged. Apply to your retailer.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

USE

The device must be correctly positioned on both sides and the patient must feel completely comfortable; furthermore it must not be subject to modifications of its external appearance as long as it is used. The device must be installed by the patient itself, no assistance is needed. In case the patient is not able to position the product by himself or herself, it is necessary to train the person assisting him or her on the correct and safe use of the collar.

In case the patient shows signs of hyperpressure or discomfort due to the device stiffness, immediately remove it until the situation has been completely normalised.

In case the subject himself or herself is not able to show such discomfort or undesirable effects deriving from the presence of the collar, carry out periodical inspections to check the situation.

In case of allergic reactions due to the contact of the collar with the skin (reddening, swelling, oedema, skin irritation, itching) or bacterial infections due to lack of hygiene, apply to the family doctor.

In case the patient's weight changes, it is necessary to change the collar size, apply to your doctor.

The doctor must inform the patient about the device usage times based on the type of pathology. The success of the application depends upon the patient's care to follow the doctor's instructions and the continuous contacts with him.

MAINTENANCE

The product can be washed with cold or warm running water and neutral soap. Do not use aggressive chemical products (bleach, ammonia, solvents...), which could alter the functionality of the product.

Do not expose the product to excessive mechanical stresses to avoid altering its functionality.

Carry out periodical controls to make sure that the device has not suffered from modifications, which could jeopardize its functionality.

The product life can vary according to the therapeutic application; to guarantee a longer life of the product it is recommended to correctly use, maintain and periodically check it.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

| | | | | | |
|--|--------------|---|--|---|---|
|  REF | Product code |  | Keep away from sunlight |  | Consult instructions for use |
|  LOT | Lot number |  | Keep in a cool, dry place |  | Caution: read instructions (warnings) carefully |
|  | Manufacturer |  | Medical Device compliant with Regulation (EU) 2017/745 |  | Medical Device |

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Gima S.p.A.