

INSTRUCTIONS FOR USE

STERILE ADHESIVE PLASTERS

PURPOSE OF USE

Medical device for the administration of medicinal substances.

INSTRUCTIONS FOR USE:

Wash your hands thoroughly

Open the package and remove the protective paper.

Center the absorbent compress on the wound, avoiding touching it, adapt the adhesive part to clean, dry skin.

Renew the dressing if necessary.

In case of reuse, infections may develop: consult a doctor.

LATEX FREE:

The device is latex free to prevent any risk of allergic reaction.

STERILIZATION METHOD:

Sterilized by Ethylene Oxide (EtO), residual amount of gas in accordance with the regulations.

DISPOSAL:

Disposal must be made in accordance with the applicable government regulations. Incineration or landfill disposal must be carried out under controlled conditions. The product that did not come in to contact with infectious agents or other hazardous materials, cannot be considered dangerous waste.

CONSERVATION:

Store in a cool dry place and far away from the any source of heat at a temperature.

These Instructions for Use note must be preserved by the user for the duration of the DM.



Health&Safety

RAYS SPA

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - www.rays.it - info@rays.it