



GIMA

PROFESSIONAL MEDICAL PRODUCTS

SUPPORTO A ROTELLE ROLLING STAND

REF 8030421049 (Gima 35148)

Importato da / Imported by:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Made in China



Instructions to User

In order to operate properly and get the best performance, please read these instructions carefully before using this rolling stand. This manual describes installation, operation, maintenance, transportation and storage.

Please keep this manual for future reference!



1 Features



Silent castors
with brakes



Basket with handle
and hook



Install Basket



Elevation angle
adjustable



Mounting
plate lock

2 Intended Use

This product is intended for transferring equipment such as Patient Monitor, ECG Monitor, etc. to help doctors work conveniently and effectively.

3 Photos



4 Cleaning and Disinfection

1. Remove the monitor before cleaning the rolling stand.
2. It is recommended to clean the rolling regularly. Only a non-corrosive cleanser is permitted. Dilute cleansing products as per the manufacturers instructions.
3. Use a cloth with alcohol or your choice of cleanser to wipe the surface of the rolling stand, and dry it with a clean cloth or leave to air-dry.

5 Dimensions and weight

Dimension: 503*461*H1050 mm
 Weight (including package): 10KG
 Dimension of package: 780*470*H185mm

6 Transportation and storage

1. Transportation
 The rolling stand should be left in original packaging when being transported and kept dry and avoided bumping.
2. Storage conditions:
 The rolling stand should be kept in a dry and well ventilated room free from dust and corrosive gases. If the stand is being stored in original packaging, stacking height should not exceed four layers, warehouse should be room temperature.

7 Packing List

Base kit *1 Mounting plate*1 Support pillar *1
 Castor *4 Installation tool *1

8 Troubleshooting

Problem	Analysis	Solution
Support pillar shakes	Fixing screws on the base may be loose	Check the screws and tighten if required
Elevation angle can be adjusted	Adjusting screws on the handle are too loose / tight	Adjust the tightness according to the requirements

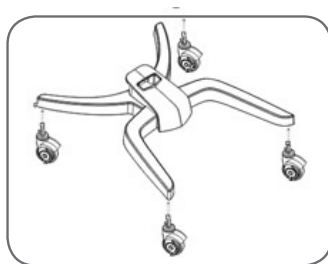
9 Installation

Part List

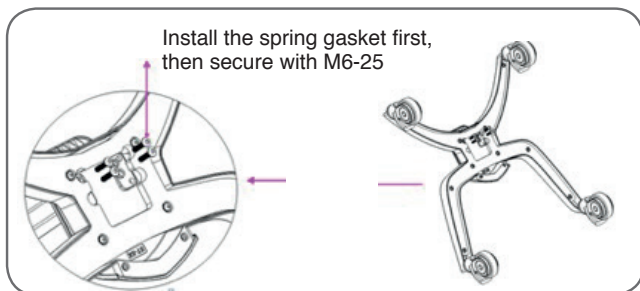
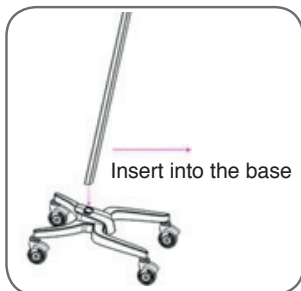
No	Part Name	Qty
1	Mounting plate	1
2	Storage Basket	1
3	Support Pillar	1
4	Base	1
5	Castor	4
6	Screw M4-10/12	8
7	Screw M5-16	5
8	Screw M6-25	4
9	3mm spanner	1
10	4mm spanner	1

Step 1:

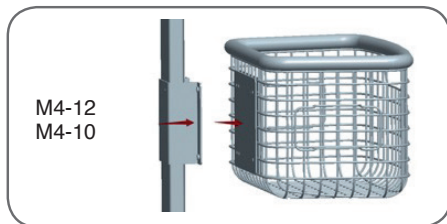
Fix four castors into the related holes on the base.

**Step 2:**

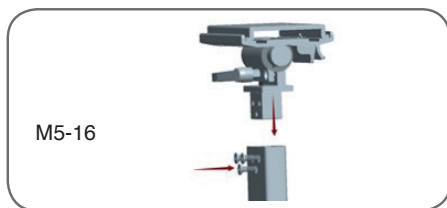
Insert the support pillar into the base and secure with M6-25 screws.

**Step 3**

Assemble the storage basket, and secure with M4-10 and M4-12 screws according to the following pictures

**Step 4**

Fix the bracket into the support pillar and secure with M5-16 screws.

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.