MANUALE D'USO
USER'S GUIDE

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Our glassbeads sterilizers are designed to allow a quick and safe sterilization of the surgical tools tip parts without envelope, in accordance with denominated method dry heat. The operating temperature 235°C assures the micro-organisms elimination quickly. In any case, the operator must take care to keep the obtained results because the instruments are not protected by envelopes. The sterilizers are equipped with :

- *electric immersion heater 60W or 150W (see technical data sheet)*
- *main switch "I/O"*
- *lights to control the status working of the equipment*
- *automatic thermostat to maintain a constant temperature*
- *fast protection fuse with removable box*
- *removable feeder with safety plug*
- *bimetallic protection thermostat for over-temperatures*



Gima Quick Plus 240 model is equipped with a thermometer which shows the rated sterilization temperature. The equipment safety is guaranteed by a design and a manufacturing observing the rules (e.g.: double thermic insulation, a very accurate electric insulation, protection fuses, feeder with safety plug, insulating material externally protected by a stainless steel plate, electric immersion heater, air circulation around the five sides of the equipment, switch and lights with safety marks, etc.).

The glassbeads removable nickel or anodized container has a limitation border to avoid any eventual glassbeads' leaks; quickly cleaned and recharged with glassbeads.

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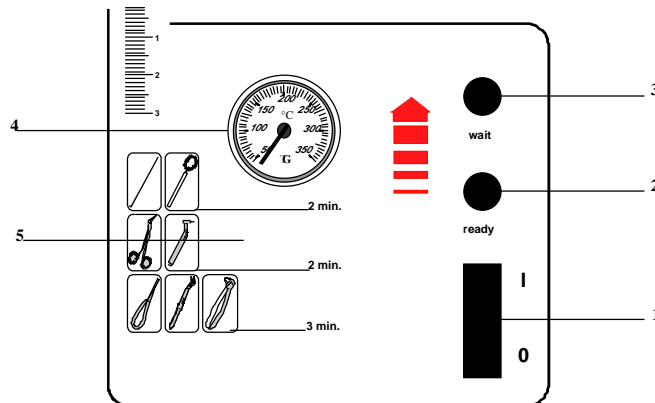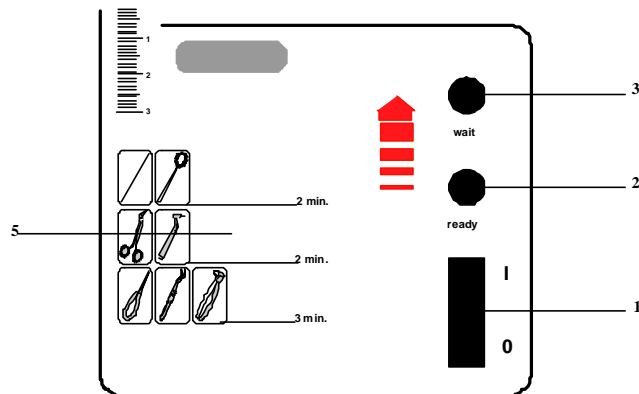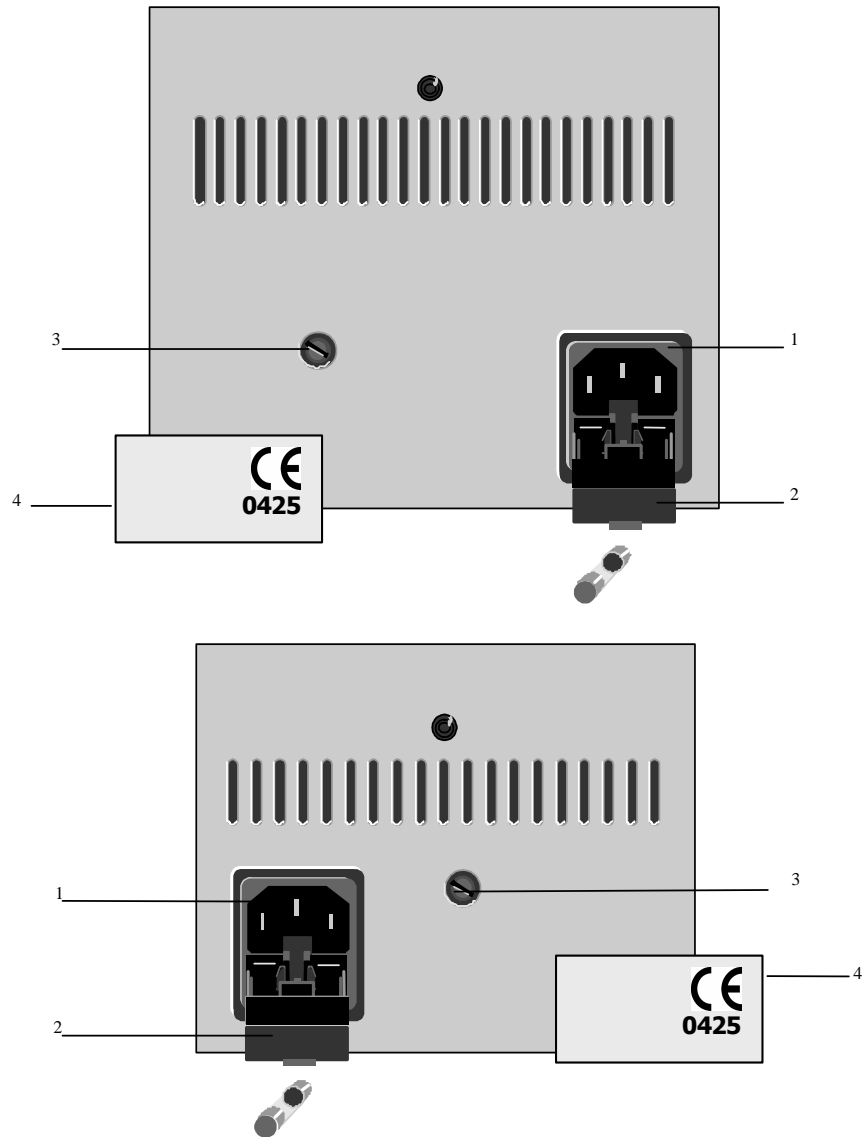
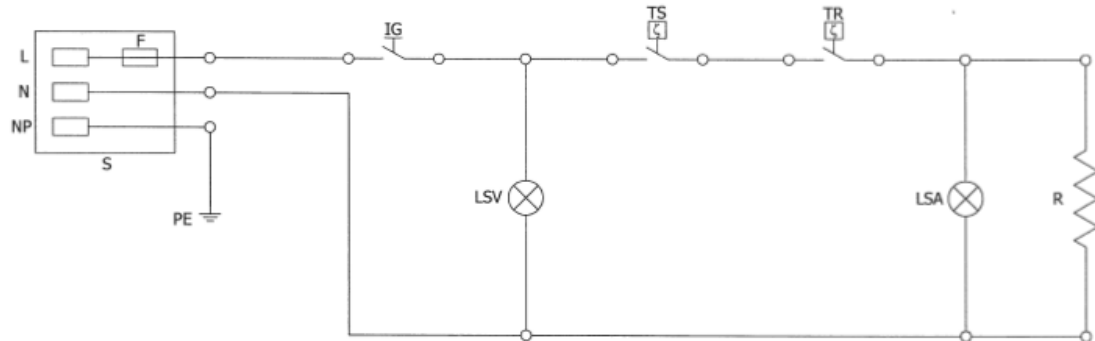
The sterilizer is shipped in an adequate package which requires the common handling cautions for fragile materials, as shown on the package itself. Use only the original package if the equipment must be returned to the manufacture.

Note : the cardboard package doesn't require particular waste disposal conditions.

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Remove the equipment from the package handling with care. Check the power supply system is correctly grounded. Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side. Keep the sterilizer at least 10cm. away from walls or others devices, to ensure an adequate ventilation. Do not place the sterilizer near heat sources, which could influence negatively the thermostat. During the installation it is essential to allow a free air circulation, to avoid the overheating of the sterilizer, taking care not to obstruct the airing slits placed on the bottom, sides and at the rear. Full the container with glassbeads but pay attention to put them 5-6mm. below the limitation border to avoid any eventual leaks. Insert the container inside its housing. Insert the feeder safety plug, at first into the equipment connection ("1" - fig.2), then insert it into the network. Press the switch ("1" - fig.1) on "I". Check the switch light and the orange lamp light up ("3" - fig.1).

Note : in case you find a damp package, it is recommended to leave the unit in a heated room to exclude humidity conditions of the components.

Note : a typical smell of heated ceramic fibre may be present at the first power on and remain for a short period of time.

Note : you can find the environmental required conditions (pressure, humidity, temperatures, etc.) inside the technical data page, as the reference rules.

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Press the switch ("1" - fig.1) on "I". Keep the upper cover closed until the use, to avoid accidental contacts. The equipment will take up 45min.approx. to reach the operating temperature (235°C inside the container). During the sterilizer's working, the orange and green lamps ("3" e "2" - fig.1) alternately light up and it is a normal situation, such as: during initial heating phase, it means a recovery of thermic inertia to avoid any over-temperatures – during full running use, it means a loss temperature under 235°C and the consequent resistance working to restore the sterilization ideal temperature. When the required temperature is reached, the sterilization times – according to the material - vary according to the directions of "Farmacopea Ufficiale" and of the medical devices manufacturers. The user must to value if the instruments' quality and characteristics, during the process, can exclude a deterioration of the products which are treated. It is important to specify the sterilization process is just only a part included in a more complex procedure which concerns the Decontamination, the Cleansing, the Sterilization and the Curing phases. The correct application of this procedure, the periodical activities of the chamber cleaning and the glassbeads control, can be exclude a crossed contamination. The Gima Quick Plus 240 model allows to check the sterilization temperature by the thermometer included inside the front panel ("4" - fig.1). About the sterilizers models without a thermometer it will be opportune a periodical check of the thermic performance to warrant the set up processes. When the working temperature is reached, the instruments can be sterilized introducing their tip parts into the glassbeads, in accordance with the required depth. The recommended sterilization time must be 2 minutes, not lower, as showed on Table "A".

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Make sure to place the instruments inside the central part of the beaker containing the glassbeads, where can be guaranteed an homogeneous and uniform temperature.

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Weigh with care if the parts of the Medical Device, which are exposed to the disinfection or sterilization treatment, are sufficient in relation to the use : states that this technique ensures the sterilization of the parts which are inserted among the glassbeads.

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About tools with small size, special pliers have to be used to avoid possible burns. Before the disinfection or sterilization process, all tools must be cleaned and dried. Thanks to the special materials used, the design characteristics and the air circulation system around five sides of the equipment, the sterilizer can be operative all working day. However, to obtain an electric power saving, it is recommended to keep the cover closed when the equipment is ready to use, but it is not working. A periodical control of the sterilization process is also recommended using the sterilization indicators or similar products on the market.

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Weigh the obtained results, further to a validation of the sterilization process.

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Before every maintenance and service, check the feeder is disconnected. The glassbeads must be periodically replaced (when they lost their transparency). To replace the glassbeads, take out the container when the sterilizer is cold. With adequate environment conditions our sterilizers need no particular maintenance, except a regular external parts' cleaning. It is recommended to clean the equipment with a soft cloth and water or neutral detergents. Check the feeder is in perfect conditions, at regular intervals. It is possible to replace the fuse, taking out the fuse box ("2" - fig.2) on the rear panel, in accordance with our technical data sheet. If the sterilizer doesn't work properly, contact the manufacturer or the authorized service. If any repairs are required, please return the equipment directly to the manufacturer or by the local distributor, using only the original package. However it is possible to contact specialist technicians, who must work in accordance with the general safety rules and accidents preventions of the country in which the equipment is used.

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No particular cautions. In any case, about the waste process, it must be applied the required procedures by the rule in force and the EU Directives. Even though the sterilizer doesn't fall inside the table of the devices provided of the Annex RAEE Directive, in any case the unit has been conceived with a selection of components according to such Directive.

N.B. *From the technical point of view, the life of the device is related to the characteristics of the components used to manufacture the device itself, and it is not related to its execution from Fast Assembler side.*

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This equipment must be only used from authorized personnel informed on the correct use, regarding especially the procedures of sterilization validation.

These sterilizers must be only used for their designed purpose : to sterilizer surgical instruments or tools in the medical and paramedical field.

The sterilizers must not be used for all treatments concerning contact lenses (Rule 15 - All.9 - Comma 1).

It is necessary to take all precautions to avoid unauthorized use. Do not modify, change or put out of order any electrical or thermal safety protection of the equipment. Check regularly the functionality of the light indicators which signal the feeding and the heating phases. When the sterilizer is operating, pay attention to the people near the equipment. The user must not attempt to carry out any repair unless authorized. Do not clean or repair the sterilizer while it is connected to an electrical system and be sure the sterilization chamber (beaker) is at room temperature. Check regularly if the power cable is damaged. When authorized technicians decide to replace spare parts, they must use original replacements and moreover they must adhere to the manufacturer's information and technical characteristics. In case of any doubt about safety rules, please contact the manufacturer or local distributor before using the equipment.

However use the sterilizer in accordance with the general safety and accident prevention rules of the country in which the equipment is installed.

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To fill the beaker use the glassbeads supplied by the manufacturer only. Avoid the accidental contact with liquids. The sterilizer does not have to be used in an environment where the explosive gases may be present.

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Place the sterilizer on a level surface so that air can freely circulate under the equipment bottom side. Keep the sterilizer at least 10cm. away from walls or others devices, to ensure an adequate ventilation. Do not place the sterilizer near heat sources, which could influence negatively the thermostat. Keep the door closed when the sterilization process is not running. Do not place any object on the upper part of the equipment. About tools with small size, special pliers have to be used. Do not clean, move or repair the sterilizer while it is connected to an electrical system and when the sterilization is still hot.

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When the sterilizer is operating, do not put hands near the beaker – the glassbeads have a 235°C temperature.

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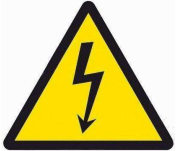
The sterilizer must be used only for its designed purpose. Improper use of the equipment is to the risk and sole responsibility of the user.

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The warranty covers only equipment with production and natural defects as indicated in the certificate that comes with the device.

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