

N Tape with change color indicator (indicators Type 1

e1)

DRY

DESCRIPTION:

Ref. TT 6120: for sterilization dry heat cycles
Change color from green to brown

Single sided dry heat sterilization indicator tape for packing purposes, with excellent bond to almost all material. The ink printed on the surface of the tape changes its color from green to brown, to show visibly that the product has undergone a dry heat cycle of sterilization. **This product does not have the aim of validating the sterilization process.**

/ OPERATING INSTRUCTIONS:

Dry heat cycles: during the sterilization process, indicator printed on the tape changes its color:
from
green to brown.

MINIMUM STERILIZATION CONDITIONS (according to UNI EN ISO 11140-1:2014 Type 1 e1)

Dry heat cycles: at 160°C minimum sterilization time is 40 min

STORAGE:

Before use: before its use, store product at ambient temperature, away from direct sunlight, heat sources or too high humidity.


After use: product is stable after sterilization and it does not require special storage conditions

EXPIRY DATE:

If stored according to instructions, expiry time is 30 months from production date.

DECLARATION:

At normal sterilization conditions, the indicators of EURO TT 6120 turn from green to brown.

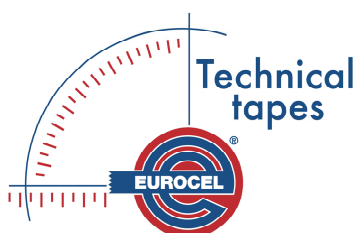

Produttore / Manufacturer:
Sicad S.p.A.
Via Caduti della Liberazione, 57
21040 Uboldo (VA)