

INDICATIONS FOR USE

Lead free indicator tape for steam sterilization
Indicators Type 1 e1 **STEAM** according to ISO 11140-1:2014

DESCRIPTION / INDICATIONS FOR USE

Ref. 6110

Single sided lead free indicator tape to close packaging used in steam sterilization processes. Resistant to high temperatures and humidity, it has an excellent bond to almost all materials, such as cloths, metals, plastics, no woven and papers. **This product does not have the aim of validating the sterilization process.**

STERILIZATION INSTRUCTIONS

Indicator according to minimum process conditions in line with ISO 11140-1:2014 for parts applicable to Type 1 e1:

<i>Initial color</i>	134°C - 2 minuti <i>light green</i>	da verde chiaro a verde medio / scuro
<i>Steam cycles</i>	121°C - 10 minutes 134°C - 2 minutes	<i>from light to dark green</i> <i>from light to medium / dark green</i>

Under more common use conditions in autoclave the indicator color turns to dark green

/ STORAGE CONDITIONS

Before use: product must be stored at room temperature (15-25°C), away from direct sunlight, heat sources or too high humidity (<65% relative humidity), protected in the original packaging

After use: product is stable after sterilization, the indicator will not change visually within 6 months when stored at above conditions

EXPIRY DATE

If stored according to instructions, expiry time is 30 months from production date

DECLARATION OF CONFORMITY

Indicators are approved according to ISO 11140-1:2014 for parts applicable to Type 1 e1



Produttore / Manufacturer:
Sicad S.p.A.
Via Caduti della Liberazione, 57
21040 Uboldo (VA)