

**CORESCRUB DRY / STERILE**

Scrub brush / with sponge used for hand cleansing disinfection.

**Description**

Dry scrub brush, single use, sterile, used for hand and forearm disinfection of sanitary personnel. Does not contain any antiseptic soap solution.

**Indications**

Device made off a nylon scrub brush, a sponge glued at the base and comes provided with a nail cleaner. Used for disinfection of sanitary personnel and surgeons' nails, fingers, hands and forearms, participating in surgery. Exclusively for external use and on healthy and unhurt skin. Used in preoperative areas only.

**Precaution**


Avoid contact with eyes once impregnated with disinfectant solution. Do not swallow. Do not use any package which presents defect sealing. DO NOT USE ON ANY OTHER PARTS OF THE BODY WHICH ARE NOT INDICATED IN THE PRESENT FILE.

**End-user**


Operating room, Urgency, Reanimation centres , dentists,...

**Warehouse**

To be protected from humidity and direct exposure to sunlight. Must be stored in clean environmental areas as by local regulation.

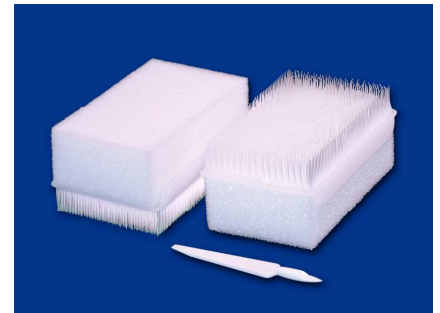

**TECHNICAL PROPERTIES**

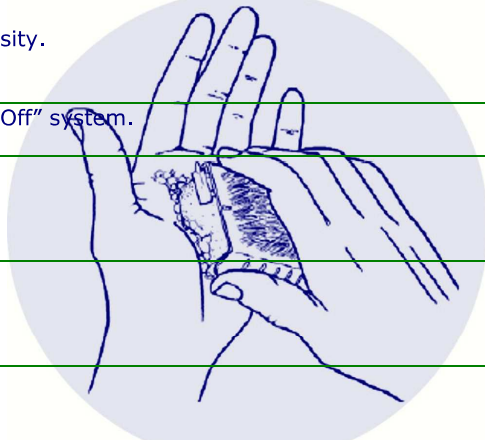Material used	BRUSH: LPD polyethylene. SPONGE: Polyurethane -medium high density. NAIL CLEANER: PET	
Packing	Individually packed, thermo sealed, "Peel Off" system.	
Brush dimensions	Lenght 80 mm. Wide 50 mm. Height of brush (hair) 12 mm.	
Sponge dimensions	lenght 80 mm. Wide 48 mm. Height 24 mm.	
Nail cleaner dimensions	Largo 65 mm. Ancho 5 to 10 mm.	
Packaging	NEW: January 2013 - 9 dispensers of 30 pieces; total: 270 pieces per outer box.CS-270-D	
Expiry date	Sterile product. Expiry 5 years.	

**Instructions of use:**

- Open the individual wrapping, where written; "open"
- Add the antiseptic solution to it and distribute soap solution equally on hands. Use the scrub only for nails and between fingers. Use the sponge for the skin areas.
- Brush and use sponge to disinfect starting from fingers, ending before reaching elbows. Take special care of areas between fingers and nails.
- Clear hands and forearms with running water holding up hands always higher then elbows. The entire procedure lasts 5 minutes.
- Discart scrub brush after use, as by residual protocols and local law application.



**LATEX FREE**

Not following the instructions may cause in a improper use of the product. Coremec reserves the right to amplify, modify and to improve the present technical file.