

## CORESCRUB WITH PVP-IODINE 7,5%

Scrub brush / with sponge used for hand cleansing disinfection.

### Description

Scrub brush with antiseptic solution, single use, used for hand and forearm disinfection of sanitary personnel. Contains 20 ml antiseptic soap solution and nail cleaner.

### Indications

Device made off a nylon scrub brush, a sponge glued at the base and comes provided with a nail cleaner. Used for disinfection of sanitary personnel and surgeons' nails, fingers, hands and forearms, participating in surgery. Exclusively for external use and on healthy and unhurt skin. Used in preoperative areas only.

### Precaution

Avoid contact with eyes. Do not swallow. Do not use any package which presents defect sealing. DO NOT USE ON ANY OTHER PARTS OF THE BODY WHICH ARE NOT INDICATED IN THE PRESENT FILE.

### End-user


Operating room, Urgency, Reanimation centres , dentists,...

### Warehouse

To be protected from humidity and direct exposure to sunlight and at room temperature. Must be stored in clean environmental areas as by local regulation.

### Instructions of use:

- Open the individual wrapping, where written; "open"
- Use the antiseptic solution and distribute soap solution equally on hands. Use the scrub only for nails and between fingers. Use the sponge for the skin areas.
- Brush and use sponge to disinfect starting from fingers, ending before reaching elbows. Take special care of areas between fingers and nails.
- Clear hands and forearms with running water holding up hands always higher than elbows. The entire procedure lasts 5 minutes.
- Discard scrub brush after use, as by residual protocols and local law application.

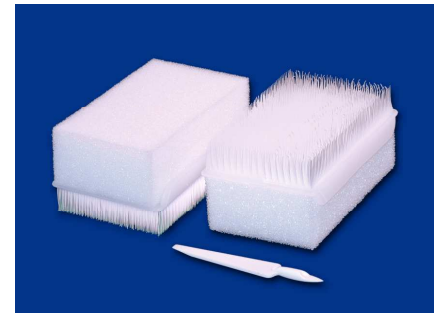

### TECHNICAL PROPERTIES

Material used	BRUSH: LPD polyethylene. SPONGE: Polyurethane -medium high density. NAIL CLEANER: PET				
Formula / Caducidad	Components	Concentration %	NUMERO CAS.	Risc sentences	Expiry
	Polivinilpirrolidone-iodine	7,5	25655-41-8	R-36/38	24 meses
	Lauril ether sodium sulfate	15	9004-82-4	R-36/38	
	Fosfate sódium	aprox. 0,4	7558-79-4	R-36/38	
	De-ionised water	100	7732-18-5	-	
Certificates	PMC Ministry of Health Italy: N° 19531- complies with EN 12054 and EN1650.				
Sponge dimensions	Lenght 80 mm. Wide 45 mm. Height 25 mm.				
Nail cleaner dimensions	Lenght 65 mm. Wide 5 to 10 mm.				
Packaging	Boxes of 270 pieces. CS-270-P 9 dispensers of 30 pieces; total: 270 pieces per outer box. CS-270-P				
Expiry date	Expiry date : 3 years				

### LATEX FREE

Not following the instructions may cause in a improper use of the product. Coremec reserves the right to amplify, modify and to improve the present technical file.