

Concentrated disinfectant
for non-invasive medical devices
with detergent action

CAUTION

Properties

GERMOCID INODORE is a concentrated solution that has a fast and effective cleaning and disinfecting action. It is active against Gram-positive and Gram-negative bacteria, fungi, mycobacteria and viruses (Adenovirus, HIV, HBV, HCV). GERMOCID INODORE is a specific product for the cleaning and disinfection of surfaces and non-invasive medical devices such as facial masks, dental assemblies and for the treatment of various equipment/ objects such as desks, armrests, handles of medical offices and dentists. The high efficiency and speed, both as a detergent and as a disinfectant in the presence of organic material (blood, saliva, organic exudates), allows it to be used on all dental instruments and not. Removing dirt does not require mechanical action.

Instructions of use: Use the product diluted to 5-10% for 15' contact for a bactericidal activity, 10% for 15' for a fungicidal activity, 10% for 60' for a mycobactericidal activity, 1% for 60' for a virucidal activity vs Adenovirus, 15% for 60' for a virucidal activity vs HIV, HVC, HBV. Apply with sponges, cloths, sprinklers, manual or mechanical pumps or any other means on walls and washable surfaces. Leave to act and rinse. The diluted solution is stable for one week if it is used correctly, without contamination by substances and/or external agents.

Composition: 100,0 g of product contain: Benzalkonium chloride 10,0 g; O-phenylphenol 1,0 g; co-formulants; purified water q.b. to 100,0 g.

Warnings: H314 Causes severe skin burns and eye damage. H400 Very toxic to aquatic life. H411 Toxic to aquatic life with long lasting effects. P260 Do not breathe dust/fume/gas/mist/vapours/ spray. P280 Wear protective gloves/protective clothing/eye protection/face protection. P303-P361-P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. P305-P351-P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P310 Immediately call a POISON CENTER/doctor/ Contains: Quaternary ammonium compounds. Shake before use. The product is incompatible with detergents and anionic surfactants. Store in a cool, dry place away from heat sources. After opening, the product retains its label validity if properly stored. Do not use damaged packaging. For professional use.

Detergent in accordance with Regulation (EC) 648/2004

Validity: 5 years

Lot and expiry date: see packaging

MEDICAL DEVICE CE 0546 European Directive 93/42/ECC

Manufacturer:

GERMO S.p.A. - Via Giotto 19/21 - 20032 Cormano (MI) Tel. 02 663 01 938

www.germo.it - info@germodis.com

MADE IN ITALY

Warnings may change with updates to the product safety data sheet