

**Alcohol-based Cleaner Disinfectant for non-invasive Medical Devices
Ready to Use****CAUTION****Properties**

GERMOCID BASIC is a hydroalcoholic solution based on Chlorhexidine digluconate and QAS. It has a broad spectrum of action and is ready to use for the rapid and effective disinfection of non-invasive medical surfaces and devices. It is suitable for use over large surface areas or those which are difficult to access on biomedical equipment and dental units (face masks, table tops, armrests, headrests, etc.). GERMOCID BASIC does not need to be diluted and provides rapid and prolonged protection against: viruses (Adenovirus, HIV, HBV, HCV), bacteria (Gram-positive, Gram-negative), mycobacteria (including *Mycobacterium tuberculosis*) and fungi (yeasts and molds). It is also effective in the presence of organic material (blood, saliva, organic exudates, etc.). The product evaporates rapidly. No corrosive effects were observed on the construction materials of medical devices or biomedical equipment. Do not use on materials which are sensitive to alcohol.

Instruction to use: Shake before use. Spray GERMOCID BASIC directly onto the surfaces and equipment to be treated. Leave the solution to act, then wipe away with a disposable cloth or paper.

GERMOCID BASIC is active against Gram-positive and Gram-negative bacteria, against fungi (yeasts and molds) and viruses (including HIV, HBV, HCV) within five minutes of contact. It is active against mycobacteria (including TB) within 60 minutes of contact.

Composition:

100.0 g of product contain:

Chlorhexidine digluconate 0.1 g;

Benzalkonium chloride 0.1 g;

Ethyl alcohol 45.0 g;

Isopropyl alcohol 15.0 g;

co-formulants;

purified water as required to 100.0g

Warnings: H226 Flammable liquid and vapour. H319 Causes serious eye irritation. H412 Harmful to aquatic life with long lasting effects. P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. Do not smoke. P280 Wear protective gloves / protective clothing / eye protection / face protection. P370+P378 In case of fire: use carbon dioxide, foam, chemical powder to extinguish. P337+P313 If eye irritation persists: Get medical advice/attention. P273 Avoid release to the environment. For professional use. Do not use on materials which are sensitive to alcohol. The product is not compatible with detergents and anionic surfactants. Once opened, if properly stored, the product retains the expiry date shown on the label. Do not use damaged packs. Store in a cool, dry place away from heat sources. For external use.

Validity: 3 years

Batch and expiry: see packaging

CE MEDICAL DEVICE 0546 Council Directive 93/42/EEC

Manufacturer:

GERMO S.p.A. - Via Giotto 19/21 - 20032 Cormano (MI) Tel. 02 663 01 938

www.germo.it - info@germodis.com

MADE IN ITALY

Warnings may change as updates are made to the product safety data sheet