

MULTILAYER DECONTAMINATING MAT

30 NUMBERED ADHESIVE SHEETS WITH ANTIBACTERIAL


REF 36696


DIMENSIONI / size 45x115 cm

CONTENUTO / content 5 TAPPETI / mats

COLORE / colour BIANCO / White

LOT 000008

 03 / 2019

 03 / 2022



INSTALLATION AND OPERATING INSTRUCTIONS

1. CLEAN AND DRY UP THE SURFACE WHERE THE MAT IS GOING TO BE PLACED

2. REMOVE THE LOWER PROTECTIVE SHEET

CAUTION: WHILE REMOVING THE LOWER PROTECTIVE SHEET MAKE SURE NOT TO DETACH THE ADHESIVE SHEET THAT HAS TO STICK THE MAT TO THE GROUND

3. MAKE THE MAT SMOOTHLY ADHERE TO THE FLOOR

4. REMOVE THE UPPER PROTECTIVE SHEET

5. PROGRESSIVELY REMOVE EACH OF THE 30 DISPOSABLE ADHESIVE

CAUTIONS

- FOR INDOOR USE ONLY
- INSTALL ON SMOOTH AND FLAT SURFACES ONLY
- AVOID CONTACT WITH WATER AND TRANSIT WITH WET SOLES: EXTREMELY SLIPPERY WHEN WET
- CARRY AND STORE IN THE ORIGINAL PACKAGE
- DO NOT FOLD THE PACKAGE
- STORE IN A DRY, COOL PLACE AWAY FROM STRONG HEAT SOURCES
- PROTECT FROM RAIN AND DIRECT SUNBEAMS
- DISPOSE IN RESPECT OF LOCAL LAWS AND REGULATIONS CONCERNING PLASTIC MATERIALS (LOW DENSITY POLYTHENE)



GIMA SPA
Via Marconi, 1
20060 Gessate (MI) - Italy

