



**GIMA**

PROFESSIONAL MEDICAL PRODUCTS

---

***TAPPETINO IN SILICONE***  
***SILICONE MAT***

---

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

**Intended use/ function:**

- Device shall be used in autoclave process.
- Silicone mat securely and gently holds instruments in place inside a sterilization tray. Can be used as is or cut to size to fit different trays. Perforations help with air circulation, aiding in the drying process.

**Component Specification:**

This device is manufactured from Silicone.

**Cleaning / Sterilization:**

All Silicone Mat have to be stored in a dry and dust free environment. Products supplied by GIMA S.p.A. are non-sterile when shipped.

**Steam Sterilization:**

- Steam sterilizer in accordance with DIN EN 13060 or DIN EN 285
- Maximum sterilization temperature 134°C/273°F; in accordance with relevant standards and regulations.
- Sterilization time (exposure time at the sterilization temperature) minimum 5 min at 134°C/273°F.












**Cleaning Agent:**

When choosing the cleaning agent and disinfectant ensure that they do not contain the following components:

- Organic, mineral or oxidizing acids (maximum permitted Ph 9.5, a neutral/ enzymatic cleaner is recommended)
- Strong alkali
- Organic solvents
- Oxidation agents (e.g. hydrogen peroxide)
- Halogens (chlorine, iodine, bromine)
- Aromatic/ halogenated hydrocarbons
- Heavy metal salts

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

	Keep in a cool, dry place		Lot number
	Product code		Caution: read instructions (warnings) carefully
	Medical Device compliant with Regulation (EU) 2017/745		Manufacturer
	Keep away from sunlight		Consult instructions for use
	Medical Device		Non-sterile
	Date of manufacture		

**REF 37320 - 37321 - 37323**



**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)  
Made in Pakistan

