

CENTRIFUGE TEST TUBES

CE IN-VITRO DIAGNOSTIC DEVICE
For professional sanitary use

Type of specimen: blood, serum or urine sample

INSTRUCTIONS FOR USE

- 1) Verify product information and package integrity.
- 2) Write on the label the adequate information (patient name, test, sample etc.).
- 3) Open the centrifuge test tube.
- 4) Using the appropriate tool, put the sample in the tube for dilution, storage, centrifugation, bacterial culture or other application.
- 5) Close the centrifuge tube with its cap effectively to avoid any leakage during centrifugation.

USE WARNINGS

- Device intended for use by healthcare professionals;
- Use protective gloves.
- Do not place the device near a flame.
- Do not use the product if expired, damaged or with opened package.
- Do not change the intended use, any other use is considered improper.
- Keep out of reach of children.
- Use the device only with suitable accessories.
- Store in a dry place, away from heat sources and direct sunlight.
- Dispose of this product in accordance with local regulations, in a biohazard waste container.
- The device is disposable. Reuse of the device will result in the loss of all performance and characteristics.
- Any serious incident occurring with the device shall be reported to the manufacturer and to the Competent Authority of the country in which the patient resides.



F.L. Medical s.r.l. Unipersonale – Via Enrico Mattei, 20- 35038 Torreglia (PD) – Italy
Phone +390495211475 – Fax +390495212566 – mail info@flmedical.com

