




GIMA

PROFESSIONAL MEDICAL PRODUCTS

KIT PODOLOGIA CON ASTUCCIO RIGIDO - 7 STRUMENTI - PODIATRY STEEL FRAME KIT - 7 PIECES KIT PODOLOGIE AVEC ÉTUI RIGIDE - 7 UNITÉS - SET PODOLOGIE MIT FESTEM ETUI - 7 INSTRUMENTE KIT DE PODOLOGÍA CON ESTUCHE DE ACERO – 7 UNIDADES - KIT DE PODOLOGIA COM ARMAÇÃO EM AÇO - 7 PEÇAS

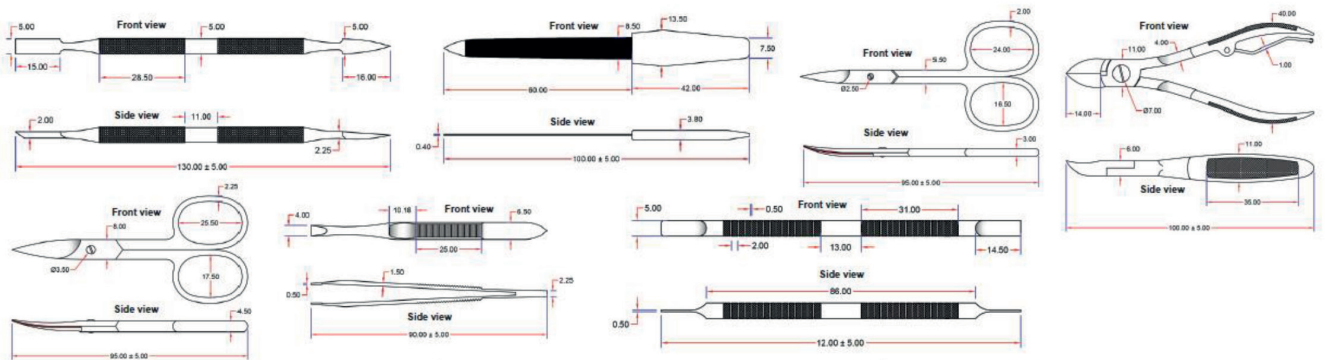

Precautions, restrictions, warnings, special cares

Methods of Proper Cleaning

Proper cleaning requires liquid soap/detergent, water and the use of a clean and disinfected scrub brush to remove all visible debris and residue.

All items should be scrubbed with a clean and disinfected scrub brush under running water. Cleaning is not disinfection; disinfection is an entirely separate step. Different items are cleaned in different ways. This often depends on the item manufacturing material and how the item has been used.

NOTE: the cleaning step must be properly performed before an item can be disinfected. All items must be thoroughly rinsed and dried with clean cloth or paper towels prior to putting them into a disinfectant.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

Description of Devices

Name: Cuticle Pusher and cleaner, 13cm

Name: Nail File With White Plastic Handle, 10cm

Name: Cuticle Scissors, 9.5cm

Name: Nail Nipper, 10cm

Name: Nail Scissors 9.5cm

Name: Tweezers, 9cm

Name: Cuticle Pusher Double Ended, 12cm

Indication of Use



- Put away from the reach of uncontrolled person.
- A trained person shall use the devices.
- For professional use only – disinfect before use.

REF 39805

Importato da / Imported by / Importé par /
Importiert von / Importado por / Importado por:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com
Made in Pakistan