

CALZE A COMPRESSIONE FUNNY
FUNNY COMPRESSION SOCKS
CHAUSSETTES DE COMPRESSION FUNNY
CALCETINES DE COMPRESIÓN FUNNY
MEIAS DE COMPRESSÃO FUNNY
FUNNY KOMPRESSIONSSTRÜMPFE
AΣΤΕΙΕΣ ΚΑΛΤΣΕΣ ΣΥΜΠΙΕΣΗΣ
FUNNY KOMPRESSIONSSTRUMPOR
FUNNY SKARPETKI UCISKOWE

Cat. No. 6115100000 (GIMA 41200)

6115100000 (GIMA 41201) 6115100000 (GIMA 41205) 6115100000 (GIMA 41206)

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante / Hersteller / Κατασκευαστής / Tillverkare / Producent Shanghai Amigo Textile Co., Ltd. Room 301, Lane 222, Yehui Road, Qingpu District, Shanghai Made in China



**ENGLISH** 

## **INSTRUCTION FOR USE**

## **Compression socks**

Thank you for having chosen a Shanghai Amigo product. Our products are synonymous with comfort and personal wellness.

The pleasant feeling of lightness and wellness is obtained thanks to the dynamic effect of the product's graduated compression.

The compression decreases from the ankle upwards.

The heel section provides for:

- The proper placement of the heel in the heel pocket thus providing for greater stability.
- Proper positioning at the maximum compression point, as well as a better fit and greater wearability and elasticity around the circumference of the ankle.

Shanghai Amigo products are not medical devices.

They are particularly effective for relieving the sensations of heaviness and fatigue in the legs.

Instructions and recommendations for wearing compression socks It is recommended to put the compression socks on as soon as you get up in the morning.

The compression socks should be protected against contact with any oils, creams or ointments applied to the legs for medical or cosmetic reasons.

Rough fingernails or rings with rough edges could potentially damage the product's fabric.

If the skin is wet, apply talcum powder to the foot in order to facilitate its insertion into the socks.

- Turn the socks inside out, with the exception of the foot section. It is best to turn the socks inside out to the point that the heel remains outside.
  - -Use both hands to widen the foot portion and slide the foot into the socks up to the heel.
  - -Turn back the inside-out portion of the socks, taking care to properly position the heel.
- 2. Insert the rest of the leg up to the knee, smoothing out the fabric with the palm of your hand.
  - Never pull directly on the top edge of the socks.
  - -In the case of tights, follow the same procedure as described above until the tights arrive at the level of the knee on both legs.
- Roll the remaining portion of the tights upward to the crotch, smoothing the fabric with your hands and checking to make sure the compression is uniform.

### **General contraindications**

Advanced state peripheral arterial occlusive disease - Massive oedema of the lower limbs of a pathological nature - Septic phlebitis

- Severe phlebitis, such as phlegmasia cerulean dolens Lymphangitis
- Advanced peripheral neuropathies (i.e., diabetes mellitus) Other sensory disorders of the limbs Intolerances/allergies related to the material of the compressive stockings.

# Relative contraindications

Ulcers and open wounds - Moist dermatosis - Immobility - Dermatitis stasis - Valvular incontinence of the superficial and deep venous sy-



stem. - Deep vein thrombosis or acute thrombophlebitis - Post thrombophlebitis syndrome – For socks with silicone edges, skin irritation may arise in cases of hypersensitivity to the material itself

#### **Special warnings**

Shanghai Amigo compression socks should only be worn after consulting a physician.

In the event that pain or skin irritation should occur when wearing the compression socks, remove the socks immediately and consult a physician or your retailer.

### Composition of the materials

Nylon 65% - polyester 30% - spandex 5%

Available sizes: small - medium - large - extra large

### **Washing instructions**

Socks be washed by hand or in the washing machine.

- Hand washing: use detergents for delicate fabrics. Rinse thoroughly. Do not wring. Air dry.
- Washing machine: place the garment in an appropriate bag or in a pillow case. Do not bleach. Do not machine dry.
- Do not dry clean. Do not iron.



## Warning

The manufacturer shall bear no responsibility for any personal injuries and/or property damage that may occur in the event that the product is used improperly or for purposes other than those for which it is intended.

#### **GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.