SEDIA DA DOCCIA con schienale e braccioli in PU
SHOWER CHAIR with PU backrest and seat
CHAISE DE DOUCHE avec dossier et siège PU
SILLA DE DUCHA con respaldo y asiento en PU
CADEIRA DE DUCHE com encosto e assento em PU
DUSCHSTUHL mit PU-Rückenlehne und -Sitz
DUSCHSTOL med ryggstöd och sits i PU
ΚΑΡΕΚΛΑ ΜΠΑΝΙΟΥ με πλάτη και κάθισμα PU

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All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

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REF HE10688023 (Gima 42909)

HEBEI HEALTHPLUS MEDICAL DEVICE CO.,LTD
NO.1,Chuangye Street,Matou Industry Park,HanDan City,HeBei
Province,China
Made in China

Y.Sung Handelsvertretung
Dusselthaler Str. 24, 40211
Dusseldorf Germany

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

HE10688023

KD ALUMINUM SHOWER CHAIR

Parts of the product

1. PU Backrest
2. PU Seat
3. Height adjustable foot tube
4. Tip

Safety instructions


- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use.
- Approved for indoor use only.
- Clothing or body parts can be pinched when sitting down, standing up, inserting the toilet bucket, as well as when raising the armrests and locking them into place.
- If you need to move over thresholds, do so slowly and carefully, otherwise the wheels can become damaged!
- When using a wheeled commode, always use the parking brakes.
- During transfer into/out of the wheeled commode, the footrests must be swiveled out of the way.
- Never use the footrests as assistance when getting up - risk of tipping!
- Prevent unauthorized use, for example, by children.
- Take note of the permitted maximum weight of the user.
- This information can be found in the technical data in these instructions for use as well as on the identification plate

Assembly

Step 1

Attach backrest to the frame, by depressing push buttons and sliding the backrest pipes into the tubes located on each side and on the back of the seat.

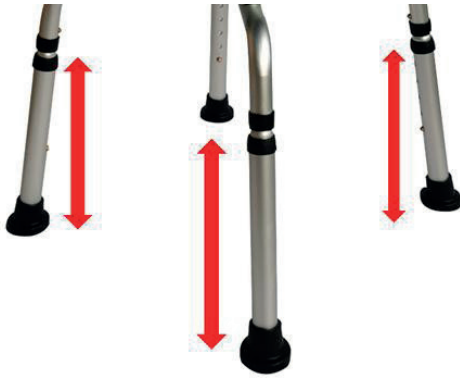**Step 2**

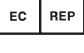
Attach the legs to the frame by depressing push buttons into the tubes located on each side of the legs.

Make sure that push button “snap” correctly through holes and are stable and adjusted the same way.

Technical data

Product	KD ALUMINUM SHOWER CHAIR
Product number	HE10688023
Weight capacity (lbs/kg)	300 lbs/ 136 kg
Product dimension (mm) L x W x H	(505-516)×(500-526)×(735-835)mm
Product weight (lbs/kg)	4.02kg
Seat dimension (inch/mm) L x W	407*337*52mm
Backrest material	PU
Seat material	PU
Frame material	Aluminum
Frame color	Sliver
Quantity in box (piece)	1
Product weight including box (lbs/kg)	5.74kg
Carton dimension L x W x H	460×200×430mm

Symbols

	Caution: read instructions (warnings) carefully		Keep in a cool, dry place
	Manufacturer		Date of manufacture
	Lot number		Product code
	Medical Device compliant with Regulation (EU) 2017/745		Authorized representative in the European community
	Medical Device		Consult instructions for use
	Serial number		Imported by

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.