



PROFESSIONAL MEDICAL PRODUCTS

STICK WITH SEAT

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

INTRODUCTION

Stick with seat, made of aluminium which is light and resistant. The tip is covered with non-slip rubber.

SAFETY RULES

- Always contact your specialist dealer to learn how to use it appropriately.
- Do not store outdoors, do not expose it to prolonged periods of sunlight.
- Always move slowly, taking small steps to ensure that the weight distribution is even and that the position is stable and safe.
- Do not use on slippery surfaces (wet floors, snow, ice) or on stairs.
- Do not use for purposes other than those for which this product has been designed.
- Always comply with the weight limits indicated on the label or in the instructions.
- Use only original tips for repairs.
- Only qualified personnel are authorised to repair this item.
- Should you not use the device for long periods of time, have it checked by your specialist dealer.
- Use caution when using on unstable and uneven ground.
- Check the wear of the non-slip rubber and replace it if necessary.

USE









To sit down, slide the chair from top to bottom.

MAINTENANCE


- The product must be cleaned regularly with a clean and dry cloth.
- The handle must be cleaned with a neutral detergent (do not use corrosive products).
- The ends must be checked and changed regularly in case of wear.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

 REF	Product code		Consult instructions for use		Manufacturer
 LOT	Lot number		Medical Device compliant with Regulation (EU) 2017/745		Medical Device
	Keep in a cool, dry place		Caution: read instructions (warnings) carefully		Date of manufacture
	Keep away from sunlight		Authorized representative in the European community		

 **FS911L (43060)**

 **Foshan Dongfang Medical Equipment Manufacturer (LTD.)**
 5 Xingfu Road, Zone A, Shishan Industrial Park,
 Nanhai District, Foshan City, China, 528000
 Made in China

 **MedNet EC-Rep GmbH**
 Borkstrasse 10. 48163 Muenster. Germany

Importato da / Imported by:
Gima S.p.A.
 Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

