

**ENGLISH**
**LIGHT ALLOY WALKING STICK**

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

**SAFETY WARNINGS**

This stick is not a tetrapod walking stick! It must be used like any walking stick and must not be intended as an aid designed to increase the patient's stability. The function of the 4 feet is a convenience that serves only to allow the stick to stand up by itself when it is not being used.

**Never use a damaged device**
**INSTRUCTIONS FOR USE**

Height adjustment procedure. (Fig. 1)

1. Disengage the locking spring. (See Fig. 2)
2. Adjust the height by removing the telescopic tubes.
3. Reposition the spring clip.

**ATTENTION**

Do not use tools to remove the fixing spring as it could get damaged. The spring is hard to remove for safety reasons. (Fig. 2)


**CLEANING AND MAINTENANCE**

**Cleaning:** regularly clean the handle with neutral soap; do not use alcohol or solvents.

**Tip:** The tip resting on the ground (Fig. 3) is subject to wear; we recommend replacing it periodically using **original OPO spare parts**.

**GIMA WARRANTY TERMS**



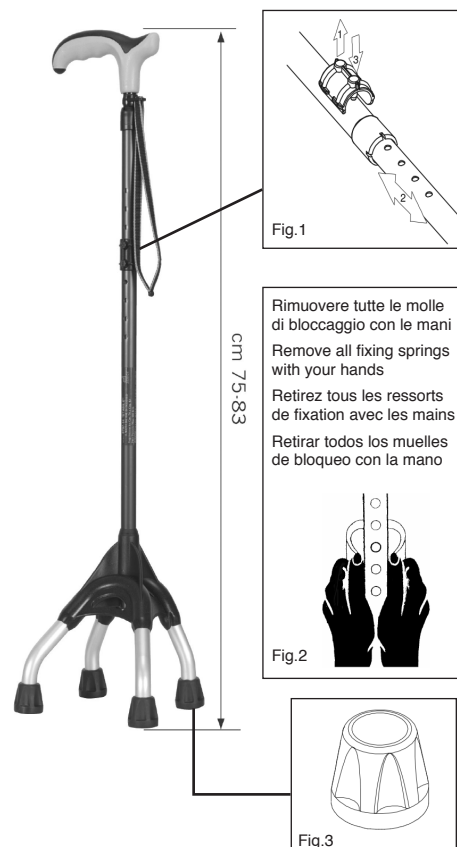






The Gima 12-month standard B2B warranty applies.




Rimuovere tutte le molle di bloccaggio con le mani  
 Remove all fixing springs with your hands  
 Retirez tous les ressorts de fixation avec les mains  
 Retirar todos los muelles de bloqueo con la mano



Peso Max. utente  
 Max. user weight  
 Poids maximal de l'utilisateur  
 Peso Máx. usuario  
**100 kg**

	<p>IT Dispositivo medico conforme al regolamento (UE) 2017/745          GB Medical Device compliant with Regulation (EU) 2017/745          FR Dispositif médical conforme au règlement (UE) 2017/745          ES Producto sanitario conforme con el reglamento (UE) 2017/745</p>		<p>IT Codice prodotto          GB Product code          FR Code produit          ES Código producto</p>		<p>IT Leggere le istruzioni per l'uso          GB Consult instructions for use          FR Consulter les instructions d'utilisation          ES Consultar las instrucciones de uso</p>
	<p>IT Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso          GB Caution: read instructions (warnings) carefully          FR Attention: lisez attentivement les instructions (avertissements)          ES Precaución: lea las instrucciones (advertencias) cuidadosamente</p>		<p>IT Numero di lotto          GB Lot number          FR Numéro de lot          ES Número de lote</p>		<p>IT Fabricante          GB Manufacturer          FR Fabricant          ES Fabricante</p>
	<p>IT Conservare in luogo fresco ed asciutto          GB Keep in a cool, dry place          FR À conserver dans un endroit frais et sec          ES Conservar en un lugar fresco y seco</p>		<p>IT Data di fabbricazione          GB Date of manufacture          FR Date de fabrication          ES Fecha de fabricación</p>		<p>IT Dispositivo medico          GB Medical Device          FR Dispositif médical          ES Producto sanitario</p>

**REF** 0767/B (43075)

 Officina Prodotti Ortopedici S.r.l.  
 Via Bellini 33, Rastignano (Bologna - Italy)  
 Made in Italy

