GIMA

PROFESSIONAL MEDICAL PRODUCTS

ENGLISH

3-LEG WALKING STICK (TRIPOD)

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

INTENDED USE

Tripod walking sticks are **Class I medical devices** that are aids for patients with walking difficulties who require a stick that provides greater stability than those of a more conventional nature. Using the walking stick will vary from person to person. A minimum of patient training by medical or health care professionals is, however, required.

CONTRAINDICATIONS AND GENERAL WARNINGS

! Using and adjusting the walking stick will depend on the user. When using for the first time, medical personnel are required to adjust the height and provide training in its use. An evaluation of suitability by medical personnel is required for persons with psychomotor disorders or those with prostheses or orthoses in their lower limbs.

In some persons prolonged use may lead to issues in the joints of the arm when using the aid. If any pain is felt in the hand, wrist, elbow or shoulder, stop using the aid and seek medical advice.

Do not use in direct contact with injured tissue

DO NOT use the device if it is damaged.

ADJUSTING THE HEIGHT

Adjustment height is shown on the label.

Proceed observing the steps in Fig.1:

1. Release the spring clip (see note in Fig.2)
2. Release the fastening ring nut
3. Adjust the height by sliding the tubes and align the holes
4. Reposition the spring clip
5. Tighten the ring nut.

Note: Do not use any tools to remove the spring clip as this may cause it to be damaged.

CORRECT USAGE

Correct usage of the walking stick is shown in Figure 3.

In order to ensure stability, users of the walking stick will have to walk using short strides to **keep the stick supported with all four of its feet on the floor at all times.**

WARNINGS

- Beware of wet, slippery or sloping surfaces.
- Take care when using on stairs by ensuring that all four feet are able to rest on the step.
- The aid is not suitable for use on soft ground or uneven surfaces. Use only on flat surfaces.

RE-USE

The aid is reusable after checking that it is structurally sound and that the hand grip does not have any defects. It also needs to be sanitised and the ferrules replaced.

MAINTENANCE AND CLEANING

Replacing the ferrules: The ground support ferrules are subject to wear and tear. Frequent replacement is recommended using original spare parts.

Disinfecting: Use sodium hypochlorite-based disinfectants at low concentrations to disinfect the hand grip.

Cleaning: Clean the hand grip frequently using mild soap.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

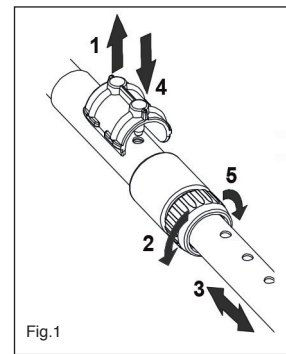
Fig.1

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Officina Prodotti Ortopedici S.r.l.