



Indicateur d'usure / Wear indicator / Verschleißanzeiger



920115  
920117



Evolution  
200 \*\*\*



Advance  
20 \*9\*\*



Progress I  
201 7\*\*



Progress II  
200 7\*\*



Tiki Turquoise-Purple: 200 510  
Tiki Orange-blue: 200 520



Désignation Designation Bezeichnung	Hauteur poignée Height handle/floor Griffhöhe	Angle poignée Support angle Winkel der Armauflagefläche	Réglage poignée Adjustable Handle Verstellung des Griffs	Largeur Cuff width Breite der Armschalen	Longueur poignée Handle length Grifflänge	Poignée Handle Griff		
Advance	76 - 101 cm	28 °	-	11 cm	10 cm	☉ 3.5 cm	0.52 kg	140 kg
Evolution			22 - 29 cm					130 kg
Progres I				150 kg				
Progres II			150 kg					
Tiki	55 - 77 cm	20°	11 - 21 cm	7 cm		☉ 3.2 cm	0.33 kg	100 kg

**Please** read these notes carefully before use. Make sure to follow the instructions and keep them in a safe place ! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

### 1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all parts are tightened (clips, handle, rubber tip..).
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..)!!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way : don't use as a lever or to move furniture.
- ♥ ALWAYS observe the weight limit on the labelling of your walker.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff, before using.
- ♥ Be careful when using: beware holes and deformations in the floor.
- ♥ The suction cap must not be too worn down. If the cap shows considerable wear, ask your supplier or nursing staff to replace it.

### 2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Height Adjustment : Take out the clip from the tubes using both thumbs, then adjust the interior pipe to the desired height and lock again with the clip. Before using the crutch, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe.

The crutches have 2 adjusting holes on the outside pipe and 5 holes on the inside pipe, what means 10 adjusting positions.

The crutch can be adjusted every 2 cm.

Keep the crutch close to your body while going to avoid accidents.

### 3. Cleaning

The crutch has to be cleaned regularly with a mild cleaning agent and a soft cloth. NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the walking aid regularly.

The rubber tips must be checked and changed regularly.

### 4. Materials

Tube in anodised aluminium and polypropylen handle. The Advance model has an handle covered with a soft material.

### 5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

### 6. Technical specifications

This product has a 24 months warranty, from the date of purchase, against any manufacturing defect. Standards: DIN-EN-12182, EN-ISO 11334-1, DIN-EN-ISO 10993-1. Biocompatible.