



GIMA

PROFESSIONAL MEDICAL PRODUCTS

**ESERCITATORE A PEDALI
BASIC PEDAL EXERCISER
APPAREIL D'EXERCICE À PÉDALE DE BASE
PEDALEADOR DE EJERCICIOS BÁSICO
EXERCITADOR DE PEDAL BÁSICO
BEWEGUNGSTRAINER MIT PEDALEN
ROTOR DO ĆWICZEŃ
MINI BICICLEȚĂ PENTRU EXERCITII
ALAP PEDÁLOZÓ GÉP
ΒΑΣΙΚΗ ΠΕΤΑΛΙΕΡΑ ΓΥΜΝΑΣΤΙΚΗΣ**

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante
Hersteller / Producent / Producător / Gyártó / Παραγωγός:
ZHEJIANG TODO HARDWARE MANUFACTURE CO., LTD
No.31th, Jinmao Building, CBD Centre,
Yongkang, Zhejiang, China
Made in China

Gima 43150

REF TD001P-2

Importato da / Imported by / Importé par / Importado por / Importado por
Eingeführt von / Importowane przez / Importat de / Importálta / Εισαγόμενα από:


Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com





1

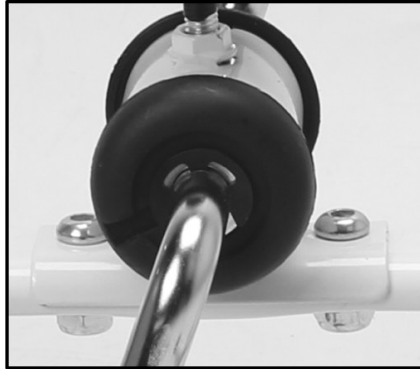

2



3

-
- ENGLISH**
1. Connect the tubes each side with screws like above.
 2. Fix the resistor onto the main frame with screws
 3. Centre the adjusting knob to hole and turn it into resistor to adjust exercise intensity

Notice: Put the product onto the smooth floor. You can exercise by hand or feet clockwise or anticlockwise.

Warning: Please don't stand on the product directly (max 40Kg). You can sit down to exercise your feet.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.
